

Car Seat Slide Pad



- The Car Seat Slide Pads will assist frail people getting in and out of cars
- It consists of a Top Pad and Bottom Pad. The person can slide their bottom back into the middle of the seat and then turn themselves around
- If they need assistance, a Carer can push them back into the seat and lift their feet into the car. If necessary, use the webbing positioning handles to turn the person to face the front correctly
- You must still do up the seat belt when driving
- Wash up to 80°C (176°F). Air or tumble dry on a cool setting, ensuring the drum is cool. Ensure the product is dry before using or storing

✔ Contracted DVA product

✔ Contract approved

Support @ Home

Function: Changing body position

Category: Prescribed

Click [here](#) for more information

Code	Display Name	Total Length	Total Width
TSE672970	Car Seat Slide Pad	580 mm	580 mm