

Oppo 1289 Wrist/Thumb Support w/Strap



- Three removable metal splints in thumb area for improved stability
- Additional strap for stabilisation during activities
- Supports the wrist and thumb, leaving the palm free
- Ideal for sprains of thumb, ligament support, and minor fractures of the thumb
- **How To Wear:**
 - Position thumb on the thumb locking area of support.
 - Be sure the metal splints are on the lateral side of thumb.
 - Secure the hook and loop strap beside thumb area to desired compression.
 - Pull the strap around the wrist and secure hook and loop strap to desired compression.

✔ Contracted DVA product

✔ Contract approved

Support @ Home

Function: Upper limb orthoses

Category: Prescribed

Click [here](#) for more information

Compliance

No applicable compliance standard

Warranty Information

- 1 Year

Code	Display Name	Material	Wrist Circumference
FBS301425	Oppo 1289 Wrist/Thumb Support - Small	Coolprene	13 - 15.2 cm
FBS3014252	Oppo 1289 Wrist/Thumb Support - Medium	Coolprene	15.2 - 17.2 cm
FBS3014253	Oppo 1289 Wrist/Thumb Support - Large	Coolprene	17.2 - 19.1 cm
FBS3014254	Oppo 1289 Wrist/Thumb Support - Xlarge	Coolprene	19.1 - 21 cm