

200 PATIENT LIFTER STEEL USER MANUAL

i



www.aspirecare.com.au 1300 133 120

TABLE OF CONTENTS

1. Overview	3
-------------	---

- 2. Safety Precautions _____ 4
- 3. Parts Glossary _____ 6
- 4. Assembly _____ 7
- 5. Preliminary Checks _____ 10
- 6. Lifter Operation _____ 10
- 7. Lifting Guide _____ 14
- 8. Sling Compatibility _____ 17
- 9. Care & Maintenance 18
- 10. Warranty & Service _____ 19
- 11. Troubleshooting _____ 20
- 12. Specifications 21
- 13. Service Log Book _____ 23

1. OVERVIEW

1300 133 120

The **ASPIRE 200 Lifter** is a compact and ergonomically designed heavy duty mobile patient lifter with a safe working load (SWL) of 200kg.

This mobile floor lifter is a safe care solution for in home care or institutional and aged care environments.

The ASPIRE 200 Lifter allows patients to be transferred from floor positions, beds, chairs and other support surfaces with comfort and safety.

Despite its compact design the hinged rotating yoke on the ASPIRE 200 Lifter generates more efficient patient space allowing a patient to be rotated through 360° at full lift height. This facilitates safe and easy transfers for both attendant and patient.

The ASPIRE range of patient lifting solutions feature industry leading LINAK Jumbo Care electronics and have been designed and tested to comply with AS/NZ ISO 10535: 2011.



2

2. SAFETY PRECAUTIONS

INTENDED USE OF THE PRODUCT

The ASPIRE 200 Lifter has been designed to lift and transfer human patients with the assistance of one carer or attendant.

All ASPIRE patient lifters should ONLY be used with Aidacare approved patient slings, which have been chosen and assessed by a competent trained health professional. For a complete list of all compliant lifting slings please contact Aidacare on 1300 133 120.

IMPORTANT - only approved slings and accessories are compatible with this lifter. If you are unsure please contact Aidacare on 1300 133 120.

The correct and appropriate use of the ASPIRE 200 Lifter reduces the risks associated with manual patient handling. The lifting and transferring of patients should always be assessed and planned by a trained health professional.



This is the direction of travel for which this lifter has been designed.

SAFETY CONSIDERATIONS

ALWAYS ensure that attendants have read this guide before operating this patient lifter.

ALWAYS use slings that are assessed and considered suitable by a trained healthcare professional.

ALWAYS ensure the area around the lifter and patient is clear of obstructions.

ALWAYS minimize the distance over which the patient is being transferred by positioning support surfaces as close as possible.

DO NOT operate the lifter without training in the safe operation of lifters and slings. IF IN DOUBT DO NOT USE .

DO NOT use the lifter where any doubt exists regarding the safety of the lifter including but not limited to wear or damage with particular importance to be placed on the yoke and lifter sling attachment points.

DO NOT use the lifter with slings showing any signs of wear or damage, including fraying.

DO NOT exceed the 200kg safe working load of the ASPIRE 200 Lifter.

DO NOT use the lifter as a means of transportation over distances in excess of 3 metres. It is not designed as a transportation device.



INTENDED USE OF THE PRODUCT

WARNING: DO NOT manually lift the boom of the lifters.

How to Use Correctly:

The boom of the lifter is designed to be lifted only by an electric actuator.

Aidacare Lifters and Standers do not pull downwards during lowering. This safety spline feature is designed to avoid entrapment and crush injuries.

This safety feature means that it is physically possible to raise the lifter boom by hand on some models with LA31 actuators fitted.

NEVER raise the lifter boom by hand or any other means except under direct drive from the electric actuator fitted to the lifter. Raising the boom can cause it to become stuck in an elevated position. Under load this can result in patient free fall and risk catastrophic injury to the patient and/or carer.

ALWAYS use the handset or control unit buttons to raise and lower the boom of the lifter. This will ensure correct and safe functioning at all times.

3. PARTS GLOSSARY



1	Boom
2	Quick Release Yoke Attachment
3	Yoke
4	Battery
5	Control Unit
6	Emergency Lowering
7	Emergency Stop
8	Handset
9	Push Handle
10	Lift Actuator
11	Mast Height Adjustment
12	Base Fastening Lever Bolt
13	Cover for Leg Actuator
14	Leg Spread Actuator
15	Rear Castor with Brake
16	Front Castor

4. ASSEMBLY

1. Unpack

Remove all parts from carton.



2. Set up mast

Insert mast into base mount and secure with base fastening lever bolt.



3. Apply thread locker / loctite

a) Locate actuator mounting hardware.



b) Install actuator hardware. A few drops of medium strength thread locker / loctite must be applied to the bolt threads.





4. Mount handle

Mount push handle onto mast in position shown and fasten screws with Allen key.



5. Attach the quick release yoke

Pull down quick release lever, engage with attachment point, release lever. Check security latch is closed.



6. Connecting the actuator

a) Connect grey actuator lead into port (1) located on base of the control unit.



b) Connect the other end into the female lead from the actuator.



7. Connecting the handset

Connect grey handset lead into port located on base of control unit.



8. Connecting the Leg Spread Actuator

Connect black actuator lead port (2) on base of control unit.





5. PRELIMINARY CHECKS

After charging the battery these checks should be completed prior to initially using the lifter:

- 1. Check handset functionality is correct.
- 2. Check to see that all indicator lights are functioning.
- 3. Check that the emergency stop button causes the lifter to stop.
- 4. Check emergency lowering function.
- 5. Check that all leads are inserted, fastened and clear of the rear castors.

6. LIFTER OPERATION CONT.

Emergency Lowering

To activate emergency lowering lift up red tab located on top of actuator. This allows the patient to be fully lowered in the event of a fault in the lifter or discharged batteries.



6. LIFTER OPERATION

Handset

Features:

- 1. Raise / Lower function buttons
- 2. Leg spread function buttons (if applicable)
- 3. Battery charge indicators
- 4. Service indicator
- Press and hold the raise / lower or leg spread (if applicable) function buttons to activate. Release to deactivate. Beware of obstacles at all times to ensure boom and legs can move freely.
- Battery charge indictors will flash when battery reaches maximum recommended discharge of 18 volts, while the lifter is under load.
- Service indictor will illuminate when lifter has reached recommended service interval, either 12 months since last service or 8000 lift cycles. At this time the lifter needs to be serviced for safety and to maintain applicable warranty.
 - A full load test should be completed at least every 12 months by an Aidacare service technician.



Charging the battery

1300 133 120

The ASPIRE 200 Lifter is supplied with a LINAK external wall mount charger and is compatible with standard Australian power outlets.

The following steps should be completed to recharge the battery pack correctly:

a) Remove the battery pack from the lifter via the top mounted release lever.

b) Connect the battery pack to the charger.







6. LIFTER OPERATION CONT.

Charger will automatically activate the charging sequence and a light will illuminate to show charging has commenced.

CHARGING INDICATOR DISPLAY

GREEN LIGHT indicates mains power is connected to 240 v power supply.

ORANGE LIGHT indicates battery is charging, and will turn off automatically when battery is fully charged.



CHARGING DOS AND DON'TS

ALWAYS

- Charge the battery whenever possible, to maximize battery life. Allowing batteries to fully discharge can reduce the useful life of a battery.
- Always inspect all cables particularly the mains power cable on the charger for any damage; replace if any signs of damage or wear are visible.
- Always stow the handset, charger and mains power cable when transporting the lifter to prevent loss and or damage.
- Always clean the actuators, control box, charger, battery and handset at regular intervals to remove dust/dirt.

NEVER

• Never allow the battery to fully discharge before recharging. Allowing batteries to fully discharge can reduce the useful life of a battery.

Castors

To **engage the castor brake** place your foot on the brake pedal and apply sufficient downward force to move it into the lock position.

To **disengage the castor brake** simply place the point of your toe underneath the brake pedal and lift-up.



Emergency Stop

To **engage the emergency stop** press the red button located on the control unit until it sits flush and remains in position.



To **disengage** the emergency stop function twist the red button clockwise until it pops out





7. LIFTING GUIDE

It is important that carers are familiar with manual handling techniques along with any sling instructions prior to completing any patient lifts. All patient's should have the correct sling and transfer techniques assessed by a qualified health professional. Explaining the lift process to the patient is important to ensure patient comfort and confidence and whenever possible two carers should conduct patient lifts for additional safety.

BED

1. Patient sling positioning

After rolling the patient on their side, position the sling along the full length of the back and head (if a head support model is being used).

The patient should be rolled into a supine position so that the sling can be retrieved and pulled out so that the patient is positioned centrally (see image)

2. Attaching the sling to the yoke

The lifter should be positioned with the legs sufficiently under the bed to allow the boom and yoke to be positioned over the patient's chest. Once in place engage the rear castor brakes and attach the loops of the sling to the hooks on the yoke.

3. Lifting the patient

Before commencing release the rear castor brakes so that the lifter can adjust its centre of gravity during the lift. As the loops pull tight check that the patient is still in a comfortable position and that the head support (if present) is providing correct support.

If the patient sling position remains correct, the lift can be commenced. Always try to position the two support surfaces that the lift is taking place between as close together as possible to avoid moving the patient over distances in the sling.







CHAIR

1. Patient sling positioning

Ensure that the patient is ready, then slowly lean them forward to allow the sling to be slid down behind the back until the bottom of the sling is positioned under the coccyx.

The sling leg straps can now be slid underneath each thigh, remaining careful to avoid creasing or bunching of the fabric. (Check with a health professional if leg straps should be crossed over for additional support or separated for enhanced accessibility for hygiene)



2. Attaching the sling to the yoke

Ensure that the lifter legs are open to the maximum width and slowly position it as close to the front of the chair as possible – be careful to keep the boom and yoke clear of the patient during this process.

Once the yoke is at chest height on the patient the sling loops can be attached

For chair lifts the rear brake castors should not be used to ensure the lifter can locate its centre of gravity easily



3. Lifting the patient

1300 133 120

Raise the yoke until the loops of the sling pull tight. Check that the patient is still comfortably positioned in the sling and that the head support is in place (if a head support model is being used).

Lift the patient only to a height needed to successfully transfer to the desired support surface.

Always try to position the two support surfaces that the lift is taking place between as close together as possible to avoid moving the patient over distances in the sling.





8. SLING COMPATIBILITY

FLOOR

Generally, a floor lift is only required in the event of a fall or emergency. Therefore it is important that a medical professional has assessed the patient and given the all clear for a lift to be carried out.

1. Patient Sling Positioning

After rolling the patient on their side, position the sling along the full length of the back and head (if a head support model is being used).

The patient should be rolled into a supine position so that the sling can be retrieved and pulled out so that the patient is positioned centrally (see image).



2. Attaching the sling to the yoke

The lifter should be positioned with the legs open to full width to ensure maximum area for the patient to be lifted.

One leg should be positioned adjacent to the patient's head and the other underneath the knees of the patient once their legs have been raised (as shown in the middle image on this page). The rear castors should be locked.

It is important to continually communicate with a patient during floor lifts as they may often be in shock or disoriented and need reassurance.

3. Lifting the patient

When carrying out a floor lift, the castors should be locked throughout the lifting procedure. As you begin vour lift and the loops pull tight, check that the patient is still in a comfortable position and that the head support (if present) is providing correct support.

If the patient sling position remains correct, the lift can be commenced. Lift the patient only to a height needed to successfully transfer to the desired support surface.

Always try to position the two support surfaces that the lift is taking place between as close together as possible to avoid moving the patient over distances in the sling.





AIDACARE manufactures a range of patient lifting slings designed for use with the ASPIRE range of electric lifting systems.

This lifter may also be suitable for use with other brands of slings provided that the design is compatible with a hook yoke/spreader bar design. To confirm if a sling is compatible with any of the ASPIRE range of patient lifting systems contact AIDACARE on 1300 133 120.







Deluxe General Purpose

Deluxe General Purpose with Head Support



1300 133 120






Mesh General Purpose with

Deluxe Access/Hygiene

Deluxe Access/Hygiene with Head Support





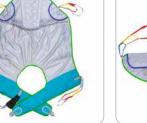

Deluxe Hammock Slings (Mesh & Super Soft Fabric Available)







9. CARE & MAINTENANCE

To prolong the working life of the lifter and for trouble free operation it is recommended that periodic inspections should be performed routinely by a person who is suitably qualified and well acquainted with the design, use and care of patient lifting lifters.

If inspection reveals that the safety of the lifter is jeopardized in any way the lifter should be IMMEDIATELY WITHDRAWN FROM SERVICE, tagged as such and the owner notified.

MAINTENANCE GUIDE & CHECK LIST

- Check castors are fastened securely to the frame
- Castors should run and swivel smoothly if hair or other fibres have built up within the castor this should be removed
- Check that the brakes on the rear castors function normally
- Clean surfaces with a pH neutral detergent chorine and phenol based detergents should not be used as these may damage lifter components
- All moving parts and pivot points should be checked for fatigue and lubricated if necessary
- All electrical components should be checked for normal function including battery and battery charger
- The actuator shaft should only be wiped with a cloth. DO NOT lubricate, contact Aidacare maintenance 1300 133 120

AS A MINIMUM A FULL SERVICE SHOULD BE COMPLETED EVERY 6 MONTHS AND THE LOAD TEST EVERY 12 MONTHS TO MAINTAIN WARRANTY.

ELECTRICAL TEST & TAG - WHERE LIFTERS ARE IN USE IN A COMMERCIAL ENVIRONMENT THEN THE TESTING REGIME APPLIED SHOULD ALIGN WITH THE RISK ASSESSMENT MADE BY THE OWNER FOR SAFETY AND OTHER COMPLIANCE REQUIREMENTS.

TO MAINTAIN AUSTRALIAN STANDARDS COMPLIANCE UNDER AS ISO 10535-2011 A FULL SWL TEST MUST BE COMPLETED EVERY 12 MONTHS.

10. WARRANTY & SERVICE

LIFTER AND ELECTRONICS

AIDACARE warrants its products to be free from defects in materials and workmanship under normal use and service and will within the period stated below, from the date of purchase, repair or replace without cost to the original customer, any part assembly or portion thereof which shall be returned to AIDACARE and from OUR inspection shows to be defective.

- 1. Patient Lifter Frame 5 years
- 2. LINAK Actuator and LINAK Controller 3 years (excluding Battery Pack)

3. LINAK Battery Pack – 12 month manufacturer's warranty subject to stringent fair wear and tear clauses (please contact AIDACARE for a copy of this information)

WARRANTY IS CONTINGENT UPON ANNUAL SERVICING BEING COMPLETED FOR THE LENGTH OF THE WARRANTY PERIOD.

AIDACARE cannot be held responsible for any personal injury, damage to the lifter or damage to property as a result of the improper or unsafe use of the product.

No warranty claim shall apply where the product or any part thereof has been modified, varied, altered or damaged either accidentally or through improper or negligent use.

No warranty claim shall apply where the lifter is repaired or serviced by parties other than those accredited and or approved by Aidacare.

Warranty does not extend to items or components which may require replacement due to normal wear and tear (e.g. castors, plastic molded components, foam padding (where fitted) and painted surfaces) for which AIDACARE nor its distributors can be held responsible.

BATTERIES

Batteries carry a limited warranty from the original manufacturer which is subject to a stringent wear and tear clause. Battery faults due to defect in original manufacture typically become apparent within the first two months of use.

Any gradual deterioration in performance after this period is typically associated with fair wear and tear, misuse and accidental damage, and as such is not covered by the LINAK manufacturer's warranty.

ALL WARRANTIES MADE ARE BACK TO BASE WARRANTIES MEANING THAT ANY PARTS OR WHOLE LIFTERS REQUIRING REPAIR NEED TO BE RETURNED TO THE NEAREST AIDACARE SERVICE CENTRE AT THE CUSTOMER'S EXPENSE – PLEASE CONTACT AIDACARE FOR A CURRENT LIST OF SERVICE CENTRE LOCATIONS.

SPARE PARTS

1300 133 120

SPARE PARTS LIST AVAILABLE UPON REQUEST FROM AIDACARE ON **1300 130 122 or www.aidacarelifters.com.au**



18

11. TROUBLESHOOTING

WARNING: TROUBLESHOOTING SHOULD ONLY BE CARRIED OUT BY AIDACARE AUTHORISED SERVICE TECHNICIANS WHO ARE TRAINED IN SERVICING THE ELECTRICAL AND MECHANICAL FEATURES OF THE PATIENT LIFTER.

The following is a guideline only to general troubleshooting. Before carrying out any troubleshooting:

- CHECK TO ENSURE the RED emergency stop button is NOT pushed in.
- CHECK TO ENSURE maintenance procedures have been followed.
- CHECK TO ENSURE all lead connections are secure.
- CHECK TO ENSURE the battery is fully charged.


PROBLEM	POTENTIAL FAULT	ACTION
Warning alarm can be heard in control unit	Battery is discharged Battery may be faulty	Recharge battery Replace battery
Power LED on charger does not illuminate	Not connected to power Fuse blown Faulty Charger	Check plugged in to power point and switched on Replace fuse Repair or replace charger
Actuator not running/no click heard from control unit when buttons pressed	Faulty control unit Faulty handset	Repair or replace control unit
Actuator not running but control unit clocks when buttons pressed	Actuator plug not inserted fully into control unit Faulty actuator Faulty control unit	Check plugs are completely inserted into control unit ports Repair or replace actuator and or control unit
Actuator not lifting properly or running slowly	Discharged or faulty battery Weight in excess of safe working load (SWL) Faulty actuator	Recharge or replace battery Repair or replace actuator
No clicking in control box when buttons pressed	Emergency stop button pressed Battery completely discharged Battery fuse faulty Faulty control unit	Release emergency stop button Recharge battery Repair or replace control unit
Actuator only works in one direction	Faulty handset Faulty control unit Emergency lower release activated	Repair or replace handset Repair or replace control unit Check emergency actuator release is reset

12. PRODUCT SPECIFICATIONS

Product Code	LSS390620
Safe Working Load (SWL)	200kg
Base Length	1320mm
External Base Width	665-1120mm
Internal Base Width	580-1040mm
Base Width Adjustment	Electric
Base Height / Under Bed Clearance	110mm
Minimum Height of Spreader Bar	520mm
Maximum Height of Spreader Bar	1720mm
Lifter Weight	40kg
Warranty	Зуrs
Under Base Height	50mm
Maximum Overall Height	1935mm
Front Castor Diameter	75mm
Rear Locking Castor Diameter	100mm
Turning Diameter	1398mm
Weight Of Heaviest Part (Base Assembly)	23.0kg
Handle Height	1220mm
Sound Level	<40Db@ 1m
Electrical Approval	EN 60601-1-2:2007; EN 61000-3-2:2006; EN 6100-3-3:2008

Box Contains: Base with 4 castor wheels, mast and boom with yoke, LINAK handset, LINAK battery charger, LINAK battery, LINAK controller, push handle. Screws, bolts, nuts, washers, plastic caps and tool kit.



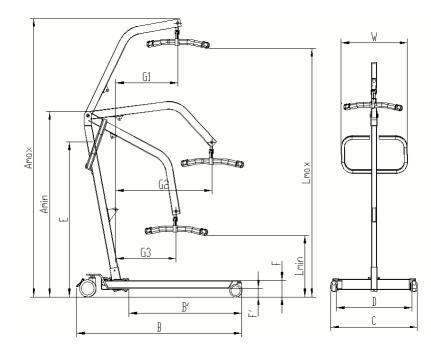

Max. Load	Max. Load and Net Weight in kg, the rest of the measurments are in cm (unmarked tolerance +/- 1cm)																	
Item No.	Max. Load	L	L	4	4	в	B'	с	D	E	F	F'	G			w	N.W	Turning Diameter
		Max.	Min.	Max.	Min.								G1	G2	G3			
7109	200	172	52	193.5	138	132	88.8	66.5-112 (tolerance +/_ 3cm)	58-104 (tolerance +/_ 3cm)	122	11	5	58 (tolerance +/_ 3cm)	73.5 (tolerance +/_ 3cm)	55 (tolerance +/_ 3cm)	53	33.5	139.5

13. SERVICE LOG BOOK

YEAR 1	Date:	
Service Type: Condition Report:		
Inspected by: Action taken:	Signature:	
	Date:	
YEAR 2 Service Type:	Date:	
Inspected by:	Signature:	
	Date:	
YEAR 3 Service Type: Condition Report:	Date:	
Action taken:	Signature: Date:	
	Date:	
Inspected by: Action taken:	Signature:	
YEAR 5 Service Type: Condition Report:		
Inspected by: Action taken:		
	Date:	



1300 133 120



AIDACARE PTY LTD BUILDING 3A/1 MOOREBANK Ave, MOOREBANK NSW 2170 1300 133 120 www.aidacare.com.au

MADE IN PRC