



1300 133 120 | aidacare.com.au

HOW TO CHOOSE YOUR LIFT RECLINE CHAIR



A. Chair/Seat Width

Determines the width at the widest part of the person (buttocks, hips, or knees). The ideal seat width would have approximately 2 - 5 cm space allowance at either side of the width measurement to ensure that it does not put any undue pressure on the sides of the body and legs.

B. Seat-to-floor Height/Seat Height

Ensure the seat height is low enough that the user can reach their bottom back to safely utilise the whole seat and safely balanced sit, whilst not being too low to prevent them getting back up. Their feet should be flat on the floor.

C. Seat Depth

Determines the ideal seat length, measure from the back of the buttocks to behind the knee and subtract 2-5 cm. When seated upright, there should be a 2-3 finger-width gap behind the knee to avoid pressure that could affect circulation.

D. Heel Placement

The end of the leg raiser should reach to the back of the ankle (in line with the ankle bone) when in the raised-up position. This is to avoid any undue pressure on the heel.

E. Armrest Height

Make sure the shoulder is supported, but not unnaturally high. The user's elbow should be bent at 90 degrees and sit comfortably on the armrest.

F. Backrest Height

Determines the length from the top of a person's head to the seat.



GABRIEL X FIVE ACTION **5 MOTOR LIFT RECLINE CHAIR**



- Zero Gravity Tilt-in-space positioning reduces forces on the lower back and improves circulation
- Five Action: Back recline, leg raise, lumbar support, independent tilt, and headrest ideal for watching TV when reclined
- Four Function Massage: Zoned air cells alternate to relieve tight, painful lower back muscles
- Electric Headrest Adjustment: Offers customisable neck support at the press of a button



Code Colour	Size	Height	Depth
CHP227591 Dark Grey	Small	475mm	450mm
CHP227596 Ebony	Large	500mm	520mm

Correct measurement and sizing of the chair are essential to meet the user's needs and prevents causing undue problems

- Legrests Extend to provide a longer support under the lower legs, beneficial for taller people
- Adjustable Seat Heating: Offers comfort and relief for sore muscles and joints
- USB A and USB C Charging: Located conveniently in the armrest so users can charge devices while browsing
- All-day Battery Backup: A lithium-ion battery ensures continued operation during power outages, supporting safety and independence

RAPHAEL X QUATTRO ACTION **4 MOTOR LIFT RECLINE CHAIR**



Image shown with standard lateral backrest.

			B
Golden Grain	Marina Grey	Nordic	Custom
Microsuede	Microsuede	Stone Vinyl	Upholstery*

Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Depth	Overall Width	Distance from Wall	SWL
CHP227640 CHP227641 CHP227642	Golden Grain Marina Grey Nordic Stone	Petite**	450mm	450 - 475mm	470mm	630mm	1060mm	840mm	820mm	420mm	130kg
CHP227650 CHP227651 CHP227652	Golden Grain Marina Grey Nordic Stone	Small	450mm	480 - 505mm	500mm	630mm	1060mm	870mm	850mm	400mm	130kg
CHP227660 CHP227661 CHP227662	Golden Grain Marina Grey Nordic Stone	Medium	470mm	500 - 525mm	525mm	680mm	1100mm	900mm	875mm	400mm	130kg
CHP227670 CHP227671 CHP227672	Golden Grain Marina Grey Nordic Stone	Large	500mm	530 - 555mm	550mm	720mm	1100mm	910mm	900mm	420mm	130kg
CHP227680 CHP227681 CHP227682	Golden Grain Marina Grey Nordic Stone	Maxi	510mm	540 - 565mm	650mm	675mm	1120mm	910mm	1020mm	420mm	180kg

**The Petite model is designed without a leg extension, catering to the needs of shorter users.

- Available in Five Sizes: Petite. Small. Medium. Large and Maxi, ensuring the perfect fit for all users
- Quattro Action: Independent back recline, lift, legrest, lumbar support and headrest adjustment
- Interchangeable Backrest Cushions with zippers allows personalised postural adjustment
- Integrated Footrest Extension helps with fatigue and improves lower limb comfort
- Premium Upholstery Options: Modern microsuede and leather-look vinyl finishes, with local customisation on request
- Backlit Hand Control with USB Charging: Illuminated buttons for low-light visibility and a built-in USB port for convenient device charging from the chair
- All-day Battery Backup: A lithium-ion battery ensures continued operation during power outages, supporting safety and independence
- Options to Bring Style and Class to any environment via custom order form*

	Eack Recline		Order Fo	rm (O) SWL 130kg Power Lumbar		
eat dth	Backrest Height	Overall Height	Overall Depth	Overall Width	Distance from Wall	SWL
)mm	630mm	1060mm	840mm	820mm	420mm	130kg
)mm	630mm	1060mm	870mm	850mm	400mm	130kg
imm	680mm	1100mm	900mm	875mm	400mm	130kg
)mm	720mm	1100mm	910mm	900mm	420mm	130kg
)mm	675mm	1120mm	910mm	1020mm	420mm	180kg

REMBRANDT FIVE ACTION **5 MOTOR LIFT RECLINE CHAIR**



- Available in Contemporary Large Size to suit larger user
- Five Action: Back recline, leg raise, lumbar support, independent tilt, and headrest ideal for watching TV when reclined
- · Zero Gravity Tilt-in-space positioning reduces forces on the lower back and improves circulation
- Integrated Wireless Phone Charger for convenient charging
- Seat and Backrest Heating for improved climate control



- Genuine Shiatsu Massage in backrest provides an exceptional seating experience
- Legrests Extend to provide a longer support under the lower legs, beneficial for taller people
- Memory Pre-set Positioning to capture the users most comfortable position with the press of a button
- All-day Battery Backup ensures continued operation during power outages, supporting safety and independence

MATISSE QUATTRO ACTION 4 MOTOR LIFT RECLINE CHAIR



- Available in Petite and Large Sizes fit to the user's body shape, improving comfort, posture and safety when standing
- Quattro Action Recliner with back recline and lift, legrest, lumbar support and headrest adjustments
- Integrated Lift and Recline Functions to assist sitting and standing
- Adjustable Headrest, ideal for watching TV whilst reclined
- Integrated Leg Extension helps to comfortably support lower legs for a wide range of user heights
- Large Side Storage Pockets ideal for books and newspapers
- All-day Battery Backup ensures continued operation during power outages, supporting safety and independence
- **Backlit Hand Control** and integrated USB charging are simple and convenient



Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	Distance from Wall	SWL
CHP22755	5 Fog Microsuede	Petite	455mm	455mm	510mm	560mm	1015mm	915mm	700mm	136kg
CHP22756	0 Fog Microsuede	Large	500mm	545mm	570mm	730mm	1065mm	965mm	580mm	159kg

DA VINCI QUATTRO ACTION 4 MOTOR LIFT RECLINE CHAIR

- Contemporary Medium Size available
- Quattro Action: Back recline, leg raise, lumbar support and headrest ideal for watching TV when reclined
- Integrated Lift and Recline Functions for easy sit-to-stand and maximum comfort







Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
CHP227500	Latte	Medium	480mm	530mm	530mm	630mm	1050mm	870mm	159kg
CHP227510	Gunmetal	Medium	480mm	530mm	530mm	630mm	1050mm	870mm	159kg

- Soft Microsuede upholstery finishes
- Backlit Hand Control with integrated USB charging point
- Chaise Padding Design fully supports legs, redistributing pressure



Latte Microsuede



Microsuede

RENOIR QUATTRO ACTION **4 MOTOR LIFT RECLINE CHAIR**

MONET DUAL ACTION 2 MOTOR LIFT RECLINE CHAIR



- Three Sizes help match the chair's fit to the user's body shape, improving comfort, posture and safety when standing
- Quattro Action: Back recline, leg raise, Lumbar support and headrest ideal for watching TV when reclined
- Integrated Leg Extension helps to comfortably support lower legs for a wide range of user heights
- Large Side Storage Pockets, ideal for books and newspapers

- Ergonomic Backrest Design reduces pressure off the lower sacral area
- Backlit Hand Control and Integrated USB Charging are simple and convenient
- All-day Battery Backup ensures continued operation during power outages, supporting safety and independence
- 4x Swivel Castors allow the chair to be moved around with ease



Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
CHP227575	Dusk Microsuede	Small	450mm	510mm	510mm	660mm	1010mm	750mm	130kg
CHP227580	Dusk Microsuede	Medium	470mm	530mm	550mm	760mm	1060mm	800mm	130kg
CHP227585	Dusk Microsuede	Large	470mm	530mm	590mm	760mm	1060mm	830mm	130kg







- and convenient
- of weightlessness

SWL 159kg	2	YEAR WARRANTY)

Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
CHP227540	Canyon Ocean	Large	495mm	535mm	560mm	570mm	1065mm	1065mm	159kg



• Contemporary Large Size available

• **Dual Action Design** creates infinite positioning via independent backrest and legrest adjustment

• Backlit Hand Control and Integrated USB Charging are simple

• Seat and Backrest Heating for improved climate control

• Zero Gravity Pre-set Positioning creates a feeling

• All-day Battery Backup ensures continued operation during power outages, supporting safety and independence

POSTUREFIT CONFIGURABLE **3 MOTOR PRESSURE CARE LIFT RECLINE CHAIR**



- Tri-Action Chair: Features powered headrest in addition to the independent back and leg motion
- Zero Gravity Positioning: Relieves lower back pressure for pain management when fully reclined
- Integrated Footrest Extension: Fits most user heights without altering the chair's legrest or height
- Tool-free Seat Depth and Width Adjustment: Easily adjust the backrest for two seat depths and interchange armrests for two seat widths, both with a 50mm range
- Backrest Cushions Have Adjustable Filling and support wedges are also available to meet unique postural requirements

- Effortless Mobility and Safety: Large castors engage via a rear foot pedal, ensuring stability and preventing tipping
- Transport Mode: Electrical functions deactivate when mobile for safety
- Medical Grade Foam: High density foam allows for prolonged periods of sitting
- Integrate Third-party Cushions like ROHO seamlessly with pre-cut seat structure, maintaining performance and infection control
- Battery Backup to maximise safety and independence



Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
CHP198250	Black	Small/Medium	490mm	475 - 525mm	485 - 535mm	740mm	1140mm	760 - 810mm	160kg
CHP198260	Black	Medium/Large	490mm	525 - 575mm	535 - 585mm	790mm	1180mm	820 - 870mm	160kg

ALTITUDE VERTICAL 2 MOTOR PRESSURE CARE LIFT RECLINE CHAIR



*Vertical lift mechanism only available on Beige and Bitter Chocolate colour way.

- Vertical Lift and Footrest Incline relieves strain on carer when attending to clinical needs, podiatry needs, assisted feeding, etc.
- USB Connectivity: Charge smart devices at leisure or during extended periods of treatment
- Built-in Battery Pack for full chair functionality without having to use a power cord
- Hand Control: Large backlit buttons make operation a simple process and enhance user independence



Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	Overall Length	SWL
CHP227700	Beige & Bitter Chocolate	Medium	560 - 700mm	520mm	510mm	710mm	1200 - 1340mm	710mm	800mm	150kg

- Designed to Accommodate a ROHO® **Cushion** in low to mid profile (or equivalent) Recommended cushion size: 18 x 20in
- Oncology Armrest Option: Designed to enhance patient comfort, the wide forearm support assists with effective administration of treatment such as blood transfusions
- Armrest Table Option for nutrition and hydration during extended treatment periods
- · TENTE Castors: 2 x 100mm Total Locking (Rear) 2 x 100mm Directional Locking (Front)

SIGNATURE 2 DUAL ACTION 2 MOTOR LIFT RECLINE CHAIR

SIGNATURE 2 SINGLE ACTION **1 MOTOR LIFE RECLINE CHAIR**



- · Available in Three Sizes: small, medium, and large
- **Dual Motor**: lift with independent back and leg adjustment
- Integrated Leg Extension helps to comfortably support lower legs for a wide range of user heights
- Depth Adjustable Seat: a choice of two depths settings in 1" (25mm) increments for custom comfort and positioning at setup
- Integrated Lithium Battery backup included
- Backlit Hand Control with integrated USB charging point
- · Four Swivel Castors allow the chair to be moved around with ease



Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
	Mink Warwick Lagoon Warwick Heritage	Small	480mm	460 - 385mm	480mm	580mm	980mm	740mm	155kg
CHP228230 CHP228240 CHP228250	Lagoon Warwick	Medium	480mm	490 - 515mm	510mm	620mm	1050mm	780mm	155kg
CHP228260 CHP228270 CHP228280	Lagoon Warwick	Large	500mm	510 - 535mm	540mm	650mm	1110mm	820mm	155kg

**Adjustable Neck Pillow CHP197781 Mink Warwick CHP197783 Lagoon Warwick CHP197782 Heritage

- Three Sizes help match the chair's fit to the user's body shape, improving comfort, posture and safety when standing
- Single Motor: lift with simultaneous back and leg adjustment
- Space Saver: the backrest and seat move forward within the chair frame during the recline action allowing closer positioning of the chair to a wall and overall smaller footprint

Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
CHP228100 CHP228110 CHP228120	Mink Warwick Lagoon Warwick Heritage	Small	480mm	460 - 485mm	480mm	580mm	980mm	740mm	145kg
CHP228130 CHP228140 CHP228150	Mink Warwick Lagoon Warwick Heritage	Medium	480mm	490 - 515mm	510mm	620mm	1050mm	780mm	145kg
CHP228160 CHP228170 CHP228180	Mink Warwick Lagoon Warwick Heritage	Large	500mm	510 - 535mm	540mm	650mm	1110mm	780mm	145kg





- Adjustable Seat Depth ensures correct positioning and safer transfers for a wide range of users
- Backlit Hand Control and integrated USB charging are simple and convenient
- Swivel Castor Wheels enable easy movement of the chair for cleaning or repositioning

**Adjustable Neck Pillow CHP197781 Mink Warwick CHP197783 Lagoon Warwick CHP197782 Heritage

MONTANA X MAXI SINGLE ACTION **1 MOTOR LIFT RECLINE CHAIR**



• Large Seating Surface for improved comfort for those users seeking a bit of extra room • Single Motor: lift with simultaneous back and leg adjustment

- Designed for Larger Users with high safe working load
- Head and Armrest covers for improved longevity
- Superior Wipe Clean Microsuede Upholstery for ease of maintenance
- Available in Bronze Microsuede leather-look

	202
2 YEAR WADDANTY	SWL 220kg

 \bigcirc

Full

Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
CHP227400	Golden Grain	Maxi	510mm	540mm	650mm	650mm	1050mm	930mm	220kg

MONTANA HD DUAL ACTION 2 MOTOR LIFT RECLINE CHAIR



Note: Lifetime warranty frame, 3 year warranty upholstery and 2 year warranty mechanism and electronics

Seat Width	Backrest Height	Overall Height	Overall Width	**Legrest SWL	SWL
650mm	810mm	1270mm	900mm	160kg	254kg
350mm	810mm	1270mm	1100mm	140kg	450kg

IDAHO DUAL ACTION 2 MOTOR LIFT RECLINE CHAIR

- Contemporary Medium Size available
- Dual Action: Independent back recline and leg raise
- Generous Padding in Backrest for improved lumbar support
- Gentle Tapered Armrests suit the modern home environment







Charcoal Fabric



Latte Vinyl

OREGON SINGLE ACTION **1 MOTOR LIFT RECLINE CHAIR**

- Contemporary Medium Size available
- Single Motor: Back and legs move simultaneously
- Generous Padding in backrest for improved lumbar support
- Gentle Tapered Armrests suit the modern home environment



(O) SwL 150kg	2 YEAR WARRANTY)	
Code	Colour	Size	Seat Heigh
CHP227150	Charcoal	Medium	480mi
CHP227155	Latte	Medium	480mi

*Adjustable Neck Pillow CHP227171 Charcoal Fabric CHP227172 Latte Vinyl

	၂၀)
2 YEAR	swL
WARRANTY	150kg

Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
CHP227160	Charcoal	Medium	480mm	480mm	480mm	620mm	1100mm	800mm	150kg
CHP227165	Latte	Medium	480mm	480mm	480mm	620mm	1100mm	800mm	150kg
	*Adjustable	Neck Pillow	CHP22	7171 Charc	oal Fabric	CHP22717	2 Latte Vinyl		



n	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
n	480mm	620mm	1100mm	800mm	150kg
n	480mm	620mm	1100mm	800mm	150kg

MAUI DUAL ACTION 2 MOTOR LIFT RECLINE CHAIR



Mocha Quilted Upholstery

Back Recline

Leg Raise

Contemporary Medium Size available

- Dual Action: Back recline and leg raise
- Backlit Hand Control with integrated USB charging point
- Memory Foam Seating Surface: Improves comfort and helps to facilitate pressure redistribution

							2 YEAR WARRAN		⊙) sw∟ 30kg
Code	Colour	Size	Seat Height	Seat Depth	Seat Width	Backrest Height	Overall Height	Overall Width	SWL
CHP227110	Mocha	Medium	500mm	500mm	540mm	630mm	1100mm	780mm	130kg

COMPLIMENTARY PRODUCTS

OVERCHAIR TABLE

Nestles around your favourite recliner, chair or bed.

A contoured design with a curved edge for arm comfort and added support. Its adjustable width (up to 1120mm) and height cater to recliners, chairs, or beds, while locking castors provide mobility and stability.



Aspire Overchair Table Details Beech Laminate Table Top Grey Frame

UNIVERSAL LIFT RECLINE CHAIR PROTECTOR

Keeps your premium lift recliner chair pristine and protected.

Crafted from heavy-duty, waterproof fabric with a soft quilted finish, this durable shield guards against spills, scratches, and everyday wear while adding a layer of plush comfort.





Universal Chair Protector Details Waterproof Fabric

Visit our website to explore our full range of products and available resources.













aspire SPECIFICATIONS





				GOLD RANG	E				
_	Gab	riel X		Raphael X					
Functions	Five A	Action			Quattro Action				
Motor	5 M	lotor	4 Motor						
Size	Small	Large	Petite	Small	Medium	Large	Maxi		
Seat Height	475mm	500mm	450mm	450mm	470mm	500mm	510mm		
Seat Depth	450mm	520mm	450 - 475mm	480 - 505mm	500 - 525mm	530 - 555mm	540 - 565mm		
Seat Width	490mm	530mm	470mm	500mm	525mm	550mm	650mm		
Backrest Height	710mm	790mm	630mm	630mm	680mm	720mm	675mm		
Overall Height	1100mm	1170mm	1060mm	1060mm	1100mm	1100mm	1120mm		
Overall Width	850mm	900mm	820mm	850mm	875mm	900mm	1020mm		
SWL	150kg	155kg	130kg	130kg	130kg	130kg	180kg		
Warranty	2 Years	2 Years	2 Years	2 Years	2 Years	2 Years	2 Years		
AUST R ARTG No.			306075	306075	306075	306075	306075		







				-				
		GOLD RANGE		S	SILVER RANGE			
_	Postu	ıreFit	Altitude	Signatu	re 2	ignature 2		
Functions	Pressur Config	,	Pressure Care/ Vertical Lift	Single Action		Dual Action		
Motors	3 Mo	otor	2 Motor	1 Motor		2 Motor		
Size	Small	Medium	Small/Medium	Small	Medium	Large		
Seat Height	490mm	490mm	560 - 700mm	480mm	480mm	500mm		
Seat Depth	475 - 525mm	525 - 575mm	520mm	460 - 485mm	490 - 515mm	510 - 535mm		
Seat Width	485 - 535mm	535 - 585mm	510mm	480mm	510mm	540mm		
Backrest Height	740mm	790mm	710mm	580mm	620mm	650mm		
Overall Height	1140mm	1180mm	1200 - 1340mm	980mm	1050mm	1110mm		
Overall Width	760 - 810mm	820 - 870mm	710mm	740mm	780mm	780mm		
SWL	160kg	160kg	150kg	Single145kg Dual 155kg	Single145kg Dual 155kg	Single145kg Dual 155kg		
Warranty	1 - 2 Years	1 - 2 Years	1 - 2 Years	2 Years	2 Years	2 Years		
AUST R ARTG No.				230186	230186	230186		



GOLD RANGE								
Rembrandt	Matisse		Da Vinci	Renoir			Monet	
Five Action	Quattro Action		Quattro Action	Q	uattro Actio	on	Dual Action	
5 Motor	4 M	otor	4 Motor	4 Motor			2 Motor	
Large	Petite	Large	Medium	Small	Medium	Large	Large	
495mm	455mm	500mm	480mm	450mm	470mm	470mm	495mm	
535mm	455mm	546mm	500mm	510mm	530mm	530mm	535mm	
560mm	510mm	571mm	530mm	510mm	550mm	590mm	560mm	
600mm	560mm	730mm	630mm	660mm	700mm	700mm	570mm	
1095mm	1015mm	1067mm	1050mm	1010mm	1060mm	1060mm	1065mm	
1015mm	915mm	965mm	870mm	750mm	800mm	830mm	890mm	
159kg	136kg	159kg	159kg	130kg	130kg	130kg	159kg	
2 Years	2 Years	2 Years	2 Years	2 Years	2 Years	2 Years	2 Years	
216484	216484	216484	216484	306075	306075	306075	216484	



BRONZE RANGE						
Montana X	Montana HD	Idaho	Oregon	Maui		
Single Action	Dual Action	Dual Action	Single Action	Dual Action		
1 Motor	2 Motor	2 Motor	1 Motor	2 Motor		
Maxi	HD254 HD450	Medium	Medium	Medium		
510mm	500mm 500mm	480mm	480mm	500mm		
540mm	500mm 500mm	480mm	480mm	500mm		
650mm	650mm 850mm	480mm	480mm	540mm		
650mm	810mm 810mm	620mm	620mm	630mm		
1050mm	1270mm 1270mm	1100mm	1100mm	1100mm		
930mm	900mm 1100mm	800mm	800mm	780mm		
220kg	254kg 450kg	150kg	150kg	130kg		
2 Years	2 - 3 Years	2 Years	2 Years	2 Years		
306075	306075 306075	306075	306075	306075		













UPHOLSTERY AND SWATCHES

GENUINE LEATHER LA CASA RANGE

Luxurious artisan leathers with a soft glove-like texture will retain their unique suppleness, provided they are regularly cared for. These leathers are produced and tested to international standards for properties such as rubbing, flex resistance, light fastness, thickness and strength. Commercial quality fabric for long lasting durability.



SOFT TOUCH FABRIC MANISA RANGE

The soft touch weave and diverse colour palette makes the Manisa ranges highly practical and functional solution. These fabrics are comforting, easy to clean and durable, yet still soft to touch. Commercial quality fabric for long lasting durability.

Licorice	Stone	Charcoal	Fossil	Thunder
Vapour	Spice	Marble	Caramel	Espresso
Acacia	Spruce	Mouse	Peacock	Atlantic
Atoll	Seagrass			
				2 YEAR WARRANTY Fabric

MICROSUEDE EASTWOOD RANGE

Suede like texture combined with high-performance characteristics ensures this fabric remains easy to clean and maintain. Commercial quality fabric for long lasting durability.



HEALTHCARE VINYL LUSTRELL RANGER RANGE

High performance vinyl with HealthGuard Antibacterial and Antimicrobial treatment to assist with infection control and hygiene. Commercial quality fabric for long lasting durability.



Note: All colour swatches shown may not be a true representation of the actual product and should be used as a guide only. Warranty applies when fabric is used in accordance with Warwick performance ratings and care instructions.





Donkey



Tan





Dove



Fawn





