



# BATH TRANSFER BENCH ASSEMBLY GUIDE

BTS108200 / BTS110900

1. Slide two main body sections together and lock in place with spring pins on each side




2. Install 4x legs by removing black plastic cap on main frame then inserting leg tube into opening.



**NOTE:** You must make sure to install the suction legs onto the side of the transfer bench which holds the armrest. The main frame and the legs are marked with A and B respectively so make sure to install the A legs on side A and vice versa.

3. Install backrest by sliding 2x tubes through the loops on the underside of the frame then inserting the backrest onto these tubes. The backrest should be held in place by lining up the spring pin with the hole in the loops the tubes were inserted through.

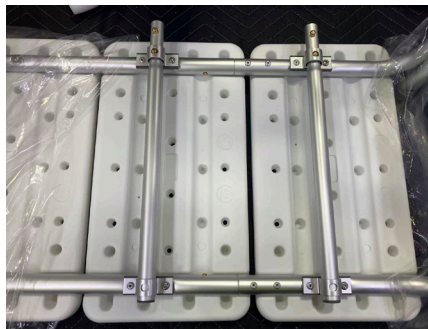

4. Insert armrest into the top of the frame, make sure the spring pins are clicked in place.

