
ACCESSORIES

HOW TO FIT TO BED

BED RAIL / BED STICK

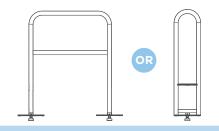

ACCESSORIES

BED RAIL BEA830210

CROOK HANDLE BEA830200




For further installation and operating instructions, refer to the User Manual.


Tools Required:

- Allen Key (5mm)
- Adjustable Shifter/Spanner

KIT CONTENTS

Bed Rail Crook Handle

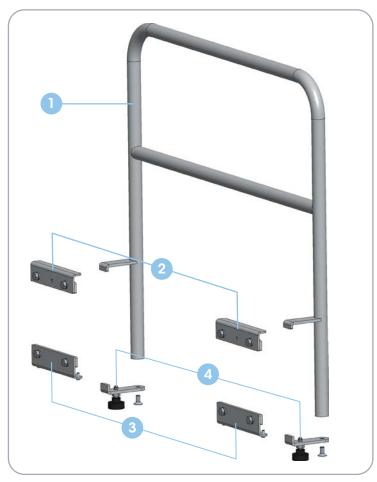

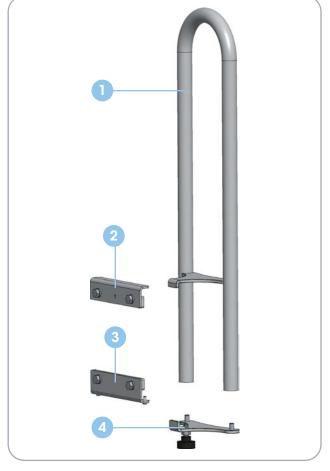

ITEM	QTY (BED RAIL)	QTY (CROOK HANDLE)	DESCRIPTION
1	1	1	Rail
2	2	1	Top Guide Rail
3	2	1	Bottom Guide Rail
4	2	1	Mounting Plate
5	2	1	Hand Bolts
6	8	4	M8 x 20 Bolts
7	8	4	M8 Washers
8	4	2	M5 x 10 Bolts
9	2	2	M8 x 16 Countersunk Bolts


 $\textbf{NOTE:} \ \mathsf{Some} \ \mathsf{items} \ \mathsf{not} \ \mathsf{shown} \ \mathsf{in} \ \mathsf{images} \ \mathsf{above}$

Raise bed height, backrest and knee break to maximum height for ease of access.

To adjust:




2 ComfiMotion beds have pre-installed mounting points for accessories.

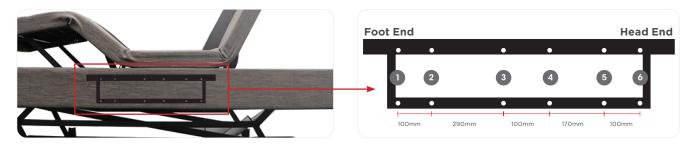

NOTE: Frame not visible through upholstery, for instructional purposes only.

To install bed rail (BEA830210) - use positions 1, 2 and 5, 6.

3 Assemble the top 2 and bottom 3 guide rail(s) onto the frame using the M8 x 20 bolts 6 and washers 7.

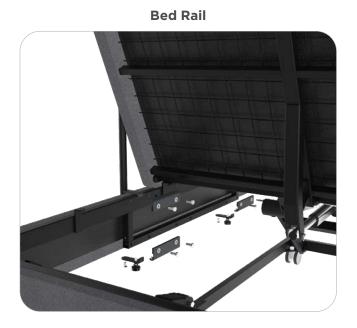



Slide the rail 1 between the mattress deck and frame and slide into the top guide rail(s). Slide the mounting plate(s) 3 into the bottom guide rail(s).

Bed Rail



Crook Handle



- 5 Attach the rail 1 and mounting plate(s) 3 using the M8 x 16 countersunk bolts 9.
- Adjust the rail 1 within the rails to desired position. Once in place, lock in place by tightening the hand bolt(s) 5.







