
8.3 BED RAIL & CROOK HANDLE

HOW TO FIT TO BED

BED RAIL / BED STICK

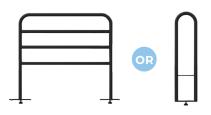

ACCESSORIES Bed Rail BEA830212 Crook Handle BEA830211	ACCESSORIES		ACCESSORIES	Bed Rail BEA830212	Crook Handle BEA830211	
---	-------------	--	-------------	--------------------	------------------------	--

For further installation and operating instructions, refer to the **User Manual.**




TOOLS REQUIRED:


• Allen Key (5mm)

KIT CONTENTS

Bed Rail

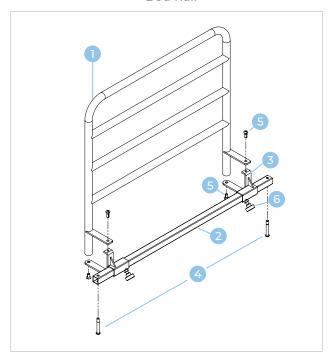

Crook Handle

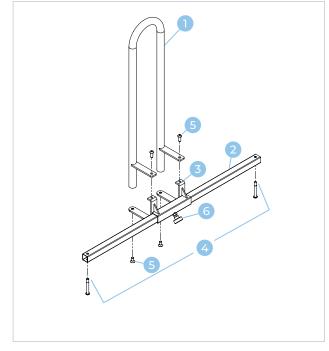


Item	Qty (Bed Rail)	Qty (Crook Handle)	Description
1	1	1	Rail
2	1	1	Mounting Tube
3	2	1	Mounting Bracket
4	2	2	M8 x 40 Bolt
5	4	4	M8 x 20 Bolt
6	2	1	Handwheel




Raise bed height, backrest and legrest to maximum height for ease of access.




Slide the mounting bracket(s) 3 onto the mounting tube 2.
For bed rail: Ensure handwheel bolts are pointing towards each other as shown.

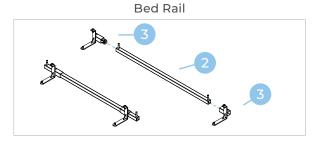

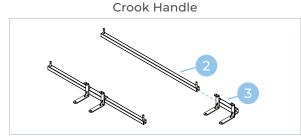


3 Assemble the mounting tube 2 onto the bed using the 40mm bolts 4.




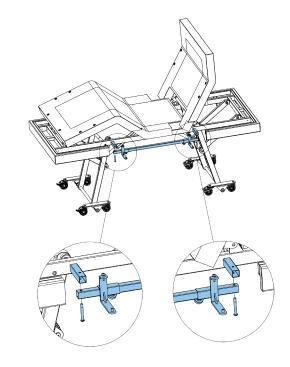

Mount the rail 1 to the mounting brackets 3 using the bolts 5.

NOTE: Images show assembly without the bed for clarity.

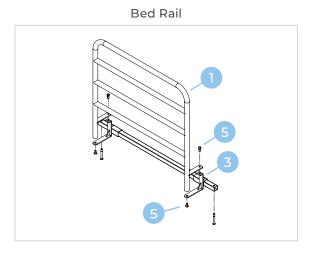

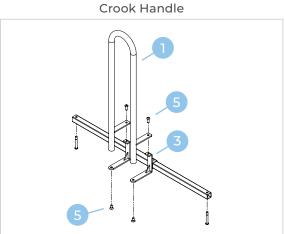

Loosen the hand wheel(s) 6 and gently slide rail 1 to desired position. Once in place, tighten the hand wheel(s) to lock in place.

