
HEADBOARDS


HOW TO ASSEMBLE OR MOUNT


ALLURE / CHESTERFIELD HEADBOARDS

| HEADBOARDS | ALLURE (LS) BEA830120 | CHESTERFIELD (LS) BEA830100 |
|------------|-----------------------|-----------------------------|
| | ALLURE (KS) BEA830130 | CHESTERFIELD (KS) BEA830110 |

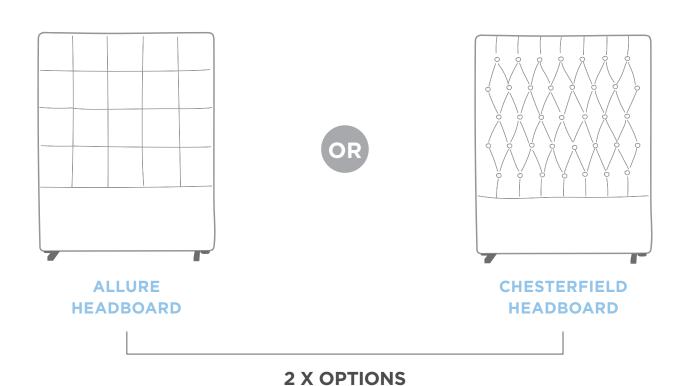

TO FIT HEADBOARDS

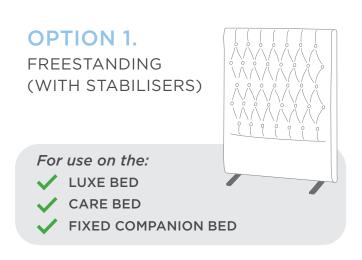


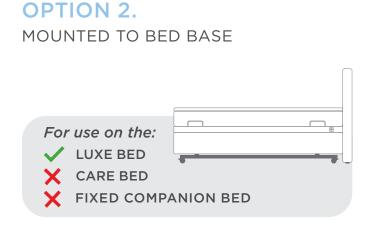



OPTION 1.

HOW TO ASSEMBLE - FREESTANDING WITH STABILISERS

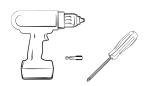


Tools Required:


- Impact driver or drill (5mm attachment)
- Phillips head screw driver (5mm)



- 1 Flip headboard upside-down to show the black upholstery.
- Using the leg stabiliser, align the holes to the white markings. **NOTE:** The short end of the stabiliser should face the back of the headboard (the side that will be positioned against the wall).



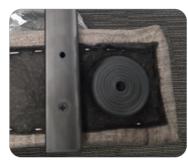

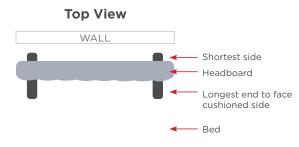

3 Secure the stabilisers in place using the screws provided.




4 Finally, flip the headboard back to its original position, enabling it to stand freely.



OPTION 2.

HOW TO MOUNT TO BED



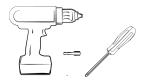

Tools Required:

- Impact driver or drill (5mm attachment)
- Phillips head screw driver (5mm)

NOTE:

It is suggested that an impact driver or drill is used to speed up this process. Using a Phillips head screwdriver will work but will take more time. If the impact Driver/Drill has a clutch setting, it should be set to low torque to minimise any risk of bolt misalignment or mount point damage.

NOTE: Headboards can be mounted to the LUXE Bed only.

CAUTION: When bed is in raised position, Headboard will raise with bed and poses risk to damaging of fixings or attachments on walls.

Raise bed height, backrest and knee break to maximum height for ease of access.

LUXE Handset

HI-LO





- Locate mounting holes for screw insertion. To locate holes:
 - a. Locate threaded mounting holes (2 per side) at head end of bed.
 - b. Line up bracket over threaded mounting holes and mount in place by rotating phillips head bolt through the top and bottom mounting holes.
 - c. Repeat on both sides, ensuring 4 x bolts (2 per side) are fastened in place.








When mounting brackets to bed end:

- a. Line bracket up with foot stopper (on base of headboard).
 - i. Ensure bracket is spaced approx. 40mm from edge of headboard.
 - ii. Ensure bracket is spaced approx. 60mm from base of headboard.
- b. Through rotation the screw will pierce through the fabric and thread into wood frame (self-tap). Repeat on both sides, ensuring 8x screws (4 per side) are fastened in place.
- c. Using an impact driver, rotate screw through the upholstery inwards towards frame.
- d. **NOTE:** Position of bracket should be as per measurements otherwise it will not fit securely to the bed frame.







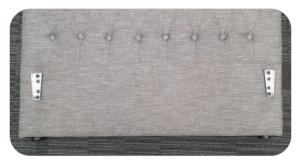

4

Align bracket fixings (headboard bracket will slide into bed frame bracket), Slide headboard down until both brackets click into place, the headboard is now in place.

NOTE: Headboard foot supports will rest on the ground when in lowest height.

