## **ASPIRE HOMECARE OVER TOILET AID**

## **ASSEMBLY INSTRUCTIONS**

BTT145025



## **ASSEMBLY INSTRUCTIONS**

- 1 Open main frame 1 and place on an even surface.
- 2 Slide the hook on the support frame 2 over the rear bar on the back of the main frame 1.



- 3 Secure support frame 2 in place by inserting into the receivers located on the main frame 1. A notable 'click' sound will be heard once the frame is secured.
- 4 Turn the frame upside down. Insert adjustable legs 3 onto the main frame leg supports by depressing the push buttons and sliding into place. A notable 'click' sound will be heard once the legs are secured.



- Turn the frame around so it is now standing on the adjustable legs. Insert the backrest 4 into the receivers whilst depressing the push buttons. A notable 'click' sound will be heard once the backrest is secured.
- 6 Attach the toilet seat 5 to the back of the frame.
  The toilet seat hooks should locate onto the frame by pressing downwards.





Your Aspire Homecare Over Toilet Aid is now ready for use.



## LEG HEIGHT ADJUSTMENT



Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration.