


# MAXI DAY CHAIR ADJUSTMENT QUICK USE GUIDE

Aspire Maxi Day Chair - CHP208010

## SEAT HEIGHT ADJUSTMENT


1. Push and hold the spring located on the outside of the rear leg.
2. While holding the button, pull the leg down to increase the seat height from the floor, or push it up to decrease the height.
3. Release the button and check that it fully pops out of the hole. Repeat steps 1 to 3 for the other legs so that all four legs are at the same height.

**Note:** All legs **MUST** be set to the same height to keep the frame stable and ensure all four wheels.

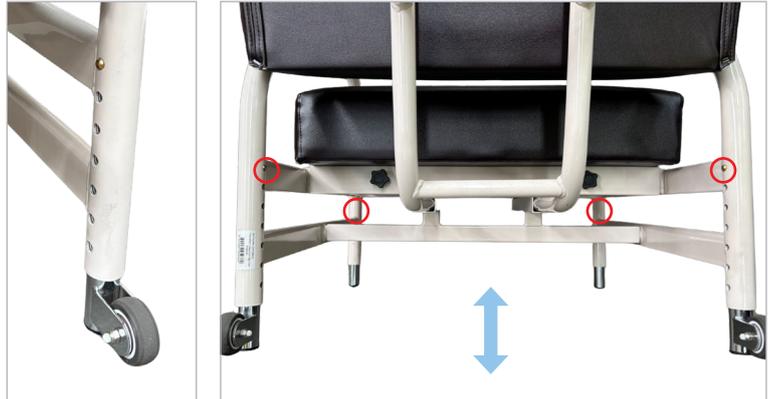

## SEAT DEPTH ADJUSTMENT

1. Loosen the adjustment knob.
2. Slide the support bar behind the seat up or down to the desired height.
3. Adjust forward or backward for depth.
4. Tighten the knob to secure.

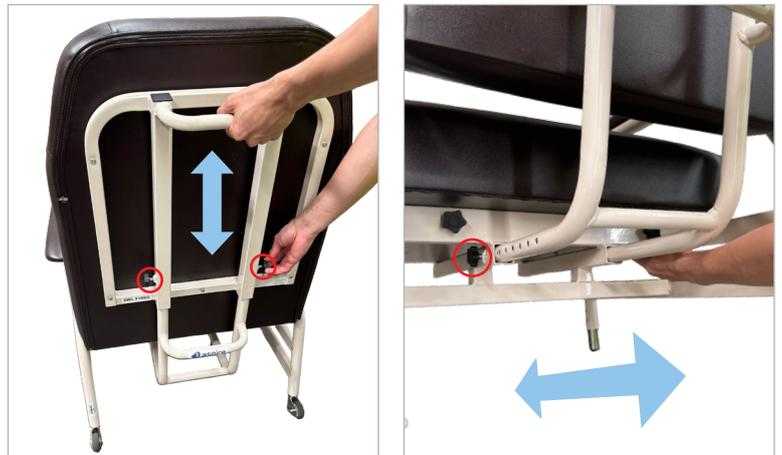

## SEAT WIDTH ADJUSTMENT

1. Locate the width adjustment levers or knobs beneath the backrest and loosen the adjustment knob.
2. Slide the armrest inwards or outwards to fit the user.
3. Ensure both sides are evenly adjusted.

