

## STAR PATIENT MOVER

The Aspire Star Mover offers a safe and comfortable solution for patients and carers doing sit to stand transfers. The compact and versatile design is perfect for Homecare, Institutional and Aged Care environments.



	Overall Length	Overall Height	Leg Length	Knee Pad Height	Seat Pad Height	Handle Height (Carer)	Handle Height (User)	Leg Width Range	Footplate Height From Floor	Turning Circle
STAR MOVER LSS390580	920	1030	780	310 – 580	850	1025	865/960	520 – 740	70	1055
	36"	40"	31"	12 – 23"	33.5"	40"	34/38"	21 – 29"	3"	41"

\*Range of motion dimensions are accurate to a +/-10mm tolerance

## **FEATURES & BENEFITS**

- · Wide seat entry suits a large range of body shapes
- Low footplate height makes patient entry and exit easier
- Large 100mm castors (two rear locking) have low rolling resistance and offer a smooth ride during transportation
- · Narrow base improves access in tight spaces
- Height adjustable knee support ensures correct positioning for a larger patient height range
- Easy access seat pad handles provide reach-around access to carers
- Soft touch grab bar provides a comfortable patient support for improved stability and confidence during transfers
- Width adjustable legs with tapered close position allow access between and, or around chair legs

## **HOW TO USE**



1 Open legs if required and lift seat pads. Ask the user to place feet on footplate and apply brakes.



2 Ask the user to hold the grab handle and stand up.




3 Swing the seat pads back into place and ask the user to sit down. Unlock brakes and commence the transfer.

## **SLING COMPATIBILITY**

The Star Mover features clip attachment points that work with Aspire Deluxe Standing Slings.

CODE	DESCRIPTION	GIRTH	EDGING	SWL
LSS390920	Deluxe Padded Standing Sling – Polyester – Clip – Small	65 – 95cm	•	200kg
LSS390925	Deluxe Padded Standing Sling – Polyester – Clip – Medium	77 – 110cm	•	200kg
LSS390930	Deluxe Padded Standing Sling – Polyester – Clip – Large	98 – 135cm	•	300kg
LSS390935	Deluxe Padded Standing Sling – Polyester – Clip – Extra Large	118 –1 57cm	•	300kg

For men and children, girth is the chest circumference. For women, girth is the hip circumference.

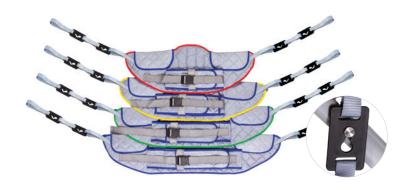

Star Patient Mover is registered in the ARTG, AUST R ARTG 373695

