


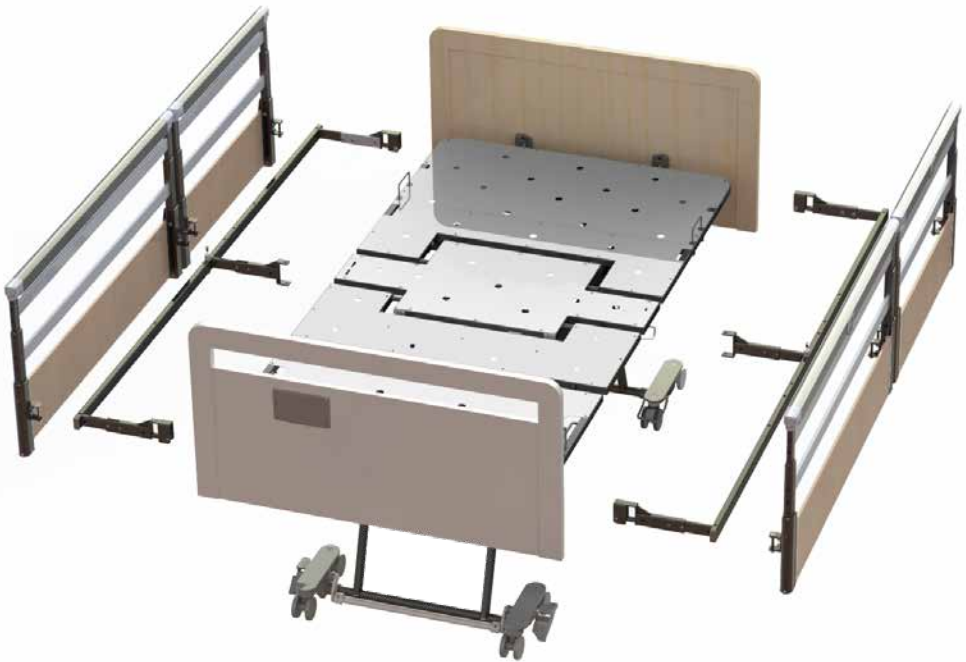

# TELESCOPIC RAILS

## ASSEMBLY GUIDE



# TELESCOPIC RAILS

## ASSEMBLY GUIDE



Mounting bar must be used to mount Telescopic Side Rails.



Warning: Blue handset (BEB046602) must be used with Telescopic Side Rails.

## KIT OPTIONS (BED NOT INCLUDED)

Kits contain all parts required to fit a full bed. Choose a telescopic side kit and bed end to complete a set.

### TELESCOPIC SIDE RAIL KITS (CHOOSE ONE OF THE BELOW OPTIONS)

BEB048800	Sublime Teak	Telescopic Side Rails Full Kit 2 x 1/3 Rail	
BEB048805	Milano Walnut	2 x 2/3 Rail	
BEB048810	Sublime Teak	Telescopic Side Rails & Trim Full Kit 2 x 1/3 Rail	
BEB048815	Milano Walnut	2 x 2/3 Decorative Trim	
BEB048820	Sublime Teak	Telescopic Side Rails & Trim Full Kit 2 x 2/3 Rail	
BEB048825	Milano Walnut	2 x 1/3 Decorative Trim	

### BED ENDS (CHOOSE ONE OF THE BELOW OPTIONS)





BEB048830	Sublime Teak	Bed End Set <b>SINGLE</b>	
BEB048835	Milano Walnut	For Telescopic Rails – Como R	
BEB048840	Sublime Teak	Bed End Set <b>KING SINGLE</b>	
BEB048845	Milano Walnut	For Telescopic Rails – Como R	

## CUSTOMISATION





The customisation option allows customers to choose what they need.

### MANDATORY

(REQUIRED TO FIT AND USE SIDERAILS - SELECT EITHER SINGLE OR KING SINGLE BED ENDS)

BEB048900		Mounting Bar & Hardware <b>PAIR</b>	
BEB046602	Blue	Hand Control <b>MIN HEIGHT LIMITING</b>	
BEB048830	Sublime Teak	Bed End Set <b>SINGLE</b>	
BEB048835	Milano Walnut		
BEB048840	Sublime Teak	Bed End Set <b>KING SINGLE</b>	
BEB048845	Milano Walnut		

### OPTIONAL

BEB048850	Sublime Teak	Telescopic Rail 1/3 Length – <b>EACH</b>	
BEB048855	Milano Walnut		
BEB048860	Sublime Teak	Telescopic Rail 2/3 Length – <b>EACH</b>	
BEB048865	Milano Walnut		
BEB048870	Sublime Teak	Telescopic Rail – Decorative Trim 1/3 Length – <b>EACH</b>	
BEB048875	Milano Walnut		
BEB048880	Sublime Teak	Telescopic Rail – Decorative Trim 2/3 Length – <b>EACH</b>	
BEB048885	Milano Walnut		

## GENERAL ASSEMBLY

### Mounting Bar & Hardware – PAIR

1. End mounting bracket (x4)
2. Centre mounting bracket (x2)
3. Mounting rail (x2)


### Side Rails / Trim

4. Telescopic side rail 1/3 length (x2)  
**Or**  
Decorative trim 1/3 length (x2)
5. Telescopic side rail 2/3 length (x2)  
**Or**  
Decorative trim 2/3 length (x2)

### Bed Ends

6. Bed End Set Single  
**Or**  
Bed End Set King Single

### Recommended Tools

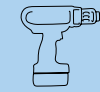

Drill



6mm  
drill bit



Allen keys  
(Size 5 & 6)



Socket  
wrench  
(Size 13)



Flexible  
drive socket  
extension



Rubber  
mallet






1. Remove all head and footboards from the bed, raise the bed to a comfortable working height and raise the backrest and leg rest to the maximum height.




2. Install End mounting brackets (1) to the head and foot ends of the frame. Use a flexible drive socket extension and ratchet to avoid damaging the bolt. **DO NOT** tighten circled bolts completely for easier rail assembly.



3. Install Centre mounting bracket (2) to the middle of the bed as shown. **DO NOT** tighten the bolt completely for easier rail assembly.

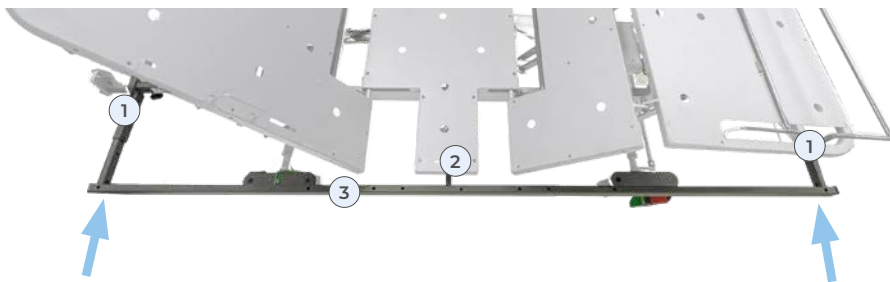


4. Insert Mounting rails (3) into the End and Centre mounting brackets (1) (2) at the same time. Use a rubber mallet to fit the rail as needed.

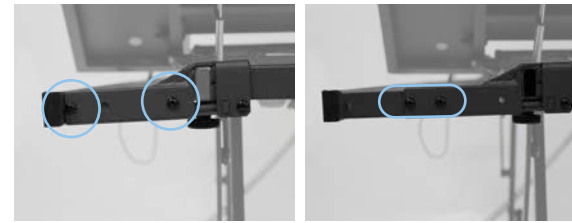


Single

King Single


5. Align holes based on the bed size. Refer to the images shown for **Single** and **King Single** rail configurations.
6. Secure Mounting rails with bolts.

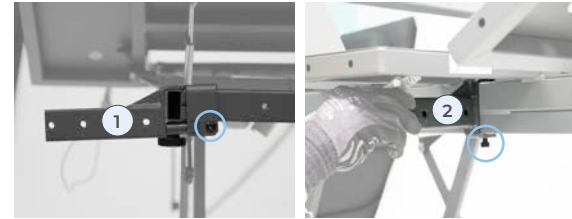

7. Tighten all remaining bolts on mounting brackets (1) & (2).

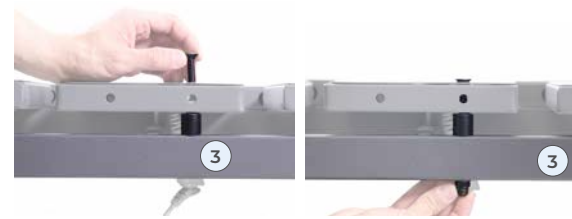

8. Secure Mounting rails (3) to the centre of the bed frame with the countersunk bolt. For serial numbers before:  
King Single: 2308-21063  
Single: 2307-30873  
**An experienced service technician will be required to drill a hole on older beds.** Refer to video instructions for more information.

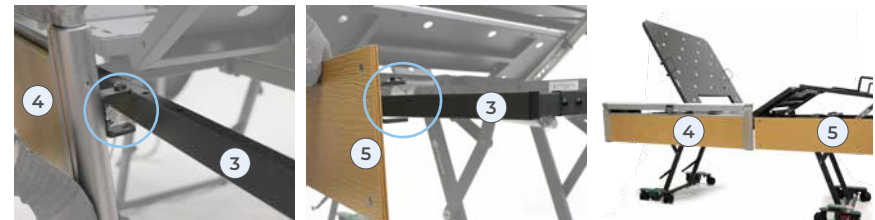

9. Assemble the Telescopic side rails (4) (5) or decorative trim on to Mounting rails (3). Ensure holes line up with both parts and secure with the provided bolts.



10. Assemble Head and Foot boards to the bed.
11. Replace the standard FL250 handset with a Blue handset to limit low height.
12. Assembly is now complete.

**AIDACARE PTY LTD**

Building 3A, 1 Moorebank Avenue,  
Moorebank NSW 2170  
Australia

1300 133 120 | [aidacare.com.au](http://aidacare.com.au)

MADE IN PRC

ABUM23.21 090924

