



Clinical Assistance Guide

Aspire Lift Recline Chairs - Premium Range

1300 133 120 | sales@aidacare.com.au | aidacare.com.au

Aspire Lift Recline Chairs – Premium Range

Five Action

The X5 range of Aspire Electric Lift and Recline Chairs offer superior comfort whilst supporting posture, mobility and independence.

All models offer the following 5 functions:

- Lift and recline
- Independent leg extension and back recline
- Head tilt
- Lumbar adjustment
- Zero gravity tilt-in-space

Independent functions provide infinite positioning enabling the user to find the optimal position to support comfort, manage pain and fatigue, and assist ease and safety of transfers.

All the X5 chairs include a host of features such as a battery back-up and a safety net to prevent under chair pet access when the legrest is raised, with some models also offering seat heating and massage. The Raphael X5 provides a greater number of custom options with 5 chair sizes from petite to maxi, choice of custom fabric, backrest options, and back cushion fill adjustability. All the X5 range have commercial grade foam, scratch resistant materials, and protective covers for increased durability and to extend the life of the chair.

Note: See comparison table on page 8



Rembrandt X5



Monet X5



Gabriel X5



Raphael X5



Renoir X5 (Coming Soon)



Matisse X5

FUNCTIONS

Five Action Lift and Recline Chair

- Independent back recline, legrest raise, lumbar support, adjustable headrest and zero gravity tilt-in-space offer complete control for users to maximise their comfort

Independent Back Recline and Legrest Raise

- Enables the user to make micro-adjustments to attain the optimal position to support comfort and activity

Electric Headrest Adjustment

- Adjustable head tilt enables the user to find the optimal position to support their head and neck when reclined, promoting chair-based activities such as watching TV, reading and relaxation

Lumbar Support

- Provides targeted, adjustable support to the lower back, to maintain a comfortable posture while seated for extended periods

Zero Gravity Tilt-in-space

- Creates a feeling of weightlessness by elevating the user's legs above their chest, with only a limited amount of space needed behind the lift recline chair

FEATURES

Safety Inclusions

- All-day Battery Backup: A lithium-ion battery ensures continued operation during power outages, supporting safety and independence
- Safety Net: Built-in safety net prevents pets from accessing the moving mechanism
- Anti-tippers: The Aspire Premium Lift Recline Chairs are fitted with anti-tippers to prevent the chair from tipping backwards when fully reclined and tilted

Integrated Leg Extension

- Provides full support under the lower legs, especially beneficial for taller users
(Note: The Raphael X5 Petite model is designed without a leg extension, catering to the needs of shorter users.)

Handset

- Easy to use, large button handset with high-visibility backlit buttons
- Optional simplified handset is available as an accessory for basic functions such as lumbar and headrest adjustments
- Handset can be set up prior to delivery for right hand (standard) or left-handed use, to meet the preferred position for the user or carer assisting with transfers
- Handset loops enable the handset to stay within reach

Chair Size Options

- Available in petite to maxi sizes to suit a wide range of body types
(See chart on page 6 for size options for different chairs.)
- Strong load capacity with a high Safe Working Load (SWL) supports a wide range of users
- Raphael X5 Maxi provides a high 190kg SWL for heavier users

Device Charging

- USB-A and USB-C charging ports in either the armrest and/or handset for phones and tablets allow the chair user to charge their devices and stay connected

Durability

- Scratch-resistant materials and protective armrest and headrest covers are included as standard to extend the life of the chair
- Commercial-grade foam engineered for high-use environments, delivering superior durability, long-lasting support, and consistent comfort over time

UNIQUE FEATURES

Rembrandt X5

- Three level backrest massage and seat and backrest heating provide comfort, warmth, and relaxation
- Adjustable airline-style headrest can be raised, lowered, or wrapped around the neck and head to support relaxation and different height users

Gabriel X5

- Adjustable seat heating provides gentle warmth and improves comfort
- Three level backrest massage provide a choice of massage settings for comfort

Monet X5

- Integrated seat and backrest heating provides gentle warmth and improves comfort

Raphael X5

- Seat depth can be extended by 25mm to support taller users
- Custom upholstery options are available in a range of finishes
- 5 size options available including petite and maxi sizes

Renoir X5 (Coming Soon)

- Ergonomic backrest design reduces pressure off the lower sacral area for improved positioning and comfort

ACCESSORIES

Care-friendly handset: Monet X5 - CHP228910 | Raphael X5 and **Renoir X5 (Coming Soon)** - CHP228900



IDEAL USER CONDITIONS

Those who may benefit from the independent functions of a Five Action Lift and Recline Chair include:

- Elderly people and people with reduced range of movement
- People recovering from surgery
- Caregivers who assist with chair transfers

CLIENT OUTCOMES

Increased Safety and Independence:

- An Electric Lift and Recline Chair (ELRC) improves safety for people with limited range of movement, or who find it difficult to stand or sit safely or independently without the risk of falling
- The chair user can control their sit-to-stand or stand-to-sit transfers by raising the chair to a comfortable height

Improved Positioning for Comfort:

- Individualised adjustment of neck, back, lumbar and leg surfaces and use of the zero-gravity function supports perfect positioning for comfort and relaxation
- The multiple seating positions can help people who spend a lot of time sitting in the chair to feel supported during specific activities such as resting, reading, watching TV, or eating
- Independent sit-to-stand transfers to allow the user to stand and move
- Adjustable headrests and lumbar area support assist the user in both an upright and reclined position, tailoring comfort to suit the needs of the user

Improved Health and Wellbeing:

- For people who find sitting uncomfortable, a Lift and Recline Chair may enable a person to find a suitable position to get out of bed and join friends or family socially, reducing the need to stay in bed because they are unable to get comfortable
- Improved independence and control can have positive impacts on health and wellbeing

Managing Energy:

- Zero gravity, recline, and leg raise functions enable resting in the chair rather than the need to, or effort of, returning to bed during the day

Improved Safety and Reduced Level of Assistance Required by Carers:

- Injury can also be sustained by the chair user when being pulled out of a chair by a carer or from overbalancing and falling when trying to stand from a low chair
- Injury can be sustained by the carer from frequent, awkward or unpredictable manual handling when assisting the chair user to stand or sit e.g. the chair-user pulling on the caregiver's arm or the carer catching a person who is falling
- For carers the lift function of the chair reduces the risk of injury to the carer from assisting the person they care for to stand
- The caregiver does not need to remain close at hand to assist with chair transfers or to assist the person to return to bed when they need a rest
- An Electric Lift Recline Chair makes transfers easier by reducing the lifting the carer is usually required to do

SPECIFICATIONS

	Seat Height mm	Seat Depth mm	Seat Width mm	Seat to Top of Back mm	Overall Height mm	Overall Depth mm	Overall Width mm	Distance From Wall mm	Armrest Height mm	Safe Working Load kg
Rembrandt X5										
Petite	455	455	510	700	1010	930	865	620	180	159
Large	500	540	570	700	1070	1020	990	640	180	159
Gabriel X5										
Small	475	450	490	710	1100	880	850	500	170	159
Large	500	520	530	780	1170	970	900	550	170	159
Monet X5										
Petite	455	455	490	700	1080	860	830	500	200	159
Large	500	550	550	750	1170	960	890	580	200	159
Raphael X5										
Petite	450	450-475	470	630	1060	840	820	420	160	159
Small	450	480-505	500	630	1060	870	850	400	175	159
Medium	470	500-525	525	680	1100	900	875	400	175	159
Large	500	530-555	550	720	1160	910	900	420	175	159
Maxi	510	540-565	650	675	1120	900	1020	420	175	190
Renoir X5 (Coming Soon)										
Small	450	510	510	660	1010	740	750	510	180	159
Medium	470	530	560	700	1060	740	800	570	180	159
Large	470	530	580	700	1060	740	830	570	190	159
Matisse X5										
Petite	455	455	510	700	1010	930	865	580	200	159
Large	500	545	570	700	1070	1020	990	700	200	159

COMPLIANCE

Test Standards:

EN 61000-3-3: 2013 +A1:2019 +A2:2021

EN IEC 65014-1: 2021

EN IEC 65014.2: 2021

EN TEC 61000-3.2: 2019 +A1:2021 +A2:2024

WARRANTY

2 years

MAINTENANCE

Please refer to the User Manual for full instructions on maintenance and safe use of the lift and recline chair.

Fabric and Upholstery Care

- Inspect the fabric on a regular basis for any pulls, tears or gaps

Electronics Care

- Inspect all wiring harnesses to make sure they are not damaged or frayed
- If damage is present, unplug the lift chair and contact your authorised Aspire Dealer for service
- Keep all electronics free from moisture and temperature extremes
- Aspire Lift Recline Chairs are intended for indoor use only

AIDACARE KNOWLEDGE HUB

Rembrandt X5



Website
Landing Page



Flyer

Gabriel X5



Website
Landing Page



Flyer

Monet X5



Website
Landing Page



Flyer

Raphael X5



Website
Landing Page



Flyer

Renoir X5

**Coming
Soon**

Website
Landing Page

**Coming
Soon**

Flyer

Matisse X5



Website
Landing Page



Flyer

Aspire Premium Range Lift Recline Chair



X5 User
Manual

Lift Recliner Assessment & Sizing Guide



How to
Choose a Lift
Recline Chair



Recliner
Sizing Guide

	Aspire Lift Recline Chair	Rembrandt X5	Gabriel X5	Monet X5	Raphael X5	Renoir X5	Matisse X5
Functions	Electric lift and recline	●	●	●	●	●	●
	Independent leg raise	●	●	●	●	●	●
	Independent back recline	●	●	●	●	●	●
	Head tilt	●	●	●	●	●	●
	Lumbar adjustment	●	●	●	●	●	●
	Zero gravity tilt-in-space	●	●	●	●	●	●
Features	Seat heating	●	●	●			
	Massage	●	●				
	Battery back-up	●	●	●	●	●	●
	Safety net	●	●	●	●	●	●
	Airline-style headrest	●					
Customisation	25mm Seat depth adjustment				●		
	Backrest style / fill adjustability				●		
	Upholstery				●		
	Simplified handset*	●		●	●	●	
Size options		2	2	2	5	3	2

* Accessory



220426