



CLAMP-ON BED SUPPORT RAIL


INTENDED USE GUIDE

Small - 425 mm L x 385 mm H - SWL 100 kg | BEA829105

The clamp-on bed support rail is compatible with the Aspire LIFESTYLE Community Bed (Single BEB046030 & King Single BEB046040), Aspire MAXI LIFESTYLE Community Bed (BEB046020) and AC4 beds (Single BEB050400 & King Single BEB050410)

INSTALLATION

1. Undo both the handwheels and fit the rail onto the bed outside frame
2. Fasten the handwheels to secure the rail to the bed frame. Clearance between the rail and any end board must be > 32 cm

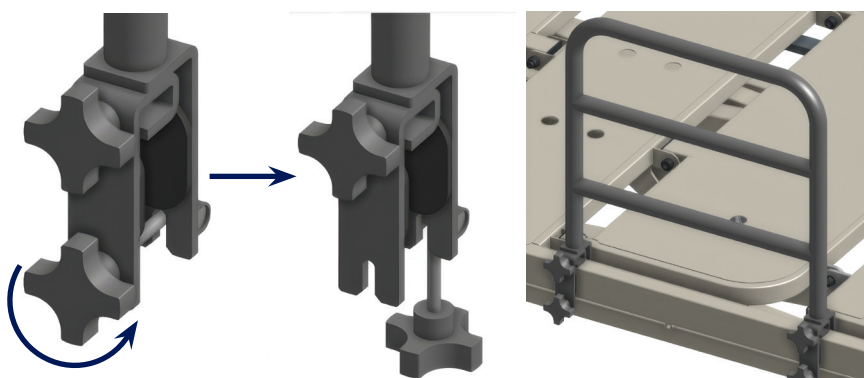

IMPORTANT



WARNING

Clearance between any point on the support rail and the head/foot board must be **greater than 32cm** at all times.



- **DO NOT** use this support rail while any other bed rail is mounted to the bed. Only one type of rail may be attached per bed side at any time
- **DO NOT** use with unsupported patient weight
- **DO NOT** install the support rail close to either the foot or head boards
- **DO NOT** tighten the thumbwheels unless it is mounted on a tube. Doing so may distort the shape of the bracket.
- **DO NOT** over-tighten the thumbwheels
- For use as a standing/repositioning aid **ONLY**
- This rail must **ONLY** be used with the Aspire LIFESTYLE Community Bed (Single BEB046030 & King Single BEB046040), Aspire MAXI LIFESTYLE Community Bed (BEB046020) and Aidacare AC4 Beds (Single BEB050400 & King Single BEB050140)

*All images are for illustration only

ASUM133.2 120225