



ACTIV CARE ADJUSTABLE BED

FIXED FOOTBOARDS INSTALLATION GUIDE

FIXED FOOTBOARD

Fixed Footboard

Long Single	BEA830160
King Single	BEA830162
Long Double	BEA830164
Queen	BEA830166

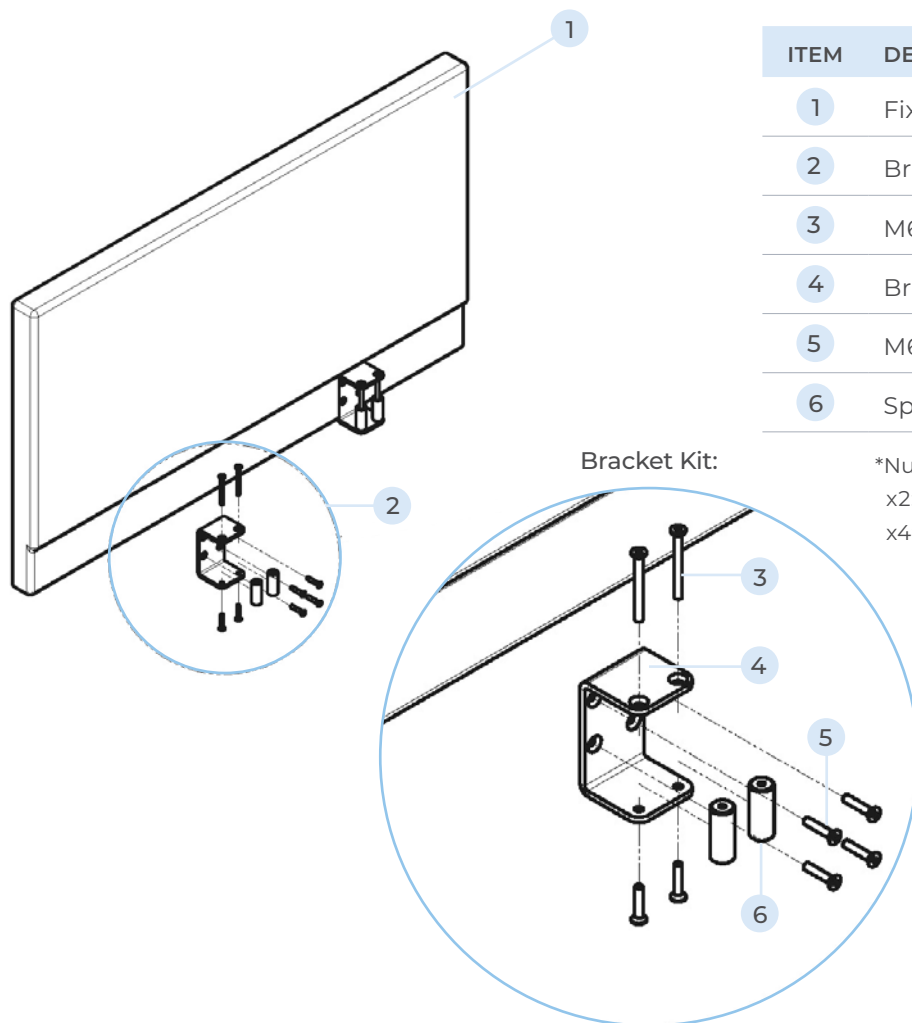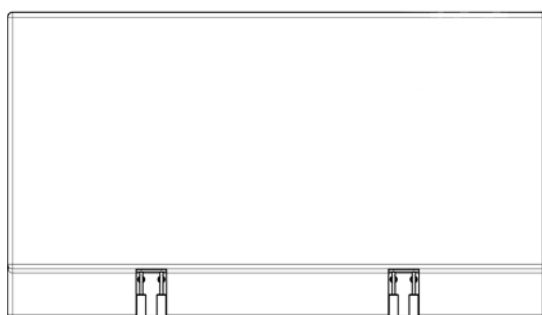


ITEM	DESCRIPTION	QTY
1	Fixed Footboard	1
2	Bracket Kit*	2 - 4
3	M6 x 60 Bolt	2
4	Bracket	1
5	M6 x 20 Bolt	2
6	Spacer	2

*Number of Bracket Kits included:
x2: Long Single & King Single
x4: Long Double & Queen

HOW TO ASSEMBLE FIXED FOOTBOARDS

TOOLS REQUIRED

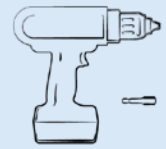

Impact driver
or drill (4mm hex bit)



4mm Allen Key (x2)

For use on the:

✓ ACTIV CARE BED




WARNING

It is recommended that assembly is carried out by TWO persons.

- 1 Fix brackets to the footboard. 4 x M6 screws per bracket

NOTE:

2x bracket kit for Long Single / King Single
4x bracket kit for Double / Queen)

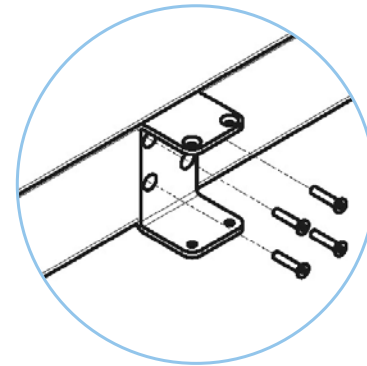

- 2 Raise the leg rest on the Comfimation bed to access the bed frame.


NOTE:

Beds with PO01145529 and later in their serial number will have pre drilled holes for mounting


Beds without mounting holes pre-drilled can have holes drilled by an experienced and competent person. Mark holes positions after step 3, remove the headboard and drill 6.5mm holes through the frame.




LEGREST
Raise to Max

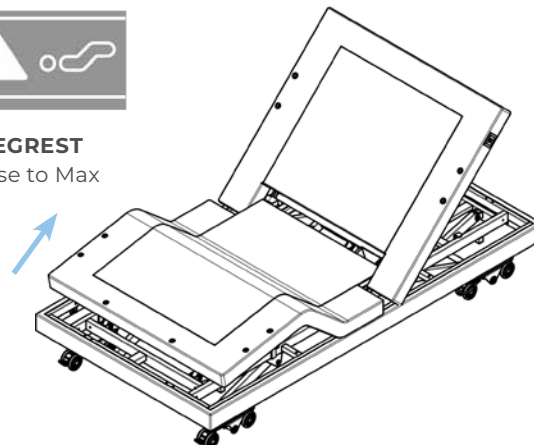


- 3 Position fixed footboard at the foot end of Comfimation bed.
The C shaped bracket is to be positioned around the upholstery and frame.

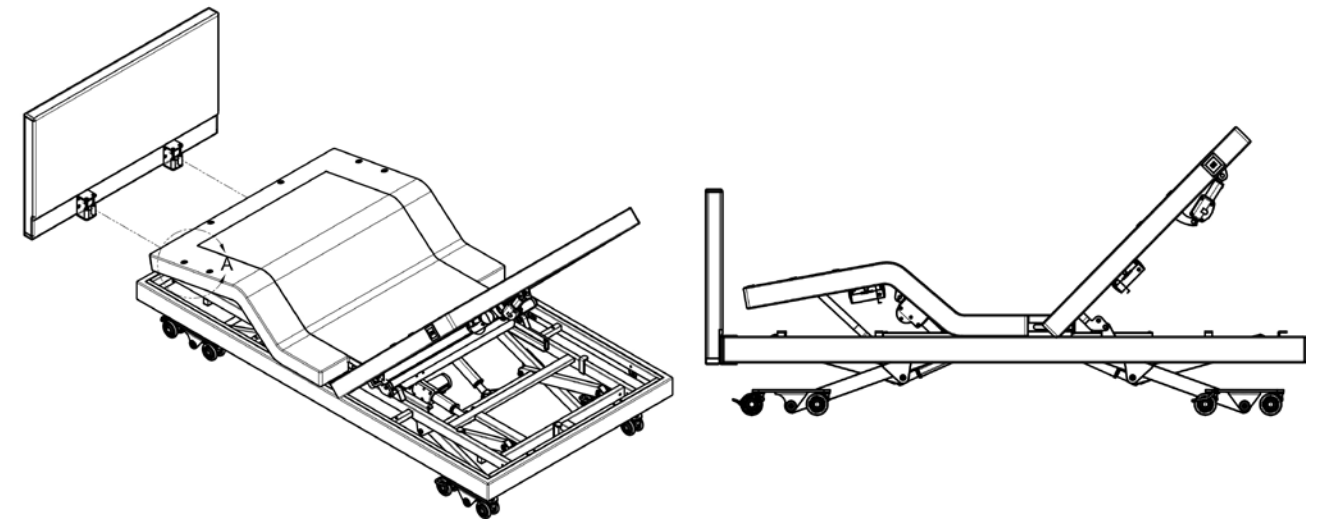

- 4 Fix the spacers to the bottom of the bracket once in position with 2x M6x20 screws.
Align to mounting holes in frame and tighten both top and bottom bolts to secure brackets in place.

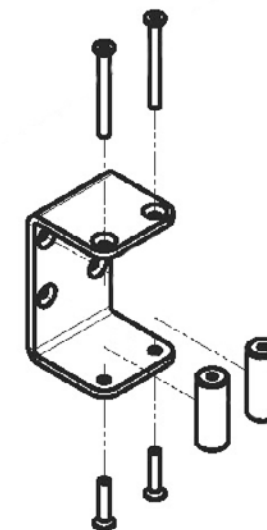

- 5 Make sure the footboard is secure with minimal movement between bracket and frame.

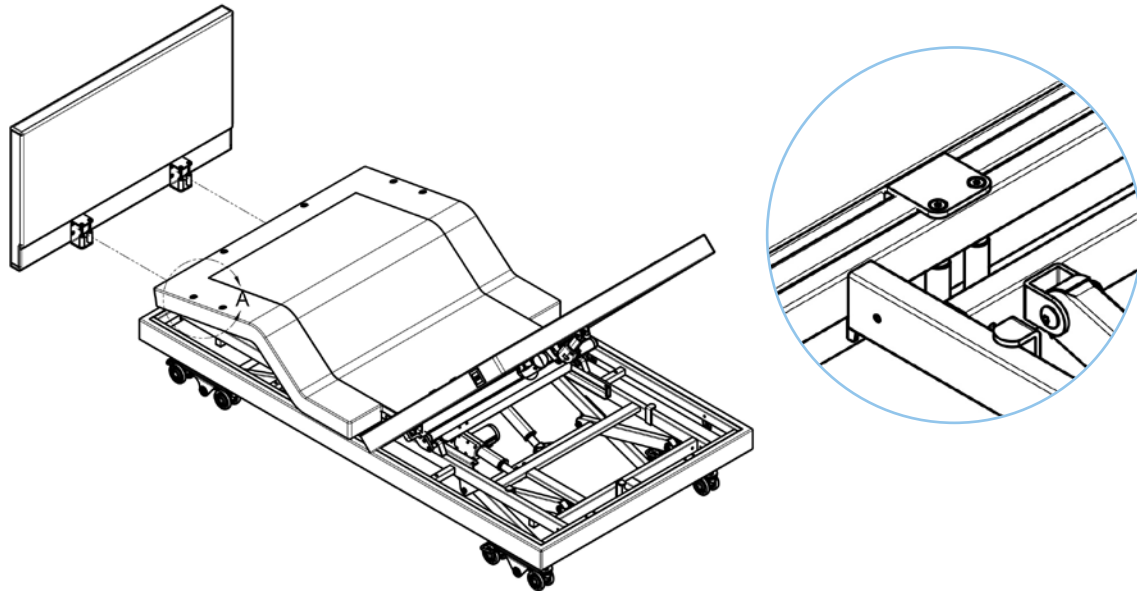

- 6 Test bed with all functions to ensure there are no collisions.
NOTE: Bed shown with both head and footboards installed

