DRUM WHEEL BRAKE KIT


QUICK INSTALLATION GUIDE

This guide is intended to explain the correct set up procedure for the Drum Wheel Brake Kit for the Aspire Evoke 2 Wheelchair.

For further operating instructions, refer to the product User Manual.

CODE	DESCRIPTION	SUITS				
		Evoke 2	Evoke 2 JNR	Rehab RX JNR	Rehab RX	Rehab RS
DRUM BRAKES						
WCA848510	Rear Wheels w/ Drum Brakes Kit - 16in	_	•	•	•	
WCA848520	Rear Wheels w/ Drum Brakes Kit - 20in	_		0		
WCA848500	Rear Wheels w/ Drum Brakes Kit - 24in	•	_	_	0	0

O - Comes as Standard Feature

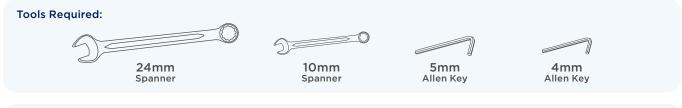



INSTALLATION GUIDE

Removing Camber Kit







- Remove quick release axle from wheelchair by pushing in spring button and pulling wheel outwards.
- 2 Lay the wheelchair on its side, remove camber kit from wheelchair frame using a 24mm spanner.



Installing Drum Brakes

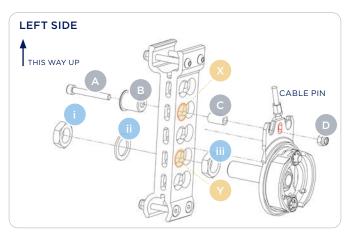

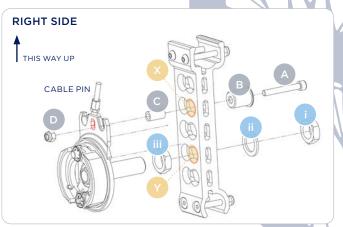

- 3 Ensure the direction of the brake kit is facing upwards towards the top of the wheelchair.
- Remove nut $\hat{\mathbf{0}}$ and washer $\hat{\mathbf{0}}$, Slide drum brake through hole $\mathbf{\hat{V}}$.
- 5 Stand wheelchair back onto it's wheels (from side lying position) ensuring cable pin stays pointing towards the top of the wheelchair.
- 6 Mount hardware ♠, ℮, ℮, ℮, through hole ♥ on wheelchair and drum brake, this must be done in alphabetical order as shown. Secure in place with 10mm Spanner & 5mm Allen key.

Installing Hand Brakes





- 7 Mount hand brake to wheelchair handle & fasten in place with a 4mm Allen key.
- 8 Fasten wheel to wheelchair by locking in place with the quick release axle provided in the box.
- 9 Repeat steps 1 to 8 for the other side.

Adjusting Brake Strength





If brakes require tightening or loosening, follow below steps.

- Remove wheel and quick release axle from wheelchair, slide out brake cable.
- 2 Adjust brake strength: Tighten (Clockwise) & Loosen (Anti-Clockwise).
- Reposition brake cable & wheel and lock into place with quick release axle.