

EVACUATION SHEET

Designed for the emergency evacuation of the non-ambulant person, the Aidacare LIFECOMFORT Evacuation Sheet helps rescue teams, carers or nurses to slide the mattress and resident along any floor surface or even downstairs, increasing resident safety and preventing injury to staff.







EVACUATION SHEET



Corner Straps

Keeps the Evacuation Sheet in position



Cross Straps

Secured over the resident & mattress (covered by protective velcro cover)



Pulling Straps

Located at head & foot-ends

EASY INSTALLATION









USING THE EVACUATION SHEET

- Release the Cross Straps and buckle them over the resident. A pillow may be placed under the Cross Straps, at the resident's chest and/or legs, for comfort.
- Pull the Cross Straps tightly to secure the resident and their bed linen prior to evacuation.
- Lower the mattress and resident carefully to the ground, then use the Pulling Straps to pull and slide the resident to safety.
- Before unbuckling the Cross Straps, release the tension first to prevent the Cross Straps snapping back.

Note: One carer should be at the head-end and another at the foot-end. If there is only one carer available, evacuate the resident using the Pulling Strap at the head-end of the mattress.

SPECIFICATIONS

Code	TSE672526-S	TSE672526
Description	Single	King Single
Length	1900mm	1900mm
Width	720mm	900mm
SWL	300kg	300kg

CARE

Washable up to 95°c with water and mild detergent

Do Not use solvents or non-approved bleaches

Do Not iron

Do Not twist or wring out