




Watson and Adjustable Day Chair

Quick Assembly Guide

CARTON CONTAINS:




1. Seat base and frame
2. Side frame left
3. Side frame right
4. Backrest frame lower
5. Backrest frame upper + hardware
6. Extension legs front (x2)
7. Extension legs with wheels rear (x2)
8. Head cushion
9. Hand wheel knob x6 (located on frame)

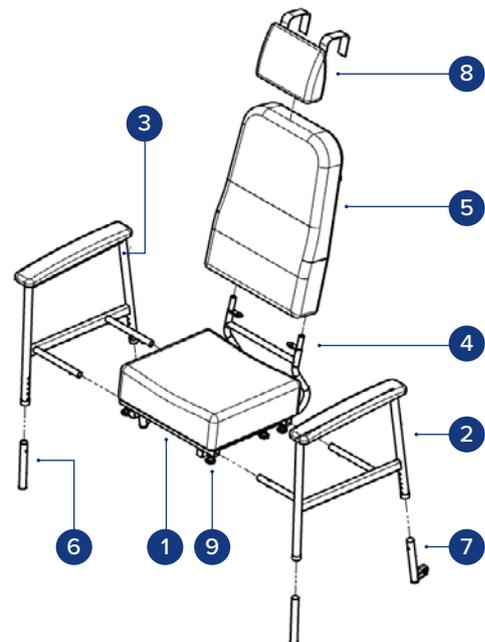

ASSEMBLY

SIDE FRAME / ARMRESTS

1. Put the left and right side frames into the base until you hear the push pins click.
2. Press the push pins underneath if you need to adjust.
3. Tighten the hand wheel knobs to hold the side frames in place.

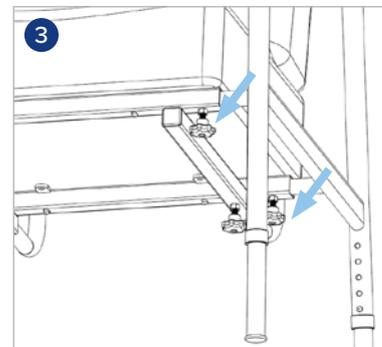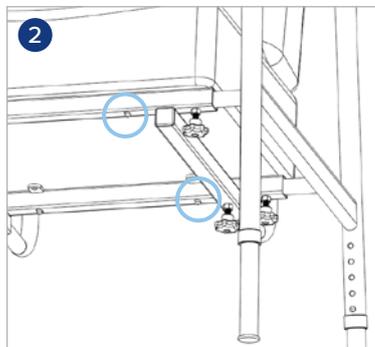
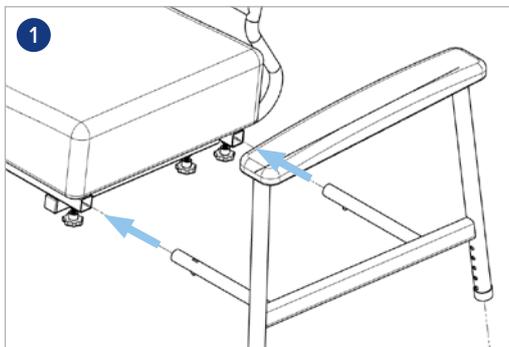


EXTENSION LEGS

1. Align and insert extension without wheels to the front.
2. Align and insert extension legs with wheels to the rear.
3. Ensure all locking pins click into place and are adjusted to the same height on both left and right sides.




Note: Extension legs with wheels may only be located at the back of the frame, allowing the chair to be tilted to manoeuvre (without a person in it).

For safety, ensure button is fully protruding.

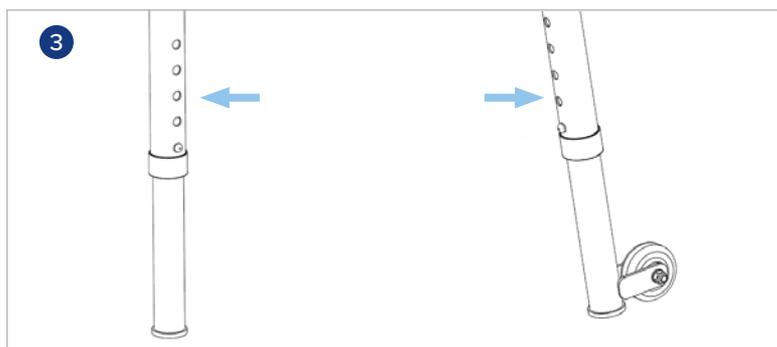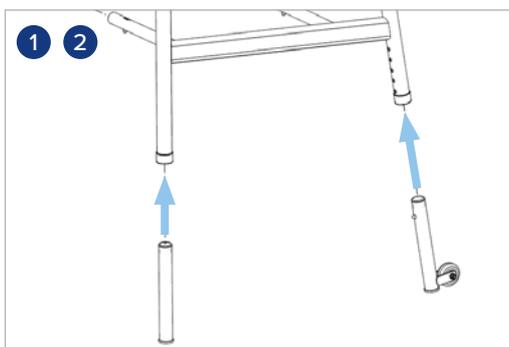



BACKREST

1. Attach the upper backrest to the lower backrest.
2. Tighten the bolt and nut to secure it.
3. Attach the head cushion straps to the backrest frame.

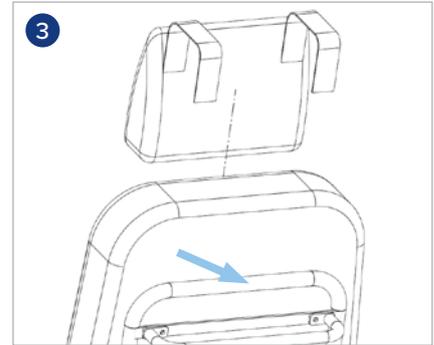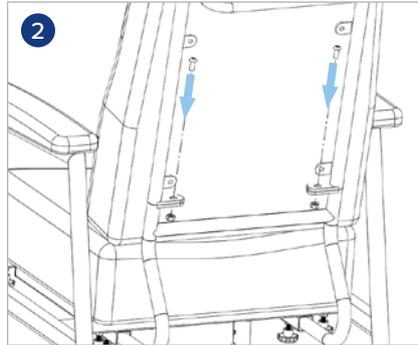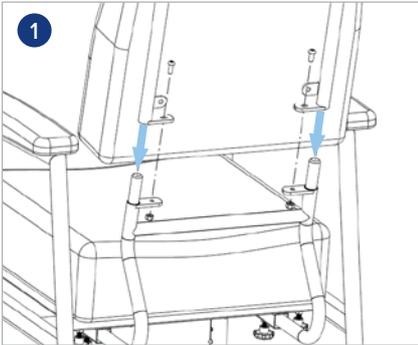

WIDTH ADJUSTMENT

1. Loosen the two hand wheel knobs on one side and move the frame to change the width. Tighten the bolt and nut to secure it.
2. The push pin will click when it reaches the widest point.

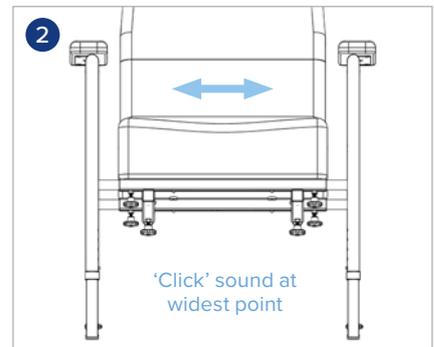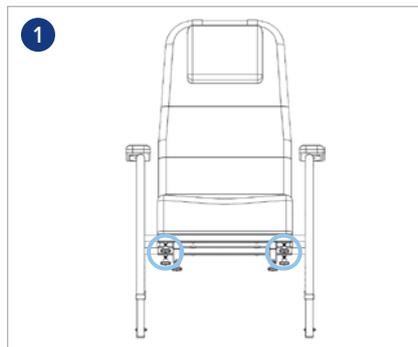

SEAT DEPTH ADJUSTMENT

1. Loosen the two hand wheel knobs holding the lower backrest frame.
2. Slide it to adjust depth.
3. The push pin will click at the maximum depth.

Once these steps are done, the pressure-reducing day chair is ready to use.

