



User Manual



W-ACM
Crash Mat with Wireless Nurse Call Alert



UK Version

The **Alertamat Crash Mat with Wireless Nurse Call Alert** is designed to be placed next to the bed to minimise injury if the user should fall from bed and activates the nurse call system when a fall occurs.

The mat incorporates sensor pads and a **Alerta Wireless Transmitter** within the foam which connects to the **Alerta Wall Point Receiver**. It can be connected to a nurse call system for centralised activation, or used on its own, independent of a nurse call system to give localised alerts through the **Alerta Wall Point Receiver** only.

Patent Number: 2203815.2

Installation

1. Follow the **Alerta Wireless Transmitter** user manual to install the batteries and power the transmitter on.
2. Install the transmitter into the **Alertamat Crash Mat** in the corner of the mat opposite the print as below;



3. Place the mat beside the bed with the print facing upwards and the zips in towards the bed as image
4. Follow Alerta Wall Point Receiver user manual to power up and set in pair mode, and then pair to the transmitter by applying pressure on the mat.

The **Alertamat Crash Mat** is quick and easy to position and benefits from a PU wipe clean surface. When unfolded the mat is 180cm long x 60cm wide x 6cm high. The mat folds in half for easy storage and transportation.



The mat must **ALWAYS** be placed with the print facing upwards and the zips in towards the bed.

UK | **CE** EU 2017/745
CA

EC | **REP**

CS Lifesciences Europe Ltd
The Black Church
St. Mary's Place
Dublin 7
D07 P4AX
Ireland

eurep@cslifesciences.com

Rev: 2024.09

Making **cutting-edge** medical equipment **affordable**

Head Office

Alerta Medical
4 Symington Place
Riverside Business Park
Irvine
KA11 5DE
United Kingdom

www.alertamedical.com

