



User Manual



W-WPR
Wall Point Receiver



UK Version

The **Alerta Wall Point Receiver** can be used on its own as a stand-alone system independent of a nurse call system, giving localised alerts, or connected to a nurse call system with centralised activation and its own in-room alerts. The volume can be deactivated for silent in-room operation.


The **Alerta Wall Point Receiver** can be wirelessly paired with up to 5 Alerta wireless devices; Alerta Wireless Bed and Chair Alertamats, Wireless Floor+ Alertamat, Wireless Deluxe Alertamat, Wireless Detect PIR, Wireless Alarm Crashmat, Wireless WetSense Bed and Chair Alertamats, Wireless Window/Door Sensor and Wireless Nurse Call Button. It uses radio frequency technology to connect, with a wireless range of up to 95 meters. Once paired with Alerta devices, it will only transmit to those devices and will not interfere with other Alerta Wall Point Receivers or devices in use nearby.

Ensure no other previously paired devices are being activated in other rooms when pairing new devices to a Wall Point Receiver.

Installing the Battery

Remove the battery cover on the back of the monitor by loosening the locking screw, then sliding the cover downwards. Insert 2 new 1.5V AA-type alkaline batteries (not supplied), taking care to observe proper battery polarity (positive to positive, negative to negative) or, the **Alerta Wall Point Receiver** can be powered using a mains AC/DC adaptor. After fitting batteries, replace the locking screw.

- When device is operative, the power indicator will flash green every 10 seconds.
- If the battery is low, the power indicator will flash red and an audible alarm will be triggered.
- Remember to remove batteries if not in use for a period of time.

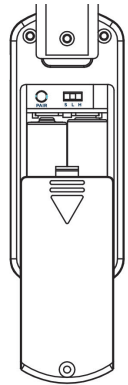


CAUTION



Improper installation of the battery or the installation of the incorrect battery type can result in damage to the contacts and cause improper operation.

How to use the Alerta Wall Point Receiver

1. Before connecting to the nurse call system, pair the wall point receiver to your desired Alerta wireless device(s). Firstly check the wireless device is powered and switched on. Next, refer to 'Pairing' section below and then come back to point 2;
2. Once successfully paired, select desired volume setting in the back under the battery cover; S=Silent, L=Low, H=High
3. If using **Alerta Wall Point Receiver** as a stand-alone system and not connecting to nurse call, then it is now ready to use. If using with a nurse call system then now plug in to the relevant wall point.

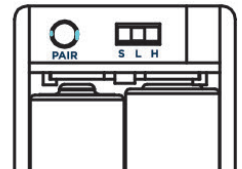

Pairing

The **Alerta Wall Point Receiver** has 5 Status indicators, 1 for each live device.

1. 'Pair' button is under battery cover. Press and hold 'Pair' button for 3 seconds. 5 LEDs flash twice and the unit will beep, release the 'Pair' button and the wall point receiver is now in pair mode.

When in pair mode, power indicator light is illuminated in bright green. Status lights light up to show number of currently paired devices.

2. Activate the wireless devices you wish to pair using the following methods depending on the device;

Apply pressure if;

- | | |
|---------------------------|----------------------------|
| Wireless Bed Alertamat | Wireless Chair Alertamat |
| Wireless Floor+ Alertamat | Wireless Deluxe Alertamat |
| Wireless Alarm Crashmat | Wireless Nurse Call Button |

Apply water using wet wipe or a drip from a cup of water if;

- | | |
|---------------------------------|-----------------------------------|
| Wireless WetSense Bed Alertamat | Wireless WetSense Chair Alertamat |
|---------------------------------|-----------------------------------|

Create movement if;

- Wireless Detect PIR

Separate the 2 parts from each other if;

- Wireless Window/Door Sensor


3. Once pairing successful, the corresponding LED will light up and then another device can be paired if spare LED lights left unlit.

4. Failed pairing;

- Device Repeat: If the wall point receiver gives a double beep and flashes for 6 seconds, repeat previous steps.
- If the wall point receiver gives a triple beep and blinks for 6 seconds then there are already 5 devices paired, if you wish you can clear all devices and re-pair.

5. To quit pairing mode, either press 'Pair' button to exit, or leave for 20 seconds to exit automatically.

6. To clear all paired devices, follow step 1 to enter pairing mode, then press and hold 'Pair' button for 6 seconds, 5 LEDs will flash twice followed by 1 long beep and all records will be cleared.

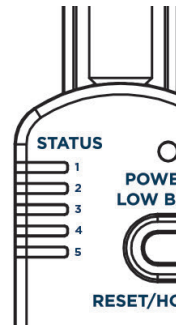

SCAN FOR HOW TO PAIR VIDEO

Alarming / Pause Alarming

When **Alerta Wall Point Receiver** receives a wireless signal from an Alerta wireless device, the corresponding alarm Status indicator flashes and alarm sounds.

1. To reset alarm, briefly press the 'Reset/Hold' button

2. To pause alarm activations for 3 minutes;

- Press and hold 'Reset/Hold' button for 3 seconds, 5 Status indicators will light up and the unit will give a double beep, release the 'Reset/Hold' button and the unit is now in pause alarm mode.
- The 5 Status indicators will continue flashing slowly.

3. To exit pause alarm mode either press 'Reset/Hold' button briefly, or it will automatically exit after 3 minutes.



UK | **CE** EU 2017/745
CA

EC | **REP**

CS Lifesciences Europe Ltd
The Black Church
St. Mary's Place
Dublin 7
D07 P4AX
Ireland

eurep@cslifesciences.com

Rev: 2025.11

Making **cutting-edge** medical equipment **affordable**

Head Office

Alerta Medical
4 Symington Place
Riverside Business Park
Irvine
KA11 5DE
United Kingdom

www.alertamedical.com

