









Multistander 1 by Jenx

A highly versatile standing system, providing 3 in 1 positioning in both neutral or abducted standing for children with moderate to complex needs, from approx. 9 months - 6 years



Always Supportive...



Multistander 1 by Jenx

This highly versatile standing frame is suitable for children from approx. 9 months to 6 years and provides a high level of postural support in upright, prone or supine with the opportunity to promote neutral or abducted standing posture.

An unrivalled amount of adjustment for height and width teamed with a maximum user weight of 35kg, make Multistander 1 the perfect choice for growing children in multi-user environments. Smooth and easy angle adjustments take the Multistander 1 from horizontal through to upright, facilitating easy transfers in supine and unlimited scope for setting the precise standing angle required for every child.

The deliberately open design ensures visibility of how the user is positioned each and every time. The capacity for outstanding thoracic and pelvic support, coupled with highly adjustable leg positioning options, make the Multistander 1 a great option for almost every child. If you need flexibility, the Multistander has got it!





Always Encouraging...

Standing is a great way to improve your child's outlook on the world and their physical development.

Improves blood pressure & circulation



Helps digestion, bowel & bladder emptying



Encourages bone & muscle development



Gives a new perspective and ways to participate



Children also gain huge psychological advantages from standing, as well as those important physical and development benefits.





Aids respiration & speech



Helps relieve pressure created during sitting



Supports pelvis, thighs & feet so children can join in everyday activity



Provides opportunities to increase social skills



PU Support Pads

Easy to clean and impervious to fluids! The boards and support pads on the Multistander 1 are made from polyurethane that has an integral antimicrobial agent within it.

This ensures Multistander 1 is quick and easy to keep clean and hygienic, which aids with re-issue for stores or for meeting infection control for multiple users.



Iways Configurable Accessories

Multistander 1 by Jenx is a modular product, allowing you to choose specific accessories to meet the exact requirements of each child.

Multistander 1 Frame

The Multistander frame includes:

- Base with 4 lockable swivel castors
- Knee block mounting brackets*
- Single angle adjustable footplate (Neutral standing) or split angle adjustable footplate (Abducted standing)

*NB: Knee Blocks must be specified separately.





Support Boards

PU chest & pelvic **Support** Boards in a range of 6 colours.







Multigrip™ Headrest (supine standing only)

The original and innovative Jenx Multigrip™ Headrest is available in 3 sizes and offers adjustment for height, depth and angle.



Waistcoat Harness

Provides additional trunk and shoulder stability for children with low tone. An innovative zip system which zips from the bottom up allows for easy-access to gastronomy feed tubes. Available in 2 sizes with a wipe-clean outer surface fabric.



Oval or Moulded PU Pad Headrest (supine standing only)

Headrests for children who require minimal head support. Comes mounted on a flexible fixing bracket, giving adjustment for height depth and angle.





in prone standing.

Tray

Chest Prompt

Tray for use in prone or

depth, complete with a

supine. Easy to attach and adjust for height, angle and

removable tray infill for use

Small foam sternal prompt, used to encourage extension of the thorax and help improve head control. Requires Support Board Covers for use.



Support Board Covers

Removable covers which wrap around the centre of the chest and pelvic Support Boards to add a splash of colour and for use with Chest Prompt.





PU Support Pads can be used at the hips or chest and are adjustable for height, width and angle and can be off-set to brace a scoliosis or accommodate and support unique positioning needs.











Available in 2 sizes, these

provide outstanding

super flexible support pads

positioning with comfort in mind. Independently









Tray Elbow Blocks

Used to encourage arms into a midline position and/or to block excessive retraction.



Abduction Accessory

This innovative accessory allows 0-30° of individual leg abduction in 5° increments. The tool-free mechanism allows simple angle adjustments with minimal effort. Available as a standalone accessory which can be retrofitted to existing Multistanders manufactured after 2015.



Knee Blocks

Innovative and flexible PU **Knee Blocks** available in 2 sizes, adjustable for height, width, angle and depth. Complete with padded, de-rotational knee straps.



Colour options available.

Sandals

Foot positioning available in 2 sizes to accommodate different size feet, even when wearing splints. Jenx **Sandals** are interchangeable across the entire range of Jenx seating and standing products.



Ankle Snugs

An optional foot accessory to the standard **Sandals**, the **Ankle Snugs** offer a unique magnetic buckle fastening mechanism, which can be operated single-handed.



Sandal Raising Blocks

Available for every size of Jenx **Sandals**, the raising block kits give the **Sandals** that extra bit of height where needed! Use to accommodate leg length discrepancy or to raise the height of the footplate to maximise the available growth.



NB: One kit needed per individual sandal.

Multistander 1 Colour Options





Multistander 1 by Jenx Technical Specifications

IMPORTANT INFORMATION: Please be aware, some minimum and maximum measurements are achieved by the use of additional accessory components.	Measurement	Units	Size 1 MUS01	Size 1 Abduction MUS01-FABD
Age Range	approx.	mths-yrs	9mths - 6yrs	9mths - 6yrs
User Height	min - max	mm	620 - 1160	600 - 1160
User Weight	max	kg	35	35
Chest Width	min - max	mm	140 - 275	140 - 275
Hip Width	min - max	mm	140 - 275	140 - 275
Height (sandal plate to top of upper support board)	min - max	mm	495 - 900	500 - 890
Height (sandal plate to top of head support: Supine only)	min - max	mm	620 - 1160	600 - 1160
Height (floor to top of upper support pad)	min - max	mm	670 - 940	610 - 940
Kneeblock Width (centre to centre)	min - max	mm	120 - 200	130 - 175
Footplate Angle	dorsi - plantar	degrees	15° - 15°	15° - 15°
Individual Leg Abduction (to achieve full 30° abduction footplate is required to be 140mm off floor level)	min - max	degrees	-	0° - 30°
Tilt-In-Space Angle	horizontal - upright	degrees	10° - 90°	10° - 90°
Base Footprint	width x length	mm	565 x 710	565 x 710

Find Your Distributor

To locate one of our carefully selected distributors near you, visit: jenx.com/where-to-buy







@JenxLtd

@JenxLtd

@JenxLtd



Jenx Limited, Wardsend Road, Sheffield S6 1RQ, United Kingdom tel: +44 (0) 114 285 3376 email: export@jenx.com www.jenx.com

