

POWER ASSISTED WHEELCHAIR SYSTEMS





1300 133 120 aidacare.com.au



CONNECTS AUTOMATICALLY AT THE TOUCH OF A BUTTON

No clumsy levers, wheelchair modification or clunky brackets are required.



Patented Automatic Connection Process

- 1. Fasten PAWS to frame with electric clamps
- 2. Raise wheelchair to DRIVE position





Parking position

Drive position

Solutions for: The daily commute

CITY

Let PAWS be a part of your everyday mobility. Get to work with ease and take care of the shopping on the way home. Seize your daily opportunities with confidence. Get busy with PAWS!

PAWS City comes with a 12" front wheel and is designed primarily for city travelling. Choose 12" for the smoother roads. With a 1m turning circle, it is great indoors as well. It likes hard ground and even surfaces, but can also cope with uneven pavement and curbs.





AUTOMATIC CLAMPING

With Automatic Easy Clamps just roll your wheelchair into the clamps and press the button for automatic clamping and lifting to the driving position. From start to go in 10 to 15 seconds!

Configuration shown: WCA872100

2

Solutions for: The open road

CRUISER

A great solution for a variety of mobility purposes. Whether it is around town or along a forest trail, this model is for you. The big 16" drive wheel eats up the bumps for a really smooth ride. Get your PAWS on the track and easily overcome the obstacles in the terrain.



With Automatic Easy Clamps just roll your wheelchair into the clamps and press the button for automatic clamping and lifting to the driving position. From start to go in 10 to 15 seconds!

Configuration shown: WCA872120

Solutions for: The road less travelled

TOURER

With PAWS you can feel confident even in unexplored terrain. PAWS Tourer has the longest wheelbase so while the big wheel overcomes roots and uneven terrain, it is also makes a smooth ride while touring. PAWS Tourer is the true off-road stager. With a 20" wheel and 4" thick tyre, you will easily get to places that would be difficult for other models. Designed primarily for off-road purposes, it is not afraid of sand, gravel, stones.. Get your PAWS in the dirt!



Rider Controls

CLASSIC CONTROLS

Classic control interface (supplied as standard) offers motorcycle experience with twist grip for acceleration and hand activated braking.



QUADRIPLEGIC CONTROLS

Quad Control interface integrates acceleration and braking into one accessible hand control lever system, for those with limited hand & finger dexterity or coordination.



INSTRUMENT CLUSTER

The instrument cluster provides an intuitive layout offering:

- · 5 preset speed modes
 - Driving lights
- Cruise control

· Traction control

· Emergency stop

 Forward and reverse · Walk mode

TOUCHSCREEN DRIVER DISPLAY

- Speed
- · Driving Mode

- Distance
- Battery Level

Performance

DRIVING LIGHTS

- · Automatically activate in low light
- · Independently adjustable
- · High powered LED for a crisp viewing outlook

WEATHERPROOF

· IP54 waterproof rating means rain, puddles and the great outdoors are all within reach

TRAVEL RANGE

- · Global reach with optional IATA approved flight-safe
- · Swap and Go batteries mean no waiting for a charge
- The powerful lithium ion battery offers up to 40kms





(Tourer shown.)

Specifications	City		Cruiser		Tourer	
	WCA872100 Standard	WCA872110 Tetra	WCA872120 Standard	WCA872130 Tetra	WCA872140 Standard	WCA872150 Tetra
Max Speed	10km/h		10km/h		10km/h	
Unit Weight	17.5kg		20.5kg		25.5kg	
Overall Length	800mm		820mm		1000mm	
Overall Width	490mm		490mm		520mm	
Overall Height	810mm		860mm		920mm	
Connection/Release Time	10-15 secs		10-15 secs		10-15 secs	
Minimum Turning Circle	1000mm		1200mm		1500mm	
Drive Wheel Size	14"		16"		20"	
Battery Charge Time	5hrs					
Frame/Drive Motor Warranty	2 years					
Battery Warranty	1 year					
Compliance	AS/NZS 3695.2:2013*					

^{*}Tested on specific compatible chairs, contact Aidacare to confirm if your chair meets these requirements.

ACCESSORIES

STORAGE BAG

WCA872280

Secures firmly with velcro fasteners. Zip top and side pockets keep valuables safe and essentials close to hand.



STORAGE BASKET

WCA872270

Easy clip on and off basket that allows you to load up for your trip. With sturdy heavy gauge steel rod construction, it is dip coated to prevent corrosion.



PHONE HOLDER

WCA872260

Allows phone to be safely stored and visible when in use. Facilitating hands free functions such as navigation.



SMART TYRE INFLATOR & PHONE CHARGER

WCA87221

The mini compressor with rechargeable battery can automatically inflate tyres to specific pressures and offers USB charging for other devices.





QUICK SWAP BATTERIES

Lithium Ion Battery Pack - 11.6AH - WCA872220 Lithium Ion Flight Battery Pack - 5.6AH - WCA872230

Secured under lock and key, the powerful lithium ion battery packs can be conveniently swapped avoiding the need to wait for charging. Full charge time is 5 hours and each has an inbuilt charge indicator.

Air travel is possible with the optional Flight battery. IATA approved for travel on passenger planes, this smaller battery pack meets the maximum power levels allowed by commercial carriers.



Exclusively Distributed by:

| Oldacare | Compared | Co

