



EXCLUSIVELY DISTRIBUTED BY





# Move More. Live More.

Experience the world with WHILL and live life without boundaries.

Innovative products built to challenge the perception of what a mobility device can be.

WHILL Model C2







# Convenience

The Model C2 is simple to operate and can make tight turns to get you wherever you need to go.



The basket<sup>1</sup> is conveniently located under the seat, making it easy to get your items in and out.

Dimensions: 24 cm x 29 cm x 43 cm

<sup>1</sup>Optional accessory: WCA870440



Lift the arm to easily sit down in the chair from the side. Stand on the foot support for added stability.



Turning Radius  
76cm

Adventure with WHILL - the Model C2 can make tight turns, allowing you to go places that a typical scooter can't take you.



Turn the power on and adjust the speed with one hand. The device will automatically brake when you release the controller, even on an incline.








## Powerful

Impressive power that allows you to see and do more. Live the life you aspire to with the WHILL Model C2.

Get out and adventure!



Effortlessly climb over obstacles up to 5cm in height with two powerful motors and large front omni-wheels.

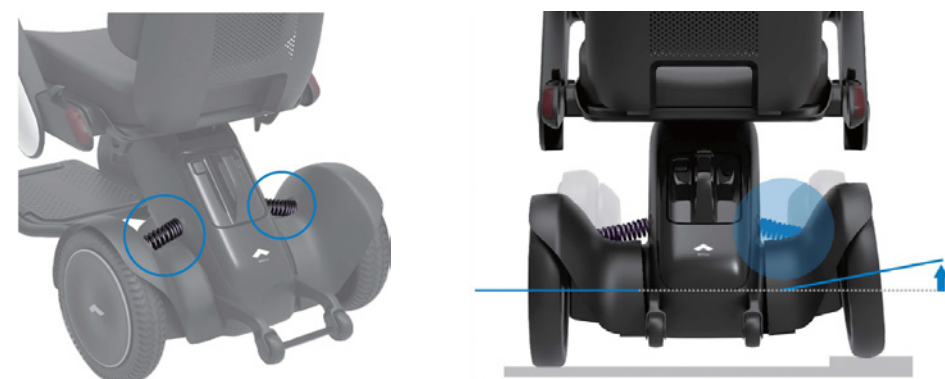

Rear wheel suspension absorbs the impact when driving over bumpy roads or obstacles.





Easily disassemble  
to fit in the boot  
of your car

\*Image shown above are for  
illustrative purposes only

# Portable

Disassembles in four easy steps.

The Model C2 can be disassembled for easy transportation in the boot of any car. Whether you're shopping on the weekend or traveling with your family, the Model C2 allows you to live life to the fullest.



Remove the battery.



Raise the lever under the seat.



Pull the seat upwards.



Pull the lever on the side of the battery towards you to separate the parts.





## Colour Variation

### Drive in Style.

Choose from 8 different arm colour options and customise the Model C2 to match your personal style.



Navy Blue

LEFT / WCA870540 | RIGHT / WCA870550



White

LEFT / WCA870580 | RIGHT / WCA870590



Grey

LEFT / WCA870500 | RIGHT / WCA870510



Black (supplied standard)

LEFT / WCA870460 | RIGHT / WCA870470



Red

LEFT / WCA870600 | RIGHT / WCA870610



Pink

LEFT / WCA870560 | RIGHT / WCA870570



Light Blue

LEFT / WCA870520 | RIGHT / WCA870530



Gold

LEFT / WCA870480 | RIGHT / WCA870490



We've thought of it all!

Feedback from our customers has helped shape the Model C2. We've packed in all the features you're looking for, including a remote lock and the ability to adjust the max speed for added safety.



Smart Key  
Remotely lock the device.

Tail Light



The tail light is clearly visible even with a bag hung over the back support, keeping you safe on the road at night.



The controller can be positioned on the left or right side.



Prevents drifting

Drive straight on a tilted path.

Battery  
Detachable battery.

5  
hour charge



Model C2 can travel 18km on a 5-hour charge, giving you ample power to go about your day.




2.7 kg

The battery can be removed and charged in a regular household outlet.




Check your total mileage, distance travelled, battery level, full charge cycles and more with the WHILL app\*. You can also lock and unlock the unit.

\*You will need to install the application beforehand.

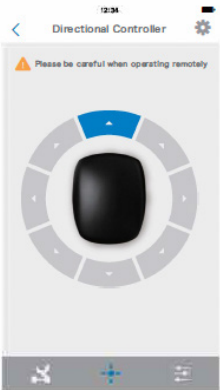



The intuitive controller screen allows you to control the unit remotely.

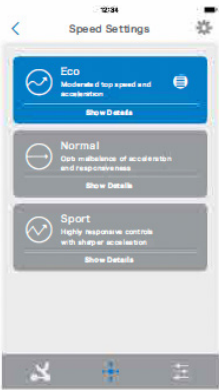

Set the drive mode to "Eco", "Normal", or "Sport" depending on your needs.

# The Right Fit

The Model C2 can be easily adjusted from seat height and back support angle to controller position (right or left drive) to create a comfortable driving experience.

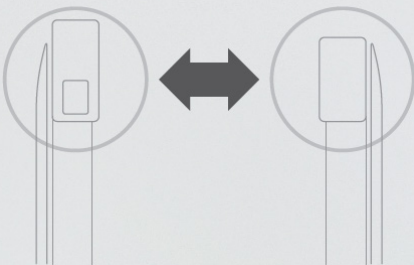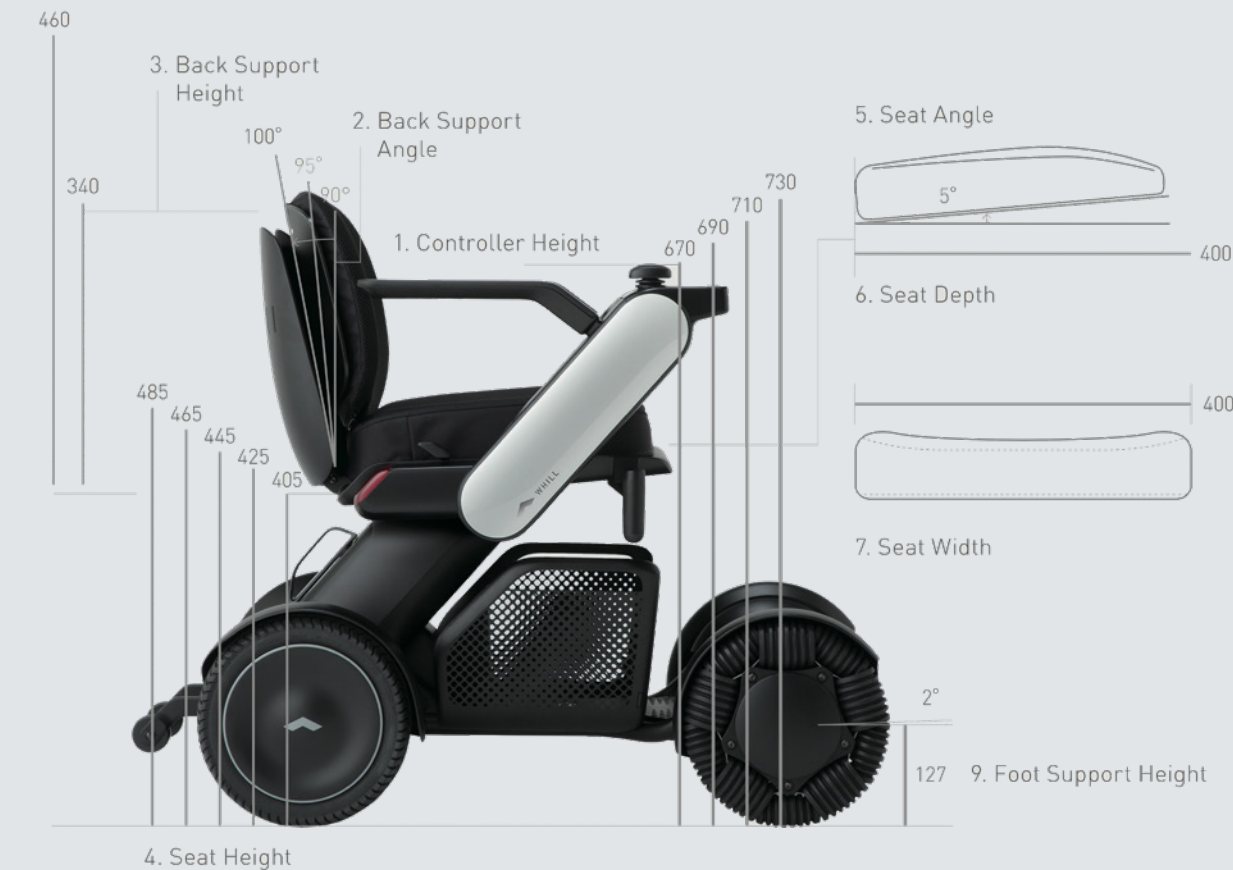

The controller can be positioned on the left or right side.

- 1. Controller Height.....670-730mm
- 2. Back Support Angle.....90-100°
- 3. Back Support Height...340-460mm
- 4. Seat Height.....405-485mm
- 5. Seat Angle.....5°
- 6. Seat Depth.....400mm
- 7. Seat Width
  - PWS655400.....450mm
  - PWS655410.....500mm
- 8. Foot Support Angle.....2°
- 9. Foot Support Height....127mm

Warning: When using your own cushion, for safety, be sure to use one that has sufficient flame retardancy.

# Performance Specifications

- Driving Range ..... 18 km
- Maximum Speed ..... 8 km/h
- Drive System ..... Two Motor Drive System
- Incline Capability ..... 10°
- Turning Radius ..... 760 mm
- Obstacle Clearance ..... 50 mm
- Ground Clearance ..... 57 mm
- Braking System ..... Electromagnetic
- Max. Weight Capacity ..... 136 kg

- Total Length (including foot rest): ..... 985 mm
- Total Width:
  - PWS655400 ..... 604mm
  - PWS655410 ..... 654 mm
- Total Height: ..... 745 - 945 mm

- Seat Assembly
  - Length: ..... 570 mm
  - Width:
    - PWS655400 ..... 600 mm
    - PWS655410 ..... 650 mm
  - Height: ..... 480 mm


- Rear Drive Base
  - Length: ..... 590 mm
  - Width: ..... 554 mm
  - Height: ..... 300 mm

- Front Drive Base
  - Length: ..... 720 mm
  - Width: ..... 554 mm
  - Height: ..... 270 mm

- Front Wheel: .... Omni-Wheel
- Rear Wheel: ..... Standard: Airless Tire

- Other Specifications
  - Charging Time ..... 5 Hours
  - Battery ..... Lithium-ion (25.3 V x 10.6 Ah)
  - Battery Charge Cycles ..... ~1.000
  - Battery Wh ..... 269 Wh
  - Operating Conditions ..... -15 - 40°C
  - Seat Cushions ..... 450 mm x 450 mm
  - Tail Lamps ..... LED Light (red)
  - Horn ..... Built-in, sound level > 65dB
  - Wheelchair Class ..... A



 \*Do not use the device as a seat in an automobile or other vehicle.

\*The unoccupied device is suitable for land and air transport.



# Accessories

Purchase accessories for even more convenience and personalisation.



Arm Covers

SEE PAGE 10-11 FOR CODES

You can change the colour by purchasing additional arm covers.

(Black is included as standard)



Lap Belt

WCA870420

Firmly stabilises the sitting position even on slopes.

Note: Standard inclusion



Battery

WCA870400

The 2.7kg lightweight battery can also be used as a spare battery.

Note: Standard inclusion



Smart Key

It provides you with the ability to lock and unlock your Model C2 from a distance, similar to a car's key fob.

Note: Standard inclusion



Battery Charger

WCA870410

A great compliment to a spare battery, it helps let you go from one fully charged battery to the next fully charged battery to keep you moving.

Note: Standard inclusion



Dust Cover

WCA870260

Protects the device from dust and dirt when stored outdoors\*.

\*It is not waterproof. Do not use for protection from rain.



Custom Backrest Frame


STANDARD WCA870456

TALLER UPRIGHT WCA870457



Side Storage Bag

WCA870430

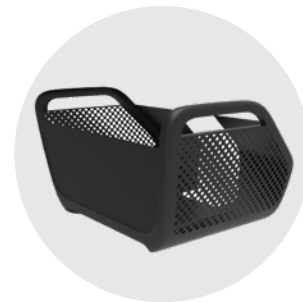

Storage Basket

WCA870440







Move More.  
Live More.





**AIDACARE PTY LTD**

Building 3A, 1 Moorebank Ave  
Moorebank NSW Australia 2170  
1300 133 120 | [www.aspirecare.com.au](http://www.aspirecare.com.au)