



# Convenience

The Model C2 is simple to operate and can make tight turns to get you wherever you need to go.



The basket¹ is conveniently located under the seat, making it easy to get your items in and out.

Dimensions: 24 cm x 29 cm x 43 cm <sup>1</sup>Optional accessory: WCA870440



Lift the arm to easily sit down in the chair from the side. Stand on the foot support for added stability.



Adventure with WHILL - the Model C2 can make tight turns, allowing you to go places that a typical scooter can't take you.



Turn the power on and adjust the speed with one hand. The device will automatically brake when you release the controller, even on an incline.





# Powerful

Impressive power that allows you to see and do more. Live the life you aspire to with the WHILL Model C2.

Get out and adventure!



Effortlessly climb over obstacles up to 5cm in height with two powerful motors and large front omni-wheels.



Rear wheel suspension absorbs the impact when driving over bumpy roads or obstacles.



## Portable

Disassembles in four easy steps.

The Model C2 can be disassembled for easy transportation in the boot of any car. Whether you're shopping on the weekend or traveling with your family, the Model C2 allows you to live life to the fullest.



Remove the battery.



Raise the lever under the seat.



Pull the seat upwards.



Pull the lever on the side of the battery towards you to separate the parts.



# Colour Variation

### Drive in Style.

Choose from 8 different arm colour options and customise the Model C2 to match your personal style.







White LEFT / WCA870580 | RIGHT / WCA870590



Grey
LEFT / WCA870500 | RIGHT / WCA870510



Black (supplied standard)
LEFT / WCA870460 | RIGHT / WCA870470



Red
LEFT/WCA870600 | RIGHT/WCA870610



Pink
LEFT/WCA870560 | RIGHT/WCA870570



Light Blue
LEFT / WCA870520 | RIGHT / WCA870530



Gold
LEFT/WCA870480 | RIGHT/WCA870490

LEFT / WCA870540 | RIGHT / WCA870550

10 11

### We've thought of it all!

Feedback from our customers has helped shape the Model C2. We've packed in all the features you're looking for, including a remote lock and the ability to adjust the max speed for added safety.






Smart Key
Remotely lock the device.



The tail light is clearly visible even with a bag hung over the back support, keeping you safe on the road at night.

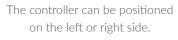





Drive straight on a tilted path.

#### Battery

Detachable battery.

bour charge



Model C2 can travel 18km on a 5-hour charge, giving you ample power to go about your day.



The battery can be removed and charged in a regular household outlet.









Check your total mileage, distance travelled, battery level, full charge cycles and more with the WHILL app\*. You can also lock and unlock the unit.

\*You will need to install the application beforehand.



The intuitive controller screen allows you to control the unit remotely.




Set the drive mode to "Eco", "Normal", or "Sport" depending on your needs.

### The Right Fit

The Model C2 can be easily adjusted from seat height and back support angle to controller position (right or left drive) to create a comfortable driving experience.



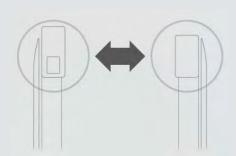

The controller can be positioned on the left or right side.

- 1. Controller Height......670-730mm
- 2. Back Support Angle.....90-100°
- 3. Back Support Height...340-460mm
- 4. Seat Height......405-485mm
- 5. Seat Angle.....5°
- 6. Seat Depth.....400mm
- 7. Seat Width
  - PWS655400.....450mm
  - PWS655410.....500mm
- 8. Foot Support Angle.....2°
- 9. Foot Support Height....127mm

## Performance Specifications

Driving Range ...... 18 km

Maximum Speed ...... 8 km/h

Drive System ...... Two Motor Drive System

Incline Capability ...... 10°

Obstacle Clearance ...... 50 mm

Ground Clearance ...... 57 mm

Braking System ..... Electromagnetic

Max. Weight Capacity ..... 136 kg

Total Length

(including foot rest): ...... 985 mm

Total Width:

PWS655400 ...... 604mm

PWS655410 ...... 654 mm

Total Height: ...... 745 - 945 mm

Seat Assembly

Length: ..... 570 mm

Width:

PWS655400 ..... 600 mm

PWS655410 ...... 650 mm

Height: ..... 480 mm

Rear Drive Base

Length: ..... 590 mm

Width: ..... 554 mm

Height: ...... 300 mm

Front Drive Base

Length: ..... 720 mm

Width: ..... 554 mm

Height: ..... 270 mm

Front Wheel: .... Omni-Wheel

Rear Wheel: ..... Standard: Airless Tire

#### Other Specifications

Charging Time ...... 5 Hours

Battery ..... Lithium-ion (25.3 V x 10.6 Ah)

Battery Charge Cycles ..... ~1.000

Battery Wh ...... 269 Wh

Operating Conditions ...... -15 - 40°C

Seat Cushions ...... 450 mm x 450 mm

Tail Lamps ..... LED Light (red)

Horn ..... Built-in, sound

level > 65dB

Wheelchair Class ...... A





\*Do not use the device as a seat in an automobile or other vehicle.

\*The unoccupied device is suitable for land and air transport.

15

4

Warning: When using your own cushion, for safety, be sure to use one that has sufficient flame retardancy.

### Accessories

Purchase accessories for even more convenience and personalisation.



Arm Covers
SEE PAGE 10-11 FOR CODES

You can change the colour by purchasing additional arm covers.

(Black is included as standard)



Smart Key

It provides you with the ability to lock and unlock your Model C2 from a distance, similar to a car's key fob.

Note: Standard inclusion



Side Storage Bag wca870430



Lap Belt wca870420

Firmly stabilises the sitting position even on slopes.

Note: Standard inclusion



Battery wca870400

The 2.7kg lightweight battery can also be used as a spare battery.

Note: Standard inclusion



Dust Cover

Protects the device from dust and dirt when stored outdoors\*.

\*It is not waterproof. Do not use for protection from rain.



Battery Charger WCA870410

A great compliment to a spare battery, it helps let you go from one fully charged battery to the next fully charged battery to keep you moving. Note: Standard inclusion



Storage Basket







#### AIDACARE PTY LTD

Building 3A, 1 Moorebank Ave Moorebank NSW Australia 2170 1300 133 120 | www.aspirecare.com.au