

Aspire Aura Floor Lifter Spare Parts Manual



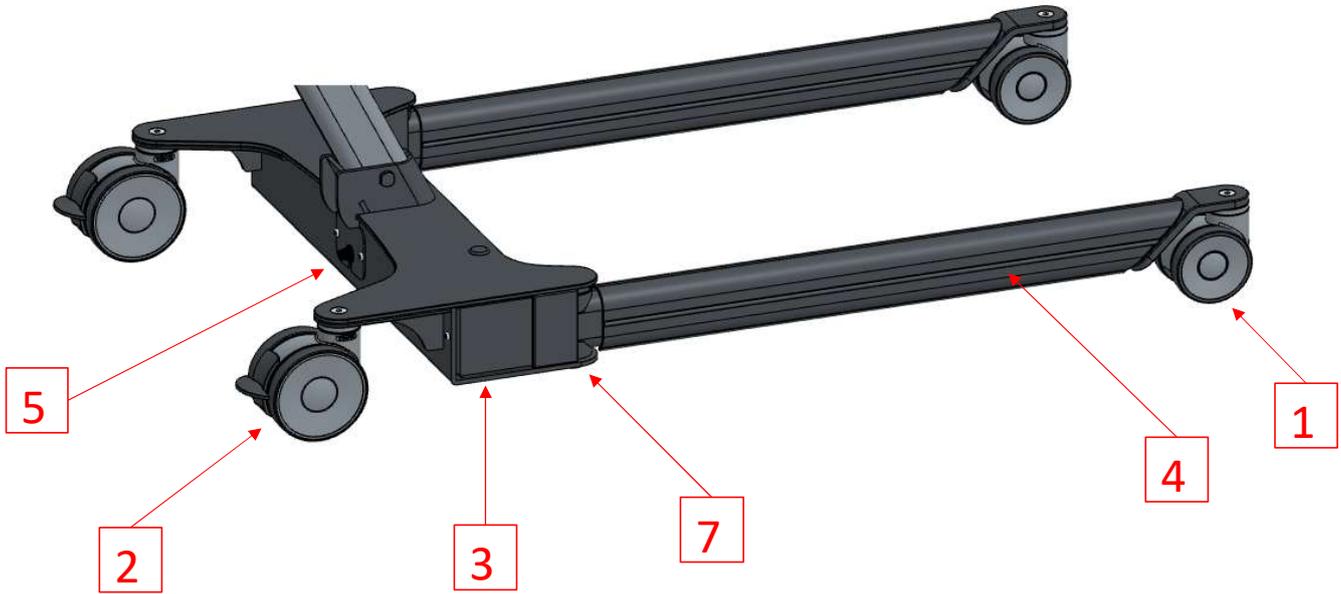
Item Number	Item Description
LSS390615	Aspire Aura - Floor Lifter - 250kg

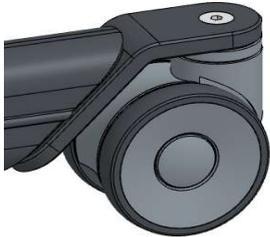
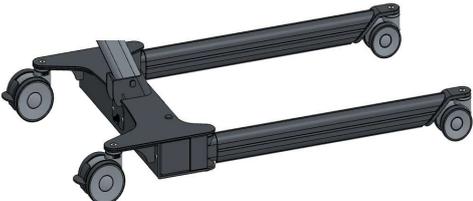
This manual is for internal use only

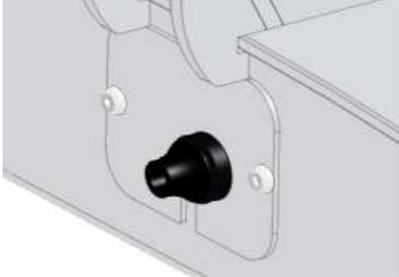
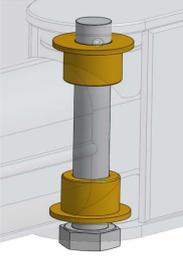
Spare Parts Diagram



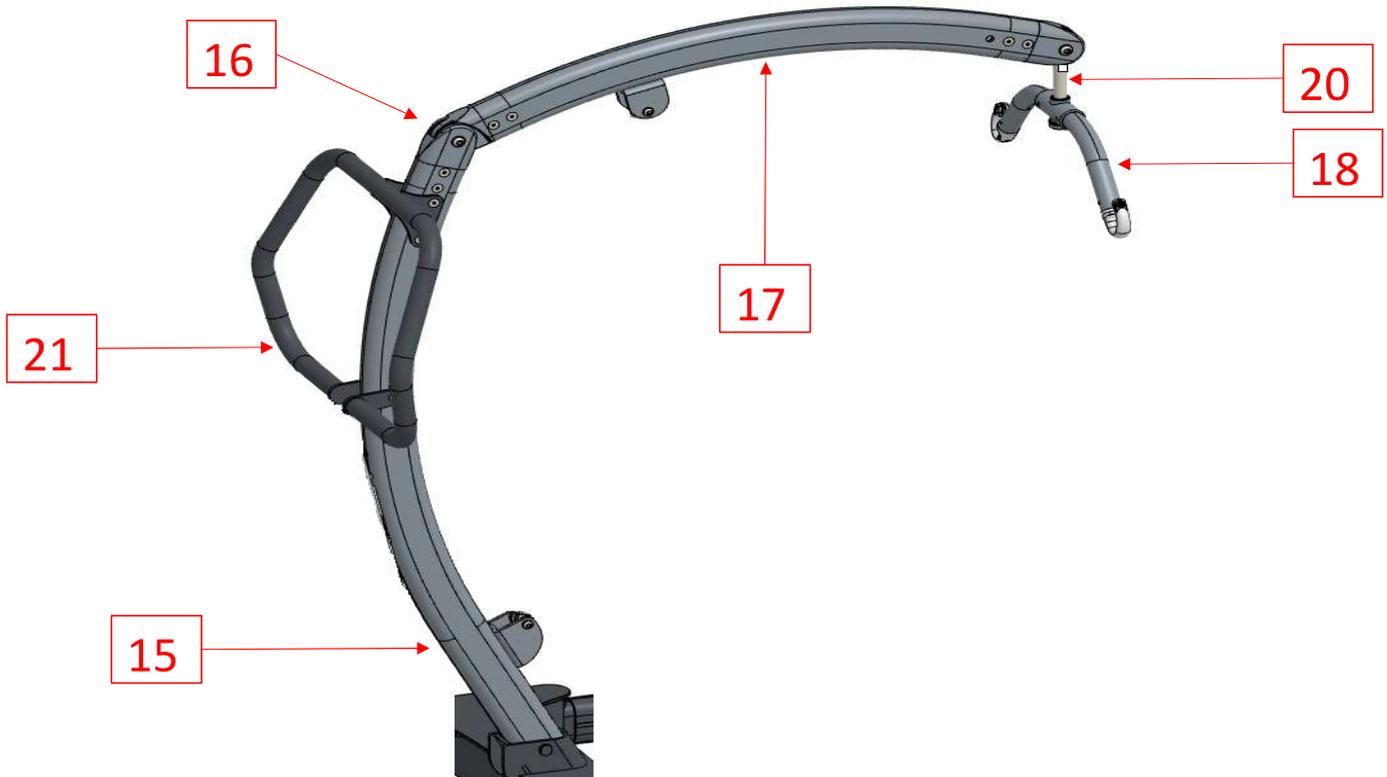
1 Base and Castors

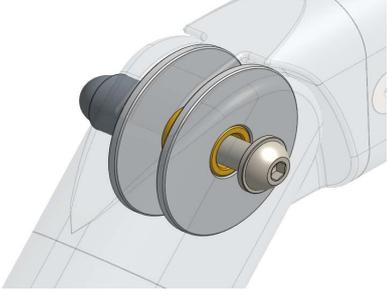


Ref	Item Number	Item Description	Image	Type
1	GSP242800	Front Castor - w/Mounting Hardware - Aspire Nova Standing / Aura Flor Lifter		Spare Part
2	GSP242802	Rear Castor - w/Mounting Hardware - Aspire Nova Standing / Aura Flor Lifter		Spare Part
3	GSP242900	Base Frame - Complete - w/Castors - w/o Electronics - Aspire Aura Floor Lifter		Spare Part

1 Base and Castors				
4	GSP242902	Plastic Covers - Legs - Pair - Aspire Aura Floor Lifter		Spare Part
5	GSP242808	Rubber Grommet - on Base Frame - Aspire Nova Standing / Aura Floor Lifter		Spare Part
7	GSP242814	Hardware - Legs Pivot Point - Aspire Nova Standing / Aura Floor Lifter		Spare Part

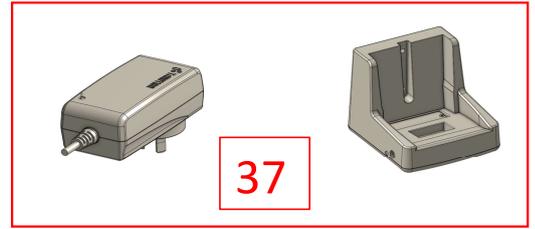
2 Mast, Boom, Sling Bar and Handle

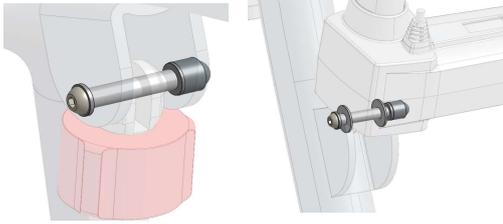


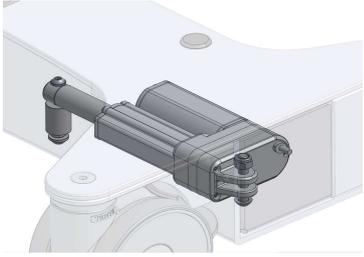
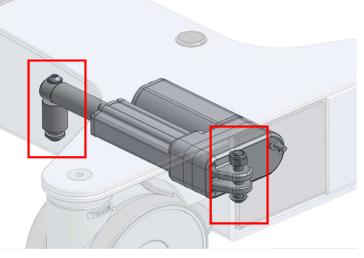
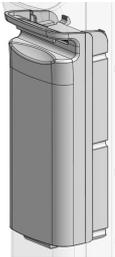
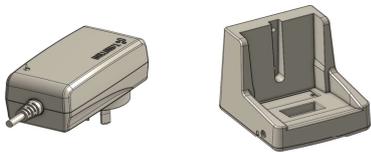
Ref	Item Number	Item Description	Image	Type
15	GSP242906	Mast Frame - w/Pivot Insert (on Top End) - Aspire Aura Floor Lifter		Spare Part
16	GSP242826	Mounting Hardware only - Hinge Assembly - Aspire Nova Standing / Aura Floor Lifter		Spare Part
17	GSP242910	Boom Frame - w/Hinge inserts on both ends - Aspire Aura Floor Lifter		Spare Part

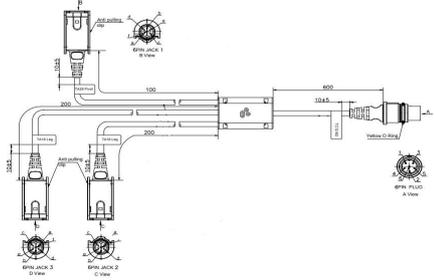
2 Mast, Boom, Sling Bar and Handle				
18	GSP242912	Sling Bar - 2 Point - Aspire Aura Floor Lifter		Spare Part
18	LSS361000	Sling Bar - 4 Point - Aspire Aura Floor Lifter		Spare Part
20	GSP242913	Adapter Pin - w/Hardware - for Sling Bar - Aspire Aura Floor Lifter		Spare Part
21	GSP242914	Handle Bar - w/Mounting Hardware - Aspire Aura Floor Lifter		Spare Part

3 Electronics and Hardware



Ref	Item Number	Item Description	Image	Type
32	GSP242916	Lifting Actuator - Aspire Floor Lifter		Spare Part
32A	GSP242854	Mounting Hardware - Top and Bottom - w/Cap - Lifting Actuator - Aspire Nova Standing / Aura Floor Lifter		Spare Part
32B	GSP242856	Plastic Cap - (Bolt Cover) - Aspire Nova Standing / Aura Floor Lifter		Spare Part

3 Electronics and Hardware				
33	GSP242858	Leg Actuator (Only) - Aspire Nova Standing / Aura Floor Lifter		Spare Part
33A	GSP242860	Mounting Hardware - w/Bushing - Leg Actuator - Aspire Nova Standing / Aura Floor Lifter		Spare Part
34	GSP242862	Control Box - Aspire Nova Standing / Aura Floor Lifter		Spare Part
36	MBT401050	Battery - Aspire Nova Standing / Aura Floor Lifter		Spare Part
37	MBT401055	Charging Cradle - w/Transformer Cable - Aspire Nova Standing / Aura Floor Lifter		Spare Part
37A	MBT401060	Power Transformer - w/Cable - for Charging Cradle - Aspire Nova Standing / Aura Floor Lifter		Spare Part

3 Electronics and Hardware				
38	GSP242872	Handset - 4 Button - Aspire Nova Standing / Aura Floor Lifter		Spare Part
38	GSP242873	Handset - 6 Button - for Lifter + Pivot Frame - Aspire Aura Floor Lifter		Spare Part
39	GSP242874	Splitter Cable - 3 Channel - (Wiring Harness) - Aspire Nova Standing / Aura Floor Lifter		Spare Part