

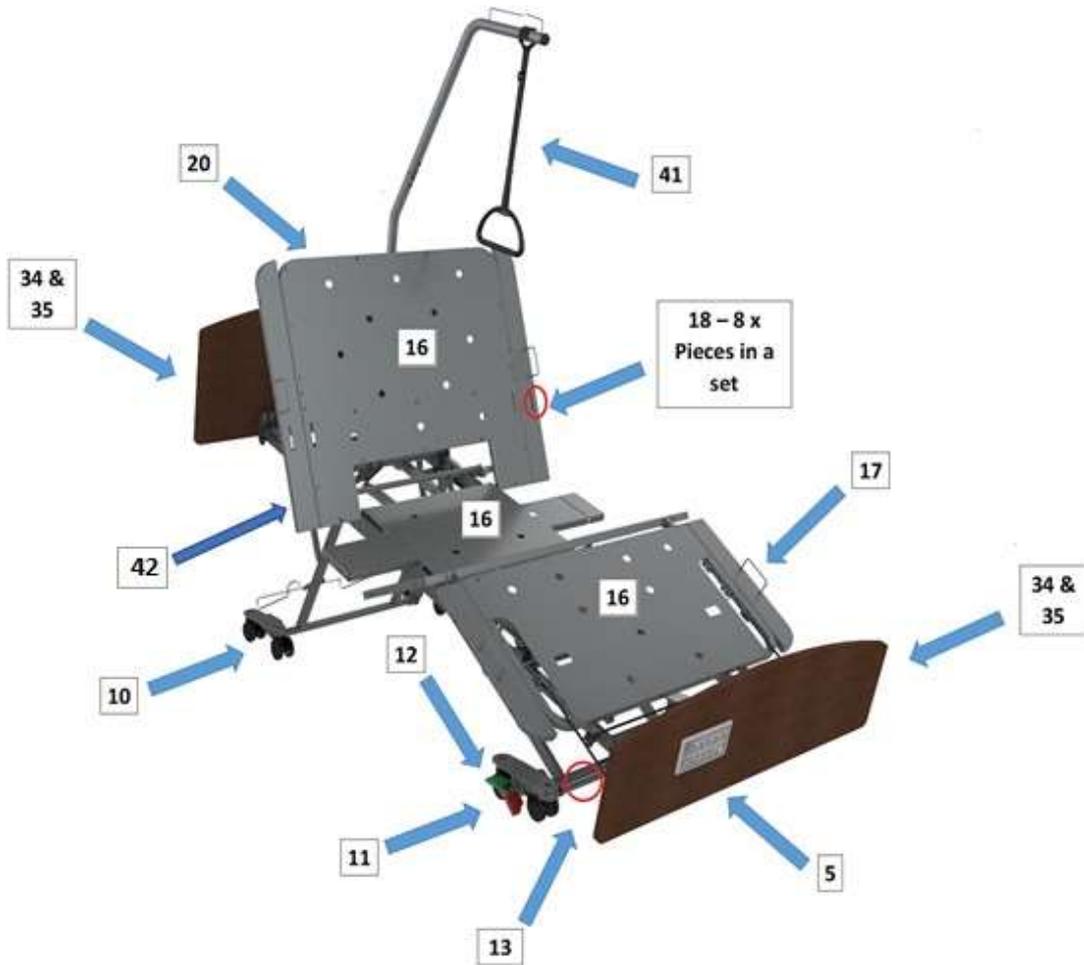
Aidacare FL250 Floorline Bed – Spare Parts Manual



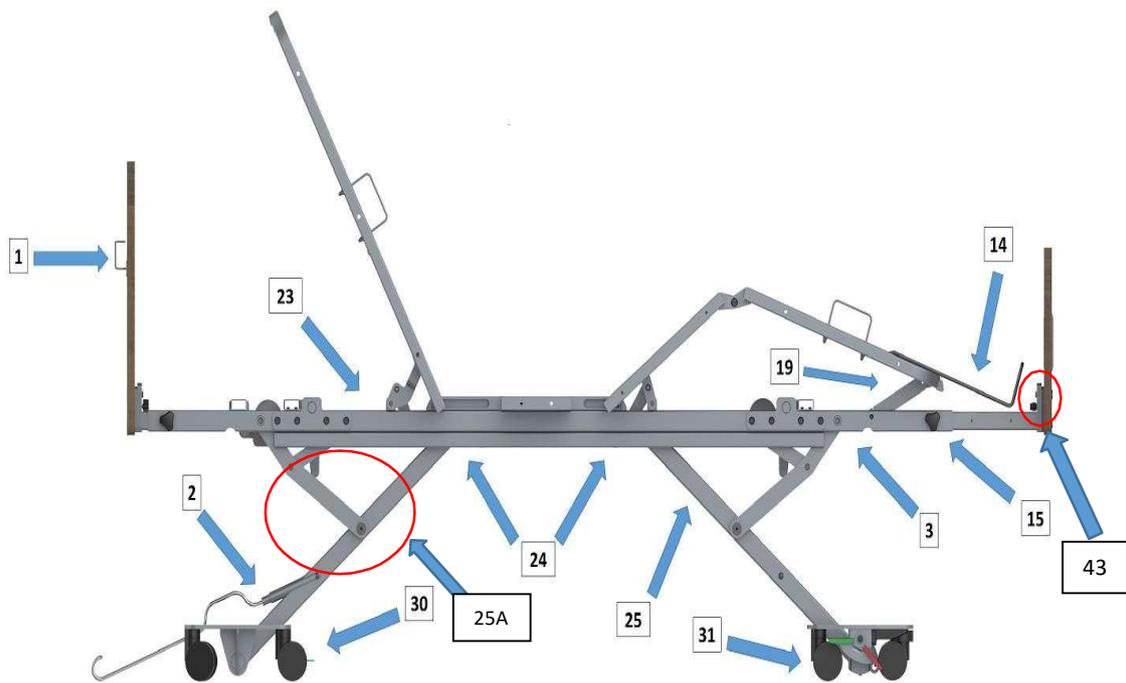
BEB046100	Aidacare FL250 Floorline Bed - Single
BEB046150	Aidacare FL250 Floorline Bed – King Single

**This manual is for internal use only.**

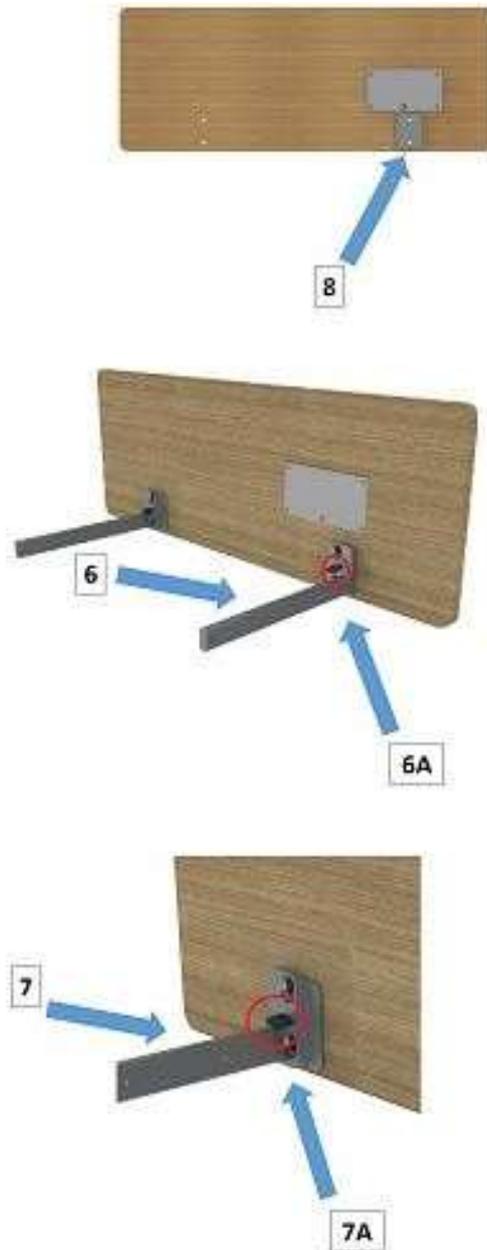
**SPARE PARTS DIAGRAM 2**

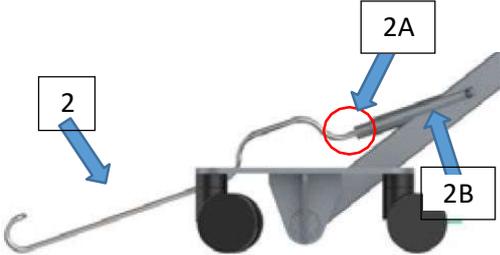
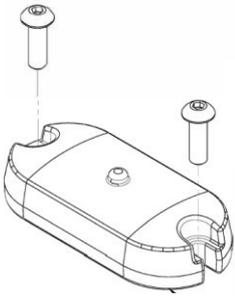


**SPARE PARTS DIAGRAM 3**

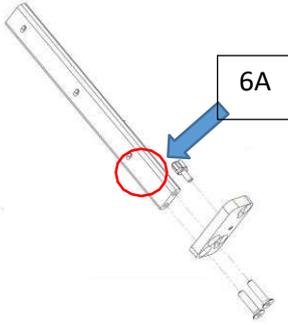


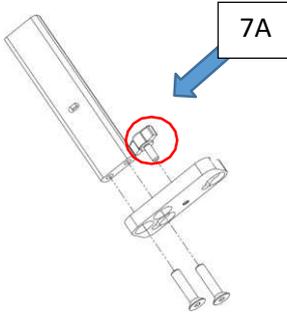
**SPARE PARTS DIAGRAM 4**



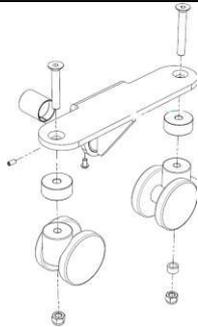
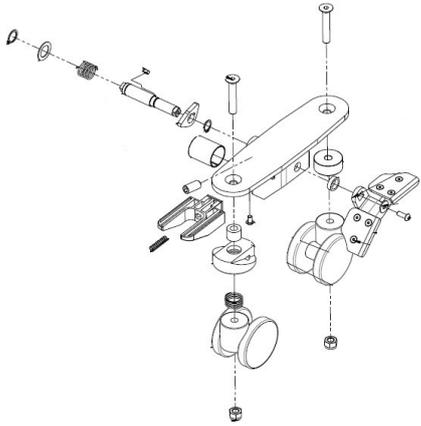
Ref #	Product Code	Product Description	Type
1	BEA030030	Hand Control Holder Bracket	Spare Part
			
2	BEB046560	Wall Buffer Bar Only – STANDARD	Spare Part
2	BEB046561	Wall Buffer Bar Only – SHORT	Spare Part
2A	BEB046555	Wall Buffer Retainer Springs Only (NOT WALL BUFFER) – PAIR	Spare Part
2B	BEB046562	Mounting Barrel – for Wall Buffer	Spare Part
			
3	BEB046575	Under Bed Light & Hardware	Spare Part
			

Ref #	Product Code	Product Description	Type
4	BEB046600	Hand Control – Old Style - Safety Key	Spare Part
	BEB046599	Hand Control – New Style – No Safety Key	Spare Part
	BEB046595	Hand Control – With CPR function – Acute	Spare Part
	BEB046602	Hand Control – BLUE – MIN HEIGHT LIMITING – No Safety Key	Spare Part
	BEB046605	MO Handset – Safety Egress Height Adjustment	Spare Part
	GSP391550	Safety Key & Chain for Hand Control <b>(NOT ON DIAGRAM)</b>	Spare Part
	GSP391534	Y-Cable for Handset	Spare Part
	GSP391537	Cable Locking Latch	Spare Part
	GSP391538	Plastic Hook for handset	Spare Part
			
5	BEB046610	Attendant Control Panel	Spare Part
	BEB046615	Attendant Control Panel – With CPR and Hook – Acute	Spare Part
			
5A	BEB046490	Bed end set – Moulded ABS – For Acute	Spare part
5B	BEA029816	Mounting Bracket & Hardware Bed Ends- For Acute	Spare part
			

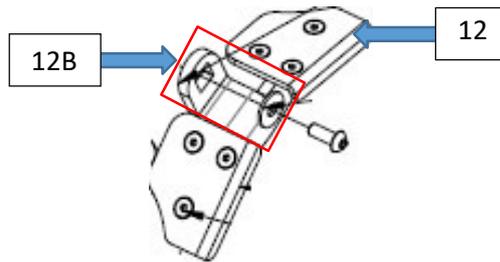
6	GSP391240	Bed End Extension Insert – Foot End – Foot Left	Spare Part
	GSP391250	Bed End Extension Insert – Foot End – Foot Right	Spare Part
6A	GSP391590	Wing Nut for Fastening Bed Ends	Spare Part
			

Ref #	Product Code	Product Description	Type
7	GSP391260	Bed End Extension Insert – Head End – Head Left	Spare Part
	GSP391270	Bed End Extension Insert – Head End – Head Right	Spare Part
7A	GSP391590	Wing Nut for Fastening Bed Ends	Spare Part
			
8	BEB046620	Bed End Mounting Plats & Hardware – Complete Set – Set of 4	Spare Part
	GSP391280	Bed End Mounting Plates & Hardware – Foot End	Spare Part
	GSP391290	Bed End Mounting Plates & Hardware – Head End	Spare Part
8A	GSP391275	Head Board Hex Bolt – Silver Colour	Spare Part
8B	GSP391620	Flat Bed End Nutsert – Silver Colour	Spare Part
			

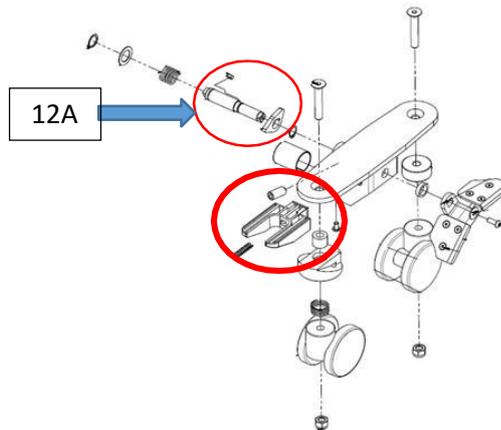
9	BEB046590	Battery Backup <b>PRIOR SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Accessory
	BEB046592	Battery Backup <b>AFTER SN* 2204-24495 (Single) / 0000-0000(KS)</b>	Accessory
			

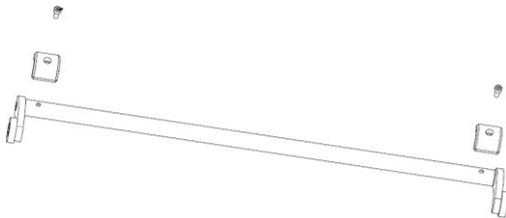
Ref #	Product Code	Product Description	Type
10	GSP391330	Castor Mounting Device & Hardware – Left – Head End	Spare Part
	GSP391335	Castor Mounting Device & Hardware (w Latch) – Left – Head End	Spare Part
	GSP391340	Castor Mounting Device & Hardware – Right – Head End	Spare Part
	GSP391345	Castor Mounting Device & Hardware (w Latch) – Right – Head End	Spare Part
			
11	GSP391350	Castor Mounting Device & Hardware – Left – Foot End	Spare Part
	GSP391360	Castor Mounting Device & Hardware – Right – Foot End	Spare Part
			

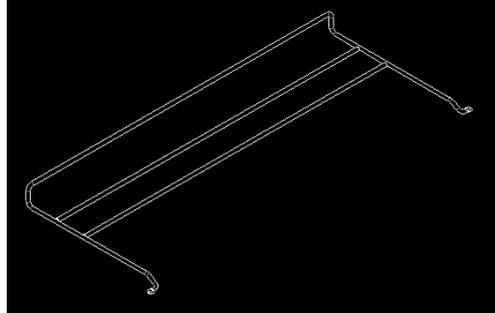
12	GSP391300	Brake Lever & Hardware – Left – Red / Green	Spare Part
	GSP391310	Brake Lever & Hardware – Right – Green / Red	Spare Part
12A	GSP391325	Brake Cam	Spare Part
12B	GSP391326	Brake Crank	Spare Part



12C	GSP391327	Braking Wedge
-----	-----------	---------------



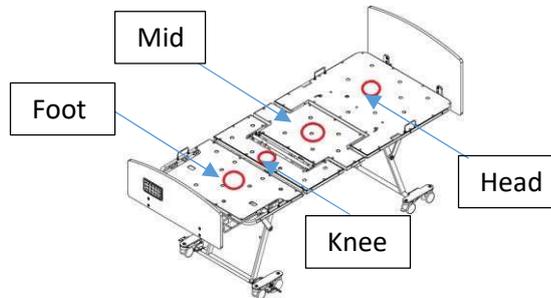
Ref #	Product Code	Product Description	Type
13	GSP391320	Brake Locking Bar & Hardware	Spare Part
			
14	GSP391470	Mattress Holder - for Bed Extension <b>Note: use only when using with a SINGLE size bed and King Single Conversion Kit</b>	Spare Part
	GSP391475	Mattress Holder - for Bed Extension - SS bed – V2 bed	Spare Part
	GSP391477	Mattress Holder - for Bed Extension - KS bed – V2 bed	Spare Part

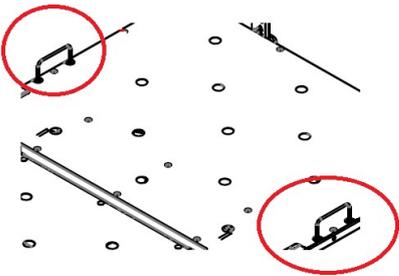
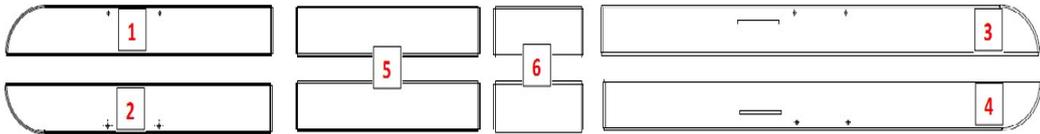


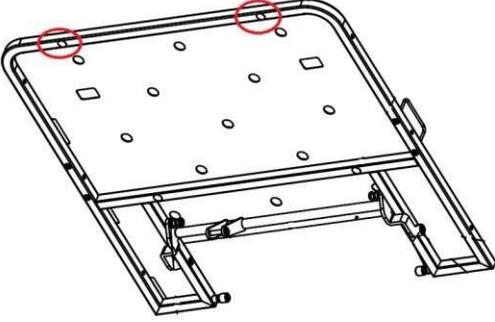
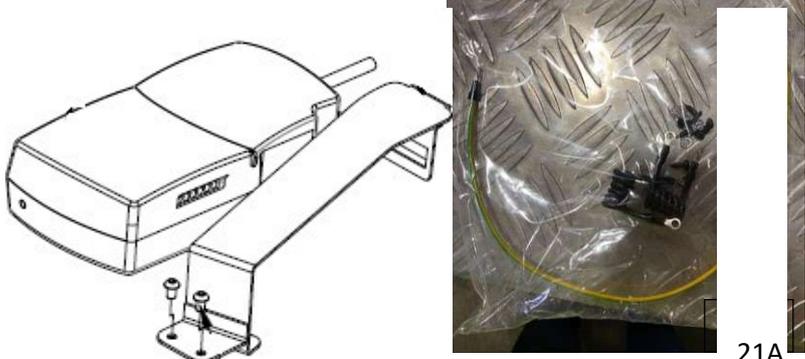
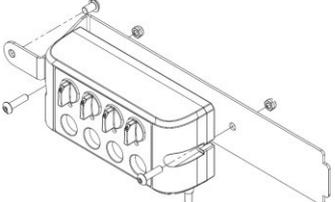
15	GSP391390	Hand Wheel for Fastening Bed Extension	Spare Part
----	-----------	--	------------



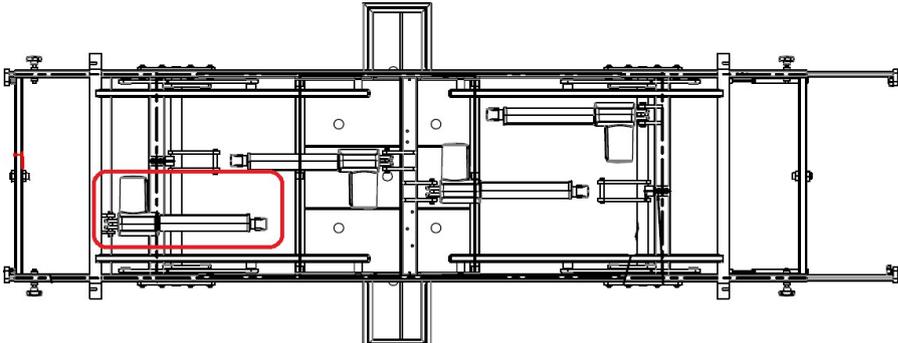
16	GSP391480	Mattress Platform – Foot Section <b>PRIOR SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Spare Part
	GSP391485	Mattress Platform – Foot Section – Single <b>AFTER SN* 2204-24495 (Single)</b>	Spare Part
	GSP391487	Mattress Platform – Foot Section – King Single <b>AFTER SN* 0000-0000 (KS)</b>	Spare Part
	GSP391490	Mattress Platform – Head Section <b>PRIOR SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Spare Part
	GSP391495	Mattress Platform – Head Section – Single <b>AFTER SN* 2204-24495 (Single)</b>	Spare Part
	GSP391497	Mattress Platform – Head Section – King Single <b>AFTER SN* 0000-0000 (KS)</b>	Spare Part
	GSP391500	Mattress Platform – Knee Section <b>PRIOR SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Spare Part
	GSP391505	Mattress Platform – Knee Section – Single <b>AFTER SN* 2204-24495 (Single)</b>	Spare Part
	GSP391507	Mattress Platform – Knee Section – King Single <b>AFTER SN* 0000-0000 (KS)</b>	Spare Part
	GSP391510	Mattress Platform – Mid Section	Spare Part

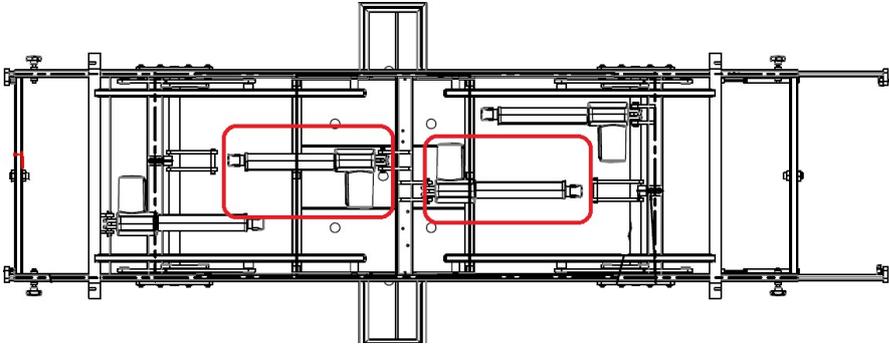
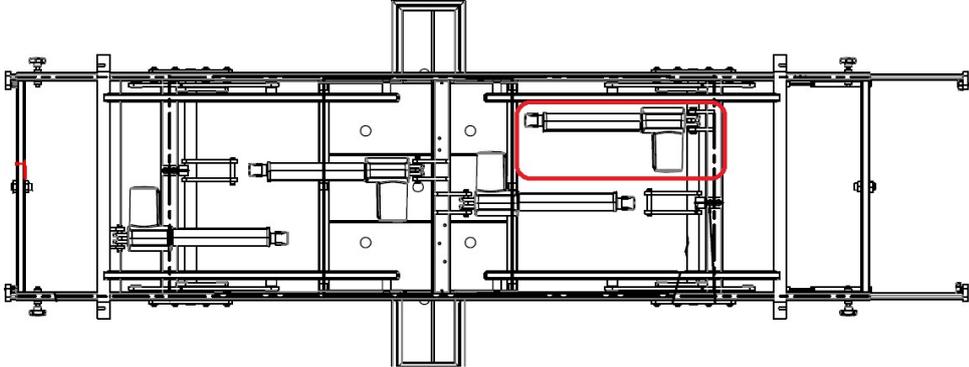
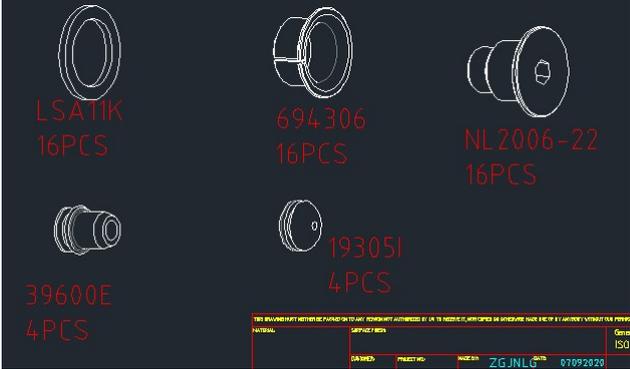


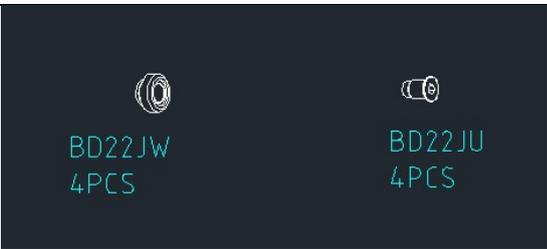
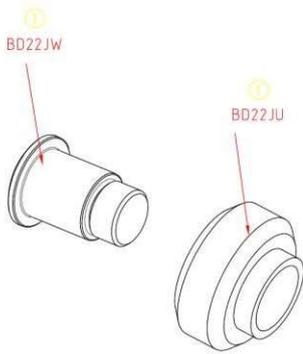
Ref #	Product Code	Product Description	Type
17	GSP391525	Mattress Retainers – Set of 2	Spare Part
			
18	BEB046500	King Single Extension Kit – Set of 8 x Pieces	Accessory
	GSP391410	King Single Extension Piece – Left – Foot - <b>(1)</b>	Spare Part
	GSP391420	King Single Extension Piece – Right – Foot - <b>(2)</b>	Spare Part
	GSP391430	King Single Extension Piece – Left – Head - <b>(3)</b>	Spare Part
	GSP391440	King Single Extension Piece – Right – Head - <b>(4)</b>	Spare Part
	GSP391450	King Single Extension Piece – Universal – Knee - <b>(5)</b>	Spare Part
	GSP391460	King Single Extension Piece – Universal – Mid - <b>(6)</b>	Spare Part
			
19	GSP391540	Ratchets & Hardware – Manual Leg Raise	Spare Part
			

Ref #	Product Code	Product Description	Type
20	GSP391380	Adhesive Foam Dampeners – Set of 4	Spare Part
			
21	GSP391370	Control Box & Hardware <b>PRIOR SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Spare Part
	GSP391372	Control Box & Hardware <b>AFTER SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	
21A	GSP391660	Earth Cable Kit – Packet contains - 2x standard earth cables and 1x control box earth cable.	Spare Part
		 <p style="text-align: right;">21A</p>	
22	GSP391400	Junction Box & Hardware <b>PRIOR SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Spare Part
	GSP391402	Junction Box & Hardware <b>AFTER SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Spare Part
			

23	BEB046616	USB Port Upgrade Kit	Spare Part
			

Ref #	Product Code	Product Description	Type
23	GSP391200	Backrest Actuator	Spare Part
	GSP391206	Cable – Backrest Actuator	Spare Part
			
24	GSP391210	Hi-Lo Actuator	Spare Part
	GSP391018	Cable – Hi Lo Actuator – Footend – 1400mm	Spare Part
	GSP391019	Cable – Hi Lo Actuator – Headend – 1100mm	Spare Part

			
25	GSP391220	Leg Rest Actuator	Spare Part
	GSP391016	Cable – Leg Rest Actuator	Spare Part
			
25A	GSP391690	Replacement Head / Foot End Leg Section	Spare Part
	Diagram on Page 3		
25B	GSP391657	Bolt and Spacers Mounting Hardware – Head/Foot End Bed Frame	Spare Part
			
25C	GSP391658	Bolt and Spacers Mounting Hardware – Middle Bed Frame	Spare Part

		 <p>BD22JW 4PCS</p> <p>BD22JU 4PCS</p>	
	GSP391545	Roller – Head & Leg Hi Lo	Spare Part
		 <p>BD22JW</p> <p>BD22JU</p>	
	GSP391547	Roller – Head End Platform Roller	Spare Part
			

Ref #	Product Code	Product Description	Type
26	GSP391230	Actuator Locking Pin & C-Clip ( <b>NOT ON DIAGRAM</b> ) – All Actuators	Spare Part
			

27	GSP391530	Mains Power Cable -Curly - 90 Degrees Rotate Plug	Spare Part
27	GSP391529	Mains Power Cable – Curly – 180 Degrees Rotate Plug	Spare Part
			
27	GSP391532	Power Cable Extension - 300mm - Aidacare FL250 Mental Health	Spare part
			
27	GSP391533	Bracket Support - w/Hardware - for Mains Power Cable - Aidacare FL250 Bed - Mental Health	Spare Part
			

Ref #	Product Code	Product Description	Type
29	GSP391570	Underbed Cable Fastener – 10mm	Spare Part
	GSP391580	Underbed Cable Fastener – 8mm	
			

31	BEB046640	Swivel Castor – Head & Foot End <b>(2) – 8 Units per Bed</b>	Spare Part
32	BEB046650	Total Lock Castor – Head & Foot End – Option – <b>(3)</b>	Accessory
32A	BEB046660	Directional Locking Latch Assembly – <b>(4) – 2 Units per Bed</b>	Spare Part



33	BEA030050	Bed Mover/Transporter – Set of 2 <b>(Exploded Diagram last page)</b>	Accessory
----	-----------	--	-----------



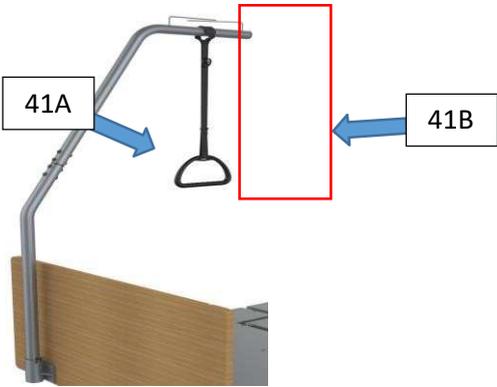
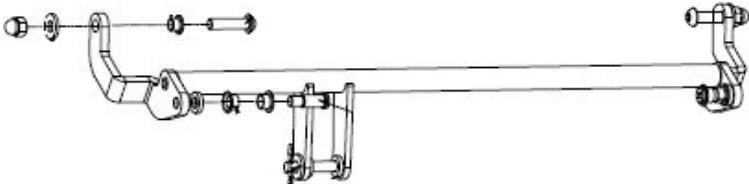
Ref #	Product Code	Product Description	Type
34	BEB046200	Bed End Set – TAHOE – Sublime Teak - Single	Accessory
	BEB046220	Bed End Set – TAHOE – Sublime Teak – King Single	Accessory

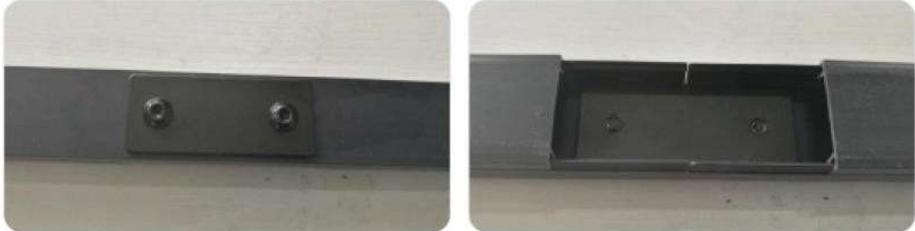


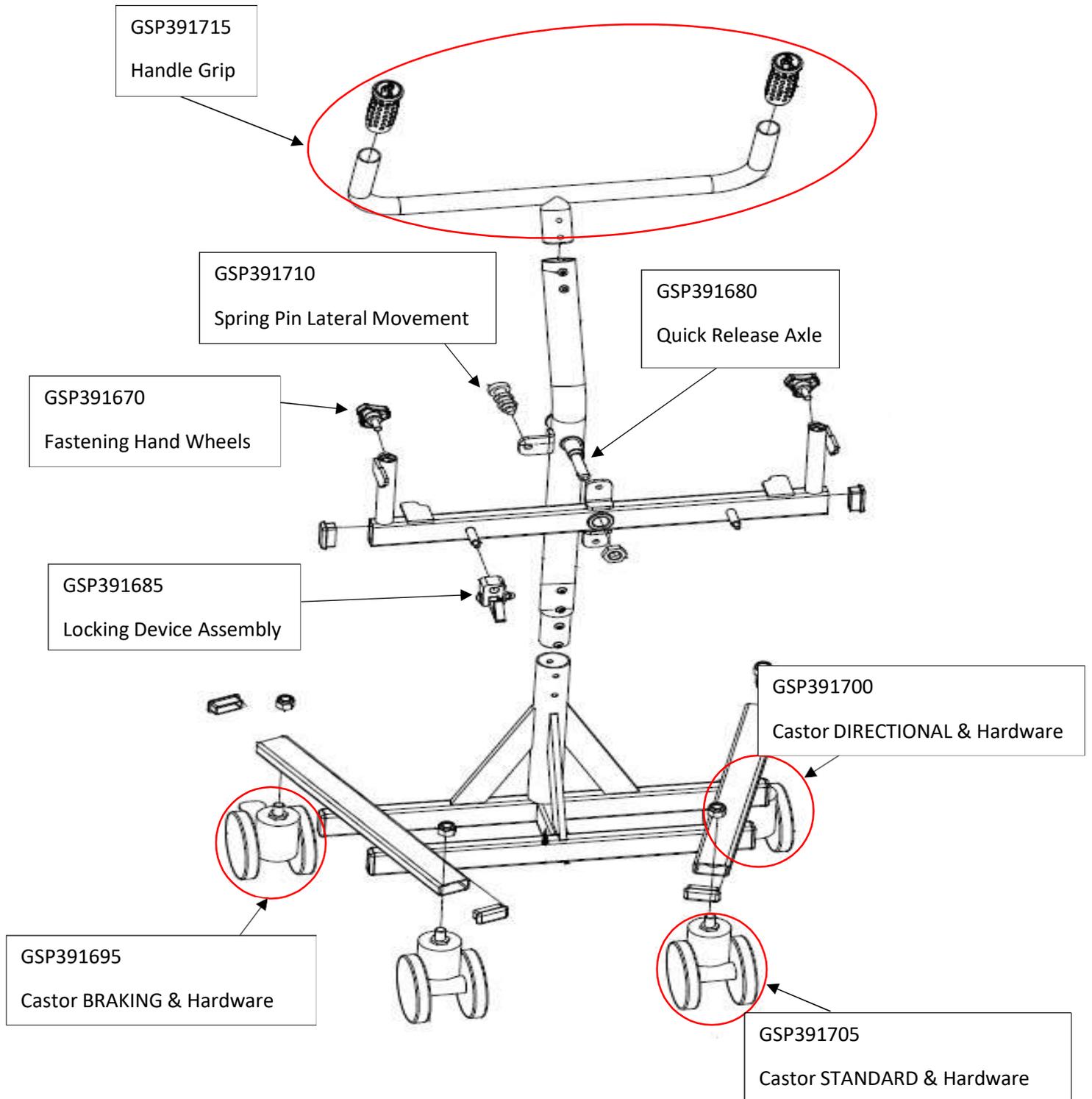
35	BEB046210	Bed End Set – VAIL – Milano Walnut - Single	Accessory
	BEB046230	Bed End Set – VAIL – Milano Walnut – King Single	Accessory
			
36	BEB046240	Bed Side Set – TAHOE – Sublime Teak – Hardware Separate ( <b>NOT ON DIAGRAM</b> )	Accessory
			

Ref #	Product Code	Product Description	Type
37	BEB046250	Bed Side Set – VAIL – Milano Walnut – Hardware Separate ( <b>NOT ON DIAGRAM</b> )	Accessory
			
37A	GSP391630	Bed Side Set Mounting Hardware – Single – Set of 16	Spare Part
	GSP391641	Bed Side Set Mounting Hardware – King Single - Set of 16	Spare Part
38	BEB046520	Folding Grab/Assist Rail - EACH ( <b>NOT ON DIAGRAM</b> )	Accessory

			
39	BEB046525	Folding Side Rails – Pair	Accessory
	GSP391600	Folding Side Rails – Fold Down – Fit Head Left/Foot Right ( <b>NOT ON DIAGRAM</b> )	Spare Part
	GSP391610	Folding Side Rails – Fold Down – Fit Head Right/Foot Right ( <b>NOT ON DIAGRAM</b> )	Spare Part
			
39	GSP391615	Folding Side Rails – Mechanism – Including Pins	Spare Part
39	GSP391616	Black Lever – for Side Rail – Aidacare FL250 Bed	Spare Part
		 	
39A	GSP391531	Power Cable Holder / Retainer Bracket	Spare Part
			

Ref #	Product Code	Product Description	Type
40	BEB046530	IV Pole ( <b>NOT ON DIAGRAM</b> )	Accessory
			
41	BEB046540	Self Help Pole	Accessory
41A	GSP391720	Self Help Pole – POLE Only	Spare Part
41B	GSP391725	Handle bar – Self Help Pole	Spare Part
			
42	GSP391650	Axis & Assembly Hardware for Backrest Actuator <b>PRIOR SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Spare Part
	GSP391652	Axis & Assembly Hardware for Backrest Actuator <b>AFTER SN* 2204-24495 (Single) / 0000-0000 (KS)</b>	Spare Part
	GSP391655	Axis & Assembly Hardware for Hi/Lo Actuator Bar	Spare Part
			

43	GSP391675	Axle Receiver – Bed Foot end - For Bed Mover	Spare Part
			
44	GSP391665	Rubber Frame Insert	Spare Part
			
45	BEB046547	Cable Conduit & Mounting Hardware	Spare Part
			
46	GSP391730	Hose Harness - Aidacare FL250 - for Air Mattress Hoses	
			



BEA030050	Bed Mover / Transporter – Aidacare FL250 Bed
-----------	--