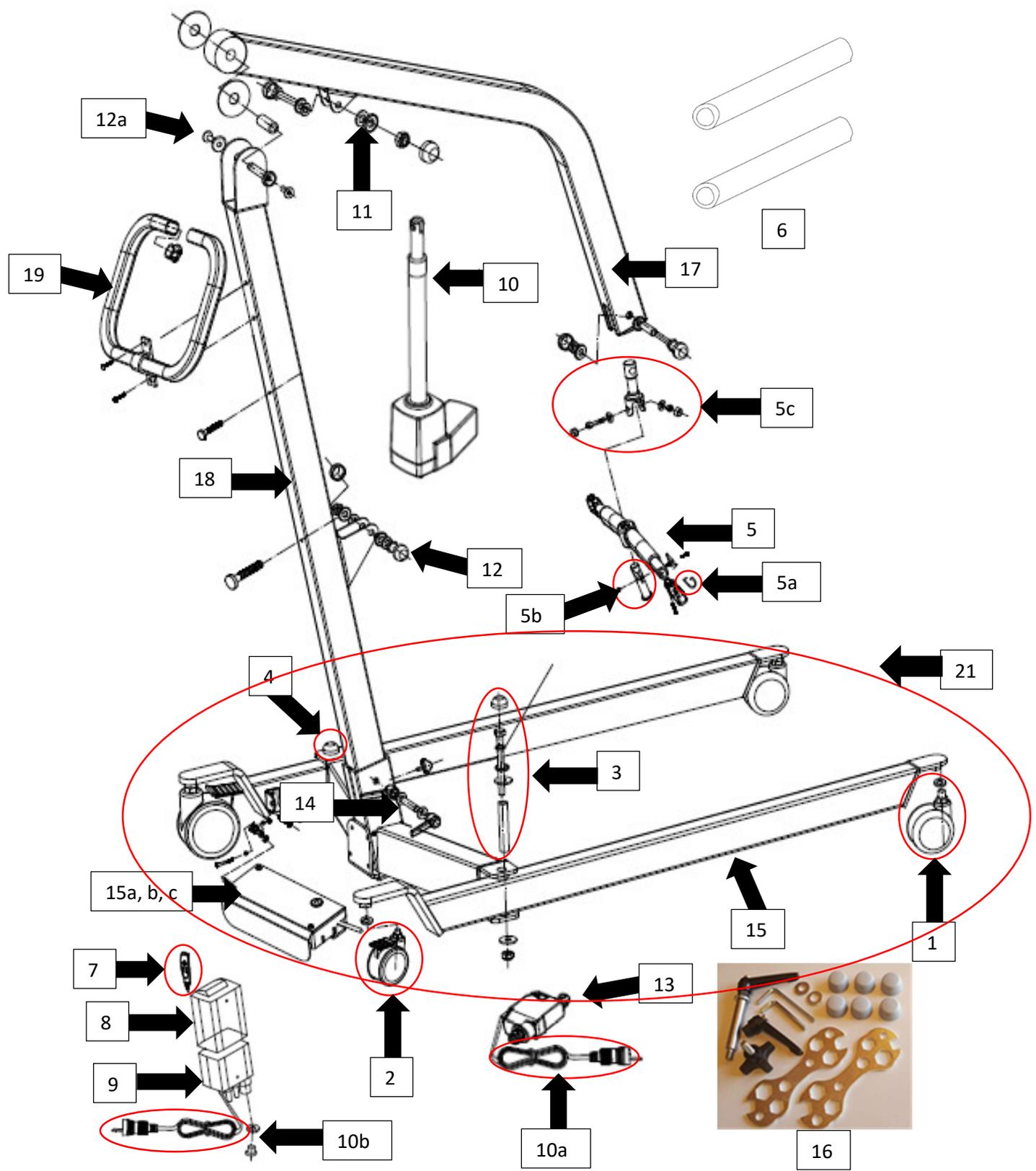


# Aidacare Aspire 200 Patient Lifter (LSS390620)

## Spare Parts Manual

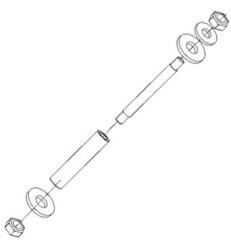
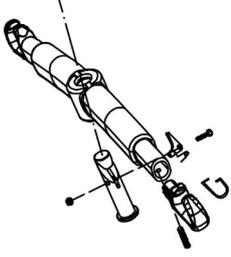
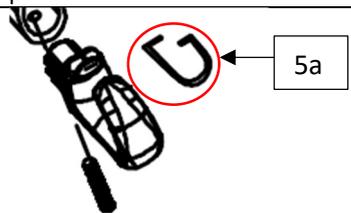


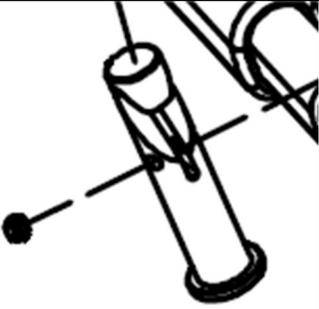
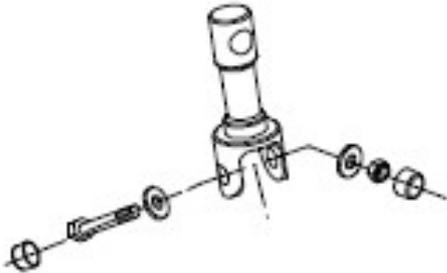
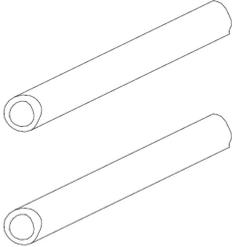
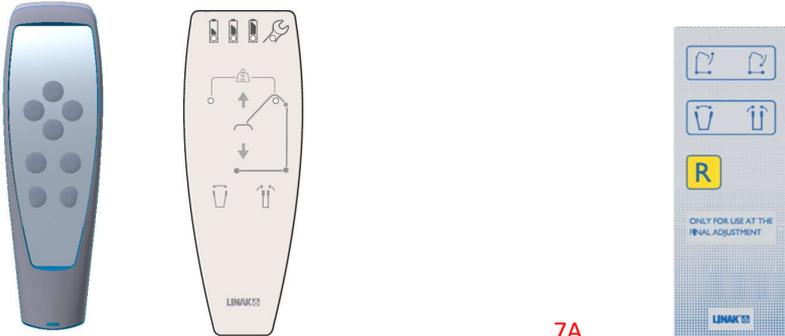
**Aidacare Aspire 200 Patient Lifter – Exploded Parts Diagram**

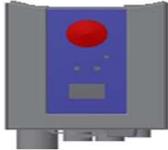
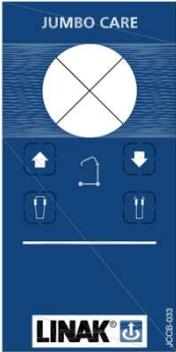


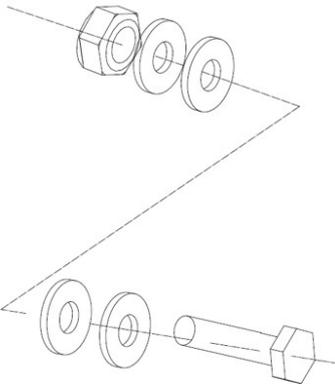
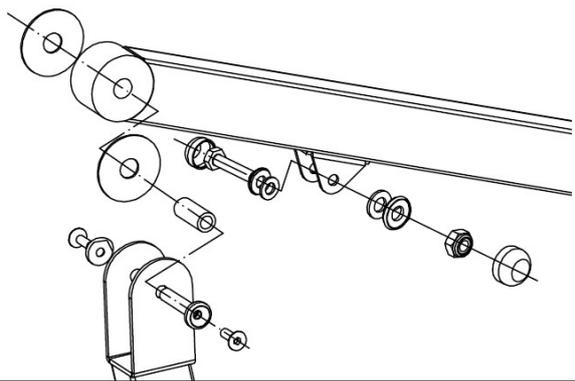
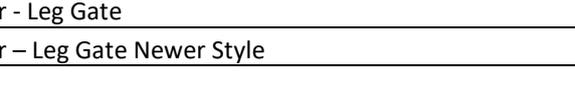
Aidacare Product Code	Aidacare Product Description
LSS390620	Patient Lifter - Aidacare Aspire 200 - Steel - Electric Leg - 200kg

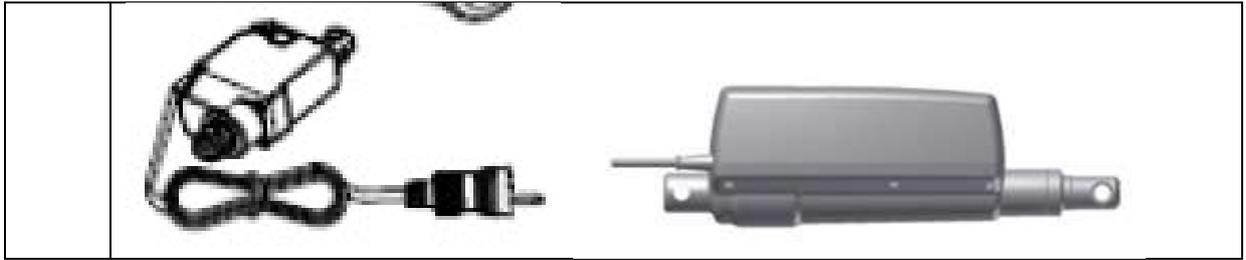
**Spare Parts List**

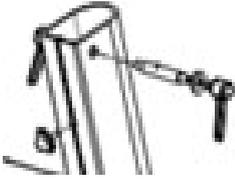
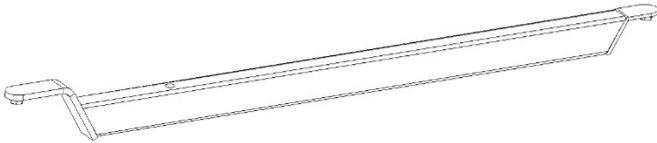
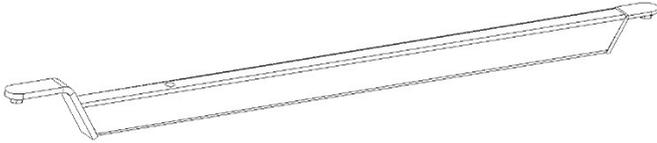
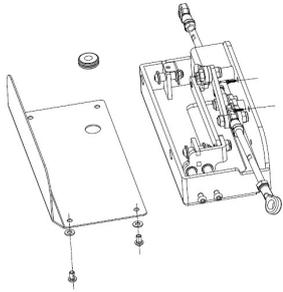
Ref #	Item Number	Item Description	Type
1, 2	Various	Castor Assembly - Tente	Spare Part
<p>PLEASE REFER TO FILE NAMED 'TENTE CASTOR ASSEMBLY' ON INTRANET FOR CORRECT PART NUMBERS AND CONSUMABLES TO USE FOR CASTORS</p>			
3	GSP390931	Leg Hinge Bolt Assembly	Spare Part
			
4	GSP390911	Plastic Cap - Bolt Cover	Spare Part
			
5	GSP390909	Sling/Spreader Bar	Spare Part
			
5a	GSP390976	Sling Bar Retainer Clip	Spare Part
			

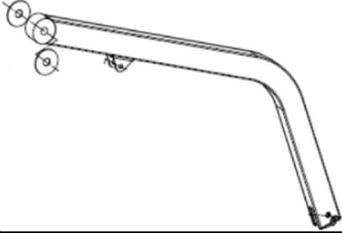
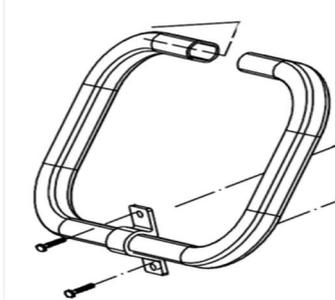
5b	GSP390968	Adaptor Pin	Spare Part
			
5c	GSP390995	Link Pin Spreader Bar to Mast Assembly	Spare Part
			
6	GSP390938	Handgrip Padding – PAIR	Accessory
			
7	GSP390691	4 Button Handset	Spare Part
7A	GSP390692	Programming Handset - for Aspire Lifters	Spare Part
	 <p>7: <span style="margin-left: 300px;">7A</span></p>		
8	MBT401200	Battery with Port	Spare Part
8	MBT401190	Battery without Port	Spare Part

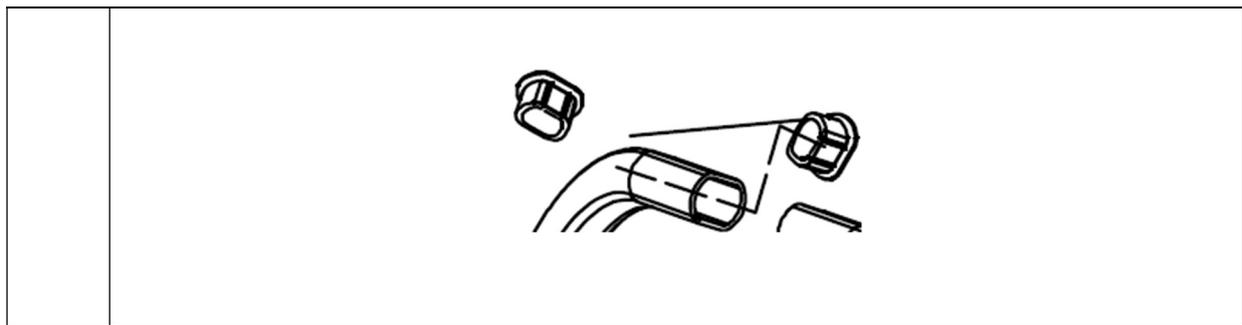
			
8a	MBT401220	Battery Charger – Plug Pack	Spare Part
8b	MBT401210	Battery Charger – Wall Mount	Spare Part
8c	MBT401012	Battery Charger - Linak - For Lifters - Including Mounting Brackets & Cable	Spare Part
9	GSP390694	Control Box	Spare Part
		 	
9a	MSP500906	Bracket – Short - for Control Box & Battery	Spare Part
10a	GSP390680C	Cable Straight – Linak – for Leg Actuators	Spare Part
			
10b	GSP394172	Cable Curly – Linak – for Lifter Actuators	Spare Part
			

10	GSP390686	Actuator - Lift	Spare Part
			
11	GSP390923	Actuator Bolt Assembly – Upper	Spare Part
12	GSP390928	Actuator Bolt Assembly – Lower	Spare Part
			
12a	GSP390985	Hinge Assembly Mounting Hardware	Spare Part
			
13	GSP390680	Actuator - Leg Gate	Spare Part
13a	GSP390679	Actuator – Leg Gate Newer Style	
			

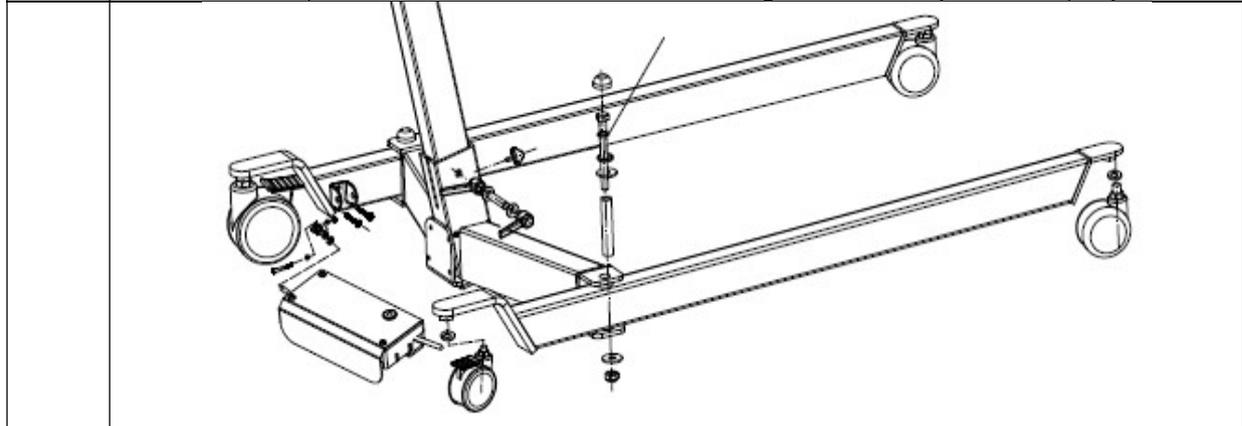


14	GSP390942	Adjustment Lever – Mast	Spare Part
			
15	GSP390948	Replacement Leg – Left	Spare Part
	GSP390948A	Replacement Leg w Bracket Welded – Left	Spare Part
			
15	GSP390937	Replacement Leg – Right	Spare Part
	GSP390937A	Replacement Leg w Bracket Welded – Right	Spare Part
			
15a	GSP390681	Electric Leg Kit Bracket	Spare Part
			
15b	GSP390675	Internal Links connection for Leg Spreader	Spare Part
			

15c	GSP390671	Leg Spreader Tie Rods newer design	Spare Part
			
16	GSP390963	Tool Assembly Kit	Spare Part
			
17	GSP390882	Boom Frame	Spare Part
			
18	GSP390883	Mast Frame	Spare Part
			
19	GSP390996	Handle Bar	Spare Part
	GSP390855	Handle Bar – Adjusted Angle	Spare Part
			
20	GSP390997	End Caps - Handle Bar	Spare Part



21      GSP390862      Base Frame w Tente Castors and Leg Actuator - Aspire 200      Spare Part



22      GSP390782      Replacement Sticker Pack      Spare Part

**aidacare 200**  
aspire

**SWL 200KG**

**KEEP CLEAR**

**DANGER**  
PINCH POINT

**CAUTION!**  
ONLY USE ASPIRE Lifter with its own base. Be careful when assembling & keep fingers clear of the mast receiver bracket when inserting the mast.

**aidacare**  
HEALTHCARE EQUIPMENT  
1300 133 120 www.aidacare.com.au

**SAFE WORKING LOAD 200KG**

**WARNING!**  
ALWAYS read the instruction manual if you are unfamiliar with how to use this lifter.  
ALWAYS use Aidacare approved slings with this lifter. Incorrect slings may cause serious injury and or equipment damage voiding all warranty. If in doubt contact Aidacare 1300 133 120.  
ALWAYS inspect lifter and sling prior to use.  
DO NOT use lifter on uneven surfaces or in situations where there is cause to believe the lifter could tip over.  
DO NOT USE if damage is suspected to or visible on the lifter or sling.  
IF IN DOUBT - DO NOT USE THIS LIFTER.

**MANUFACTURE AND IMPORT**  
Aidacare Pty Ltd  
100-102  
Bridgeway St  
Northmead, NSW 2150  
Australia

**CONTACT US**  
T: 1300 133 120  
F: 62 96 138 5111  
E: info@aidacare.com.au

**SERVICES**  
It is recommended that you have this product serviced regularly.

**PACKAGE FLOOR CLIPS**

<b>SMALL</b>	<b>XL</b>
Part No. 20014	Part No. 20015
<b>USE</b>	<b>USE</b>
Label 19919	Label 19919