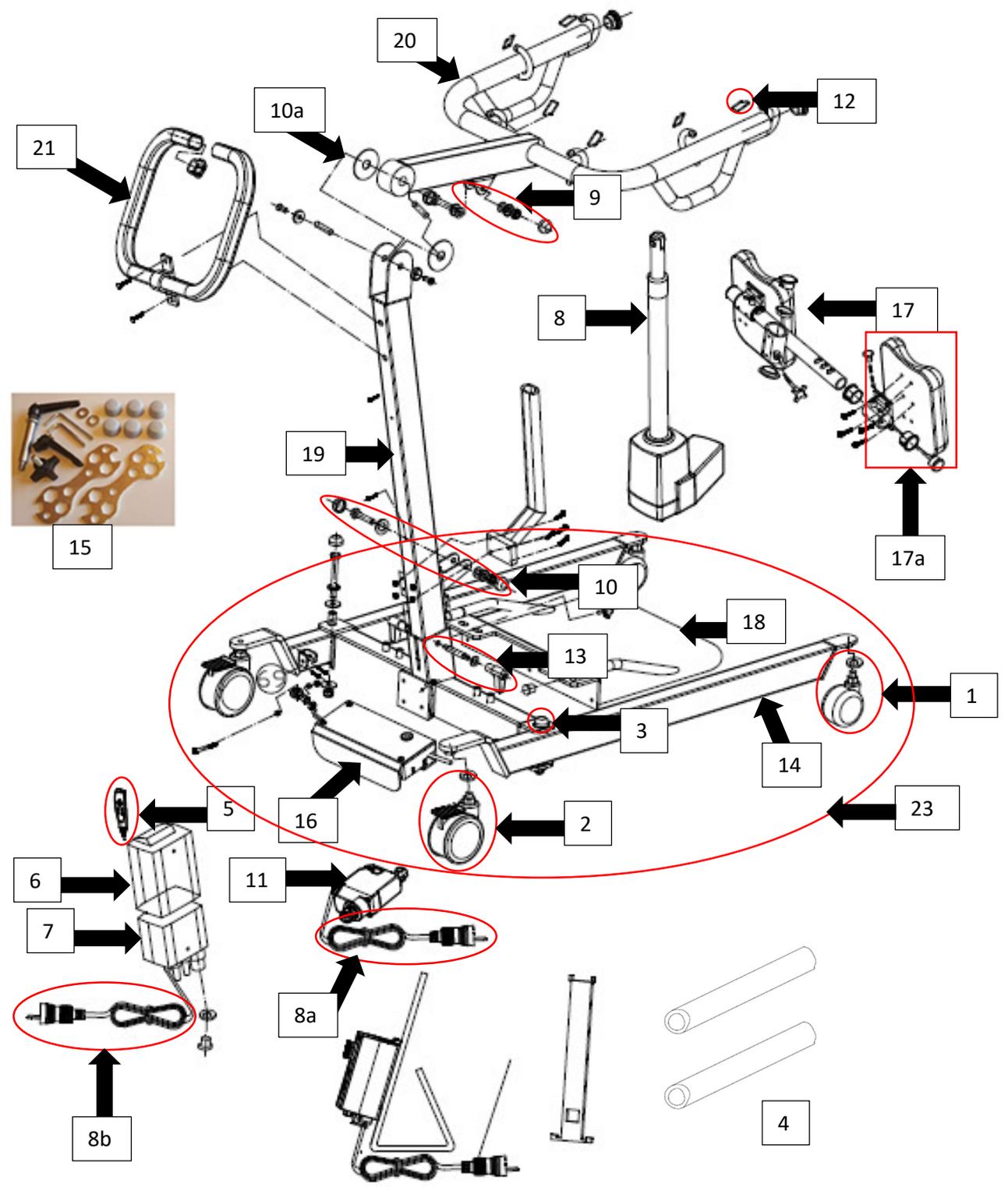


Aidacare Aspire 200S Standing Lifter (LSS390650)
Spare Parts Manual

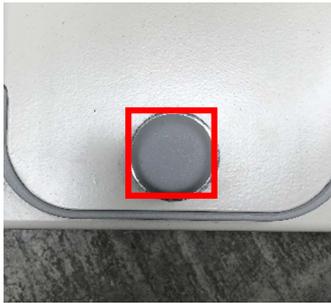
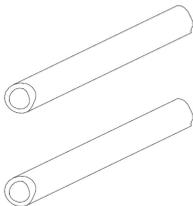
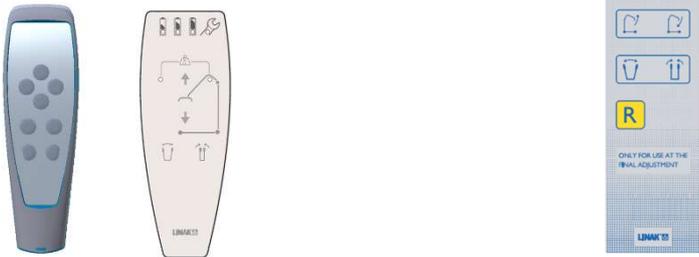


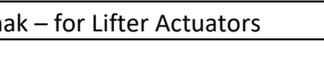
Aidacare Aspire 200S Standing Lifter – Exploded Parts Diagram

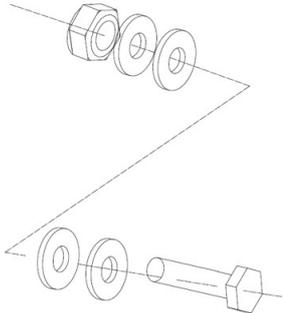
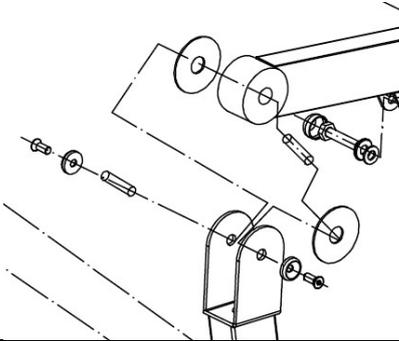


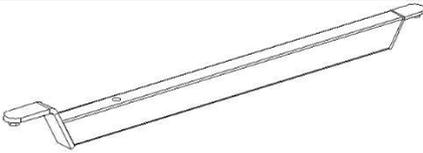
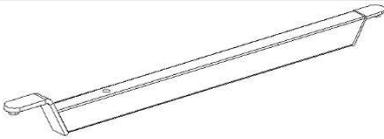
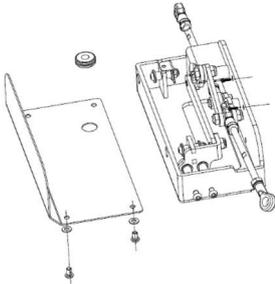
Aidacare Product Code	Aidacare Product Description
LSS390650	Standing Lifter - Aidacare Aspire 200S - Steel - Electric Leg - 200kg

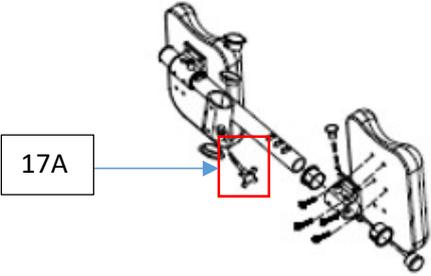
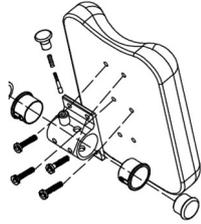
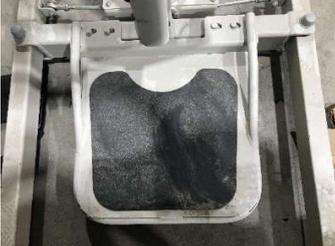
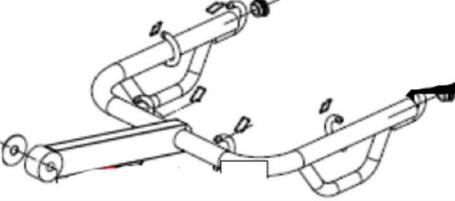
Spare Parts List

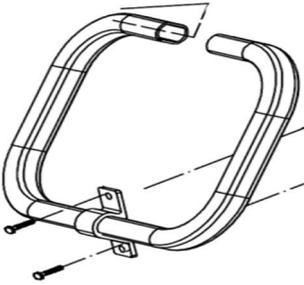
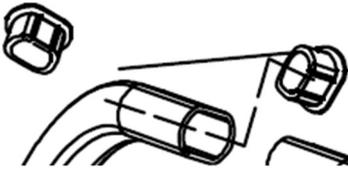
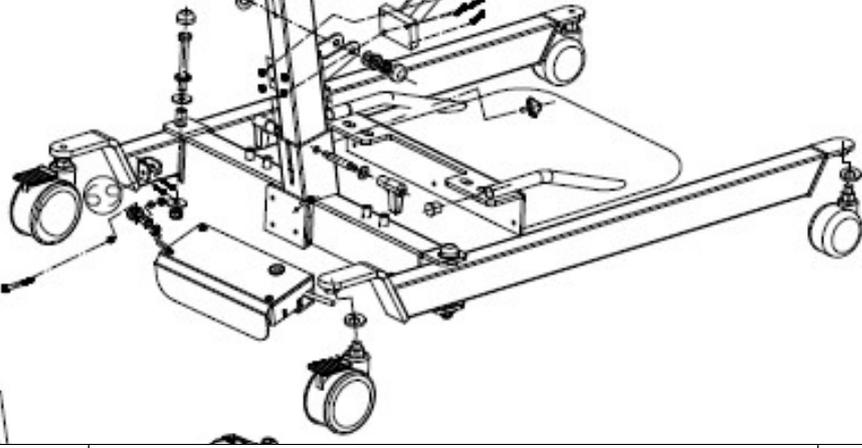
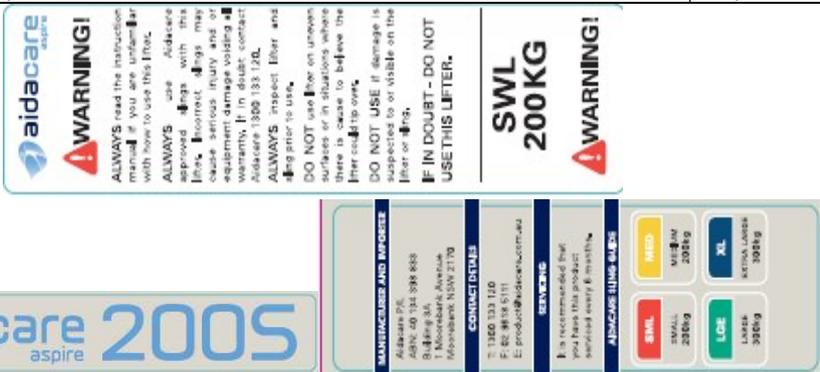
Ref #	Item Number	Item Description	Type
1, 2	VARIOUS	Castor Assembly - Tente	Spare Part
<p>PLEASE REFER TO FILE NAMED 'TENTE CASTOR ASSEMBLY' ON INTRANET FOR CORRECT PART NUMBERS AND CONSUMABLES TO USE FOR CASTORS</p>			
3	GSP390911	Plastic Cap - Bolt Cover	Spare Part
			
4	GSP390938	Handgrip Padding – PAIR	Accessory
			
5	GSP390691	4 Button Handset	Spare Part
5A	GSP390692	Programming Handset - for Aspire Lifters	Spare Part
			
6	MBT401200	Battery with Port	Spare Part
6	MBT401190	Battery without Port	Spare Part
			

6a	MBT401220	Battery Charger Plug Pack	Spare Part
6b	MBT401210	Battery Charger – Wall Mount	Spare Part
6c	MBT401012	Battery Charger - Linak - For Lifters - Including Mounting Brackets & Cable	Spare Part
7	GSP390694	Control Box	Spare Part
			
7a	MSP500906	Bracket – Short - for Control Box, Battery	Spare Part
8	GSP390686	Actuator - Lift	Spare Part
			
8a	GSP390680C	Cable Straight – Linak – for Leg Actuators	Spare Part
			
8b	GSP394172	Cable Curly – Linak – for Lifter Actuators	Spare Part
			

			
9	GSP390923	Actuator Bolt Assembly – Upper	Spare Part
10	GSP390928	Actuator Bolt Assembly – Lower	Spare Part
			
10a	GSP390985	Hinge Assembly Mounting Hardware	Spare Part
			
11	GSP390680	Actuator – Leg Gate – for 200/200S/A200S	Spare Part
			
12	GSP390960	Sling Bar Retainer Clip	Spare Part
			
13	GSP390942	Adjustment Lever – Mast	Spare Part

			
14	GSP390949	Replacement Leg – Left	Spare Part
	GSP390949A	Replacement Leg – Left – Aluminium - New	Spare Part
			
14	GSP390974	Replacement Leg - Right	Spare Part
	GSP390974A	Replacement Leg – Right – Aluminium – New	Spare Part
			
14A	GSP390898	Lifter Spares - Plastic Castor Cover - Front - PAIR - Aspire A205/200S	Spare Part
14A	GSP390899	Lifter Spares - Plastic Castor Cover - Rear - PAIR - Aspire A205 / 200S	Spare Part
15	GSP390966	Tool Assembly Kit	Spare Part
			
16	GSP390681	Electric Leg Kit bracket	Spare Part
			
16a	GSP390675	Internal Links Connection for Leg Spreader	Spare Part
			

17	GSP390982	Knee Pad Assembly	Spare Part
17A	GSP390850	Handwheel Bolt	Spare Part
			
17a	GSP390972	Knee Pad - Left	Spare Part
17a	GSP390984	Knee Pad – Right	Spare Part
			
18	GSP390958	Foot Board Panel	Spare Part
18A	GSP390977	Rubber Liner - w/Peel off Sticker Glue - Aspire A200S/200S Lifters	Spare Part
			
19	GSP390887	Mast Frame	Spare Part
			
20	GSP390886	Boom Frame	Spare Part
			
21	GSP390996	Handle Bar	Spare Part

	GSP390855	Handle Bar – Adjusted Angle	Spare Part												
															
22	GSP390997	End Caps - Handle Bar	Spare Part												
															
23	GSP390861	Base Frame w Tente Castors and Leg Actuator	Spare Part												
															
24	GSP390784	Replacement Sticker Pack	Spare Part												
		 <p>aidacare aspire</p> <p>WARNING! ALWAYS read the instructions manual if you are familiar with how to use this lift. ALWAYS use Aidacare approved lifts with this lift. Incorrect lifts may cause serious injury and are not covered by our warranty. If in doubt contact Aidacare 1300 333 110. ALWAYS inspect lift and legs prior to use. DO NOT use lift on uneven surfaces or in situations where there is a risk of tipping over. DO NOT USE if damage is suspected to or visible on the lift or legs. IF IN DOUBT - DO NOT USE THIS LIFT.</p> <p>SWL 200KG WARNING!</p> <p>aidacare 200S aspire</p> <p>MANUFACTURER AND IMPORTER Aidacare Pty. ARN: 40 134 938 633 84 King St. 1 Moorebank Avenue Moorebank, NSW 2170</p> <p>CONTACT US T: 1300 333 110 F: 62 9638 5111 E: product@aidacare.com.au</p> <p>WARNING It is recommended that you have this product serviced every 6 months.</p> <p>AIDACARE LIFT WEIGHTS</p> <table border="1"> <tr> <td>SWL</td> <td>MED</td> <td>XL</td> </tr> <tr> <td>200kg</td> <td>150kg 200kg</td> <td>ACTUAL LIFT 300kg</td> </tr> <tr> <td>LEG</td> <td></td> <td></td> </tr> <tr> <td>LARGE 300kg</td> <td></td> <td></td> </tr> </table>	SWL	MED	XL	200kg	150kg 200kg	ACTUAL LIFT 300kg	LEG			LARGE 300kg			
SWL	MED	XL													
200kg	150kg 200kg	ACTUAL LIFT 300kg													
LEG															
LARGE 300kg															

