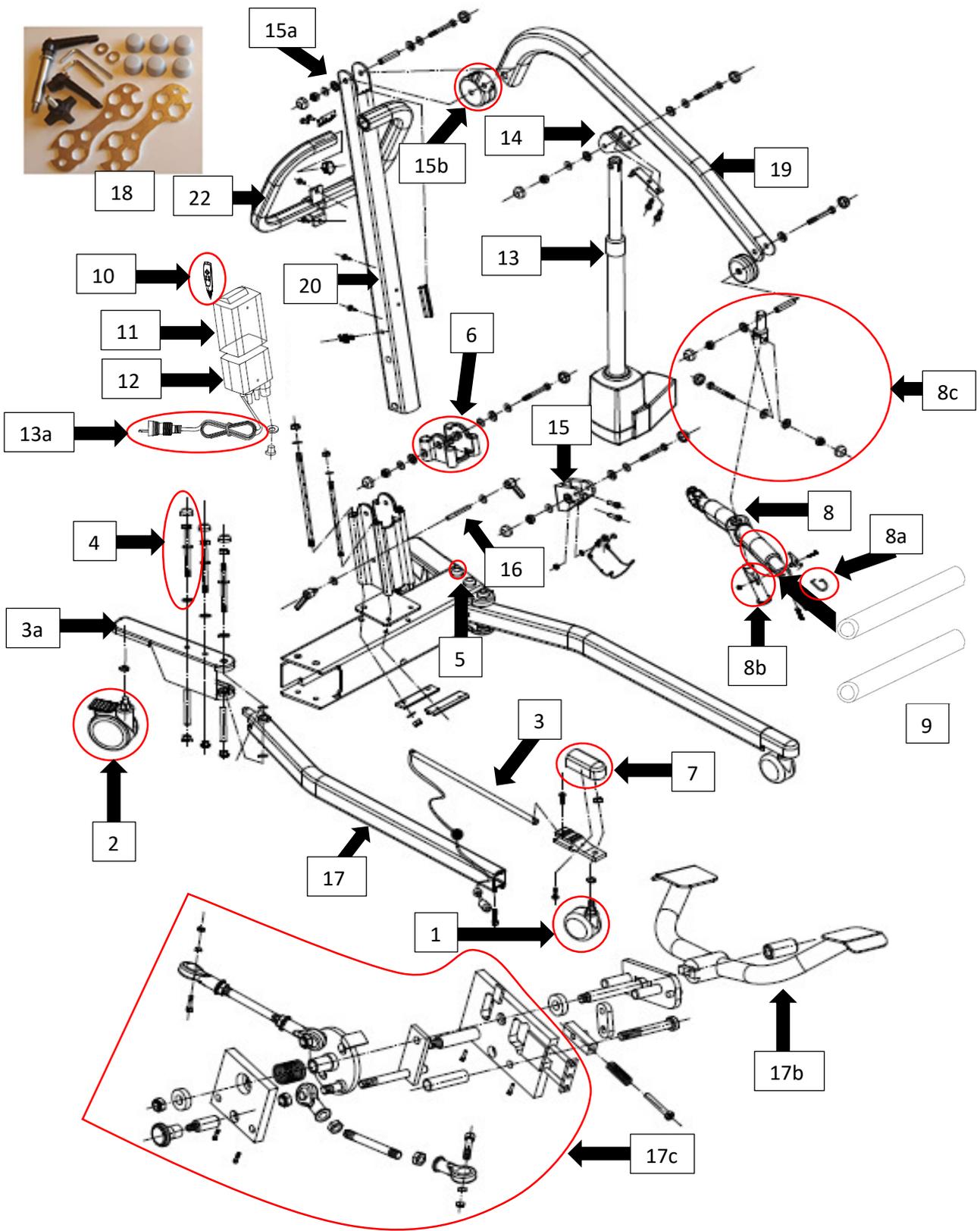


# Aidacare Aspire A150F Patient Lifter (LSS390600) Spare Parts Manual

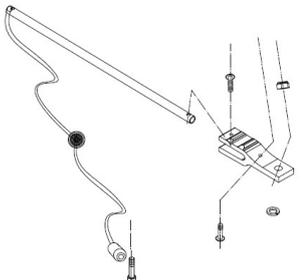
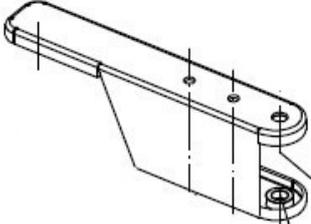
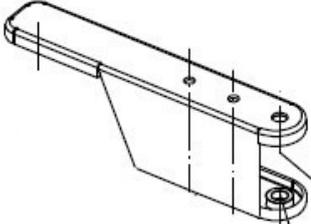


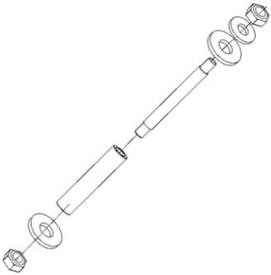
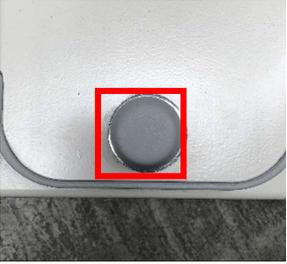
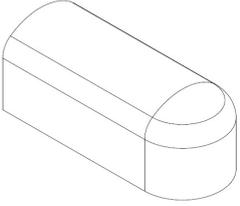
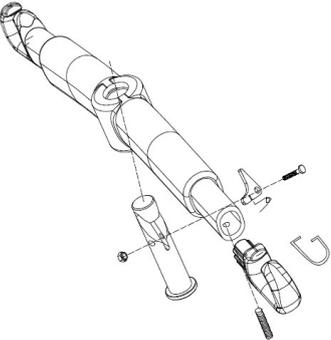
# Aidacare Aspire A150F Patient Lifter – Exploded Parts Diagram

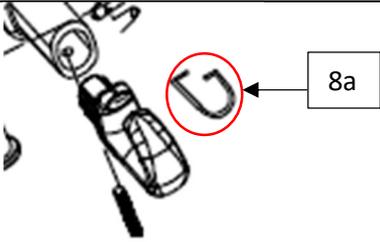
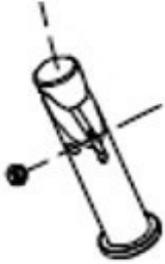
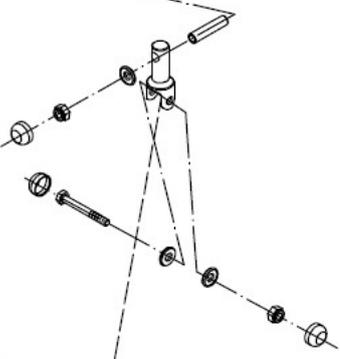
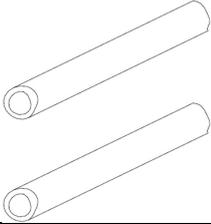


Aidacare Product Code	Aidacare Product Description
LSS390600	Patient Lifter - Aidacare Aspire A150F - Alum - Folding - Manual Leg - 150kg

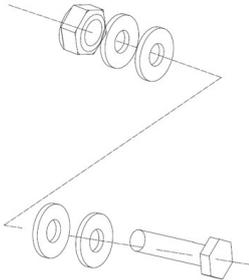
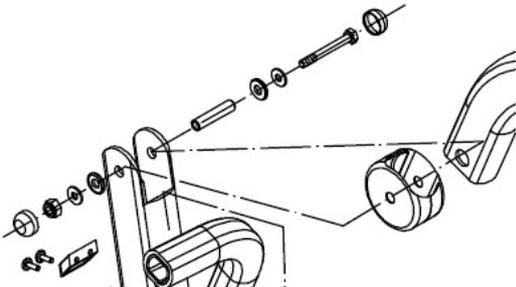
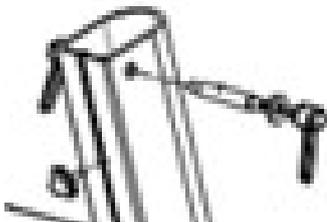
## Spare Parts List

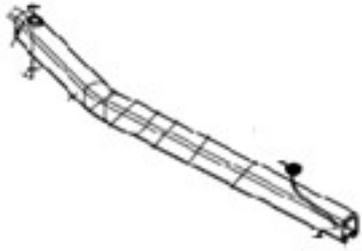
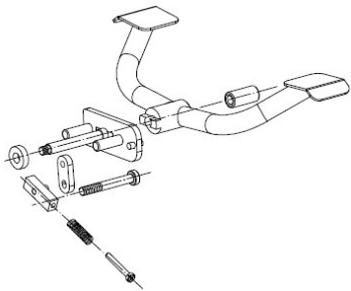
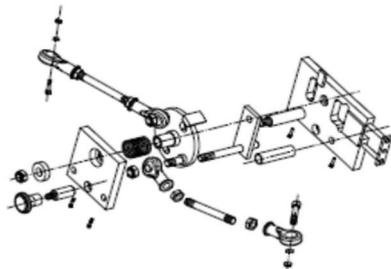
Ref #	Item Number	Item Description	Type
1	VARIOUS	Castor Assembly - Tente	Spare Part
<p>PLEASE REFER TO FILE NAMED 'TENTE CASTOR ASSEMBLY' ON INTRANET FOR CORRECT PART NUMBERS AND CONSUMABLES TO USE FOR CASTORS</p>			
3	GSP390935	Metal Castor Bracket – Including folding locking strap (front) - PAIR	Spare Part
			
3a	GSP390872	Metal Castor Bracket – Rear – Right	Spare Part
			
3a	GSP390874	Metal Castor Bracket – Rear – Left	Spare Part
			

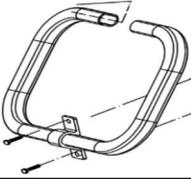
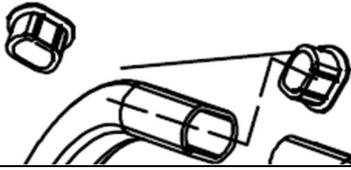
4	GSP390932	Leg Hinge Bolt Assembly	Spare Part
			
5	GSP390840	Plastic Cap - Bolt Cover	Spare Part
			
6	GSP390914	Plastic Shroud for Base of Mast	Spare Part
			
7	GSP390916	Plastic Cover – Front Castor	Spare Part
			
8	GSP390909	Sling/Spreader Bar	Spare Part
			

8a	GSP390976	Sling Bar Retainer Clip	Spare Part
			
8b	GSP390968	Adaptor Pin	Spare Part
			
8c	GSP390994	Link Pin Spreader Bar to Mast Assembly	Spare Part
			
9	GSP390939	Handgrip Padding – PAIR	Accessory
			
10	GSP390690	2 Button Handset	Spare Part
10A	GSP390692	Programming Handset - for Aspire Lifters	
			

11	MBT401200	Battery with Port	Spare Part
11	MBT401190	Battery without Port	Spare Part
			
11a	MBT401220	Battery Charger Plug Pack	Spare Part
11b	MBT401210	Battery Charger – Wall Mount	Spare Part
11c	MBT401012	Battery Charger - Linak - For Lifters - Including Mounting Brackets & Cable	Spare Part
12	GSP390689	Control Box	Spare Part
			
12a	MSP500906	Bracket - Short – for Control Box & Battery	Spare Part
13	GSP390684	Actuator - Lift	Spare Part
			
13a	GSP394172	Cable Curly – Linak – for Leg Actuators	Spare Part

			
14	GSP390922	Actuator Bolt Assembly – Upper	Spare Part
15	GSP390927	Actuator Bolt Assembly - Lower	Spare Part
			
15a	GSP390989	Hinge Assembly Mounting Hardware	Spare Part
			
15b	GSP390981	Plastic Hinge Cover – for A150F	Spare Part
			
16	GSP390943	Adjustment Lever – Mast	Spare Part
			

17	GSP390954	Replacement Leg – Left	Spare Part
17	GSP390955	Replacement Leg – Right	Spare Part
			
17b	GSP390956	Manual Leg Kit Pedal	Spare Part
			
17c	GSP390674	Internal Linkage Assembly	Spare Part
			
18	GSP390962	Tool Assembly Kit	Spare Part
			
19	GSP390880	Boom Frame	Spare Part
			
20	GSP390881	Mast Frame	Spare Part

			
21	GSP390845	Handle Bar	Spare Part
			
22	GSP390997	End Caps - Handle Bar	Spare Part
			
23	GSP390786	Replacement Sticker Pack	Spare Part
	