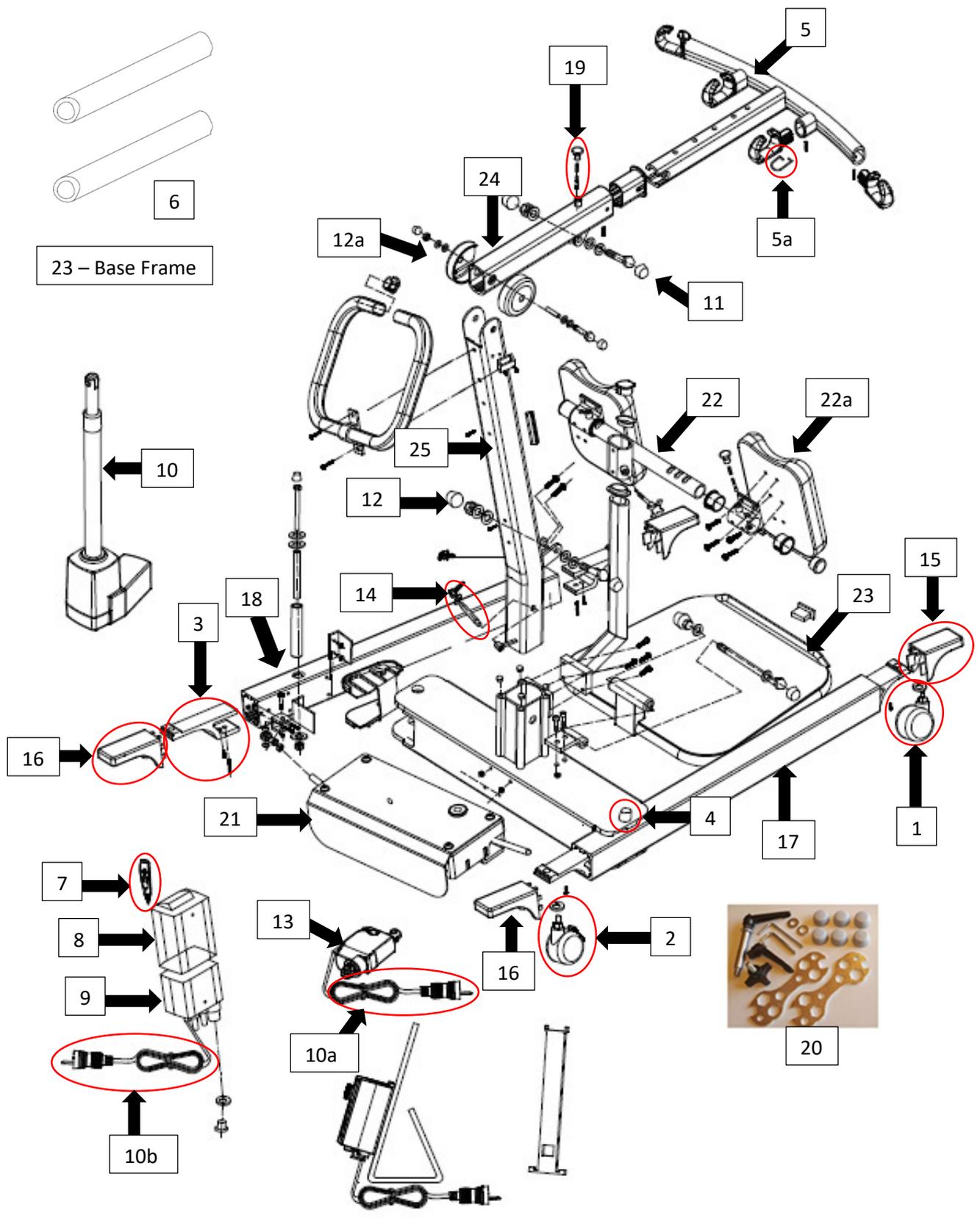


**Aidacare Aspire A200S Standing Lifter (LSS390640)**  
**Spare Parts Manual**

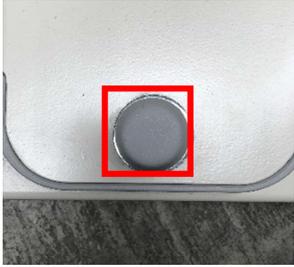
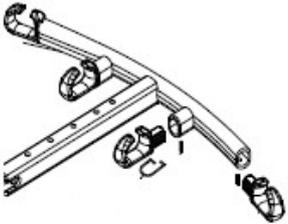
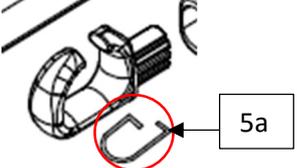


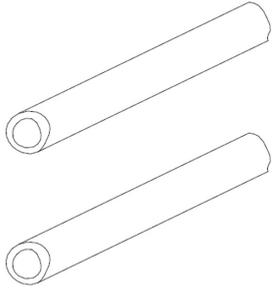
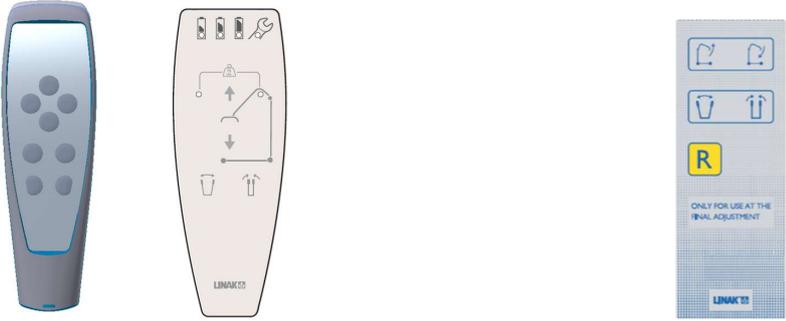
### Aidacare Aspire A200S Patient Lifter – Exploded Parts Diagram



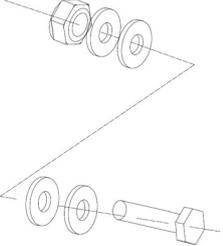
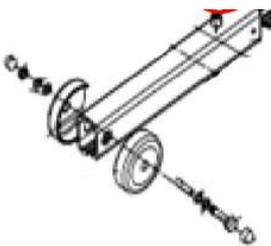
Aidacare Product Code	Aidacare Product Description
LSS390640	Standing Lifter - Aidacare Aspire A200S - Alum - Electric Leg - 200kg

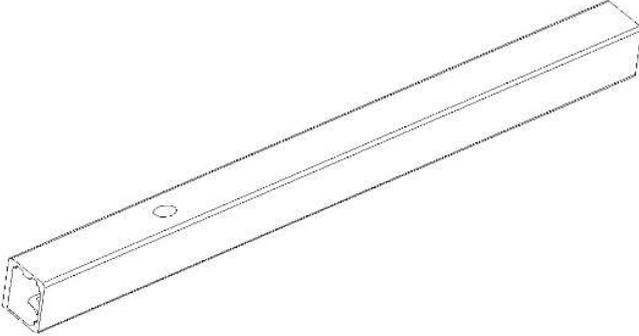
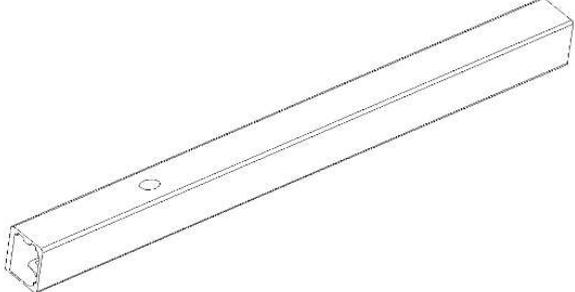
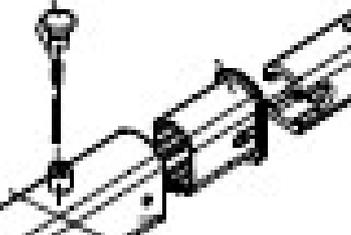
## Spare Parts List

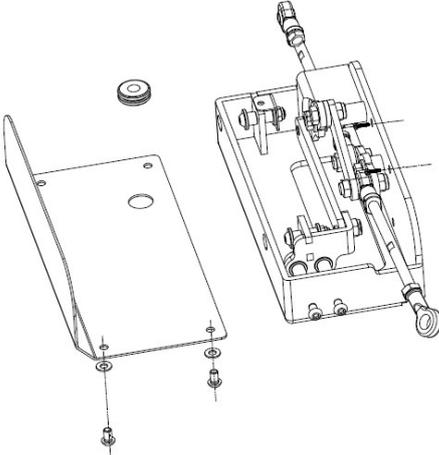
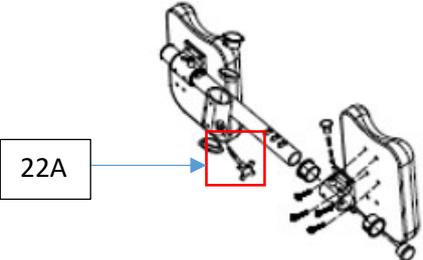
Ref #	Item Number	Item Description	Type
1, 2	VARIOUS	Castor Assembly - Tente	Spare Part
<p>PLEASE REFER TO FILE NAMED 'TENTE CASTOR ASSEMBLY' ON INTRANET FOR CORRECT PART NUMBERS AND CONSUMABLES TO USE FOR CASTORS</p>			
3	GSP390936	Metal Castor Bracket – Front and Rear	Spare Part
			
4	GSP390911	Plastic Cap - Bolt Cover	Spare Part
			
5	GSP390910	Sling/Spreader Bar	Spare Part
			
5	GSP390910H	Sling / Spreader Bar with Grip Handles	Spare Part
			
5a	GSP390976	Sling Bar Retainer Clip	Spare Part
			

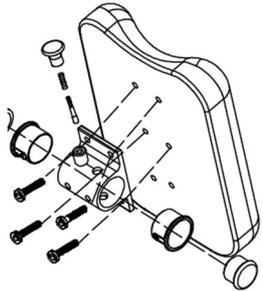
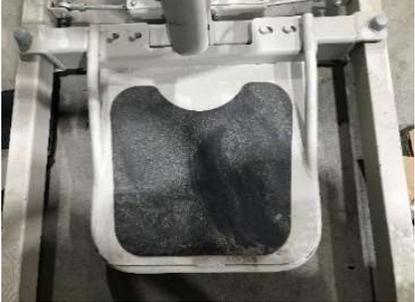
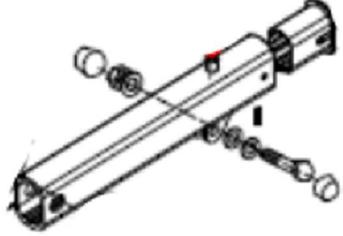
6	GSP390938	Handgrip Padding – PAIR	Accessory
			
7	GSP390691	4 Button Handset	Spare Part
7A	GSP390692	Programming Handset - for Aspire Lifters	
			
8	MBT401200	Battery with Port	Spare Part
8	MBT401190	Battery without Port	Spare Part
			
8a	MBT401220	Battery Charger Plug Pack	Spare Part
8b	MBT401210	Battery Charger – Wall Mount	Spare Part
8c	MBT401012	Battery Charger - Linak - For Lifters - Including Mounting Brackets & Cable	Spare Part
9	GSP390694	Control Box	Spare Part

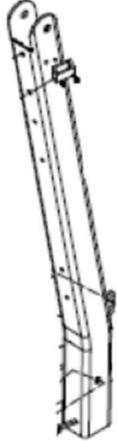
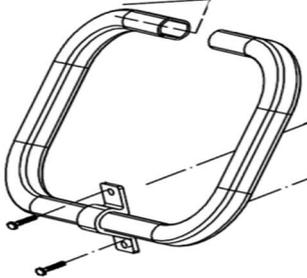
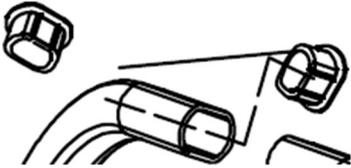
			
9a	MSP500906	Bracket – Short - for Control Box, Battery	Spare Part
10a	GSP390680C	Cable Straight – Linak – for Leg Actuators	Spare Part
			
10b	GSP394172	Cable Curly – Linak – for Lifter Actuators	Spare Part
			
10	GSP390686	Actuator - Lift	Spare Part
			
11	GSP390922	Actuator Bolt Assembly – Upper	Spare Part
12	GSP390927	Actuator Bolt Assembly – Lower	Spare Part

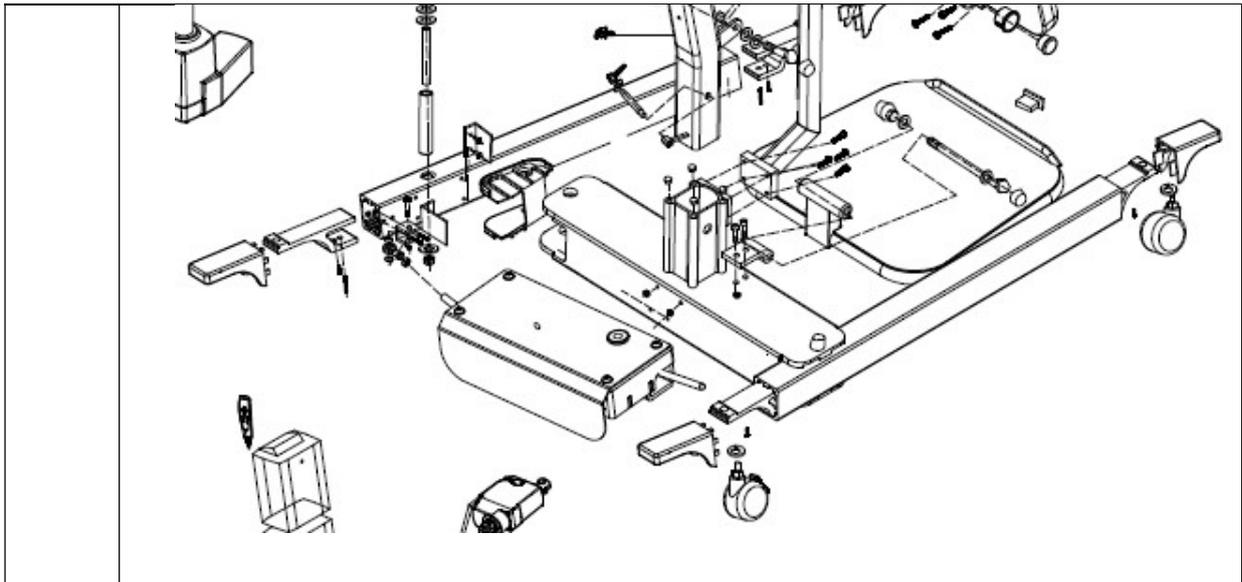
			
12a	GSP390988	Hinge Assembly Mounting Hardware	Spare Part
			
13	GSP390680	Actuator – Leg Gate	Spare Part
13a	GSP390679	Actuator – Leg Gate – Newer Style	
			
14	GSP390944	Adjustment Lever - Mast	Spare Part
			
15	GSP390917	Plastic Castor Cover – Front – PAIR	Spare Part
			
16	GSP390919	Plastic Castor Cover – Rear – PAIR	Spare Part

			
17	GSP390951	Replacement Leg – Left	Spare Part
17	GSP390951A	Replacement Leg – Left – Aluminium	Spare Part
			
18	GSP390983	Replacement Leg – Right	Spare Part
18	GSP390983A	Replacement Leg – Right – Aluminium	Spare Part
			
19	GSP390947	Spring Adjustment Bolt – Boom / Sling Bar	Spare Part
			

20	GSP390967	Tool Assembly Kit	Spare Part
			
21	GSP390682	Electric Leg Kit Bracket	Spare Part
			
21a	GSP390675	Internal Links connection for Leg Spreader	Spare Part
			
22	GSP390973	Knee Pad Assembly	Spare Part
22A	GSP390850	Handwheel Bolt	Spare Part
			

22a	GSP390972	Knee Pad – Left	Spare Part
22a	GSP390984	Knee Pad – Right	Spare Part
			
23	GSP390957	Foot Board Panel	Spare Part
23A	GSP390977	Rubber Liner - w/Peel off Sticker Glue - Aspire A200S/200S Lifters	Spare Part
			
24	GSP390884	Boom Frame	Spare Part
			
25	GSP390885	Mast Frame	Spare Part

			
21	GSP390996	Handle Bar	Spare Part
21	GSP390855	Handle Bar – Adjusted Angle	Spare Part
			
22	GSP390997	End Caps - Handle Bar	Spare Part
			
23	GSP390860	Base Frame w Tente Castors and Leg Actuator	Spare Part



24

GSP390788

Replacement Sticker Pack

Spare Part

**SAFE WORKING LOAD 200KG**

**WARNING!**  
ALWAYS read the instruction manual if you are using this lifter. ALWAYS use Aidacare approved slings with this lifter. Incorrect slings may cause equipment damage voiding all warranty. If in doubt contact Aidacare on 1300 133 130.

**ALWAYS inspect:** lifter, slings, and attachments. **DO NOT** use lifter on uneven surfaces or in situations where there is cause to believe the lifter could tip over. **DO NOT** use lifter if damage is suspected to or visible on the lifter or sling.

**IF IN DOUBT - DO NOT USE THIS LIFTER.**

1300 133 130 [www.aidacare.com.au](http://www.aidacare.com.au)

**CAUTION!**

ONLY USE ASPIRE Lifter with its own base. Be careful when assembling & keep fingers clear of the mast receiver bracket when inserting the mast.

**KEEP CLEAR**

**ANGER**

**PINCH POINT**

**MANUFACTURER AND IMPORTER**

Aidacare Pty Ltd  
 ABN: 45 134 988 889  
 Building 3A  
 Moonbank Avenue  
 Moonbank, VIC 3172

**CONTACT DETAILS**

T: 1300 133 130  
 F: 61 8 9418 2171  
 E: [product@aidacare.com.au](mailto:product@aidacare.com.au)

**SEWING**

It is recommended that you have this product serviced every 6 months.

**ASPIRE SLING GUIDE**

<b>SMALL</b>	250kg	<b>XL</b>	EXTREME LARGE 350kg
<b>MID</b>	300kg	<b>LARGE</b>	300kg