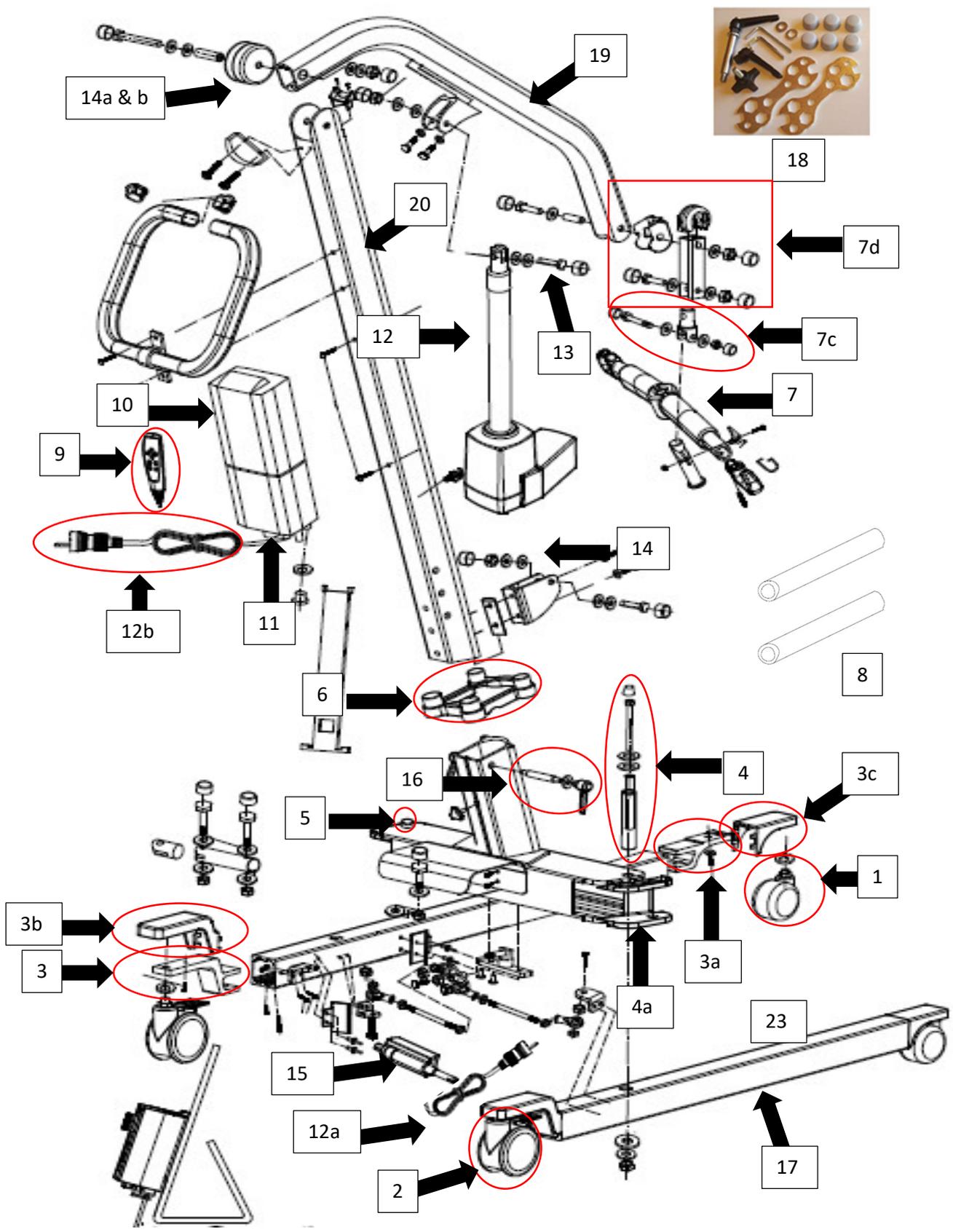


Aidacare Aspire A205 Patient Lifter (LSS390610)
Spare Parts Manual

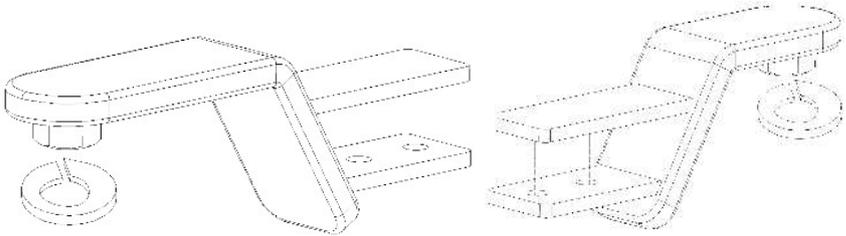
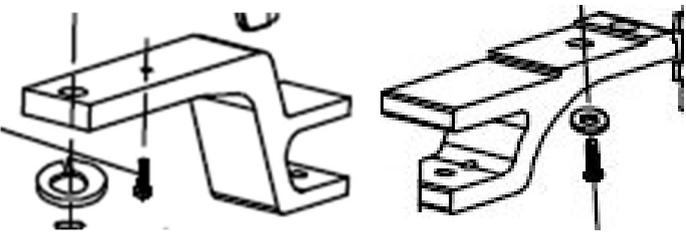
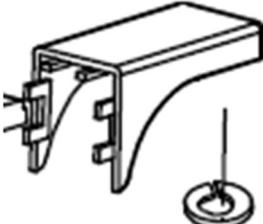


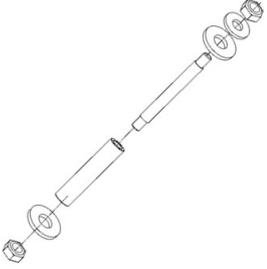
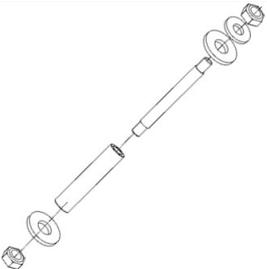
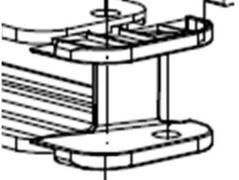
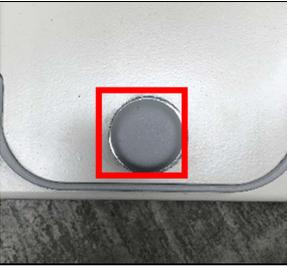
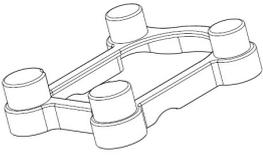
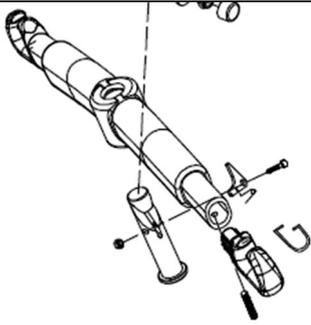
Aidacare Aspire A205 Patient Lifter – Exploded Parts Diagram

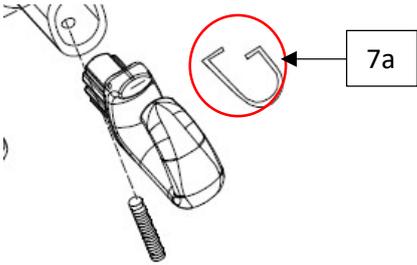
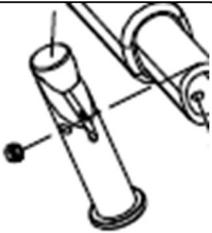
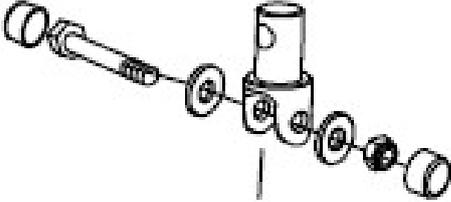
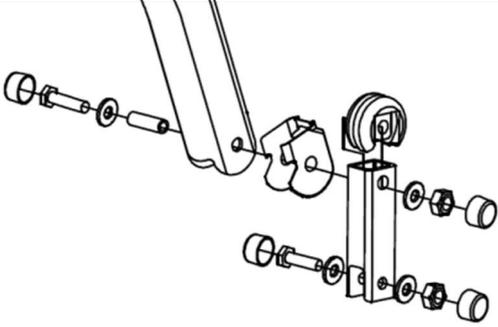
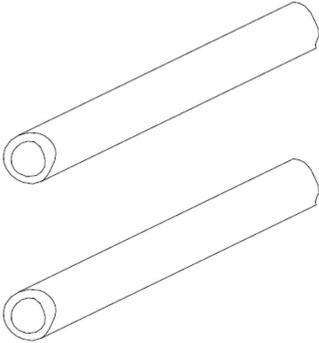


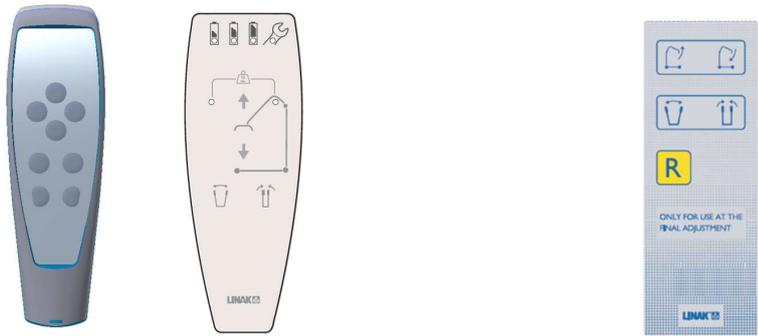
Aidacare Product Code	Aidacare Product Description
LSS390610	Patient Lifter - Aidacare Aspire A205 - Alum - Electric Leg - 205kg

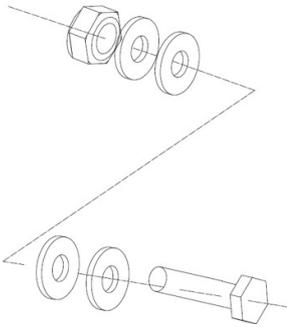
Spare Parts List

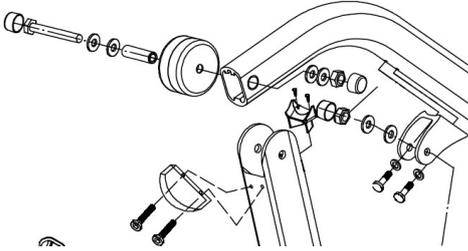
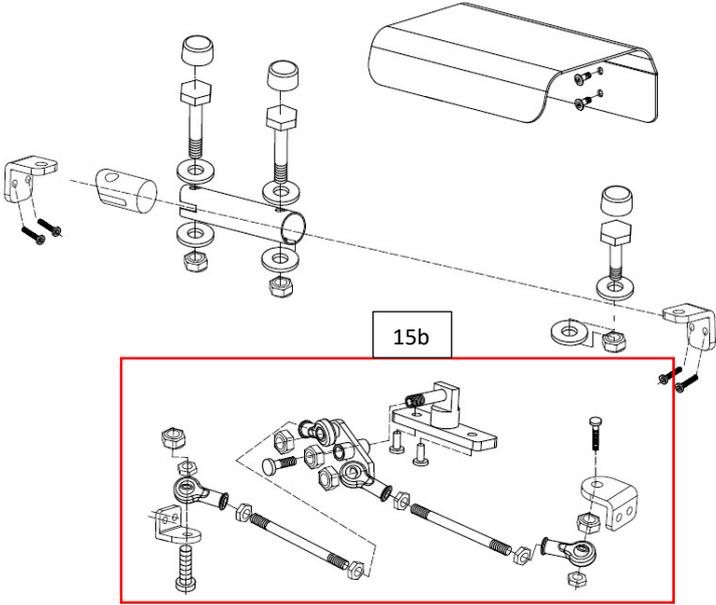
Ref #	Item Number	Item Description	Type
1, 2	VARIOUS	Castor Assembly - Tente	Spare Part
<p>PLEASE REFER TO FILE NAMED 'TENTE CASTOR ASSEMBLY' ON INTRANET FOR CORRECT PART NUMBERS AND CONSUMABLES TO USE FOR CASTORS</p>			
3	GSP390934	Metal Castor Bracket – Rear & Front – OLD – Fit Round Leg Tube	Spare Part
			
3	GSP390873	Metal Castor Bracket – Rear & Front – New – Fit Square Leg Tube	Spare Part
			
3b	GSP390899	Plastic Castor Cover – Rear – PAIR	Spare Part
			
3c	GSP390898	Plastic Castor Cover – Front – PAIR	Spare Part
			

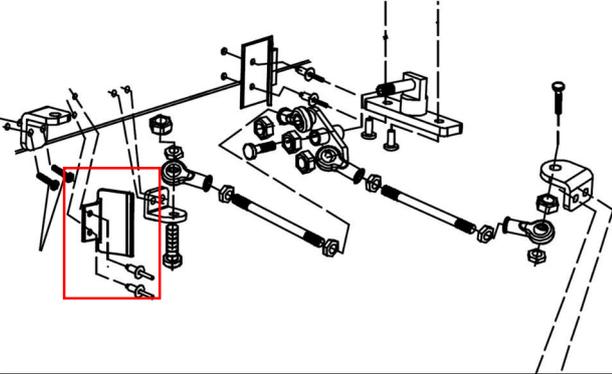
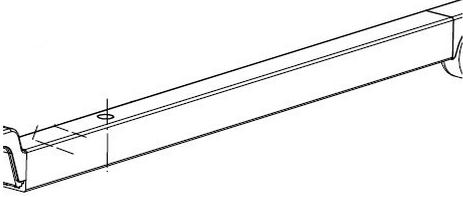
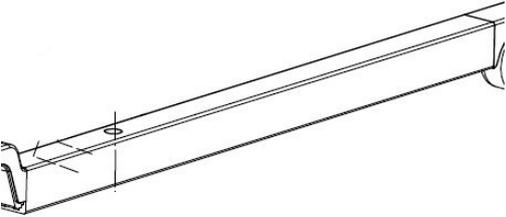
4	GSP390931	Leg Hinge Bolt Assembly – OLD – Fits Round Leg Tube	Spare Part
			
4	GSP390870	Leg Hinge Bolt Assembly – NEW – Fits Square Leg Tube	Spare Part
			
4a	GSP390959	Leg Hinge Bushing	Spare Part
			
5	GSP390911	Plastic Cap - Bolt Cover	Spare Part
			
6	GSP390913	Plastic Shroud for Base of Mast	Spare Part
			
7	GSP390909	Sling/Spreader Bar	Spare Part
			

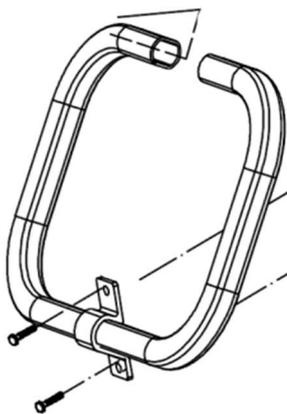
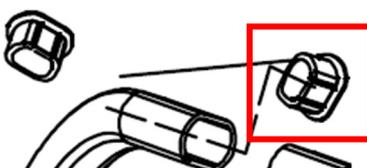
7a	GSP390976	Sling Bar Retainer Clip	Spare Part
			
7b	GSP390968	Adaptor Pin	Spare Part
			
7c	GSP390992	Link Pin Spreader Bar to Mast Assembly	Spare Part
			
7d	GSP390990	Space Link Mounting Assembly – for Sling Bar	Spare Part
			
8	GSP390938	Handgrip Padding – PAIR	Accessory
			

9	GSP390691	4 Button Handset	Spare Part
9A	GSP390692	Programming Handset - for Aspire Lifters	
			
10	MBT401200	Battery with Port	Spare Part
			
10	MBT401190	Battery without Port	Spare Part
10a	MBT401220	Battery Charger Plug Pack	Spare Part
10b	MBT401210	Battery Charger – Wall Mount	Spare Part
10c	MBT401012	Battery Charger - Linak - For Lifters - Including Mounting Brackets & Cable	Spare Parts
11	GSP390694	Control Box	Spare Part
			
11a	MSP500906	Bracket – Short - for Control Box & Battery	Spare Part
12	GSP390684	Actuator - Lift	Spare Part

			
12a	GSP394097	Cable Straight – Linak – for Leg Actuators	Spare Part
			
12b	GSP394172	Cable Curly – Linak – for Lifter Actuators	Spare Part
			
13	GSP390922	Actuator Bolt Assembly – Upper	Spare Part
14	GSP390926	Actuator Bolt Assembly – Lower	Spare Part
			
14a	GSP390978	Plastic Hinge Cover	Spare Part

			
14b	GSP390986	Hinge Assembly Mounting Hardware	Spare Part
			
15	GSP390679	Actuator – Leg Gate	Spare Part
			
15a	GSP390678	Electric Leg Kit Bracket	Spare Part
			

15b	GSP390676	Leg Linkage Assembly (inside Frame)	Spare Part
15c	GSP390693	Platic Hinge – for Base Frame	Spare Part
			
16	GSP390945	Adjustment Lever – Mast	Spare Part
			
17	GSP390952	Replacement Leg – Left	Spare Part
			
17	GSP390950	Replacement Leg - Right	Spare Part
			
18	GSP390964	Tool Assembly Kit	Spare Part
			
19	GSP390878	Boom Frame	Spare Part

			
20	GSP390879	Mast Frame	Spare Part
			
21	GSP390996	Handle Bar	Spare Part
			
22	GSP390997	End Caps – for Handle Bars	Spare Part
			
23	GSP390863	Lifter Spares - Base Frame w Tente Castors and Leg Actuator - Aspire A205	



23

GSP390780 Replacement Sticker Pack

Spare Part



A205



**SAFE WORKING
LOAD 205KG**



MANUFACTURE AND IMPORTER
Aidacare Pty Ltd
PO Box 139 839
Building 3A
1 Melbourne Avenue
Northcote, VIC 3071

CONTACT DETAILS
T: 1300 133 120
E: 02 9619 9111
F: 02 9619 9111
E: prod@aidacare.com.au

SEARCHING
It is recommended that you have the following information ready when searching for a product:

AIDACARE EMS CODE

SMALL 250kg	MID 250kg	MEDIUM 250kg	XL 250kg
LARGE 350kg	POTENTIAL 350kg		

DANGER

PINCH POINT



aidacare aspire

WARNING!

ALWAYS read the instruction manual if you are unfamiliar with how to use this lifter.

ALWAYS use Aidacare approved slings with this lifter. Incorrect slings may cause serious injury and or equipment damage voiding all warranty. If in doubt contact Aidacare 1300 133 120.

ALWAYS inspect lifter and sling prior to use.

DO NOT use lifter on uneven surfaces or in situations where there is a risk of the lifter could tip over.

DO NOT USE if damage is suspected to or visible on the lifter or sling.

IF IN DOUBT - DO NOT USE THIS LIFTER.

**SWL
205KG**

WARNING!

CAUTION!

ONLY USE ASPIRE Lifter with its own base. Be careful when assembling & keep fingers clear of the mast receiver bracket when inserting the mast.



1300 133 120 www.aidacare.com.au