

Spare Parts Diagram – Aspire Active Air 8




Product Code - Mattress


No.	Product Code	Product Description	Suited
	PTM639300	Mattress Replacement - Aspire ACTIVE AIR 8 - Single - 20 Cells - 2000x900x200mm - 200kg	8 – Single
	PTM639310	Mattress Replacement - Aspire ACTIVE AIR 8 - King Single - 20 Cells - 2000x1050x200mm - 200kg	8 – King Single

Mattress – Exploded Diagram

AFTER Serial Number SN:1209362188 (Single Mattress) & SN:1209361968 (King Single Mattress)

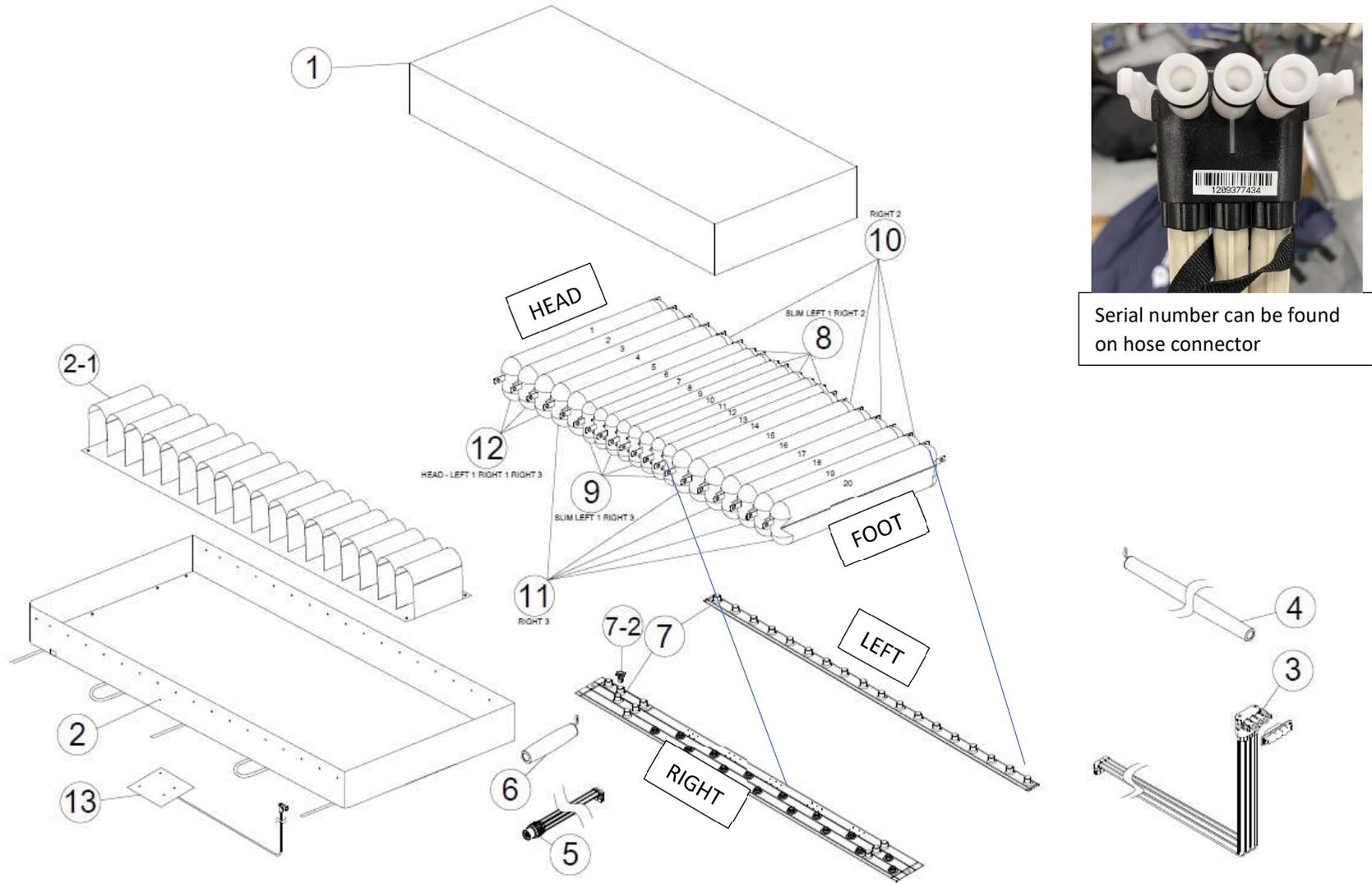

Mattress Replacement Cell Connection Guide – Manifold (Part No. 7)

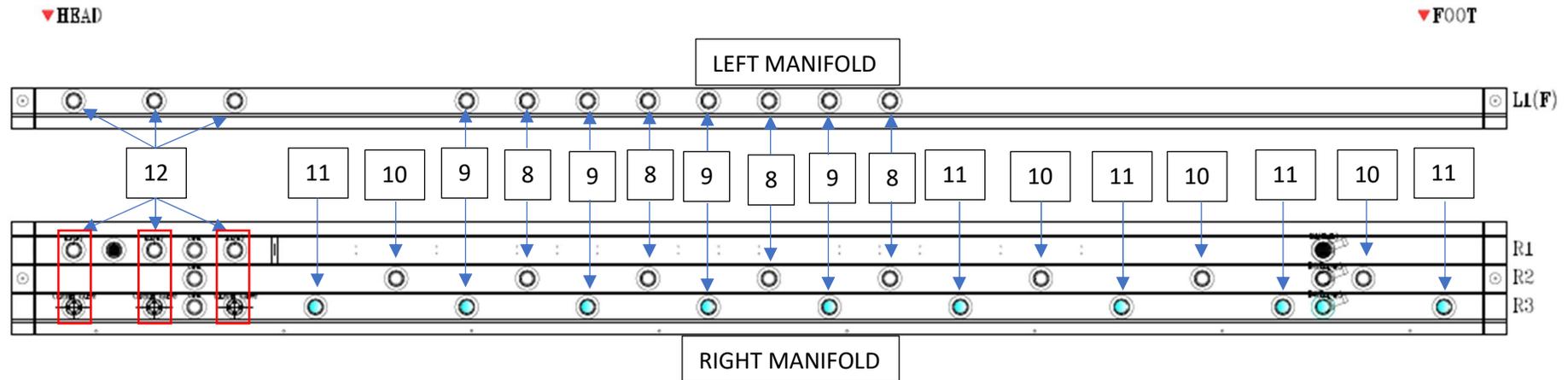

No.	Product Code	Product Description	Suited
12	GSP493245	Replacement Cell - Left (1) / Right (1) / Right (3) Connectors - Head	8 – Single
12	GSP493305	Replacement Cell - Left (1) / Right (1) / Right (3) Connectors - Head	8 – King Single
11	GSP493295	Replacement Cell - Right (3) Connectors - Body / Foot	8 – Single
11	GSP493335	Replacement Cell - Right (3) Connectors - Body / Foot	8 – King Single
10	GSP493285	Replacement Cell - Right (2) Connectors - Body	8 – Single
10	GSP493325	Replacement Cell - Right (2) Connectors - Body	8 – King Single
9	GSP493275	Replacement Cell - Left (1) / Right (3) Connectors - Body	8 – Single
9	GSP493341	Replacement Cell - Left (1) / Right (3) Connectors - Body	8 – King Single
8	GSP493359	Replacement Cell - Left (1) / Right (2) Connectors - Body	8 – Single
8	GSP493349	Replacement Cell - Left (1) / Right (2) Connectors - Body	8 – King Single



Spare Parts Code

No.	Product Code	Product Description	Suited
1	GSP493072	Top Cover – w/old style buttons - Aspire Active Air 8 Mattress	8 – Single
1	GSP493070	Top Cover – w/new style buttons - Aspire Active Air 8 Mattress	8 – Single
1	GSP493080	Top Cover - Aspire Active Air 8 King Single Mattress	8 – King Single

2	GSP493125	Base Cover - Aspire Active Air 8 Mattress	8 – Single
2	GSP493126	Base Cover - Aspire Active Air 8 King Single Mattress	8 – King Single
2-1	GSP493450	Cell Retainer - Aspire Active Air 8 Mattress	8 – Single
2-1	GSP493460	Cell Retainer - Aspire Active Air 8 King Single Mattress	8 – King Single
3	GSP493590	Mattress Pump Hose & Connector - Aspire Active Air 8/8KS Mattresses	8 – ALL
4	GSP493830	Pump Hose Sleeve Cover - Aspire Active Air Mattresses	8 – ALL
5	GSP493610	CPR Twist Valve - Aspire Active Air 8/8KS Mattresses	8 – ALL
6	GSP493620	CPR Valve Sleeve Cover - Aspire Active Air Mattresses	8 – ALL
7	GSP493398	Mattress Cell Manifold - Aspire Active Air 8 Mattress	8 – Single
7	GSP493411	Mattress Cell Manifold - Aspire Active Air 8 King Single Mattress	8 – King Single
7-2	GSP493750	Mechanical Bleed Valve - Aspire Active Air 8/8KS Mattresses	8 – ALL
13	GSP493761	Fowler Detector - Aspire Active Air 8/8KS Mattresses	8 – ALL
14	GSP493039	Mattress Cover Button/Eyelet Set – Plastic - Aspire Active Air Mattresses	8 - ALL
	PTM639020	Carry Bag – Aspire Active Air 8 / 8KS Mattress	8 - ALL


Mattress – Exploded Diagram

PRIOR Serial Number SN:1209362188 (Single Mattress) & SN:1209361968 (King Single Mattress)

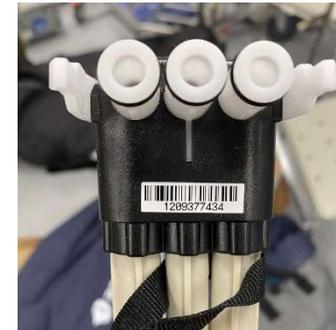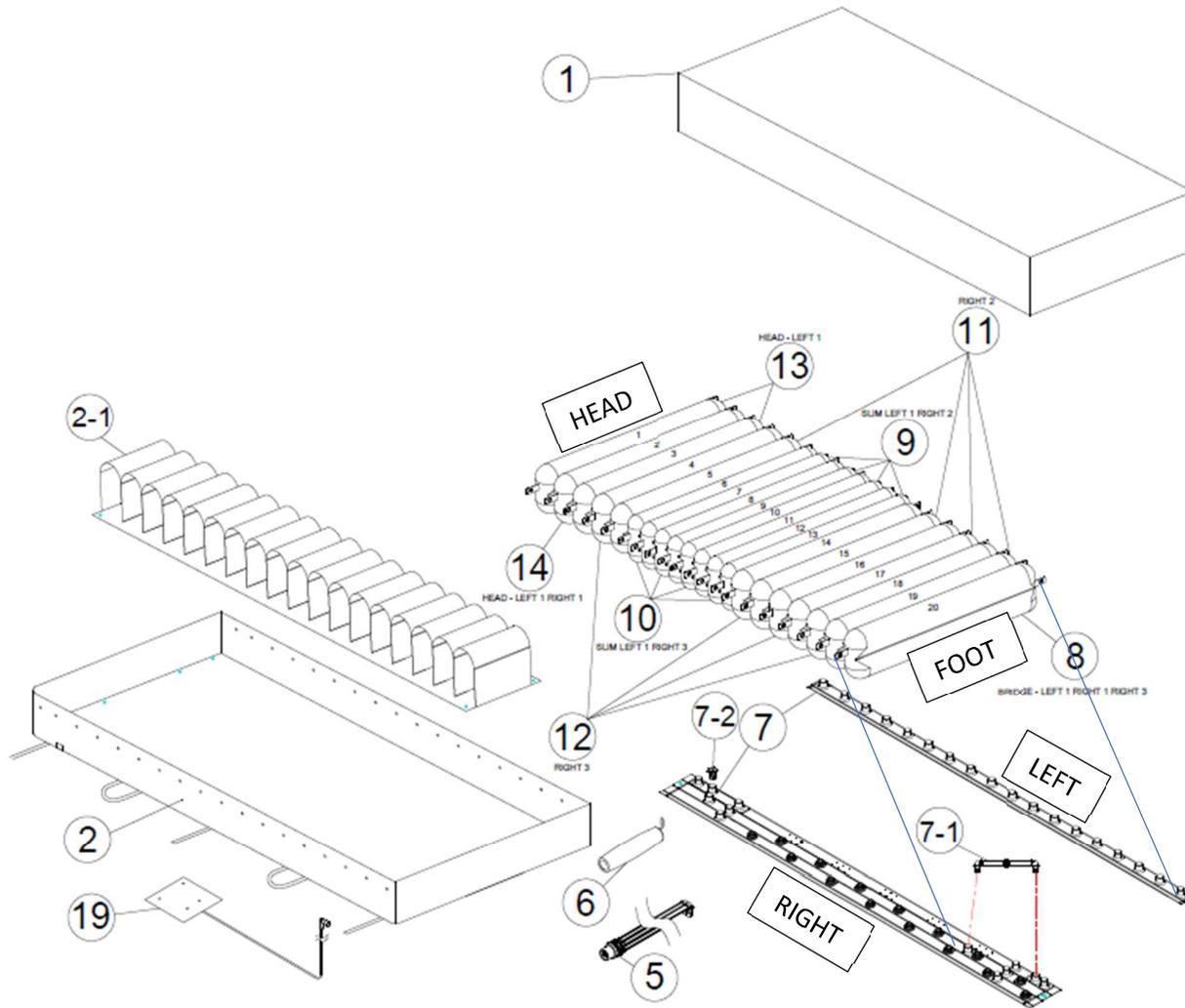

Serial number can be found on hose connector

Mattress Replacement Cell Connection Guide – Manifold (Part No.7)

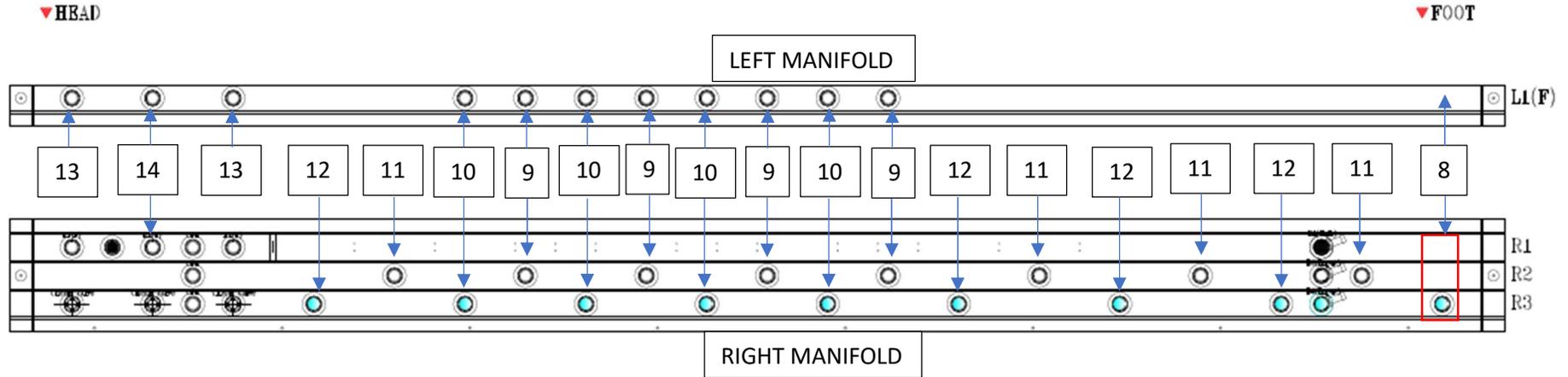


No.	Product Code	Product Description	Suited
14	GSP493240	Replacement Cell - Left (1) /Right (1) Connectors - Head	8 – Single
14	GSP493300	Replacement Cell - Left (1) /Right (1) Connectors - Head	8 – King Single
13	GSP493230	Replacement Cell - Left (1) Connector - Head	8 – Single
13	GSP493310	Replacement Cell - Left (1) Connector - Head	8 – King Single
12	GSP493290	Replacement Cell - Right (3) Connectors - Body	8 – Single
12	GSP493330	Replacement Cell - Right (3) Connectors - Body	8 – King Single
11	GSP493280	Replacement Cell - Right (2) Connectors - Body	8 – Single
11	GSP493320	Replacement Cell - Right (2) Connectors - Body	8 – King Single
10	GSP493270	Replacement Cell - Left (1) /Right (3) Connectors - Body	8 – Single
10	GSP493350	Replacement Cell - Left (1) /Right (3) Connectors - Body	8 – King Single
9	GSP493260	Replacement Cell - Left (1) /Right (2) Connectors - Body	8 – Single
9	GSP493340	Replacement Cell - Left (1) /Right (2) Connectors - Body	8 – King Single
8	GSP493250	Replacement Cell - Left (1) / Right (1) / Right (3) Connectors – Foot Bridge	8 – Single
8	GSP493360	Replacement Cell - Left (1) / Right (1) / Right (3) Connectors – Foot Bridge	8 – King Single

Spare Parts Code

No.	Product Code	Product Description	Suited
1	GSP493070	Top Cover - Aspire Active Air 8 Mattress	8 – Single
1	GSP493080	Top Cover - Aspire Active Air 8 King Single Mattress	8 – King Single
2	GSP493120	Base Cover - Aspire Active Air 8 Mattress	8 – Single
2	GSP493130	Base Cover - Aspire Active Air 8 King Single Mattress	8 – King Single
2-1	GSP493450	Cell Retainer - Aspire Active Air 8 Mattress	8 – Single
2-1	GSP493460	Cell Retainer - Aspire Active Air 8 King Single Mattress	8 – King Single
3	GSP493590	Mattress Pump Hose & Connector - Aspire Active Air 8/8KS Mattresses	8 – ALL
4	GSP493830	Pump Hose Sleeve Cover - Aspire Active Air Mattresses	8 – ALL
5	GSP493610	CPR Twist Valve - Aspire Active Air 8/8KS Mattresses	8 – ALL
6	GSP493620	CPR Valve Sleeve Cover - Aspire Active Air Mattresses	8 – ALL
7	GSP493400	Mattress Cell Manifold - Aspire Active Air 8 Mattress	8 – Single
7	GSP493410	Mattress Cell Manifold - Aspire Active Air 8 King Single Mattress	8 – King Single
7-2	GSP493750	Mechanical Bleed Valve - Aspire Active Air 8/8KS Mattresses	8 – ALL
19	GSP493760	Fowler Detector - Aspire Active Air 8/8KS Mattresses	8 – ALL
20	GSP493038	Mattress Cover Button/Eyelet Set – Brass/Plastic - Aspire Active Air Mattresses	8 - ALL

Pump – Exploded Diagram

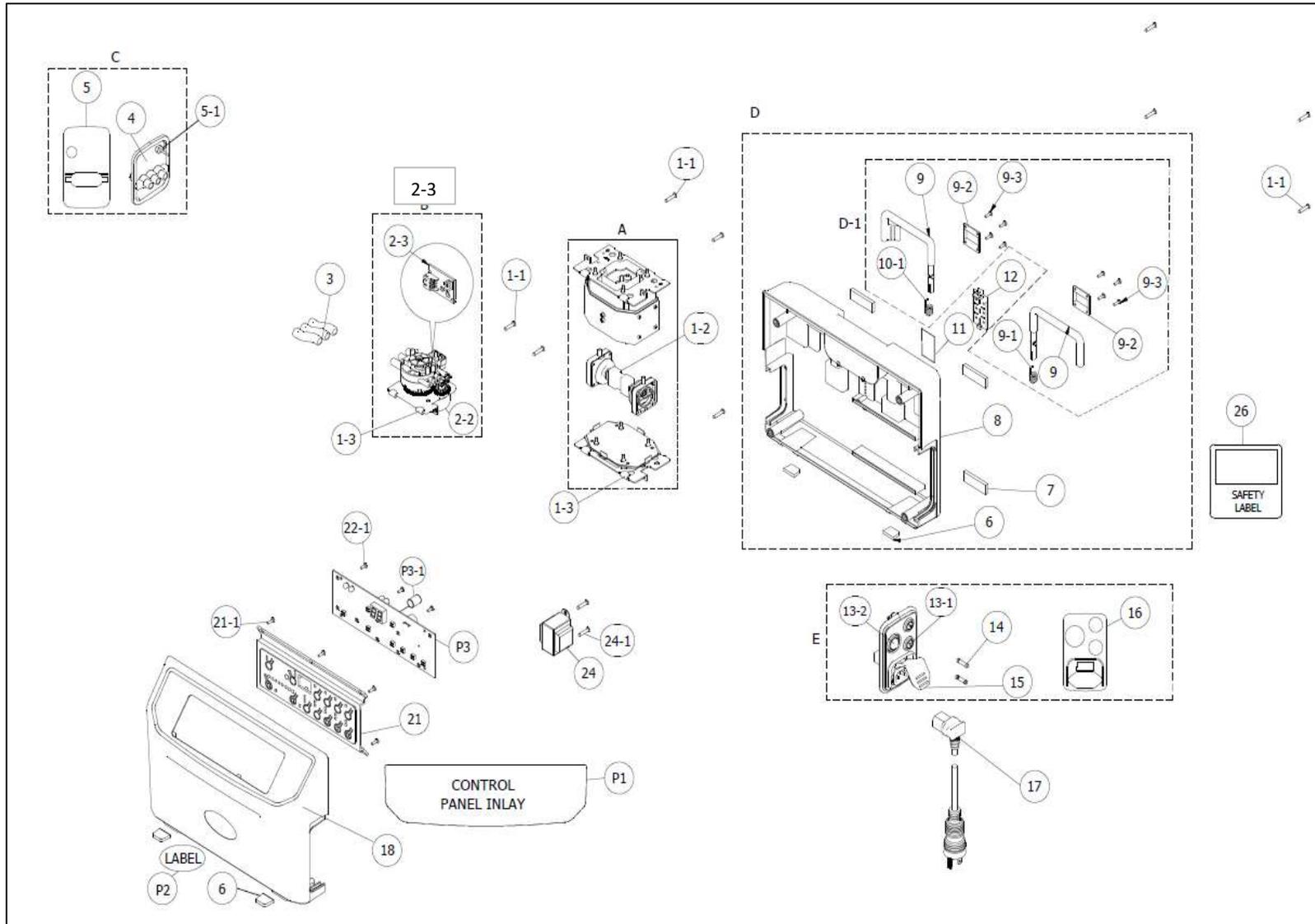

No.	Product Code	Product Description	Suited
	GSP493020	Replacement Pump - Aspire Active Air 8 Single Mattress Replacement	8 – Single
	GSP493021	Replacement Pump - Aspire Active Air 8 King Single Mattress Replacement	8 – King Single

No.	Product Code	Product Description	Suited
3	GSP493780	Inner Pump Tubing - ALL Aspire Active Air Standard / Acute / Smart Pumps	8 – ALL
6	GSP493880	Foam Feet for Pump - ALL Aspire Active Air Standard / Acute / Smart Pumps	8 – ALL
7	GSP493900	Rear Foam Pads for Pump - ALL Aspire Active Air Standard / Acute / BARI / Smart Pumps	8 – ALL
8	GSP493925	Rear Case - Aspire Active Air 8S/KS Pumps	8 – ALL
11	GSP493810	Pump Filter - ALL Aspire Active Air Standard / Acute / Smart Pumps	8 – ALL
12	GSP493820	Pump Filter Cap - ALL Aspire Active Air Standard / Acute / Smart Pumps	8 – ALL
14	GSP493960	1 AMP Fuse for Pump - ALL Aspire Active Air Standard / 8 / Smart Pumps	8 – ALL
15	GSP493862	Power Sockets IEC 2 Pins - ALL Aspire Active Air Standard/Smart/Acute Pumps	8 – ALL
15	GSP493869	Latch Red Clip - for Mains Power Cable - ALL Aspire Active Air Standard /ACUTE/BARI /Smart Pumps	8 – ALL
17	GSP493870	Mains Power Cable - ALL Aspire Active Air Standard / Acute / Smart Pumps	8 – ALL
18	GSP493915	Front Case - Aspire Active Air 8S/8KS Pump	8 – ALL
8 / 18	GSP493932	Front & Rear Case – All Aspire Active Air Pumps and Smart Pumps	8 – ALL
21	GSP493512	Front Panel Control Board Assembly - Aspire Active Air 8S/8KS and BARI 10 Pump	8 – ALL
24	GSP493680	PCB Board Transformer - ALL Aspire Active Air Standard / Acute / BARI / Smart Pumps	8 – ALL
25	GSP493690	PCB for Fowler Detector - ALL Aspire Active Air Standard / Acute / BARI / Smart Pumps	8 – ALL
2-2	GSP493730	Synchronous Motor - ALL Aspire Active Air Standard / Acute /Smart Pumps/BARI Pumps	8 – ALL
2-3	GSP493720	Air Valve PCB - ALL Aspire Active Air Standard / ACUTE / BARI / Smart Pumps	8 – ALL
A	GSP493770	Pump/Air Compressor - Aspire Active Air 8S/KS Pump	8 – ALL
B	GSP493710	Air Valve Assembly - Aspire Active Air 8S/KS Pump	8 – ALL
C	GSP493865	Air Hose Pump Connector - Aspire Active Air 8/8KS Pump and Smart Pump 5/8	8 – ALL
D-1	GSP493930	Rear Case Bracket/Hook Assembly - ALL Aspire Active Air Standard / Acute / BARI Pumps and Smart Pumps	8 – ALL
E	GSP493861	Side Power Assembly	8 – ALL
P3	GSP493660	PCB Board Assembly - Aspire Active Air 8 Single Pump	8 – Single
P3	GSP493670	PCB Board Assembly - Aspire Active Air 8 King Single Pump	8 – King Single
P3-1	GSP493696	Battery Alarm – ALL Aspire Active Air Pump	8 - ALL
5,26,P1,P2,P4	GSP493491	Sticker Label Pack - Aspire Air 8S/8KS Pump	8 – ALL