


Spare Parts Diagram – Aspire Active Air BARI 10




Product Code - Mattress

No.	Product Code	Product Description	Suited
	PTM639500	Mattress Replacement - Aspire ACTIVE AIR BARI 10 - Single - 20 Cells - 2000x900x270mm - 450kg	BARI 10 – Single
	PTM639510	Mattress Replacement - Aspire ACTIVE AIR BARI 10 - King Single - 20 Cells - 2000x1070x270mm - 450kg	BARI 10 – King Single
	PTM639520	Mattress Replacement - Aspire ACTIVE AIR BARI 10 - Super King - 20 Cells - 2000x1230x270mm - 450kg	BARI 10 – Super King Single


Mattress – Exploded Diagram

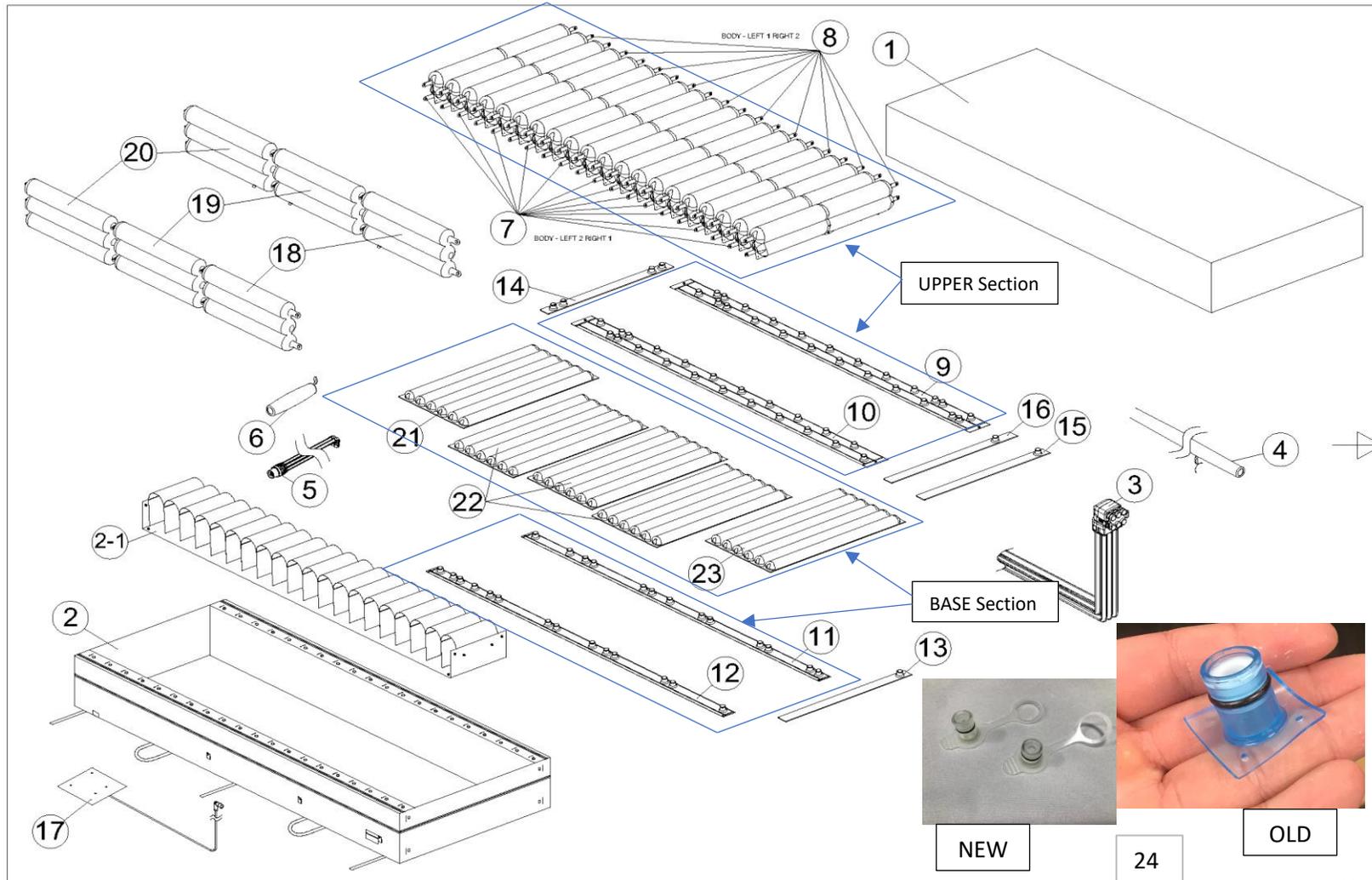

Mattress Replacement Cell Connection Guide – UPPER Section – 9,10

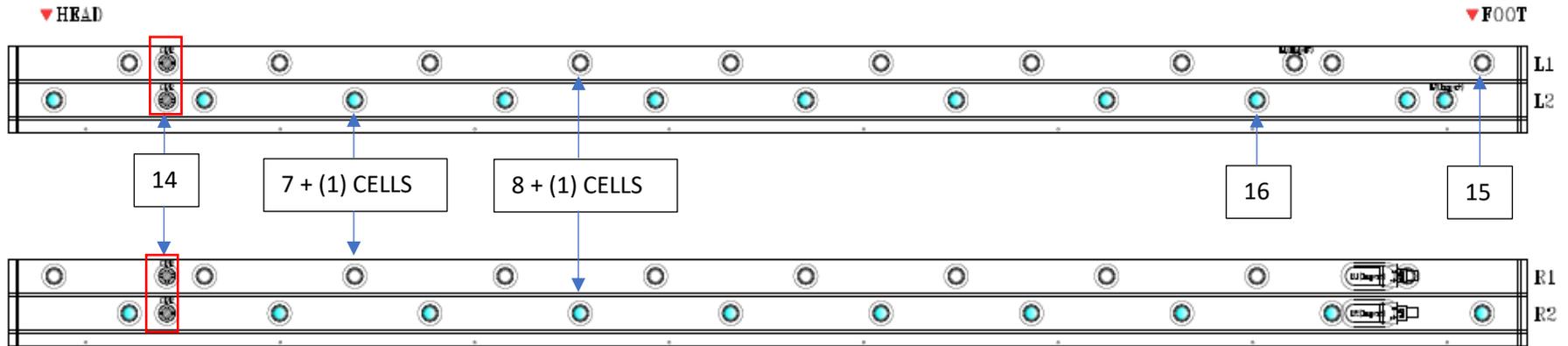

Mattress Replacement Cell Connection Guide – LOWER Section – 11,12

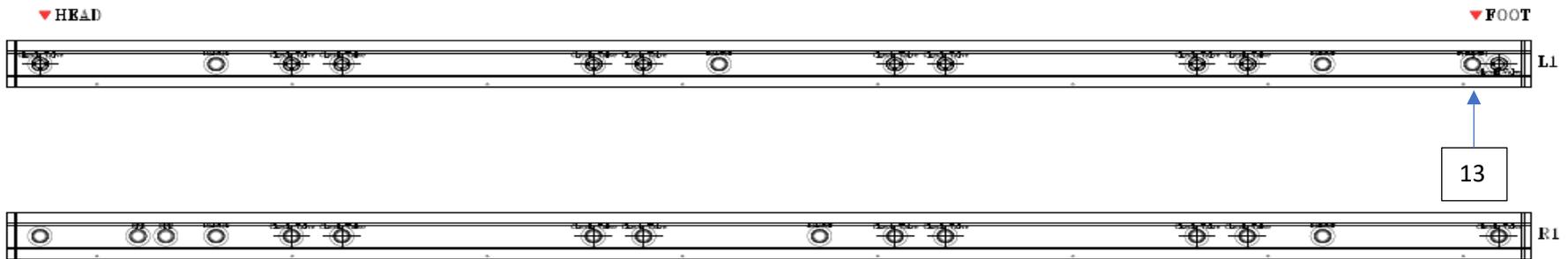

No.	Product Code	Product Description	Suited
9	GSP493401	Mattress Cell Manifold - Air Cells Left (1) Left (2) - Aspire Active Air BARI 10 S/KS/SKS Mattress	BARI 10 – ALL
10	GSP493402	Mattress Cell Manifold - Air Cells Right (1) Right (2) - Aspire Active Air BARI 10 S/KS/SKS Mattress	BARI 10 – ALL
11	GSP493403	Mattress Cell Manifold - Air Pad - Single Left - Aspire Active Air BARI 10 S/KS/SKS Mattress	BARI 10 – ALL
12	GSP493404	Mattress Cell Manifold - Air Pad - Single Right - Aspire Active Air BARI 10 S/KS/SKS Mattress	BARI 10 – ALL
13,14,15,16	GSP493405	Mattress Cell Manifold Assembly Pack - Aspire Active Air BARI 10 S Mattress	BARI 10 – Single
13,14,15,16	GSP493406	Mattress Cell Manifold Assembly Pack - Aspire Active Air BARI 10 KS Mattress	BARI 10 – King Single
13,14,15,16	GSP493407	Mattress Cell Manifold Assembly Pack - Aspire Active Air BARI 10 SKS Mattress	BARI 10 – Super King Single

Replacement Cell – Spare Parts Code


No.	Product Code	Product Description	Suited
7	GSP493361	Replacement Cell - Left (2) /Right (1) Connectors - Body - Aspire Active Air BARI 10 S Mattress	BARI 10 – Single
7	GSP493362	Replacement Cell - Left (2) /Right (1) Connectors - Body - Aspire Active Air BARI 10 KS Mattress	BARI 10 – King Single
7	GSP493363	Replacement Cell - Left (2) /Right (1) Connectors - Body - Aspire Active Air BARI 10 SKS Mattress	BARI 10 – Super King Single
8	GSP493364	Replacement Cell - Left (1) /Right (2) Connectors - Body - Aspire Active Air BARI 10 S Mattress	BARI 10 – Single
8	GSP493365	Replacement Cell - Left (1) /Right (2) Connectors - Body - Aspire Active Air BARI 10 KS Mattress	BARI 10 – King Single
8	GSP493366	Replacement Cell - Left (1) /Right (2) Connectors - Body - Aspire Active Air BARI 10 SKS Mattress	BARI 10 – Super King Single

Spare Parts Code - Mattress

No.	Product Code	Product Description	Suited
1	GSP493083	Top Cover - Aspire Active Air Acute 10 Single Mattress	BARI 10 – Single
1	GSP493084	Top Cover - Aspire Active Air Acute 10 King Single Mattress	BARI 10 – King Single
1	GSP493085	Top Cover - Aspire Active Air Acute 10 Super King Single Mattress	BARI 10 – Super King Single
2	GSP493133	Base Cover - Aspire Active Air Bari 10 Single Mattress	BARI 10 – Single
2	GSP493134	Base Cover - Aspire Active Air Bari 10 King Single Mattress	BARI 10 – King Single
2	GSP493135	Base Cover - Aspire Active Air Bari 10 Super King Single Mattress	BARI 10 – Super King Single
3	GSP493595	Mattress Pump Hose & Connector - Aspire Active Air BARI 10 S/KS/SKS Mattresses	BARI 10 – ALL
4	GSP493835	Pump Hose Sleeve Cover - Aspire Active Air BARI 10 S/KS/SKS Mattresses	BARI 10 – ALL
5	GSP493615	CPR Twist Valve - Aspire Active Air BARI 10 S/KS/SKS Mattresses	BARI 10 – ALL
6	GSP493625	CPR Valve Sleeve Cover - Aspire Active Air Acute 8 / BARI 10 Mattresses	BARI 10 & Acute 8
11	GSP493403	Mattress Cell Manifold - Air Pad - Single Left - Aspire Active Air BARI 10 S/KS/SKS Mattress	BARI 10 – ALL
12	GSP493404	Mattress Cell Manifold - Air Pad - Single Right - Aspire Active Air BARI 10 S/KS/SKS Mattress	BARI 10 – ALL
17	GSP493762	Fowler Detector - All Aspire Active Air Acute / BARI 10 Mattresses	BARI 10 – ALL
18	GSP494070	Replacement Side Bolster - Left (1) - Aspire Active Air 10 S/KS/SKS Mattress	BARI 10 – ALL
19	GSP494071	Replacement Side Bolster - Middle (1) - Aspire Active Air 10 S/KS/SKS Mattress	BARI 10 – ALL
20	GSP494072	Replacement Side Bolster - Right (1) - Aspire Active Air 10 S/KS/SKS Mattress	BARI 10 – ALL
21	GSP494080	Replacement Air Pad - Head - Aspire Air BARI 10 S Mattress	BARI 10 – Single
21	GSP494081	Replacement Air Pad - Head - Aspire Air BARI 10 KS Mattress	BARI 10 – King Single
21	GSP494082	Replacement Air Pad - Head - Aspire Air BARI 10 SKS Mattress	BARI 10 – Super King Single
22	GSP494083	Replacement Air Pad - Body - Aspire Air BARI 10 S Mattress	BARI 10 – Single

22	GSP494084	Replacement Air Pad - Body - Aspire Air BARI 10 KS Mattress	BARI 10 – King Single
22	GSP494085	Replacement Air Pad - Body - Aspire Air BARI 10 SKS Mattress	BARI 10 – Super King Single
23	GSP494086	Replacement Air Pad - Foot - Aspire Air BARI 10 S Mattress	BARI 10 – Single
23	GSP494087	Replacement Air Pad - Foot - Aspire Air BARI 10 KS Mattress	BARI 10 – King Single
23	GSP494088	Replacement Air Pad - Foot - Aspire Air BARI 10 SKS Mattress	BARI 10 – Super King Single
2-1	GSP493465	Cell Retainer - Aspire Active Air BARI 10 S Mattress	BARI 10 – Single
2-1	GSP493466	Cell Retainer - Aspire Active Air BARI 10 KS Mattress	BARI 10 – King Single
2-1	GSP493467	Cell Retainer - Aspire Active Air BARI 10 SKS Mattress	BARI 10 – Super King Single
24	GSP493805	Cell Stopper Plug PVC - Aspire Active Air Acute 8 and BARI 10 Mattress	BARI 10 & Acute 8 – OLD
24	GSP493807	Cell Stopper Plug PVC - Aspire Active Air Acute 8 and BARI 10 Mattress	BARI 10 & Acute 8 – NEW

Pump – Exploded Diagram

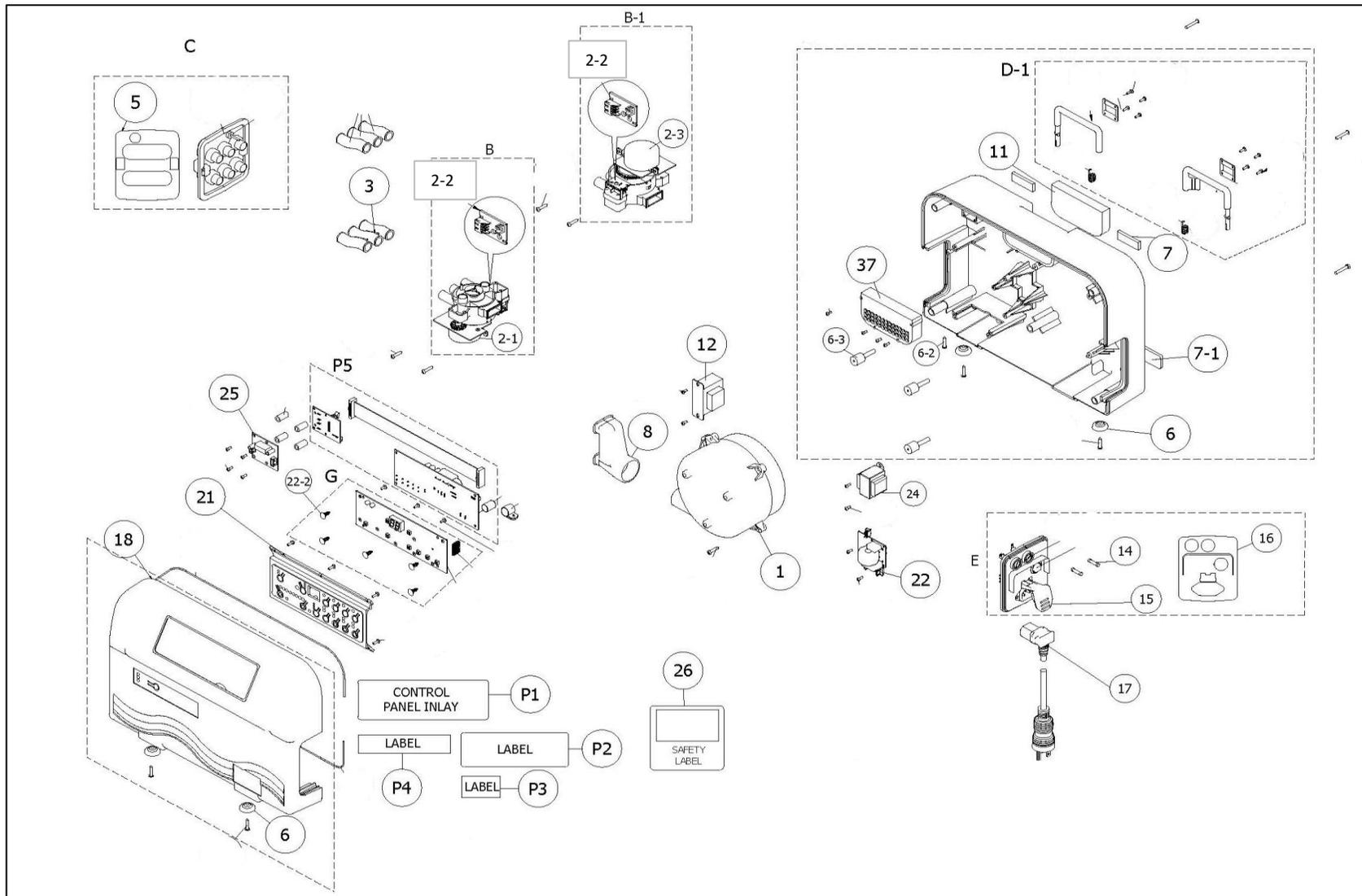

Product Code – Pump

No.	Product Code	Product Description	Suited
	GSP493034	Replacement Pump - Aspire Active Air Bari 10 Air Mattress Replacement	BARI 10

Spare Part Codes – Pumps

No.	Product Code	Product Description	Suited
1	GSP493776	Pump/Air Compressor - Aspire Active Air 10 Pump	BARI 10
3	GSP493785	Inner Pump Tubing - Aspire Active Air BARI 10 Pump	BARI 10
6	GSP493885	Foam Feet for Pump - Aspire Active Air BARI 10 Pump	BARI 10
7	GSP493900	Rear Foam Pads for Pump - ALL Aspire Active Air Standard / Acute / BARI / Smart Pumps	BARI 10
8	GSP493786	Blower to Valve Tubing - Aspire Active Air BARI 10 Pump	BARI 10
11	GSP493815	Pump Filter - Aspire Active Air BARI 10 Pump	BARI 10
12	GSP493995	Inductor 230V for EMC - Aspire Active Air BARI 10 Pump	BARI 10
13	GSP493857	Power Sockets IEC 3 Pins	BARI 10
14	GSP493965	5 AMP Glass Fuse for Pump - Aspire Active BARI 10 Pump	BARI 10
15	GSP493857	Power Sockets IEC 3 Pins - ALL Aspire Active Air Standard/Smart/Acute Pumps	BARI 10
17	GSP493876	Mains Power Cable (3 Prong) - Aspire Active Air BARI 10 Pump	BARI 10
18	GSP493916	Front Case - Aspire Active Air BARI 10 Pump	BARI 10
18 / 7	GSP493396	Front & Rear Case – Aspire Active Air BARI 10 Pump	BARI 10
21	GSP493512	Front Panel Control Board Assembly - Aspire Active Air ACUTE 8 / 8KS and BARI 10 Pump	BARI 10
22	GSP493676	Secondary PCB Board - Aspire Active Air BARI 10 Pump - for EMC	BARI 10
24	GSP493680	PCB Board Transformer - ALL Aspire Active Air Pumps and Smart Pumps	BARI 10
25	GSP493690	PCB for Fowler Detector - ALL Aspire Active Air Pumps and Smart Pumps	BARI 10
37	GSP494040	Muffler Assembly Aspire Active Air BARI 10 Pump	BARI 10
6-3	GSP494060	Anti-Vibration Feet for Aspire Active Air BARI 10 Pump	BARI 10
7-1	GSP493905	Rear Foam Pads for Pump (Bottom) - Aspire Active Air BARI 10 Pump	BARI 10
B	GSP493716	Air Valve Assembly - Aspire Active Air BARI 10 Pump	BARI 10
B-1	GSP493717	Secondary Air Valve Assembly - Aspire Active Air BARI 10 Pump	BARI 10
B-1-2-3	GSP493730	Synchronous Motor - ALL Aspire Active Air Pumps/Smart Pumps/BARI 10	BARI 10
B-2-1	GSP493735	Synchronous Motor - Aspire Active Air BARI 10 Pump - MAIN Air Valve	BARI 10
B-2-2	GSP493720	Air Valve PCB - ALL Aspire Active Air Standard / ACUTE / BARI / Smart Pumps	BARI 10

B-1-2-2	GSP493725	Air Valve PCB - Aspire Active Air Acute 10 Pump	BARI 10
C	GSP493867	Air Hose Pump Connector - Aspire Active Air BARI 10 Pump	BARI 10
D-1	GSP493930	Rear Case Bracket/Hook Assembly – ALL Aspire Active Air Standard / Acute / BARI / Smart Pumps	ALL Pumps
G	GSP493675	Secondary PCB Board Assembly - Aspire Active Air BARI 10 Pump - for Control Panel	BARI 10
P1,P2,P3,P4,5,16,26	GSP493492	Sticker Label Pack - Aspire Active Air BARI 10 Pump	BARI 10
P5	GSP493674	Main PCB Board Assembly - Aspire Active Air BARI 10 Pump	BARI 10
P5-1	GSP493696	Battery Alarm – ALL Aspire Active Air Pump	BARI 10
D-2	GSP493926	Rear Case and Foam Pads Assembly - Aspire Active Bari 10	BARI 10