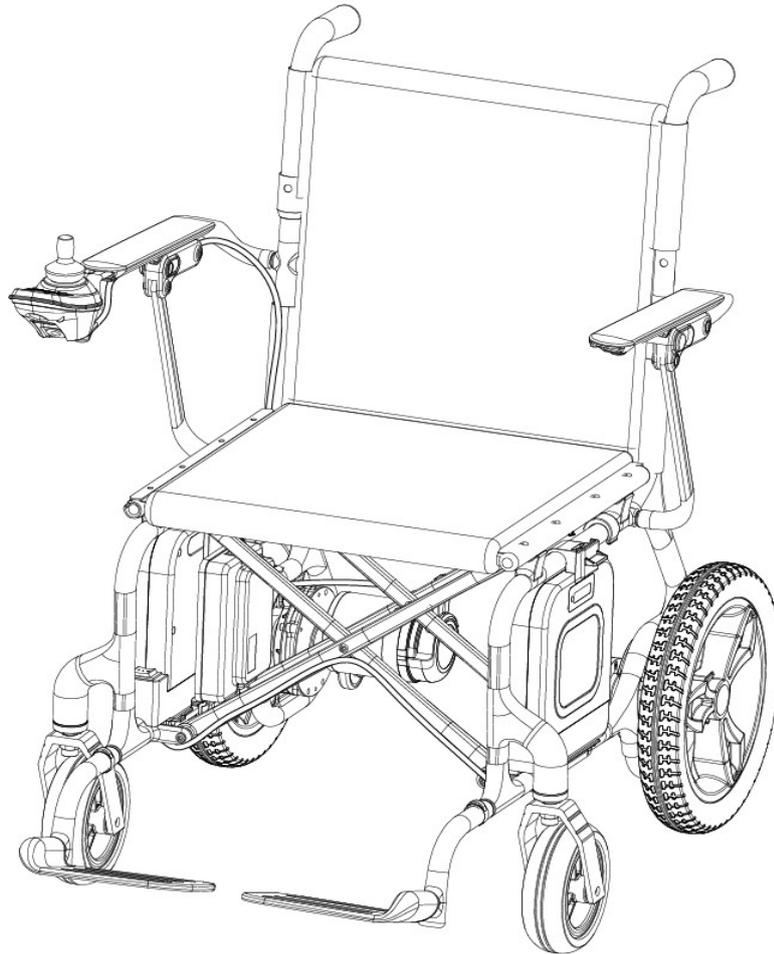


## Spare Parts Manual

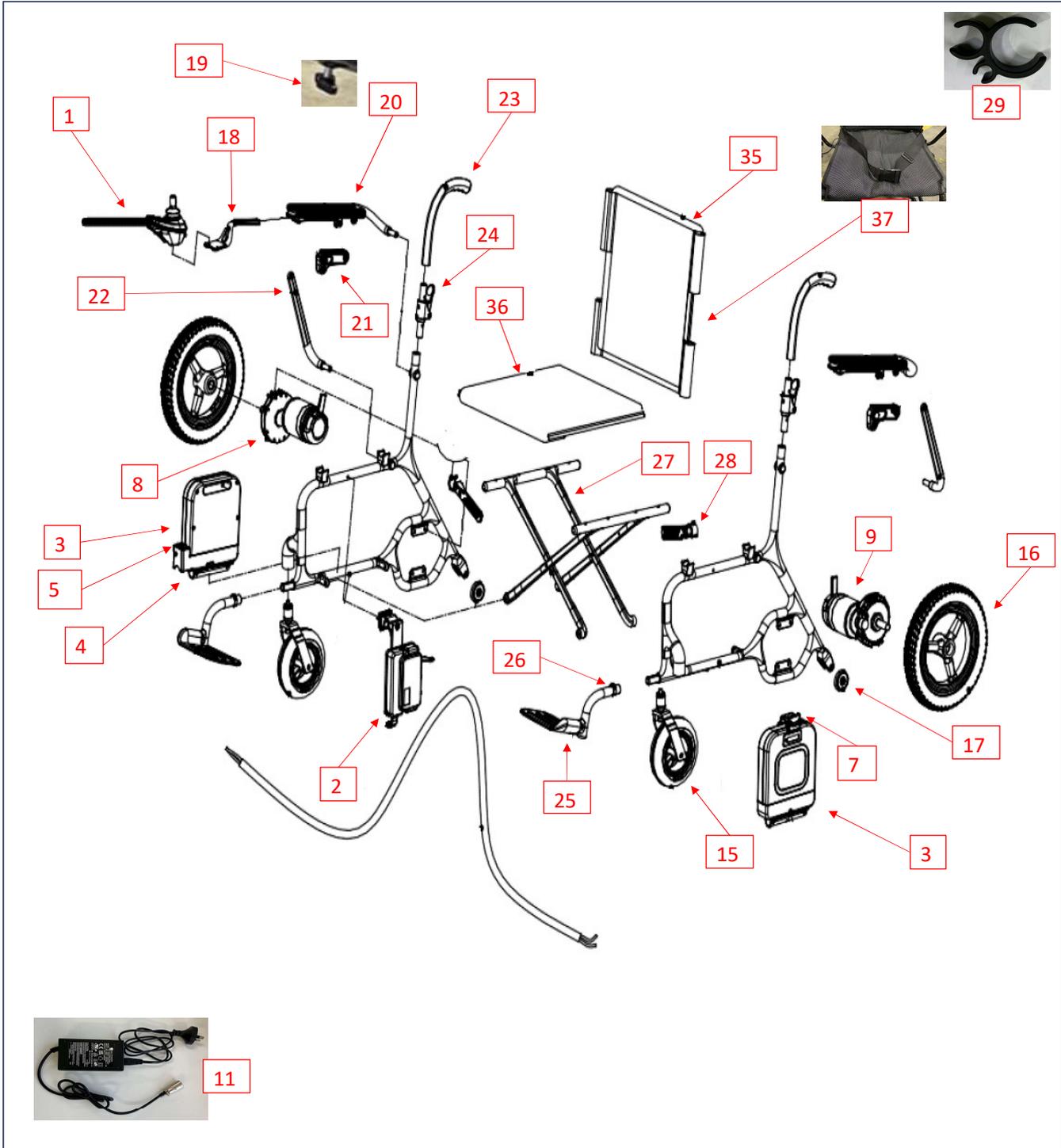
---



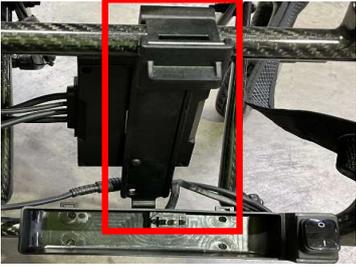
Item Number	Item Description
PWS654150	Aspire AeroLite Carbon Fibre Power Wheelchair - 120kg

**This manual is for internal use only**

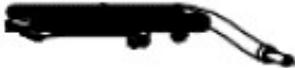
## Exploded Diagram

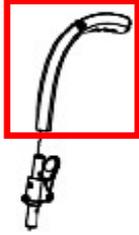
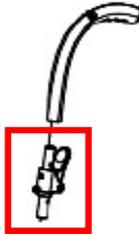


1 Electronics				
Ref	Item Number	Item Description	Image	Type
1	GSP047100	Controller - Joystick - Aspire AeroLite CF Power Wheelchair		Spare Part
2	GSP047102	Control Box - Aspire AeroLite CF Power Wheelchair		Spare Part
3	GSP047104	Battery - Aspire AeroLite CF Power Wheelchair		Spare Part
4	GSP047106	Battery Doc - Aspire AeroLite CF Power Wheelchair		Spare Part
5	GSP047108	Two Position Switch - Standard - Aspire AeroLite CF Power Wheelchair		Spare Part

1 Electronics				
7	GSP047112	Mounting Bracket - for Battery - Aspire AeroLite CF Power Wheelchair		Spare Part
8	GSP047114	Motor - <b>Right</b> - Aspire AeroLite CF Power Wheelchair		Spare Part
9	GSP047116	Motor - <b>Left</b> - Aspire AeroLite CF Power Wheelchair		Spare Part
11	GSP047120	Battery Charger - Aspire AeroLite CF Power Wheelchair		Spare Part

2 Wheels & Frame				
Ref	Item Number	Item Description	Image	Type
15	GSP047130	Front Wheel Assembly - w/Fork/Mnt Hardware - Aspire AeroLite CF Power Wheelchair		Spare Part
16	GSP047132	Rear Wheel - Aspire AeroLite CF Power Wheelchair		Spare Part
17	GSP047134	Anti Tip Wheel - with Mnt Hardware - Aspire AeroLite CF Power Wheelchair		Spare Part
18	GSP047136	Mount for Controller - Aspire AeroLite CF Power Wheelchair		Spare Part
19	GSP047138	Hand Bolt - Controller Mount - Aspire AeroLite CF Power Wheelchair		Spare Part

2 Wheels & Frame				
20	GSP047140	Armrest - Right - Aspire AeroLite CF Power Wheelchair		Spare Part
20	GSP047142	Armrest - Left - Aspire AeroLite CF Power Wheelchair		
21	GSP047144	Armrest Folding Mechanism - Right - Aspire AeroLite CF Power Wheelchair		Spare Part
21	GSP047146	Armrest Folding Mechanism - Left - Aspire AeroLite CF Power Wheelchair		Spare Part
22	GSP047148	Armrest Lower Link - Right - Aspire AeroLite CF Power Wheelchair		Spare Part
22	GSP047150	Armrest Lower Link - Left - Aspire AeroLite CF Power Wheelchair		Spare Part

2 Wheels & Frame				
23	GSP047152	Backrest Cane / Handle - suits Left / Right - Aspire AeroLite CF Power Wheelchair		Spare Part
24	GSP047154	Folding Mechanism - Backrest Cane - suits Left / Right - Aspire AeroLite CF Power Wheelchair		Spare Part
25	GSP047156	Legrest - Right - w/C-Clip - Aspire AeroLite CF Power Wheelchair		Spare Part
25	GSP047158	Legrest - Left - w/C-Clip - Aspire AeroLite CF Power Wheelchair		Spare Part
26	GSP047160	C-Clip - for Legrest - Aspire AeroLite CF Power Wheelchair		Spare Part
27	GSP047162	Frame Scissor - Aspire AeroLite CF Power Wheelchair		Spare Part

2 Wheels & Frame				
28	GSP047164	Plastic Hook (Frame Stabilizer) - <b>Suits Left / Right</b> - Aspire AeroLite CF Power Wheelchair		Spare Part
29	GSP047166	Cable Clip - Aspire AeroLite CF Power Wheelchair		Spare Part

2 Seating				
Ref	Item Number	Item Description	Image	Type
35	GSP047170	Seat Upholstery - Aspire AeroLite CF Power Wheelchair		Spare Part
36	GSP047172	Backrest Upholstery - Aspire AeroLite CF Power Wheelchair		Spare Part
37	GSP047174	Safety Seat Belt - Aspire AeroLite CF Power Wheelchair		Spare Part
38	GSP047176	Calf Support - Upholstered - Aspire AeroLite CF Power Wheelchair		Spare Part