


### Aspire Lift Recline Chair ALTITUDE - Spare Parts Diagram

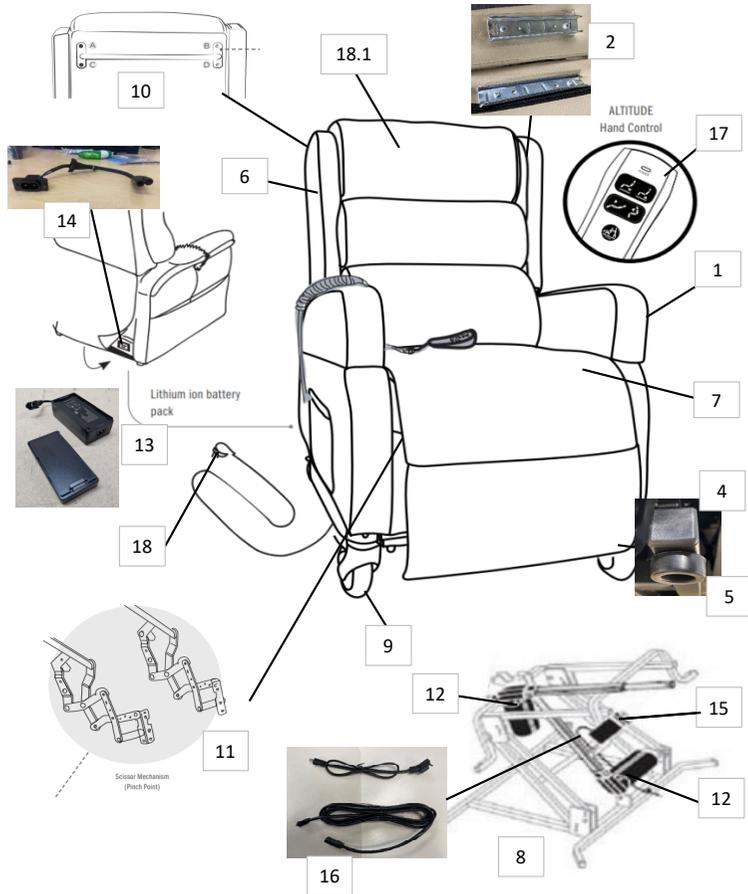

Product Code	Product Description
CHP227700	Chair Powerlift Recline - Aspire ALTITUDE Vertical Lift CHAIR - BitterChocolate/Beige - Dual Motor
CHP227705	Chair Powerlift Recline - Aspire ALTITUDE Single Action Recliner - Black/Blue

No.	Item Number	Item Description
1	GSP510326	Armrest - Standard - Aspire ALTITUDE Single Action Recliner - PAIR
1	GSP510325	Armrest - Standard - Aspire ALTITUDE Vertical / Single Action Recliner Chair - PAIF
2	GSP510327	Mounting Hardware - Wing Pad - Aspire ALTITUDE Vertical Chair - PAIF
4	GSP510358	Plastic End Cap - Aspire ALTITUDE Vertical / Single Action Recliner / Porto Chair
5	GSP510352	Plastic Foot - Aspire ALTITUDE Vertical / Porto / Signature 2 Chair
6	GSP510329	Wing Pad - Aspire ALTITUDE Single Action Recliner - PAIR
6	GSP510328	Wing Pad - Aspire ALTITUDE Vertical Chair - PAIR
7	GSP510445	Seat Detachable - Aspire ALTITUDE Vertical / Single Action Recliner Chair

No	Item Number	Item Description
8	GSP511200	Base Frame - Aspire ALTITUDE Lift Recline Chair - Dual Motor
8	GSP511210	Base Frame - Aspire ALTITUDE Lift Recline Chair - Single Action
8	GSP511340	Backrest Frame - Aspire ALTITUDE Lift Recline Chair - Dual Motor
8	GSP511342	Backrest Frame - Aspire ALTITUDE Lift Recline Chair - Single Action
9	GSP510160	Castor - Locking - Aspire ALTITUDE Vertical / Single Action Recliner Chair
9	GSP510210	Castor - Standard - Aspire ALTITUDE Vertical / Single Action Recliner Chair
10	GSP510520	Handle Bar Assembly - Aspire ALTITUDE Vertical / Single Action Recliner Chair
11	GSP511205	Seat Extension Frame - Aspire ALTITUDE Lift Recline Chair - Single Actior

No	Item Number	Item Description
12	GSP510015	Actuator - Backrest / Footrest - Aspire ALTITUDE Vertical Chair
12	GSP510017	Actuator - Hi/Lo - Aspire ALTITUDE Vertical / Single Action Recliner Chair
12	GSP510018	Actuator - Single - Aspire ALTITUDE Single Action Recliner
13	BEA830220	Battery Backup & Power Transformer- ComfiMotion Care Bed, Luxe Bed & Altitude Chair
13	GSP400649	Battery ONLY - for Battery Backup - All Aspire Lift Recline Chair / ComfiMotion Care / LUX Bed:
13	GSP400648	Transformer ONLY - for Battery Backup - All Aspire Lift Recline Chair / ComfiMotion Care / LUX Bed:
14	GSP510424	Charger Socket - for Mains Power Cord / Transformer / Plug - for Aspire ALTITUDE Vertical / Single Action Recliner Chair
15	GSP510235	Control Unit - Aspire ALTITUDE Vertical Lift Chair * AFTER SN NUMBER : AVL-PO569933-0001
15	GSP510236	Control Unit - OLD - Aspire ALTITUDE Vertical Lift Chair * PRIOR SN NUMBER : AVL-PO569933-0001
16	GSP510115	Extension Cable - Battery to Control Box - Aspire ALTITUDE Vertical Recliner Chair
16	GSP510422	Extension Cable - for Control Box to Transformer - Aspire ALTITUDE Vertical Recliner Chair
17	GSP510398	Handset - Aspire ALTITUDE Single Action Recliner
17	GSP510395	Handset - Aspire ALTITUDE Vertical Lift Chair * AFTER SN NUMBER : AVL-PO569933-0001
17	GSP510396	Handset - OLD- Aspire ALTITUDE Vertical Lift Chair * PRIOR SN NUMBER : AVL-PO569933-0001
18	GSP510423	Mains Power Pack / Transformer / Plug - Aspire ALTITUDE Vertical / Single Action Recliner Chair
18.1	GSP510421	Cable - for Handset - Aspire ALTITUDE Lift Recline Chair