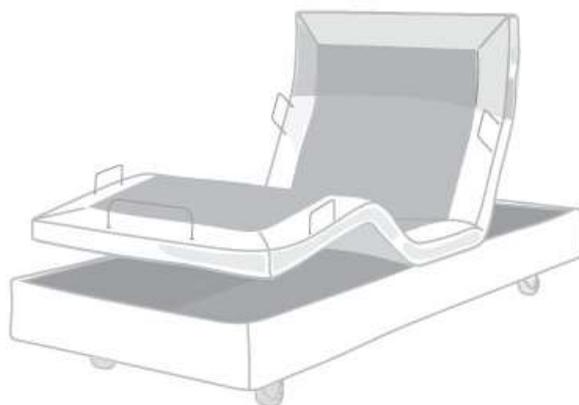



# Aspire ComfiMotion Activ Care

## Spare Parts Manual



This Manual Contains Spare Parts Information for The Following Products:

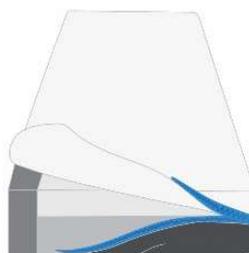Bed Collection	
Product Code	Product Description
BEB046740	Bed - Aspire - ComfiMotion ACTIV Care Bed - Portobello Grey - Long Single (Headboard Sep) - 250kg
BEB046750	Bed - Aspire - ComfiMotion Low ACTIV Care Bed - Portobello Grey - King Single (Headboard Sep) - 250kg
BEB046760	Bed - Aspire - ComfiMotion Activ Care Bed - Portobello Grey - Long Double (Headboard Sep) - 360kg
BEB046770	Bed - Aspire - ComfiMotion Activ Care Bed - Portobello Grey - Queen (Headboard Sep) - 360kg




Mattress Collection		
Product Code	Product Description	Size
BEM469800	Pocket Spring Mattress	Long Single
BEM469810	Pocket Spring Mattress	King Single
BEM469820	Pocket Spring Mattress	Double
BEM469830	Pocket Spring Mattress	Queen
BEM469910	Memory Foam Mattress – Classic	Long Single
BEM469920	Memory Foam Mattress – Classic	King Single
BEM469922	Memory Foam Mattress – Classic	Double
BEM469924	Memory Foam Mattress – Classic	Queen
BEM469930	Memory Foam Mattress – Incontinence	Long Single
BEM469940	Memory Foam Mattress – Incontinence	King Single
BEM469942	Memory Foam Mattress – Incontinence	Double
BEM469944	Memory Foam Mattress – Incontinence	Queen

## Contents

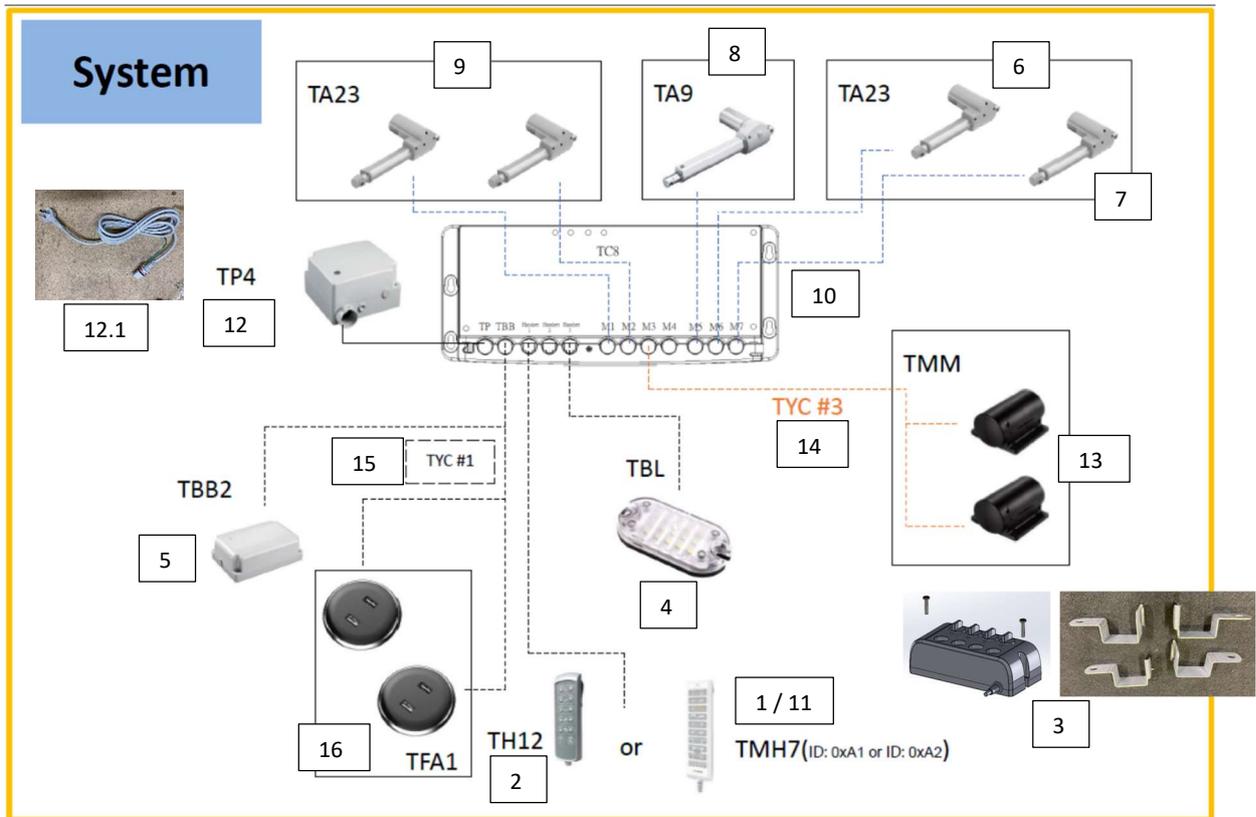
Electrical Schematics –.....	3
Exploded Diagram .....	5
Mattress Spares ONLY.....	12
Headboard Collection .....	12
Bed Accessories.....	12

Electrical Schematics –


For all Long Single and King Single Beds with S/N prior to **CMAS-PO796260-00000** or **CMAKS-PO796260-00000**



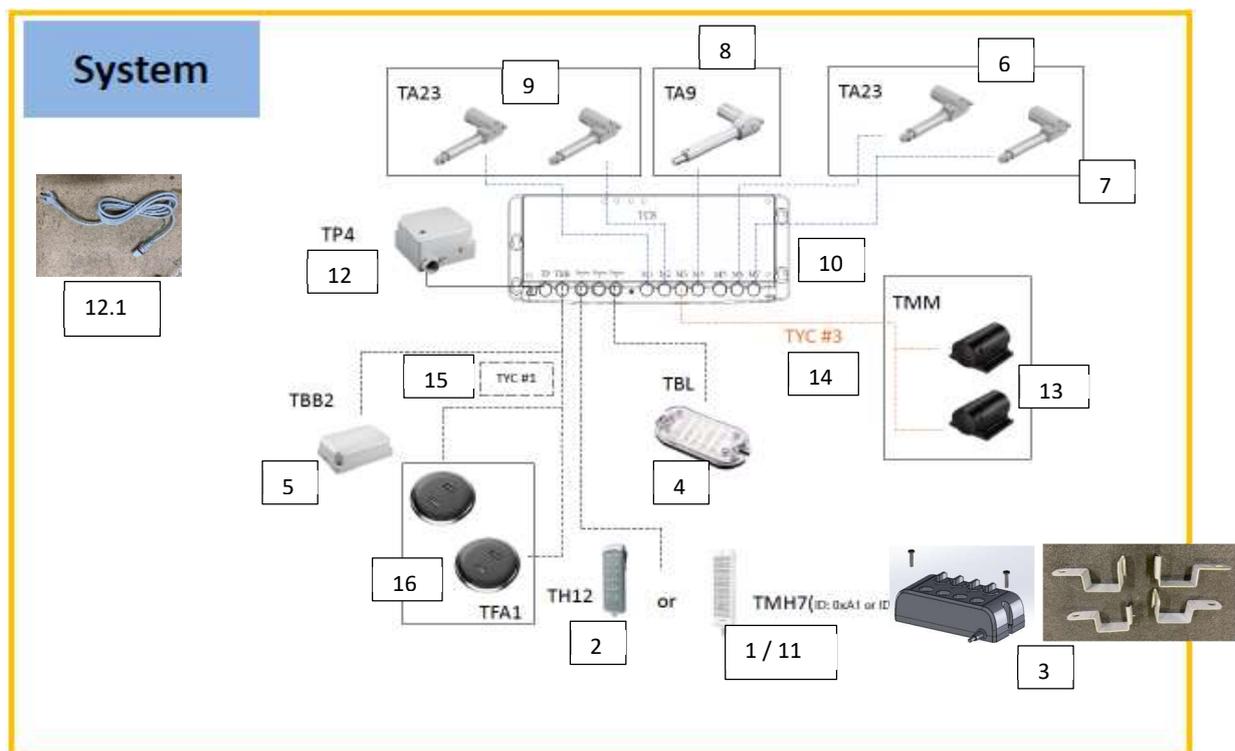For all Long Single and King Single Beds with S/N including and after to **CMAS-PO796260-00000** or **CMAKS-PO796260-00000**






For all Long Double and Queen Beds


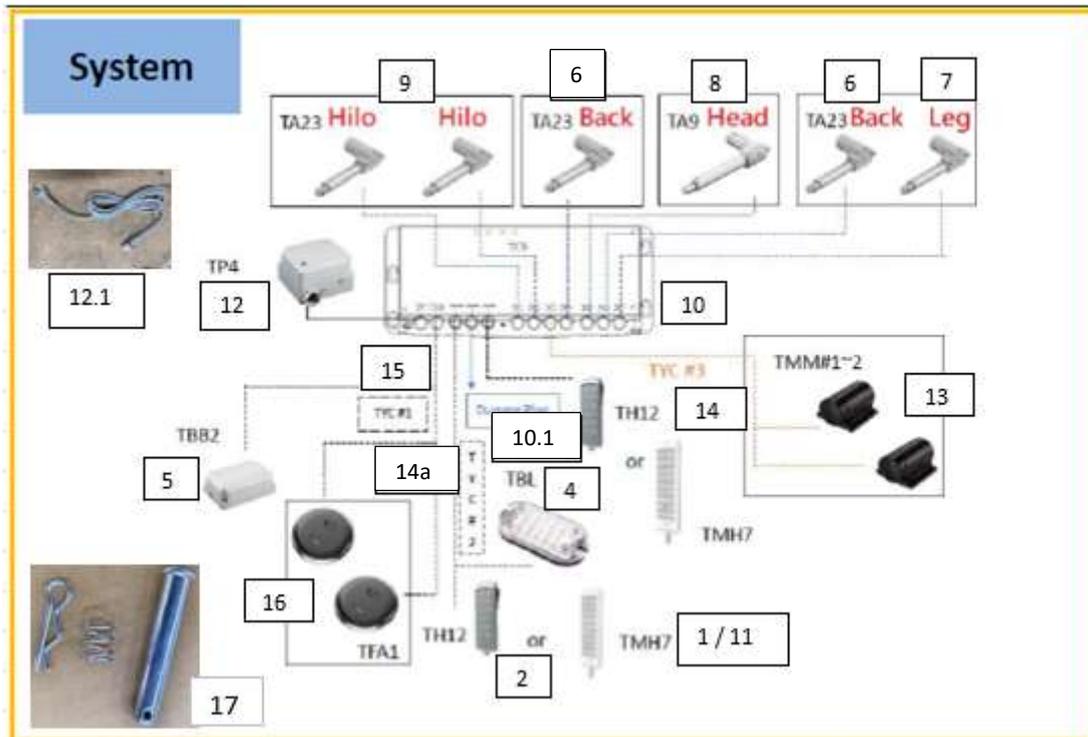





No.	Code	Description
1	BEA830213	Handset - Partner
2	BEA830214	Handset -Simplified
3	BEA830230	Split King Upgrade Kit
4	BEB046570	Under Bed Light & Hardware
5	BEB046590	Battery Backup
6	GSP640150	Actuator - Backrest - King Single / Long Single
6	GSP640151	Actuator - Backrest - Double / Queen
7	GSP640155	Actuator - Foot End
8	GSP640160	Actuator - Headrest
9	GSP640165	Actuator - Hi / Lo - King Single / Long Single
9	GSP640166	Actuator - Hi / Lo - Double / Queen
10	GSP640205	Control Box
10.1	GSP640206	Dummy Plug - for Control Box - Double / Queen
11	GSP640220	Handset - Standard
12	GSP640250	Power Supply
12.1	GSP640252	Mains Power Cable
13	GSP640255	Massage Motor Assembly - Head / Foot
14	GSP640345	Splitter Cable - 2 Channels – for Massage Function
14a	GSP640347	Splitter Cable – 2 Channel – for Handset & under bed light
15	GSP640350	Splitter Cable - 3 Channels – for Battery backup & USB
16	GSP640355	USB Port
17	GSP640356	Hardware Kit - for Actuators - 1 Pin, 1 Spring & 1 R-Clip - Aspire ComfiMotion Active Care Bed


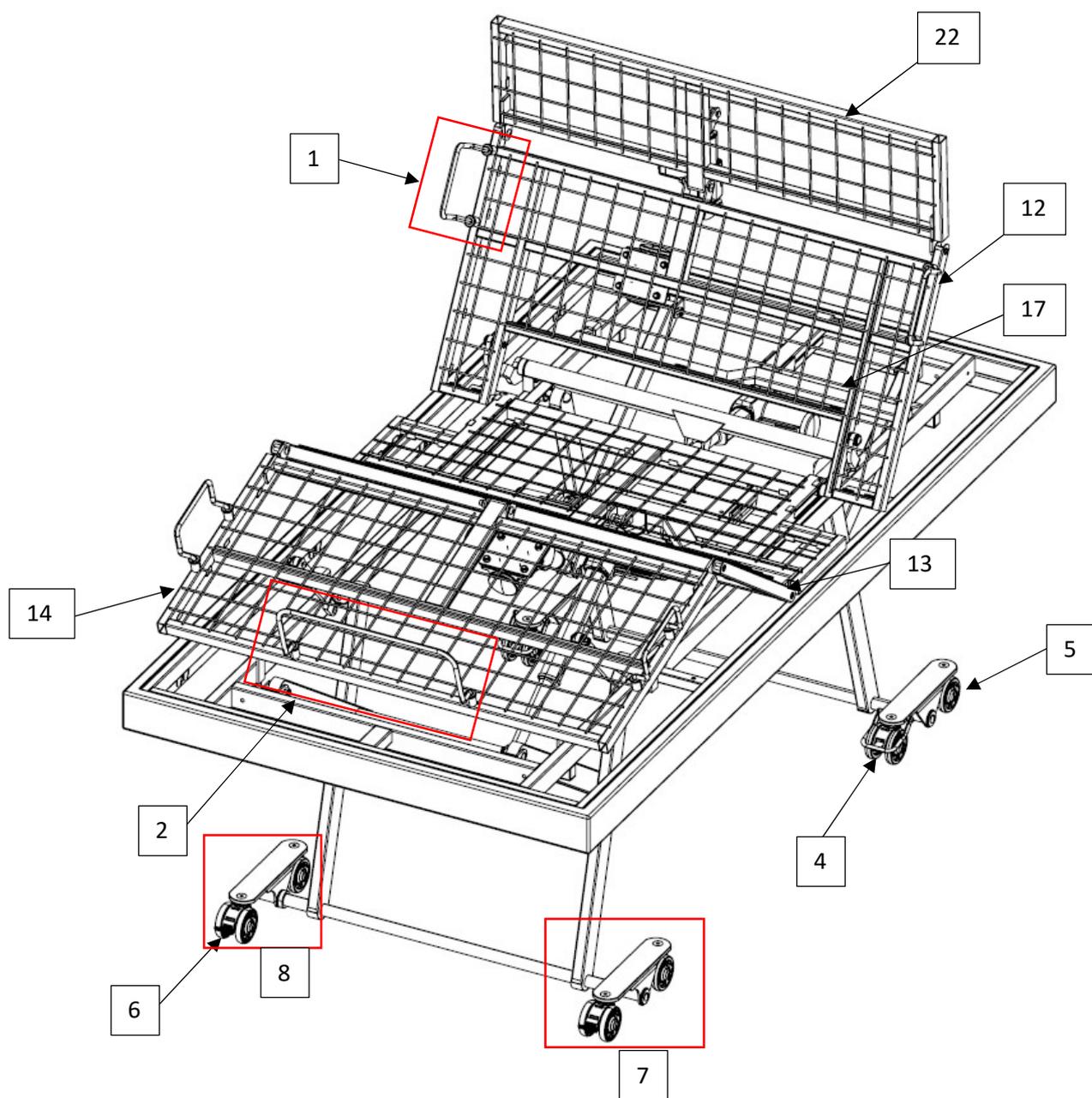
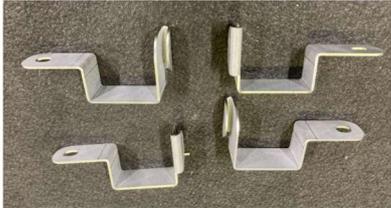
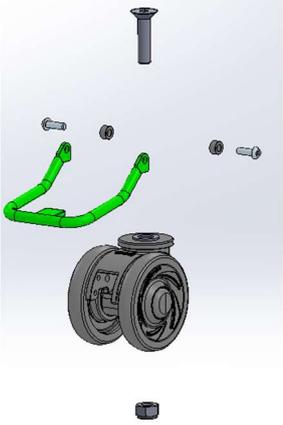
Exploded Diagram







No.	Code	Description
1	GSP639890	MattressRetainer-Sides-AspireComfiMotionBeds-Setof4
		
2	GSP639885	Mattress Retainer - Foot End - Aspire ComfiMotion Beds / Activ Care - <b>Single</b>
2	GSP639886	Mattress Retainer - Foot End - Aspire Activ Care Bed - <b>Double / Queen</b>
		
3	GSP640170	Connection Brackets - for Split King Upgrade Kit – for Long Single / King Single
		
4	GSP640180	Castor - Directional Lock
		
5	GSP640185	Castor - Swivel

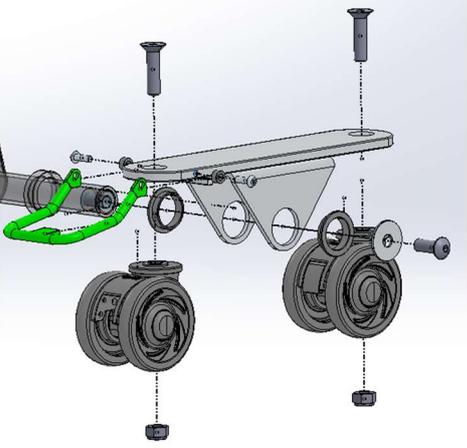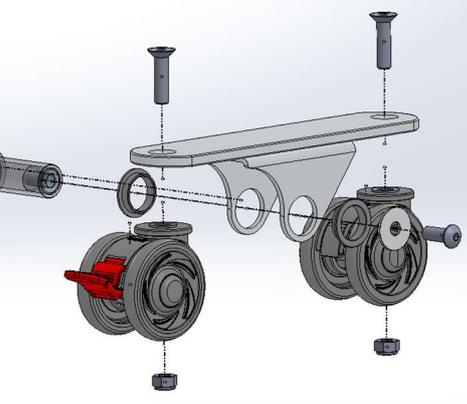			
6	GSP640190	Castor - Total Lock	
			
7	GSP640195	Castor Mounting Device and Hardware - Directional Lock - Foot End - Left / Right	
			
8	GSP640200	Castor Mounting Device and Hardware - Total Lock - Head End - Left / Right	
			
9.1	GSP640230	Leg Frame - Actuator Support - King Single	

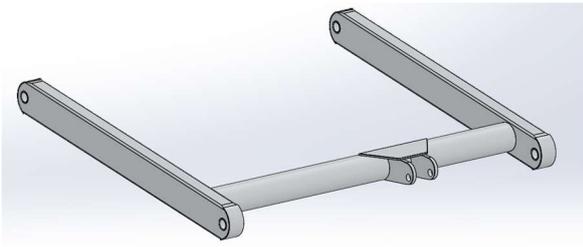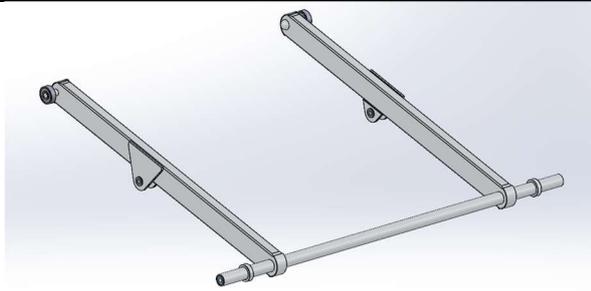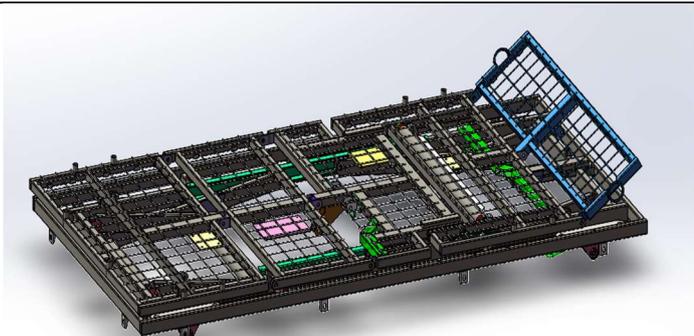9.2	GSP640235	Leg Frame - Actuator Support - Long Single
9.3	GSP640232	Leg Frame - Actuator Support - Double
9.4	GSP640233	Leg Frame - Actuator Support - Queen
		
10.1	GSP640240	Leg Frame - Roller and Castor Support - King Single
10.2	GSP640245	Leg Frame - Roller and Castor Support - Long Single
10.3	GSP640241	Leg Frame - Roller and Castor Support - Double
10.4	GSP640242	Leg Frame - Roller and Castor Support - Queen
		
11	GSP640260	Mounting Hardware Bracket - for Massage Motor
		
22.1	GSP640261	Mattress Platform - Headrest Section - King Single
22.2	GSP640262	Mattress Platform - Headrest Section - Double
22.3	GSP640263	Mattress Platform - Headrest Section - Long Single
22.4	GSP640264	Mattress Platform - Headrest Section - Queen
		
12.1	GSP640265	Mattress Platform - Backrest Section - King Single

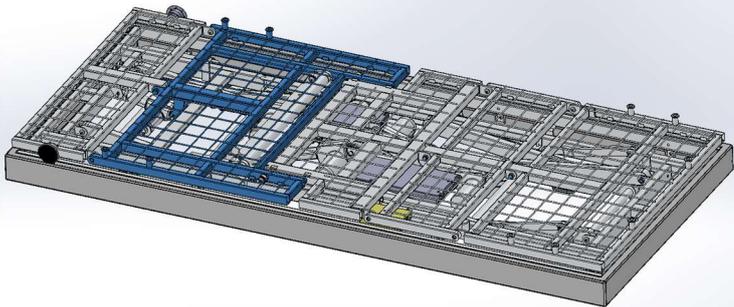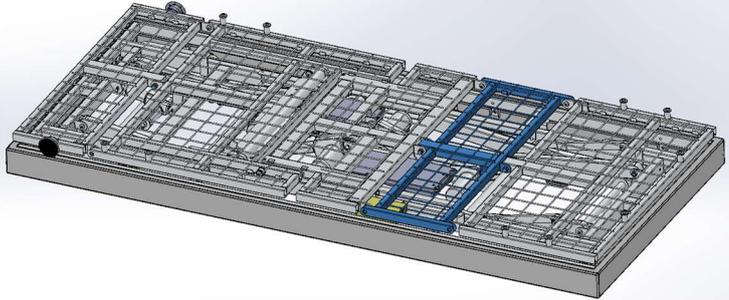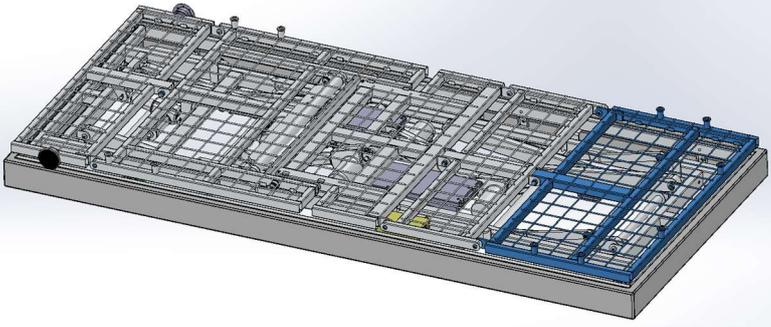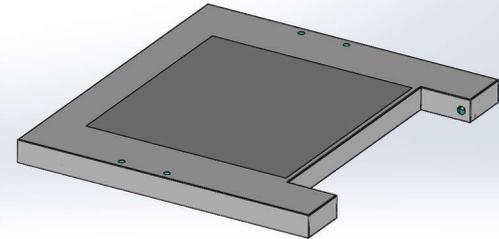12.2	GSP640270	Mattress Platform - Backrest Section - Long Single
12.3	GSP640266	Mattress Platform - Backrest Section - Double
12.4	GSP640271	Mattress Platform - Backrest Section - Queen
		
13.1	GSP640275	Mattress Platform - Knee Section - King Single
13.2	GSP640280	Mattress Platform - Knee Section - Long Single
13.3	GSP640276	Mattress Platform - Knee Section - Double
13.4	GSP640281	Mattress Platform - Knee Section - Queen
		
14.1	GSP640285	Mattress Platform - Leg Section - King Single
14.2	GSP640290	Mattress Platform - Leg Section - Long Single
14.3	GSP640288	Mattress Platform - Leg Section - Double
14.4	GSP640287	Mattress Platform - Leg Section - Queen
		
15.1	GSP640295	Mattress Platform Cover - Backrest Section - King Single
15.2	GSP640300	Mattress Platform Cover - Backrest Section - Long Single
15.3	GSP640296	Mattress Platform Cover - Backrest Section - Double
15.4	GSP640301	Mattress Platform Cover - Backrest Section - Queen
		

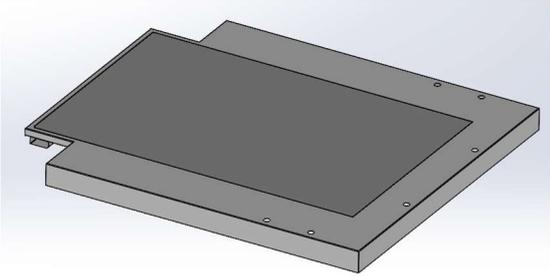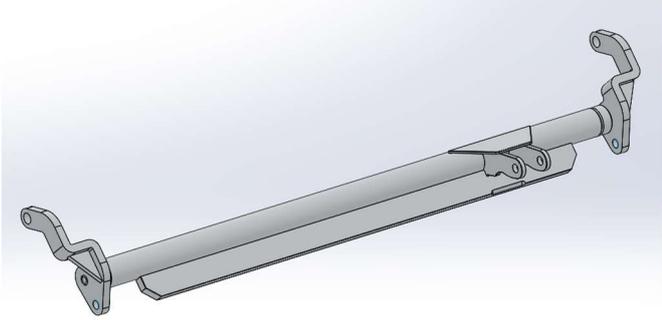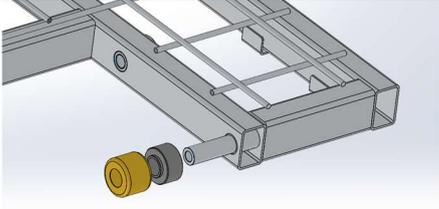16.1	GSP640305	Mattress Platform Cover - Foot End Section - King Single
16.2	GSP640310	Mattress Platform Cover - Foot End Section - Long Single
16.3	GSP640302	Mattress Platform Cover - Foot End Section - Double
16.4	GSP640311	Mattress Platform Cover - Foot End Section - Queen
		
17.1	GSP640315	Pivot Bar - Backrest - King Single
17.2	GSP640320	Pivot Bar - Backrest - Long Single
		
18.1	GSP640321	Hinge Bar – Backrest - Queen
18.2	GSP640316	Hinge Bar – Backrest – Double
		*Requires Drawings
19	GSP640325	Replacement Foam Dampener Pad
		
20	GSP640330	Roller Bearing - Auto Regression (Backrest)
		
21	GSP640335	Roller Bearing - for Hi/Lo

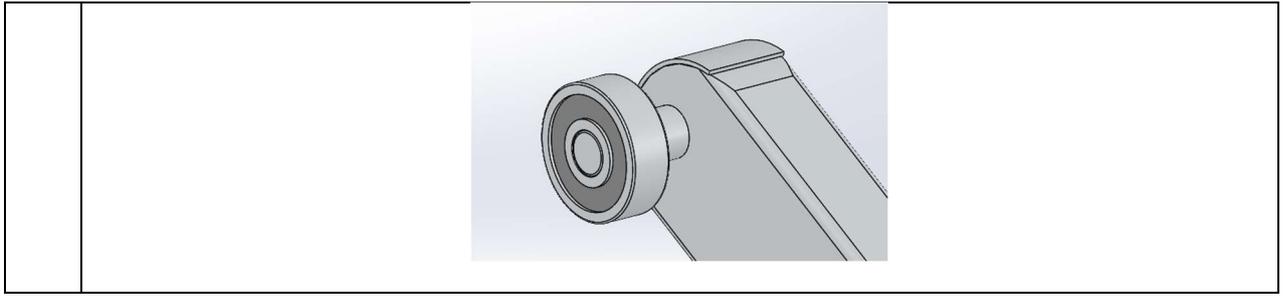

### Mattress Spares ONLY

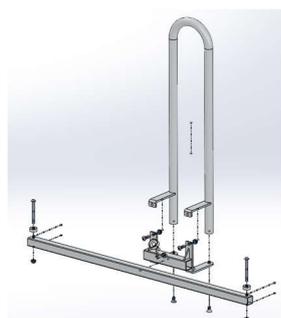
Code	Description	Size
BEM046960	Mattress Cover ONLY - Incontinence Cover	Long Single
BEM046961	Mattress Cover ONLY - Incontinence Cover	King Single
BEM046970	Mattress Cover ONLY - Incontinence Cover	Double
BEM046971	Mattress Cover ONLY - Incontinence Cover	Queen
BEM046962	Mattress Cover ONLY - Classic Cover	Long Single
BEM046963	Mattress Cover ONLY - Classic Cover	King Single
BEM046972	Mattress Cover ONLY - Classic Cover	Double
BEM046973	Mattress Cover ONLY - Classic Cover	Queen

### Headboard Collection


Code	Description	Size
BEA830120	Allure Headboard	Long Single
BEA830130	Allure Headboard	King Single
BEA830132	Allure Headboard	Double
BEA830134	Allure Headboard	Queen
BEA830136	Allure Headboard	King
BEA830100	Chesterfield Headboard	Long Single
BEA830110	Chesterfield Headboard	King Single
BEA830112	Chesterfield Headboard	Double
BEA830114	Chesterfield Headboard	Queen
BEA830116	Chesterfield Headboard	King
GSP640120	Head Board Legs with Hardware – for Headboard	Spare Part

### Bed Accessories

Code	Description	Type
BEA830212	Bed Rail	Accessory
BEA830211	Bed Rail – Crook Handle	Accessory
BEA830200-N	Bed Rail - Crook Handle - NEW DESIGN	Accessory
GSP640360	Mounting Hardware - for Bed Rail BEA830212	Spare Part
GSP640365	Mounting Hardware - for Crook Handle BEA830211	Spare Part



Crook Handle

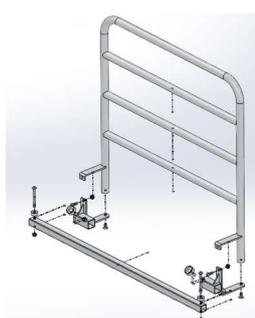

Bed Rail