



aspire

Aspire COVE Chair Spare Parts Manual

Contents

Cove Chair — Product Code	2
A. Cove Chair — Body Frame & Rear Mounts.....	3
B. Cove Chair — Base Frame & Castor Assemblies	4
C. Cove Chair — Gas Strut Assemblies	5
D. Cove Chair — Arm Pads & Wings Assemblies.....	6
E. Cove Chair — Main Body Cushion Assemblies.....	7
F. Cove Chair — Under Cushion Assemblies	8
G. Cove Chair — Outer Cushion & Cover Assemblies	9
H. Cove Chair — Accessories	10

Product Codes:


Code	Description
CHP198978	Chair Pressure Relief - Aspire COVE Chair - Teal Vinyl - Medium - 160kg
CHP198979	Chair Pressure Relief - Aspire COVE Chair - Teal Vinyl - Large - 160kg

Discontinued Chairs:



Code	Description
CHP198965	Aspire COVE Chair – Navy / Bronx Steel – Medium – 160kg [Discontinued]
CHP198975	Aspire COVE Chair – Navy / Bronx Steel – Large – 160kg [Discontinued]
CHP198960	Aspire COVE Chair – Grey / Andreas Pumice – Medium – 160kg [Discontinued]
CHP198970	Aspire COVE Chair – Grey / Andreas Pumice – Large – 160kg [Discontinued]
CHP198976	Chair Pressure Relief - Aspire COVE Chair - Grey Vinyl - Medium - 160kg
CHP198977	Chair Pressure Relief - Aspire COVE Chair - Grey Vinyl - Large - 160kg

A. Cove Chair — Body Frame & Rear Mounts

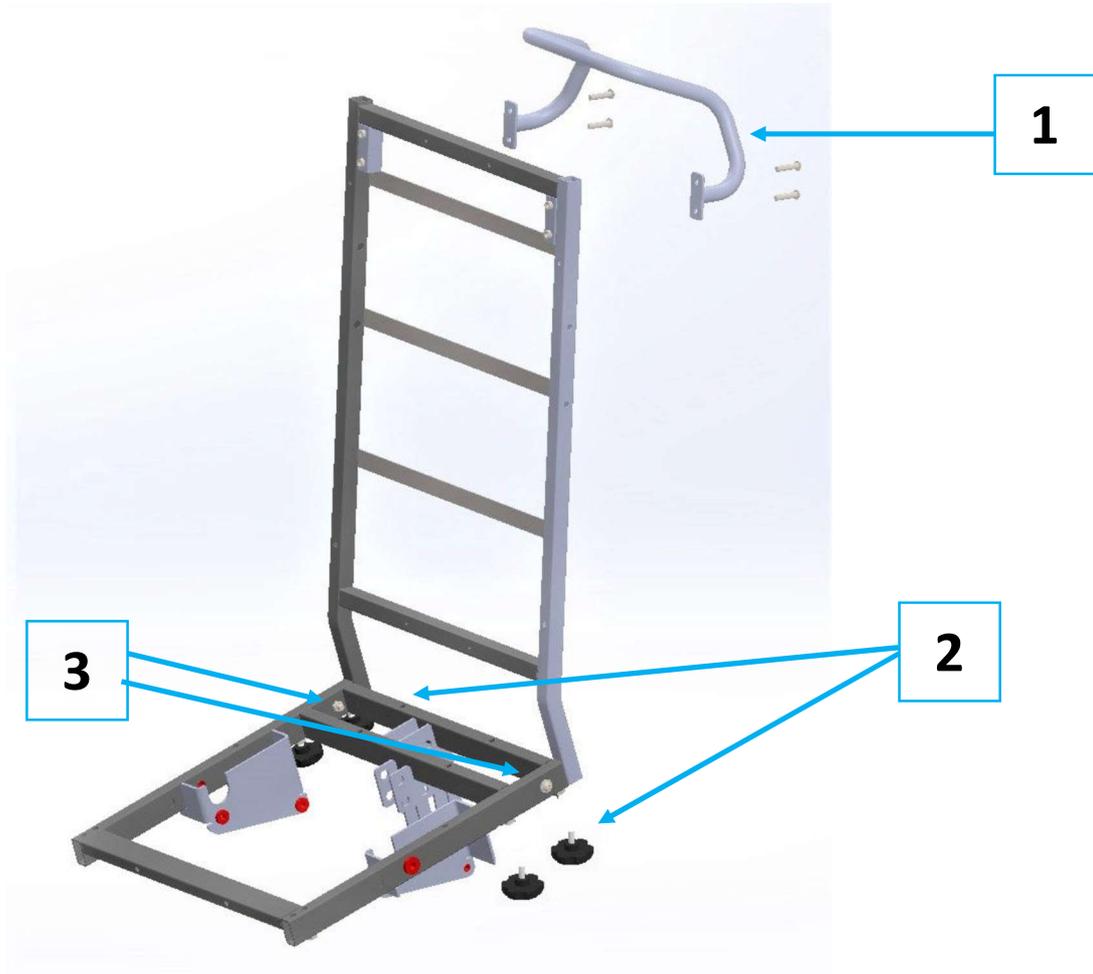

No.	Product Code	Product Description
A	GSP403000	Backrest Frame – Medium
A	GSP403005	Backrest Frame – Large
1	GSP403332	Handle Push Bar—Medium
1	GSP403333	Handle Push Bar—Large
2	GSP400840	Handwheel Bolt – for Depth Adjustment
3	GSP403320	Retention Screw – for Depth Adjustment

B. Cove Chair — Base Frame & Castor Assemblies




No.	Product Code	Product Description
B	GSP403235	Base Frame – Medium
B	GSP403240	Base Frame – Large
1	GSP403365	Castor Wheel — Total Locking
2	GSP403370	Castor Wheel — Directional Locking




1	GSP400797	Complete Leg Linkage / Scissor - w/Mounting Hardware - Medium
1	GSP400798	Complete Leg Linkage / Scissor - w/Mounting Hardware - Large

C. Cove Chair — Gas Strut Assemblies

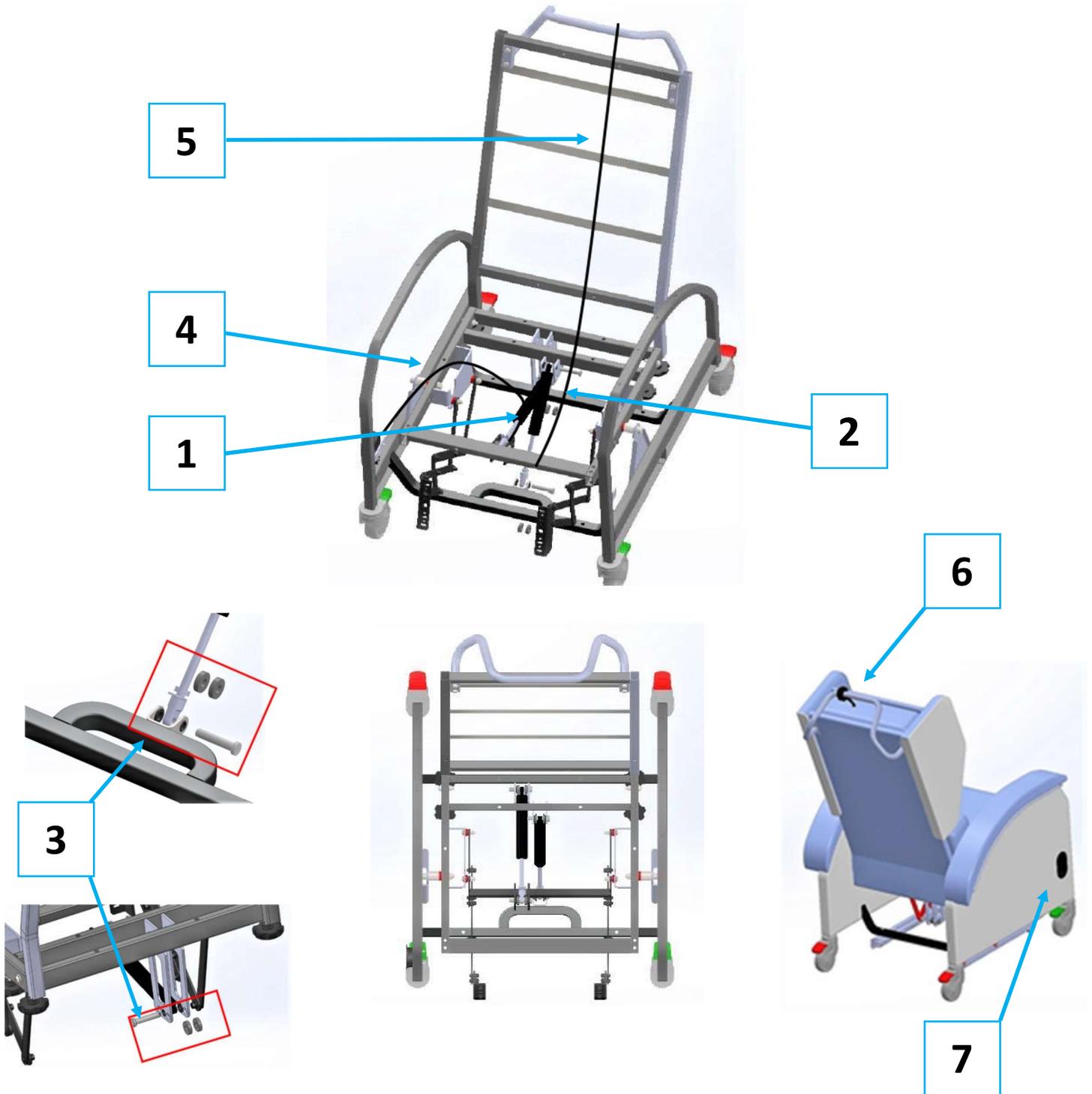


No.	Product Code	Product Description
1	GSP403345	Leg Rest Gas Strut – 280N
2	GSP403330	Tilt Gas Strut – 400N – Medium
2	GSP403325	Tilt Gas Strut – 600N – Large
3	GSP400821	Lock/Release Assembly for Gas Strut
4	GSP400800	Gas Strut Cable - Leg Rest
5	GSP403340	Gas Strut Cable - Tilt
6	GSP400790	Release Lever—Tilt
7	GSP400795	Release Lever—Leg Rest

D. Cove Chair — Arm Pads & Wings Assemblies

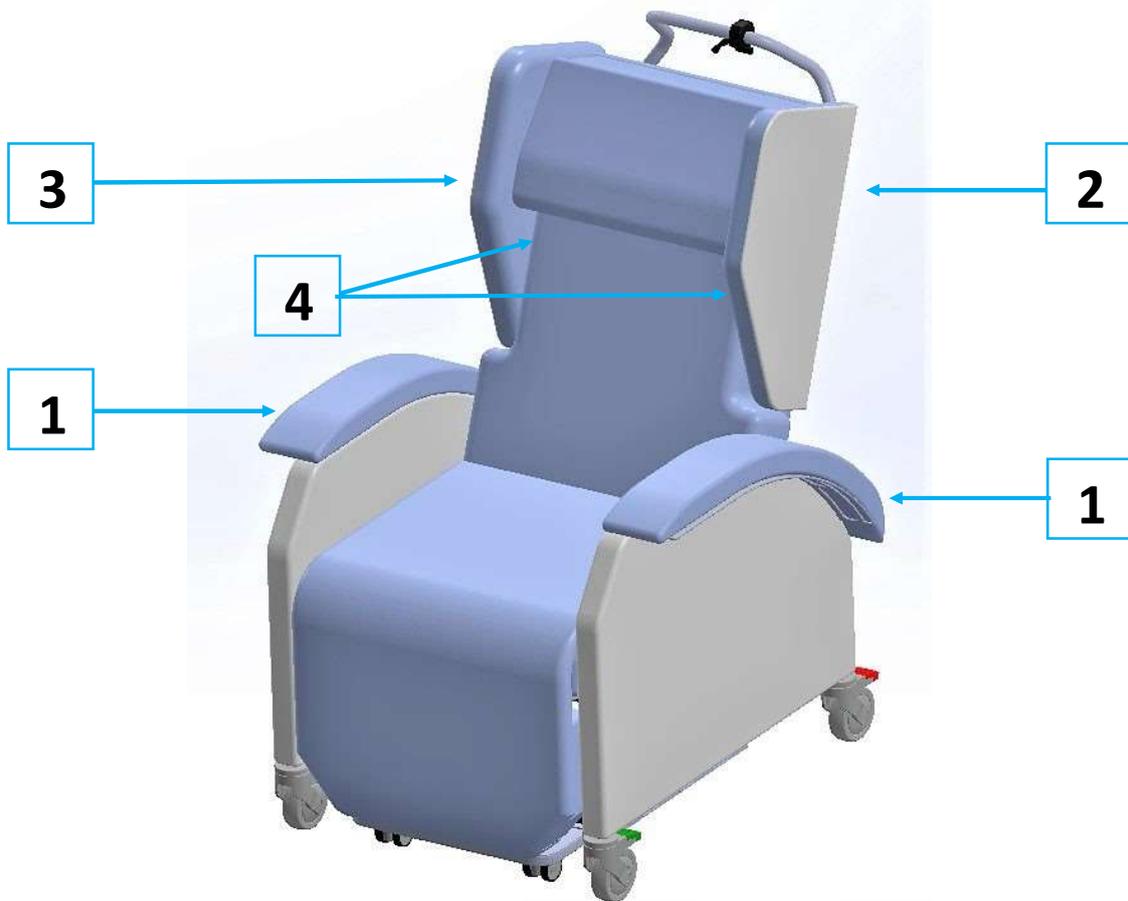


No.	Product Code	Product Description
1	GSP403265	Arm Pad Assembly – Grey
1	GSP403270	Arm Pad Assembly – Navy
2	GSP403275	Wing Cushion Assembly – Grey Fabric – Left – Large
2	GSP403276	Wing Cushion Assembly – Grey Vinyl - Left - Large
2	GSP403280	Wing Cushion Assembly – Navy – Left – Large
2	GSP403295	Wing Cushion Assembly – Grey Fabric – Left – Medium
2	GSP403296	Wing Cushion Assembly - Grey Vinyl - Left - Medium
2	GSP403300	Wing Cushion Assembly – Navy – Left – Medium
3	GSP403285	Wing Cushion Assembly – Grey Fabric – Right – Large
3	GSP403286	Wing Cushion Assembly - Grey Vinyl - Right - Large
3	GSP403290	Wing Cushion Assembly – Navy – Right – Large
3	GSP403305	Wing Cushion Assembly – Grey Fabric – Right – Medium
3	GSP403306	Wing Cushion Assembly – Grey Vinyl - Right - Medium
3	GSP403310	Wing Cushion Assembly – Navy – Right – Medium
3	GSP403277	Wing Cushion Assembly - Teal Vinyl - Left - Large - Aspire COVE Chair
3	GSP403287	Wing Cushion Assembly - Teal Vinyl - Right - Large - Aspire COVE Chair
3	GSP403297	Wing Cushion Assembly - Teal Vinyl - Left - Medium - Aspire COVE Chair
3	GSP403307	Wing Cushion Assembly - Teal Vinyl - Right - Medium - Aspire COVE Chair
4	GSP403312	Mounting Bracket Assembly – for Wing Cushion

E. Cove Chair — Main Body Cushion Assemblies

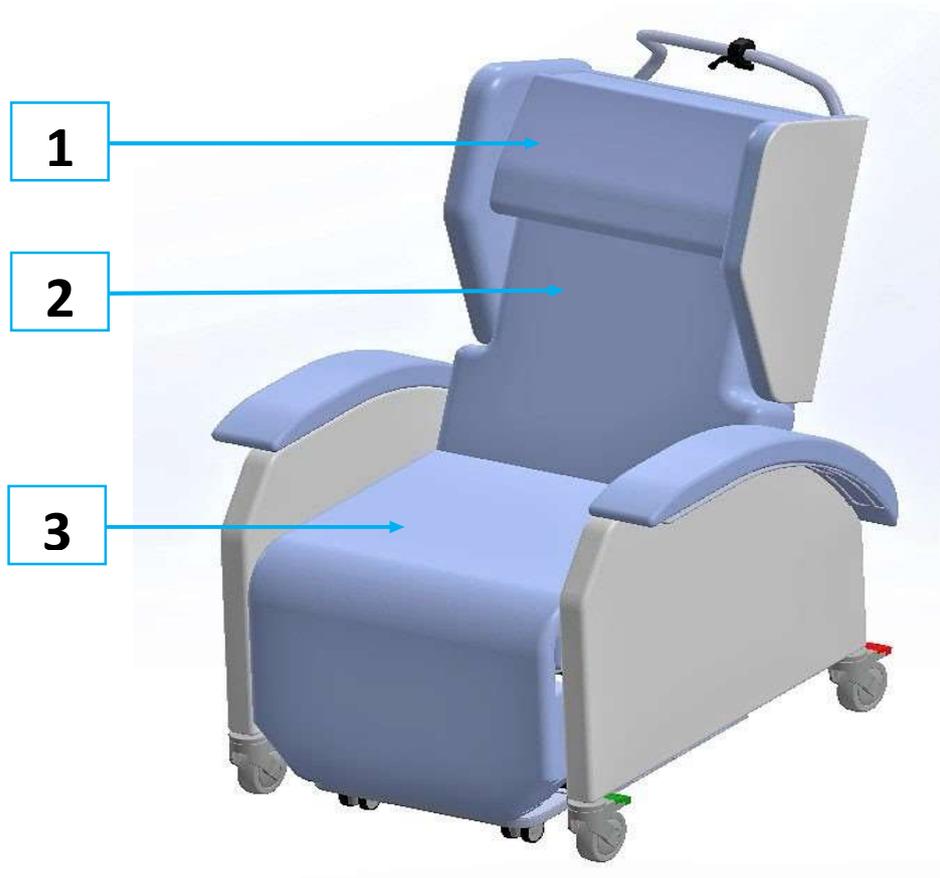

No.	Product Code	Product Description
1	GSP403130	Headrest Cushion – Grey - Large
1	GSP403135	Headrest Cushion – Navy - Large
1	GSP403140	Headrest Cushion – Grey – Medium
1	GSP403145	Headrest Cushion – Navy – Medium
2	GSP403030	Backrest Cushion – Grey – Large
2	GSP403035	Backrest Cushion – Navy – Large
2	GSP403040	Backrest Cushion – Grey – Medium
2	GSP403045	Backrest Cushion – Navy – Medium
3	GSP403070	Overlay Cushion – Grey – Large
3	GSP403075	Overlay Cushion – Navy – Large
3	GSP403080	Overlay Cushion – Grey – Medium
3	GSP403085	Overlay Cushion – Navy – Medium

F. Cove Chair — Under Cushion Assemblies

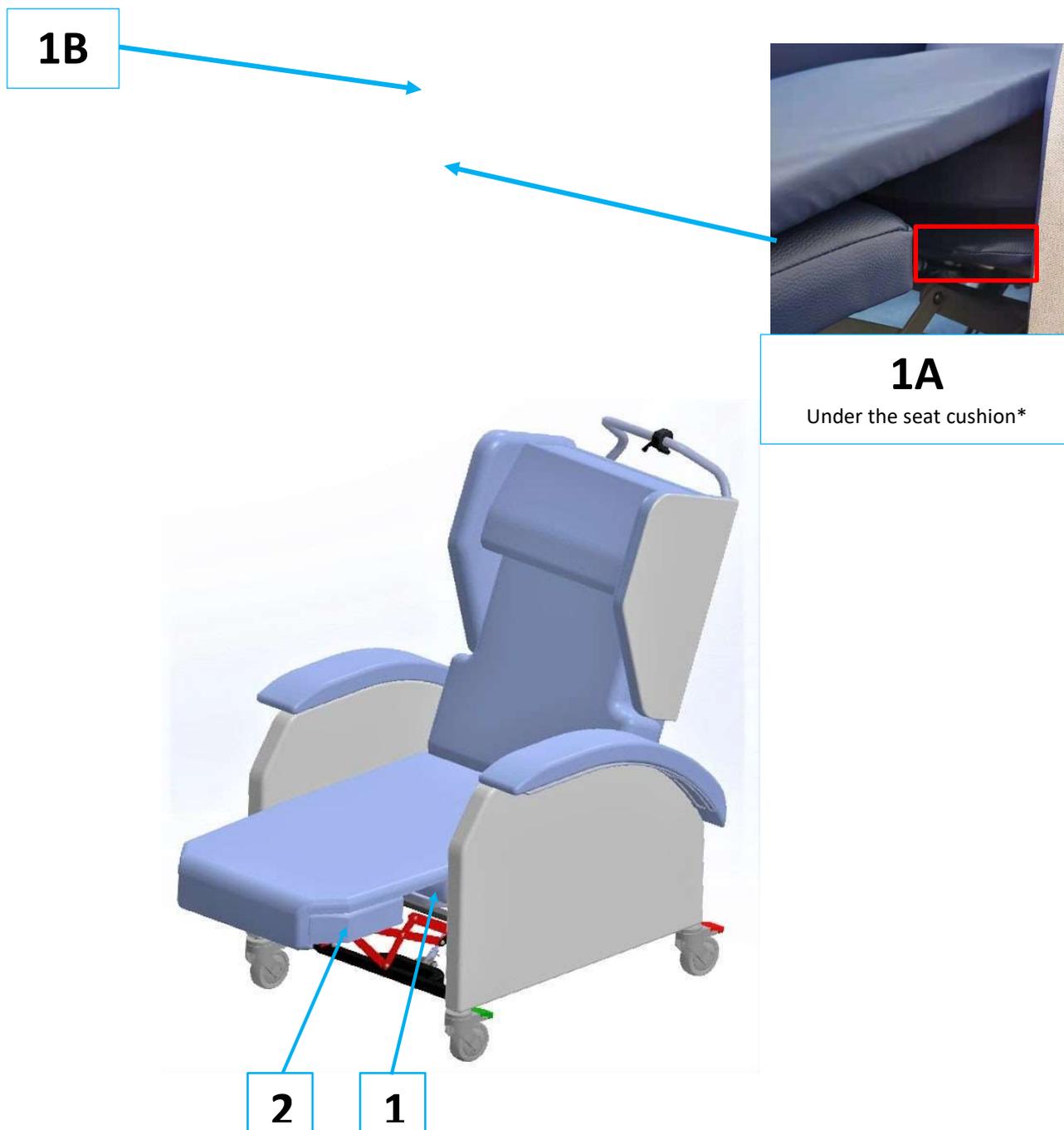


No.	Product Code	Product Description
-----	--------------	---------------------

F. Cove Chair — Under Cushion Assemblies


1	GSP403150	Seat Cushion – Grey – Large
1	GSP403155	Seat Cushion – Navy – Large
1	GSP403160	Seat Cushion – Grey – Medium
1	GSP403165	Seat Cushion – Navy – Medium
1	GSP403175	Cover – Seat Cushion – Grey – Large
1	GSP403185	Cover – Seat Cushion – Navy – Large
1	GSP403180	Cover – Seat Cushion – Grey – Medium
1	GSP403190	Cover – Seat Cushion – Navy – Medium
1A	GSP403215	Cover Upholstery – Seat Frame - Grey - Large
1A	GSP403220	Cover Upholstery – Seat Frame - Grey - Medium
1A	GSP403225	Cover Upholstery – Seat Frame - Navy - Large
1A	GSP403230	Cover Upholstery – Seat Frame - Navy - Medium
1B	GSP403170	Air Bag for Seat Cushion (*Inside Cushion)
2	GSP403195	Leg Rest Pad – Grey – Large
2	GSP403200	Leg Rest Pad – Navy – Large
2	GSP403205	Leg Rest Pad – Grey – Medium
2	GSP403210	Leg Rest Pad – Navy – Medium

G. Cove Chair — Outer Cushion & Cover Assemblies

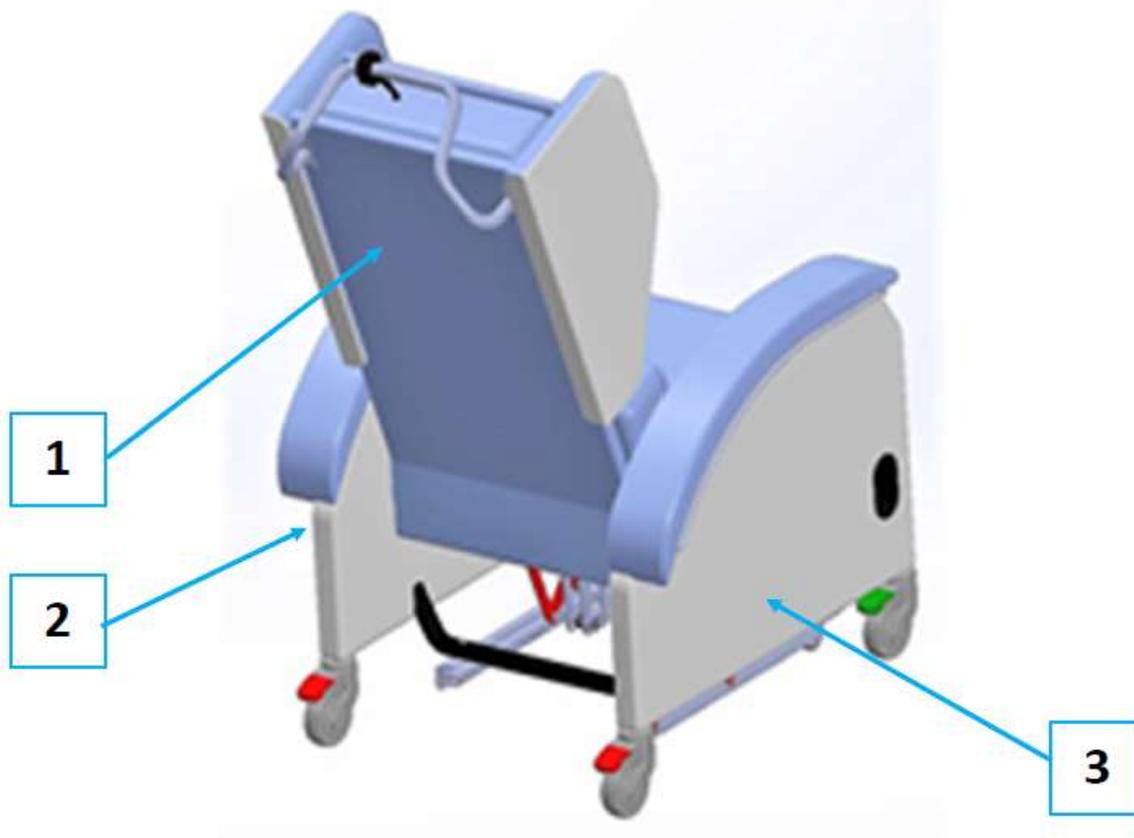

No.	Product Code	Product Description
1	GSP403010	Cover – Backrest Frame – Grey – Large
1	GSP403020	Cover – Backrest Frame – Navy – Large
1	GSP403015	Cover – Backrest Frame – Grey – Medium
1	GSP403025	Cover – Backrest Frame – Navy – Medium
2	GSP403250	Cover – Seat Base Frame – Grey Fabric – Left
2	GSP403251	Cover - Seat Base Frame – Grey Vinyl - Left
2	GSP403260	Cover – Seat Base Frame – Navy – Left
3	GSP403245	Cover – Seat Base Frame – Grey Fabric – Right
3	GSP403246	Cover - Seat Base Frame - Grey Vinyl - Right
3	GSP403255	Cover – Seat Base Frame – Navy – Right
3	GSP403247	Cover - Seat Base Frame - Teal Vinyl - Right - Aspire COVE Chair
3	GSP403252	Cover - Seat Base Frame - Teal Vinyl - Left - Aspire COVE Chair

H. Cove Chair — Accessories

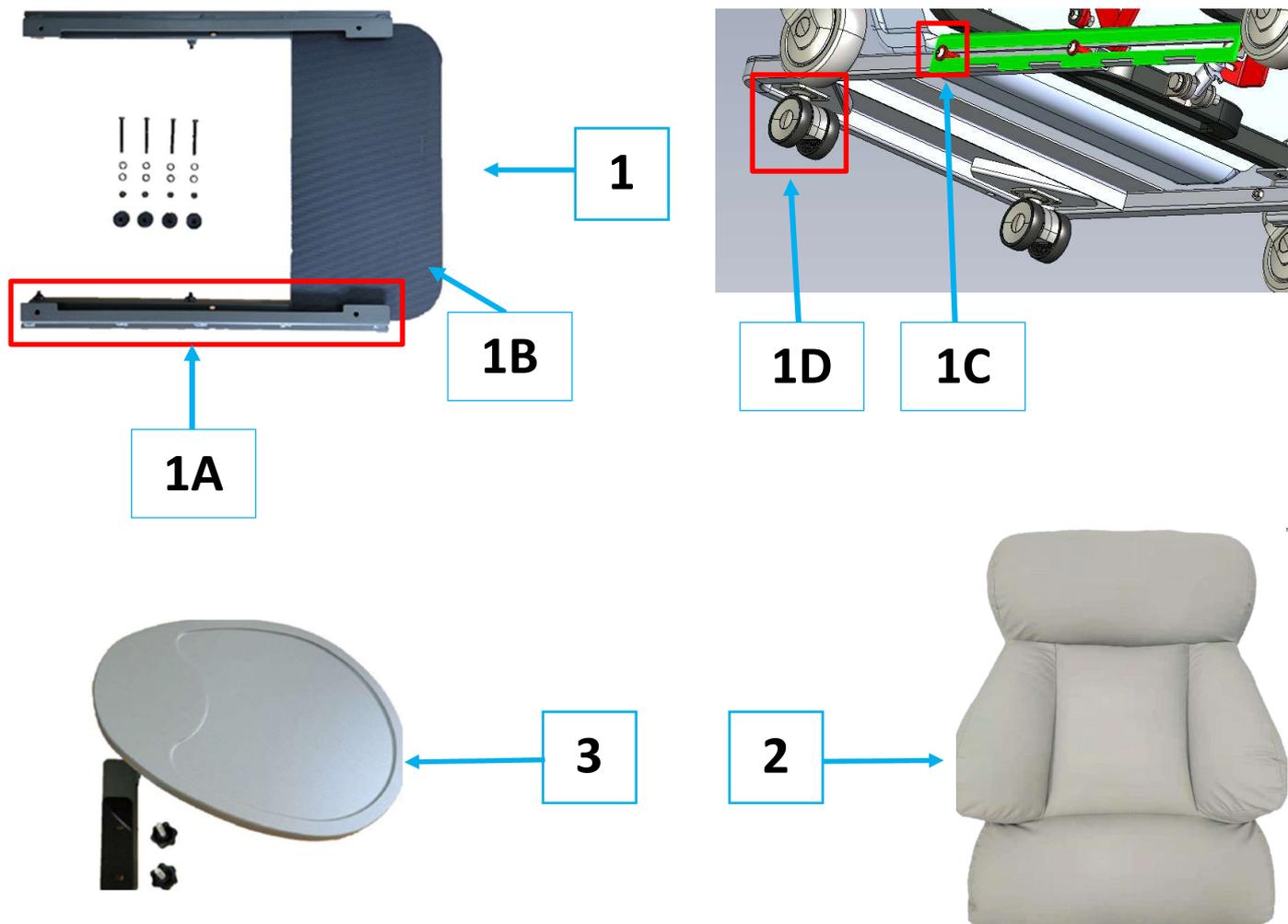

No.	Product Code	Product Description
1	CHP198985	Footrest Assembly
1A	GSP403410	Slider Assembly – Left – for Footrest
1A	GSP403412	Slider Assembly – Right – for Footrest
1B	GSP403415	Foot Pad – for Footrest
1C	GSP403400	Slide Bush – for Footrest
1D	GSP403405	Castor – 50mm – for Footrest
2	CHP198990	Contour Cushion – Grey – Medium
2	CHP198991	Contour Cushion – Navy – Medium
2	CHP198992	Contour Cushion – Grey – Large
2	CHP198993	Contour Cushion – Navy – Large
3	CHP198980	Tray Table Assembly