

Spare Parts Manual

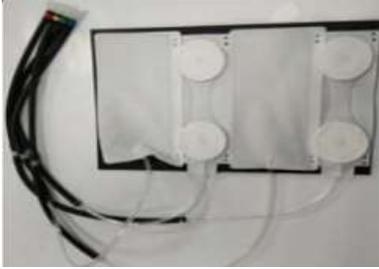
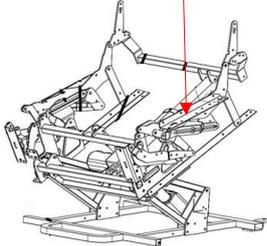
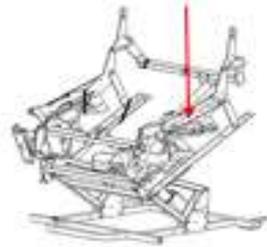
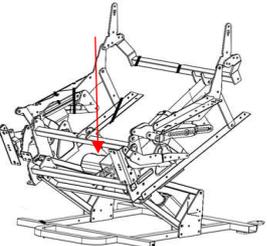


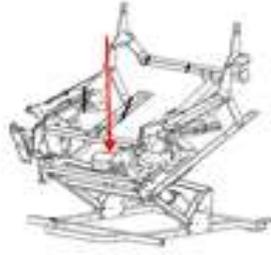
Item Number	Item Description
CHP227590	Aspire Gabriel Quattro Powerlift Recline - Small - Dark Grey - 150kg - (prior SN: 01145568-001)
CHP227595	Aspire Gabriel Quattro Powerlift Recline - Large - Ebony - 150kg - (prior SN: 01145568-001)
CHP227591	Aspire Gabriel X Quattro Powerlift Recline - Small - Dark Grey - 150kg - (after SN: 01145568-001)
CHP227596	Aspire Gabriel X Quattro Powerlift Recline - Large - Ebony - 150kg - (after SN: 01145568-001)

This manual is for internal use only

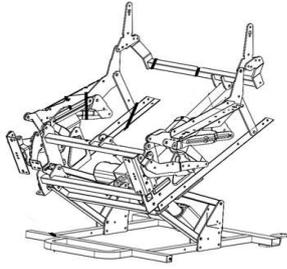
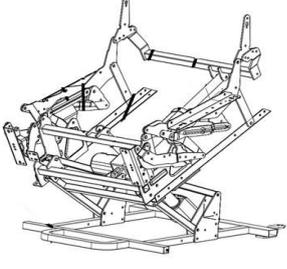
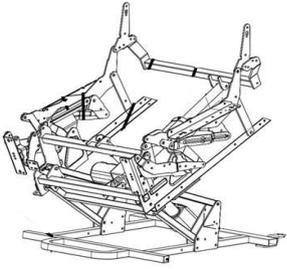
Exploded Diagram:



1 Electronics				
Ref	Item Number	Item Description	Image	Type
1	GSP146900	Actuator - Headrest - Aspire Gabriel Quattro - Recline Chair		Spare Part
2	GSP146902	Control Box - w/Cables and Tubing - Aspire Gabriel Quattro - Recline Chair		Spare Part
2A	GSP146904	Massage Pad - w/tubing - backrest - Aspire Gabriel Quattro - Recline Chair		Spare Part
3	GSP146906	Actuator - Backrest - Aspire Gabriel Quattro - Recline Chair		Spare Part
3	GSP146907	Actuator - Backrest - Aspire Gabriel X - Quattro - Recline Chair		Spare Part
4	GSP146908	Actuator - Footrest / Lift - Aspire Gabriel Quattro - Recline Chair		Spare Part

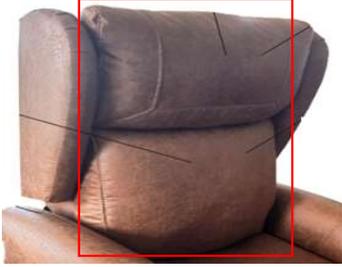
1 Electronics				
4	GSP146909	Actuator - Footrest / Lift - Aspire Gabriel X - Quattro - Recline Chair		Spare Part
4A	GSP146912	Actuator - Tilt - Aspire Gabriel X - Quattro - Recline Chair	Image Coming Soon	Spare Part
5	GSP146910	Mains Power Plug (Cord) - Aspire Gabriel Quattro - Recline Chair		Spare Part
6	GSP400648	Transformer - Aspire Gabriel Quattro - Recline Chair		Spare Part
7	GSP400649	Battery (only) - for Battery Backup - Aspire Gabriel Quattro - Recline Chair		Spare Part
8	GSP146914	Extension Cable - Aspire Gabriel Quattro - Recline Chair		Spare Part

1 Electronics				
9	GSP146916	Handset - Aspire Gabriel Quattro - Recline Chair		Spare Part
10	GSP146918	Cable - from Handset to Control Box - Aspire Gabriel Quattro - Recline Chair		Spare Part
11	GSP146920	Control Panel - Tilt - Aspire Gabriel X - Quattro - Recline Chair		Spare Part

2 Frame				
Ref	Item Number	Item Description	Image	Type
15	GSP146926	Base Frame/Mechanism - Aspire Gabriel Quattro - Recline Chair - Small/Large		Spare Part
15	GSP146938	Base Frame/Mechanism - Aspire Gabriel X - Quattro - Recline Chair - Small		Spare Part
15	GSP146940	Base Frame/Mechanism - Aspire Gabriel X - Quattro - Recline Chair - Large		Spare Part
16	GSP146928	Plastic Foot - Aspire Gabriel Quattro - Recline Chair		Spare Part
17	GSP146930	End Cap - Aspire Gabriel Quattro - Recline Chair		Spare Part
18	GSP146932	Rear Castor - Locking - Aspire Gabriel Quattro - Recline Chair		Spare Part

2 Frame				
19	GSP146934	Front Castor - Non-Locking - Aspire Gabriel Quattro - Recline Chair		Spare Part
20	GSP146936	Spring - for Legrest - Aspire Gabriel Quattro - Recline Chair		Spare Part

4 Seating				
Ref	Item Number	Item Description	Image	Type
25	GSP146948	Headsash - w/Pillow - Aspire Gabriel Quattro - Recline Chair - Dark Grey		Accessory
25	GSP146952	Headsash - Aspire Gabriel Quattro - Recline Chair - Ebony		Accessory
26	GSP146954	Headrest Wing - Left - Aspire Gabriel Quattro - Recline Chair - Small - Dark Grey		Spare Part
26	GSP146956	Headrest Wing - Right - Aspire Gabriel Quattro - Recline Chair - Small - Dark Grey		Spare Part
26	GSP146958	Headrest Wing - Left - Aspire Gabriel Quattro - Recline Chair - Large - Ebony		Spare Part
26	GSP146960	Headrest Wing - Right - Aspire Gabriel Quattro - Recline Chair - Large - Ebony		Spare Part

4 Seating				
27	GSP146962	Backrest - w/Massage Pad/Headrest - Aspire Gabriel Quattro - Recline Chair - Small - Dark Grey		Spare Part
27	GSP146964	Backrest - w/Massage Pad/Headrest - Aspire Gabriel Quattro - Recline Chair - Large - Ebony		Spare Part
28	GSP146966	Sliding/Mounting Bracket - Backrest - Pair - Aspire Gabriel Quattro - Recline Chair		Spare Part
29	GSP146968	Detachable Seat Upholestery - Incl/Heating Eliments, Cables - Aspire Gabriel Quattro - Recline Chair - Small - Dark Grey		Spare Part
29	GSP146970	Detachable Seat Upholestery - Incl/Heating Eliments, Cables - Aspire Gabriel Quattro - Recline Chair - Large - Ebony		Spare Part
30	GSP146972	Armrest Covers - Pair - Aspire Gabriel Quattro - Recline Chair - Small - Dark Grey		Accessory

4 Seating

30	GSP146974	Armrest Covers - Pair - Aspire Gabriel Quattro - Recline Chair - Large - Ebony		Accessory
----	-----------	---	--	-----------