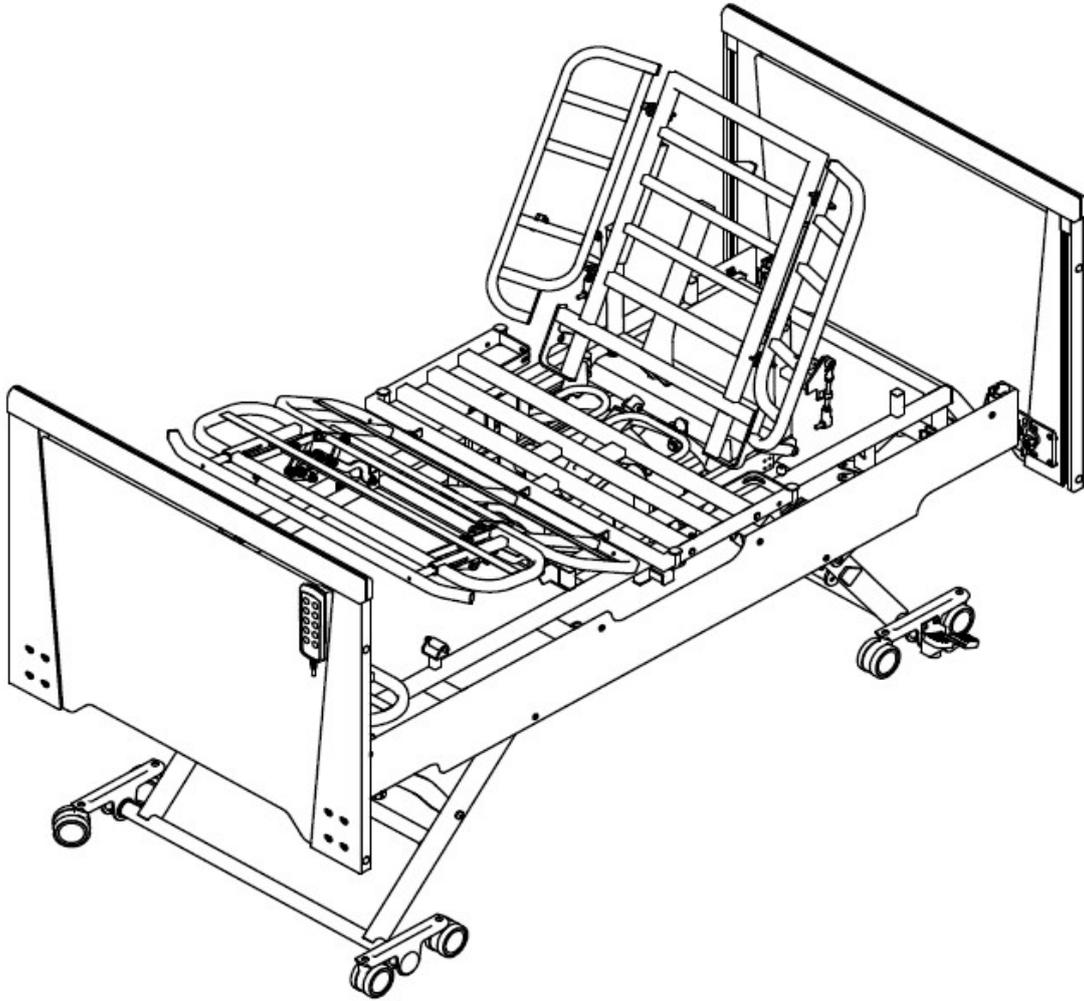


Spare Parts Manual



Item Number	Item Description
BEB046045	Aspire LIFESTYLE Rotating Bed - Single

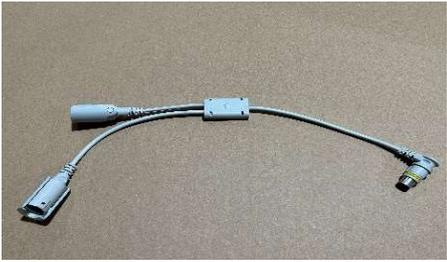
This manual is for internal use only

1 Electronics

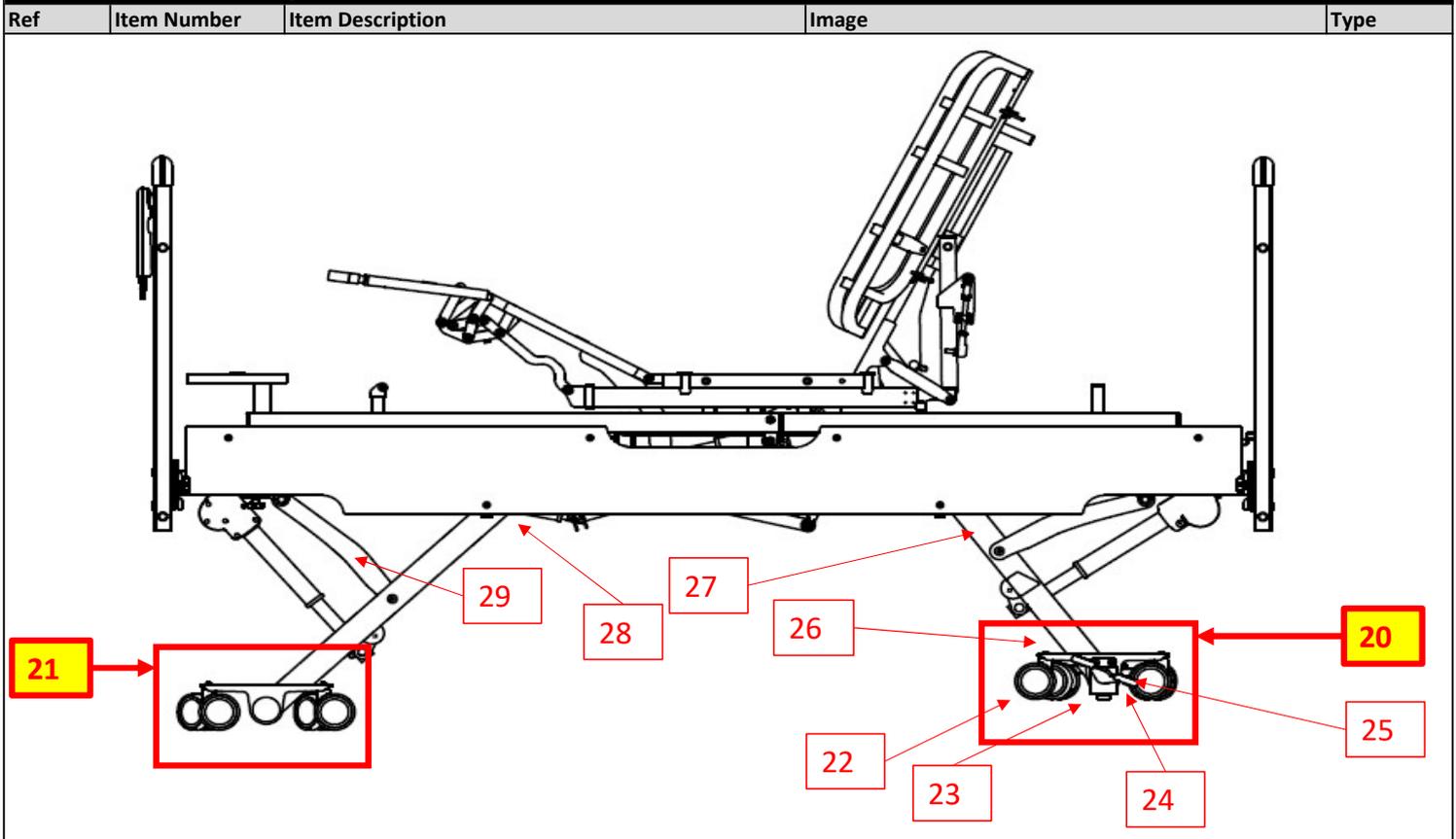
Ref	Item Number	Item Description	Image	Type

Ref	Item Number	Item Description	Image	Type
1	GSP047300	Backrest Actuator - Aspire LIFESTYLE Rotating Bed		Spare Part
2	GSP047302	Footrest / Tilting Actuator - Aspire LIFESTYLE Rotating Bed		Spare Part

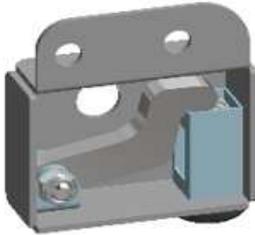
1 Electronics				
Ref	Item Number	Item Description	Image	Type
3 & 4	GSP047304	HILO Actuator - Aspire LIFESTYLE Rotating Bed		Spare Part
5	GSP047306	Cable - Connecting Control Box to Extension Box - Aspire LIFESTYLE Rotating Bed		Spare Part
6	GSP047308	Control Box - Aspire LIFESTYLE Rotating Bed		Spare Part
7	GSP047310	Extension Box - Aspire LIFESTYLE Rotating Bed		Spare Part
8	GSP047312	Rotating Actuator - Aspire LIFESTYLE Rotating Bed		Spare Part

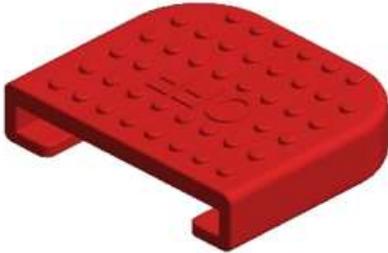
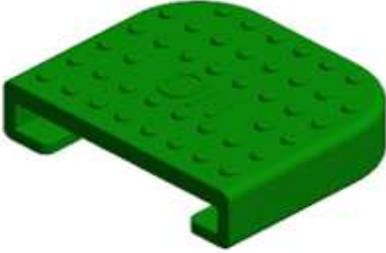
1 Electronics				
Ref	Item Number	Item Description	Image	Type
10	GSP047316	Handset - Aspire LIFESTYLE Rotating Bed		Spare Part
11	GSP047318	Battery Back up - Aspire LIFESTYLE Rotating Bed		Spare Part
12	GSP047320	Y - Cable - Control Box to Extension Box & Battery Back up - Aspire LIFESTYLE Rotating Bed		Spare Part
13	GSP047322	Power Cable - AU Plug - Aspire LIFESTYLE Rotating Bed		Spare Part

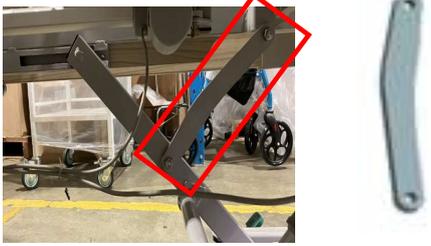
2 Castors & Leg Section



Ref	Item Number	Item Description	Image	Type
20	GSP047334	Castor Mounting Device - Complete - w/Castors, Braking Device, Brake Lever - Head Left - Aspire LIFESTYLE Rotating Bed		Spare Part
20	GSP047336	Castor Mounting Device - Complete - w/Castors, Braking Device, Brake Lever - Head Right - Aspire LIFESTYLE Rotating Bed		Spare Part
21	GSP047338	Castor Mounting Device - Complete - w/Castors - Head Left - Aspire LIFESTYLE Rotating Bed		Spare Part

2 Castors & Leg Section				
Ref	Item Number	Item Description	Image	Type
21	GSP047340	Castor Mounting Device - Complete - w/Castors - Head Right - Aspire LIFESTYLE Rotating Bed		Spare Part
22	GSP047342	Castor - swivel / No Brakes - Aspire LIFESTYLE Rotating Bed		Spare Part
22	GSP047344	Castor - Braking - Aspire LIFESTYLE Rotating Bed		Spare Part
23	GSP047346	Braking Mechanism Box - Complete - Right - Aspire LIFESTYLE Rotating Bed		Spare Part
23	GSP047348	Braking Mechanism Box - Complete - Left - Aspire LIFESTYLE Rotating Bed		Spare Part
24	GSP047350	Brake Pedal - Left - Aspire LIFESTYLE Rotating Bed		Spare Part

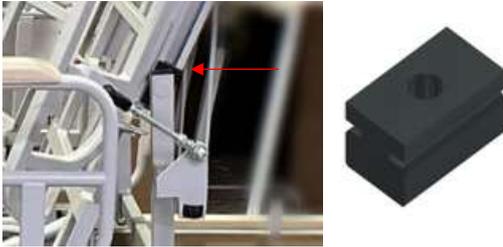
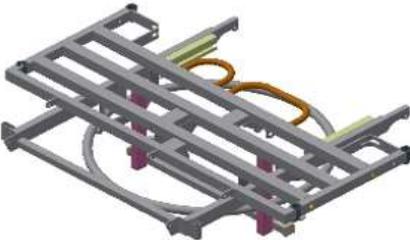
2 Castors & Leg Section				
Ref	Item Number	Item Description	Image	Type
24	GSP047352	Brake Pedal - Right - Aspire LIFESTYLE Rotating Bed		Spare Part
25	GSP047354	Brake Pedal Cap - RED - Aspire LIFESTYLE Rotating Bed		Spare Part
25	GSP047356	Brake Pedal Cap - GREEN - Aspire LIFESTYLE Rotating Bed		Spare Part
26	GSP047358	Hex Bar - Braking - Aspire LIFESTYLE Rotating Bed		Spare Part
27	GSP047360	Leg Frame - Head End - Aspire LIFESTYLE Rotating Bed		Spare Part
27	GSP047362	Leg Frame - Foot End - Aspire LIFESTYLE Rotating Bed		Spare Part

2 Castors & Leg Section				
Ref	Item Number	Item Description	Image	Type
28	GSP047364	Sliding Block - Leg Frame / Backrest - Aspire LIFESTYLE Rotating Bed		Spare Part
29	GSP047366	Cross Bar - Leg Frame - Head / Foot End - w/Hardware - Aspire LIFESTYLE Rotating Bed		Spare Part

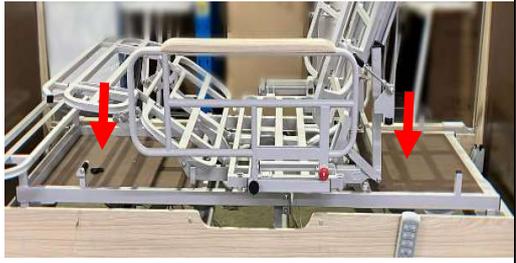
3 Frame & Deck

Ref	Item Number	Item Description	Image	Type

Ref	Item Number	Item Description	Image	Type
40	GSP047374	Frame - Lower Rectangular Frame - Aspire LIFESTYLE Rotating Bed		Spare Part
41	GSP047376	Frame - Upper Rectangular Frame - Aspire LIFESTYLE Rotating Bed		Spare Part
42	GSP047378	Roller - w/hardware - Foot Section Rest - Aspire LIFESTYLE Rotating Bed		Spare Part

3 Frame & Deck				
Ref	Item Number	Item Description	Image	Type
43	GSP047380	Backrest - Mattress Platform - Main Section - Aspire LIFESTYLE Rotating Bed		Spare Part
44	GSP047382	Backrest - Mattress Platform - Left Section - Aspire LIFESTYLE Rotating Bed		Spare Part
44	GSP047384	Backrest - Mattress Platform - Right Section - Aspire LIFESTYLE Rotating Bed		Spare Part
45	GSP047386	Tie Rod - Side Sections - Aspire LIFESTYLE Rotating Bed		Spare Part
46	GSP047364	Sliding Block - Leg Frame / Backrest - Aspire LIFESTYLE Rotating Bed		Spare Part
47	GSP047388	Rotating Frame - Aspire LIFESTYLE Rotating Bed		Spare Part

3 Frame & Deck				
Ref	Item Number	Item Description	Image	Type
48	GSP047390	Thigh Section Frame - Aspire LIFESTYLE Rotating Bed		Spare Part
49	GSP047392	Calf Section Frame - Aspire LIFESTYLE Rotating Bed		Spare Part
50	GSP047394	Extension Piece - Calf Section Frame - Aspire LIFESTYLE Rotating Bed		Spare Part
51	GSP047396	Scissor Mechanism - Foot Section - Pair (Left + Right) - Aspire LIFESTYLE Rotating Bed		Spare Part
52	GSP047398	Btacket - for Tilting Actuator - Aspire LIFESTYLE Rotating Bed		Spare Part
53	GSP047400	Tie Rod - Tilting Mechanism - Aspire LIFESTYLE Rotating Bed		Spare Part

3 Frame & Deck				
Ref	Item Number	Item Description	Image	Type
54	GSP047402	Wood Panels - for Upper Rectangular Frame - Pair (Head+Foot Section) - Aspire LIFESTYLE Rotating Bed		Spare Part

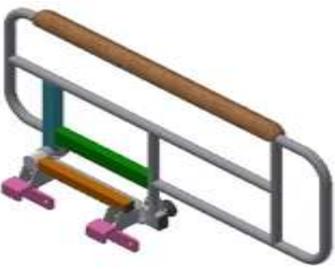
4 Bed Ends & Wooden Side Trim

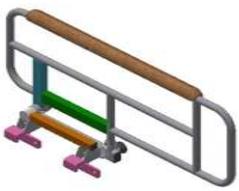
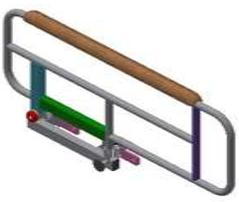
Ref	Item Number	Item Description	Image	Type

Ref	Item Number	Item Description	Image	Type
60	GSP047410	Wooden Side Trim - w/Mounting Hardware - Pair (Left + Right) - Aspire LIFESTYLE Rotating Bed		Spare Part
61	GSP047412	Bed Ends - Head and Foot Boards - Pair - Aspire LIFESTYLE Rotating Bed		Spare Part

4 Bed Ends & Wooden Side Trim				
Ref	Item Number	Item Description	Image	Type
62	GSP047414	Bed End Inserts - Foot End - Pair (Left + Right) - Aspire LIFESTYLE Rotating Bed		Spare Part

5 Side Rails - Standard Inclusions

Ref	Item Number	Item Description	Image	Type
				
				
			<div style="border: 1px solid red; padding: 5px; display: inline-block; color: red;">75 - Folding Side Rails</div>	
			<div style="border: 1px solid red; padding: 5px; display: inline-block; color: red;">76 - Full Length Side Rails</div>	

Ref	Item Number	Item Description	Image	Type
75	GSP047424	Folding Side Rail - Left - Aspire LIFESTYLE Rotating Bed		Spare Part
75	GSP047426	Folding Side Rail - Right - Aspire LIFESTYLE Rotating Bed		Spare Part
76	GSP047428	Full Length Side Rails - Complete - w/Rails + End Caps + Guides - One SET for One Bed - Aspire LIFESTYLE Rotating Bed		Spare Part
76A	GSP047430	End Caps - Complete - w/Guides - for Full Length Side Rails - One SET for One Bed - Aspire LIFESTYLE Rotating Bed		Spare Part