



## Aspire Mobile Air Chair – Spare Parts Manual

Serial number From SN#1512518001




<b><i>Aidacare Code</i></b>	<b><i>Aidacare Description</i></b>
CHP198850	Aspire Mobile Air Chair – Small
CHP198855	Aspire Mobile Air Chair – Large

## 1. Arm Assembly

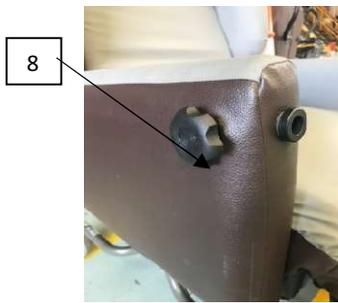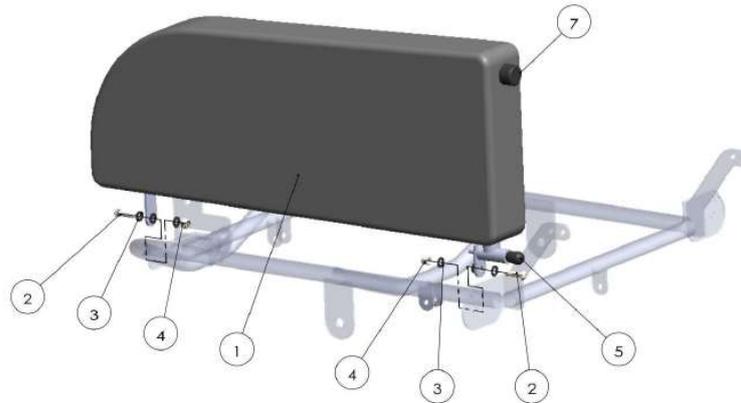


No.	Product Code	Product Description
1	GSP400740	Arm Rest Cover - Left – OLD (Prior to SN#1512518001)
1	GSP400745	Arm Rest Cover - Right – OLD (Prior to SN#1512518001)
1	GSP400748	Arm Rest Cover - Left – SMALL - NEW (From SN#1512518001)
1	GSP400749	Arm Rest Cover - Right – SMALL - NEW (From SN#1512518001)
1	GSP400746	Arm Rest Cover - Left – LARGE - NEW (From SN#1512518001)
1	GSP400747	Arm Rest Cover - Right – LARGE - NEW (From SN#1512518001)
1,2,3,4,7	GSP400698	Arm Rest Assembly - Left - SMALL - OLD (Prior to SN#1512518001)
1,2,3,4,7	GSP400699	Arm Rest Assembly - Right - SMALL - OLD (Prior to SN#1512518001)
1,2,3,4,7	GSP400700	Arm Rest Assembly - Left – OLD (Prior to SN#1512518001)
1,2,3,4,7	GSP400705	Arm Rest Assembly - Right OLD (Prior to SN#1512518001)
1,2,3,4,7	GSP400709	Arm Rest Assembly - Left - SMALL - NEW (From SN#1512518001)
1,2,3,4,7	GSP400708	Arm Rest Assembly - Right - SMALL - NEW (From SN#1512518001)
1,2,3,4,7	GSP400706	Arm Rest Assembly - Left – LARGE - NEW (From SN#1512518001)
1,2,3,4,7	GSP400707	Arm Rest Assembly - Right – LARGE - NEW (From SN#1512518001)
5	GSP400845	Arm Rest & Wing Pin Assembly
6	GSP400795	Leg Recline Lever
6	GSP400800	Leg Recline Cable
7	GSP400836	Dining Table Plastic Bush
8	GSP400835	Dining Table Adjustment Screw

## 2. Base Frame and Castor Assembly

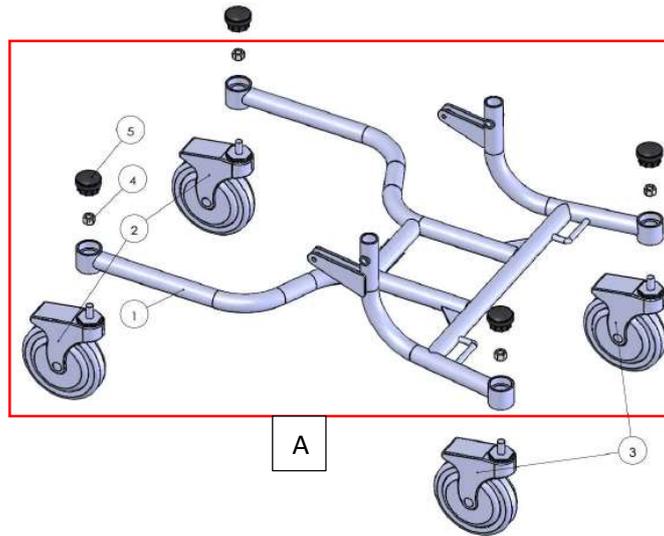

No.	Product Code	Product Description
1	GSP400850	Replacement Frame - Small
1	GSP800855	Replacement Frame - Large
2	GSP400825	6in Locking Castor
3	GSP400830	6in Directional Castor
4	GSP400831	Castor Mounting Nut M12
5	GSP400832	Castor Mounting Cap
A	GSP400850	Replacement Base Frame – Small – Lay Flat Model
A	GSP400852	Replacement Base Frame – Small – Non-Lay Flat Model
A	GSP400855	Replacement Base Frame – Large – Lay Flat Model
A	GSP400857	Replacement Base Frame – Large – Non-Lay Flat Model


### 3. Leg Rest Assembly

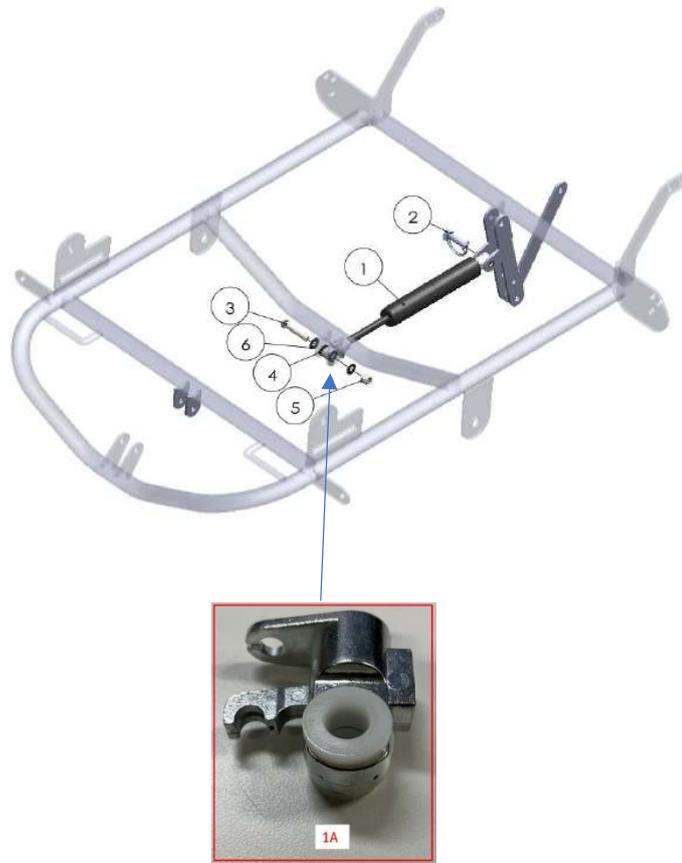

No.	Product Code	Product Description
1	GSP400810	Leg Rest Gas Strut
1A	GSP400812	Lever Pin, Boot & Bushing Assembly - for Gas Strut - Aspire Mobile Air Chair & Cove Chair
2	GSP400823	Pin with Lock for Tilt / Leg Rest Gas Strut
3,4,5,6	GSP400809	Mounting Hardware for Leg Rest Gas Strut



#### 4. Recline Assembly

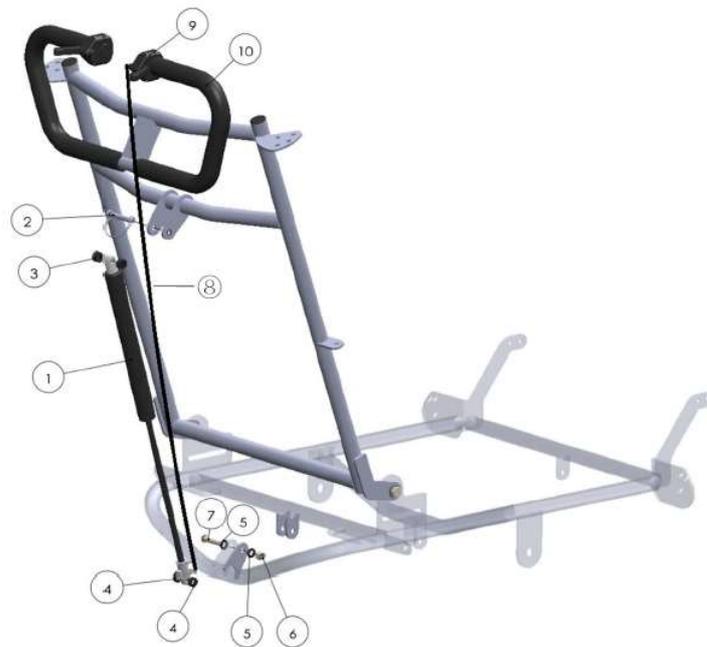


No.	Product Code	Product Description
1	GSP400815	Reclining Gas Strut (Non-Lay Flat Model)
1	GSP400816	Reclining Gas Strut (Lay Flat Model)
2	GSP400822	Pin with Lock for Recline Gas Strut
3,4,5,6,7	GSP400814	Mounting Hardware for Recline Gas Strut
8	GSP400806	Recline Cable – 1050mm
9	GSP400790	Tilt & Recline Lever
10	GSP400824	Hand Grip

## 5. Tilt Assembly




No.	Product Code	Product Description
1	GSP400820	Tilting Gas Strut
2	GSP400823	Pin with Lock for Tilt / Leg Rest Gas Strut
3,4,5,6	GSP400819	Mounting Hardware for Tilt Gas Strut
7	GSP400805	Tilt Cable -1300m
8	GSP400790	Tilt & Recline Lever
9	GSP400824	Hand Grip

## 6. Wing Assembly

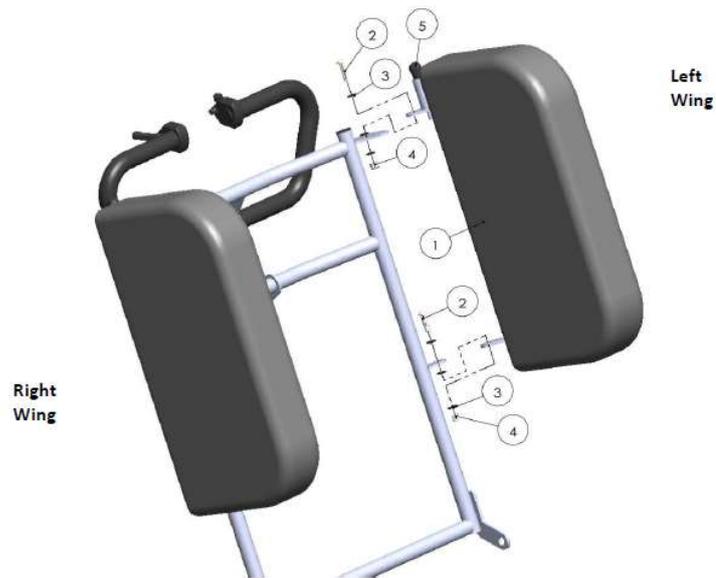

No.	Product Code	Product Description
1	GSP400770	Wing Cushion Cover - Left - OLD (Prior to SN#1523818001)
1	GSP400775	Wing Cushion Cover - Right – OLD (Prior to SN#1523818001)
1	GSP400776	Wing Cushion Cover – Left – NEW (From SN#1523818001)
1	GSP400777	Wing Cushion Cover – Right – NEW (From SN#1523818001)
1,2,3,4,5	GSP400730	Wing Cushion Assembly – Left - OLD (Prior to SN#1523818001)
1,2,3,4,5	GSP400735	Wing Cushion Assembly - Right – OLD (Prior to SN#1523818001)
1,2,3,4,5	GSP400736	Wing Cushion Assembly – Left – NEW (From SN#1523818001)
1,2,3,4,5	GSP400737	Wing Cushion Assembly - Right – NEW (From SN#1523818001)
5	GSP400845	Arm Rest & Wing Pin Assembly

## 7. Cushions and Accessories

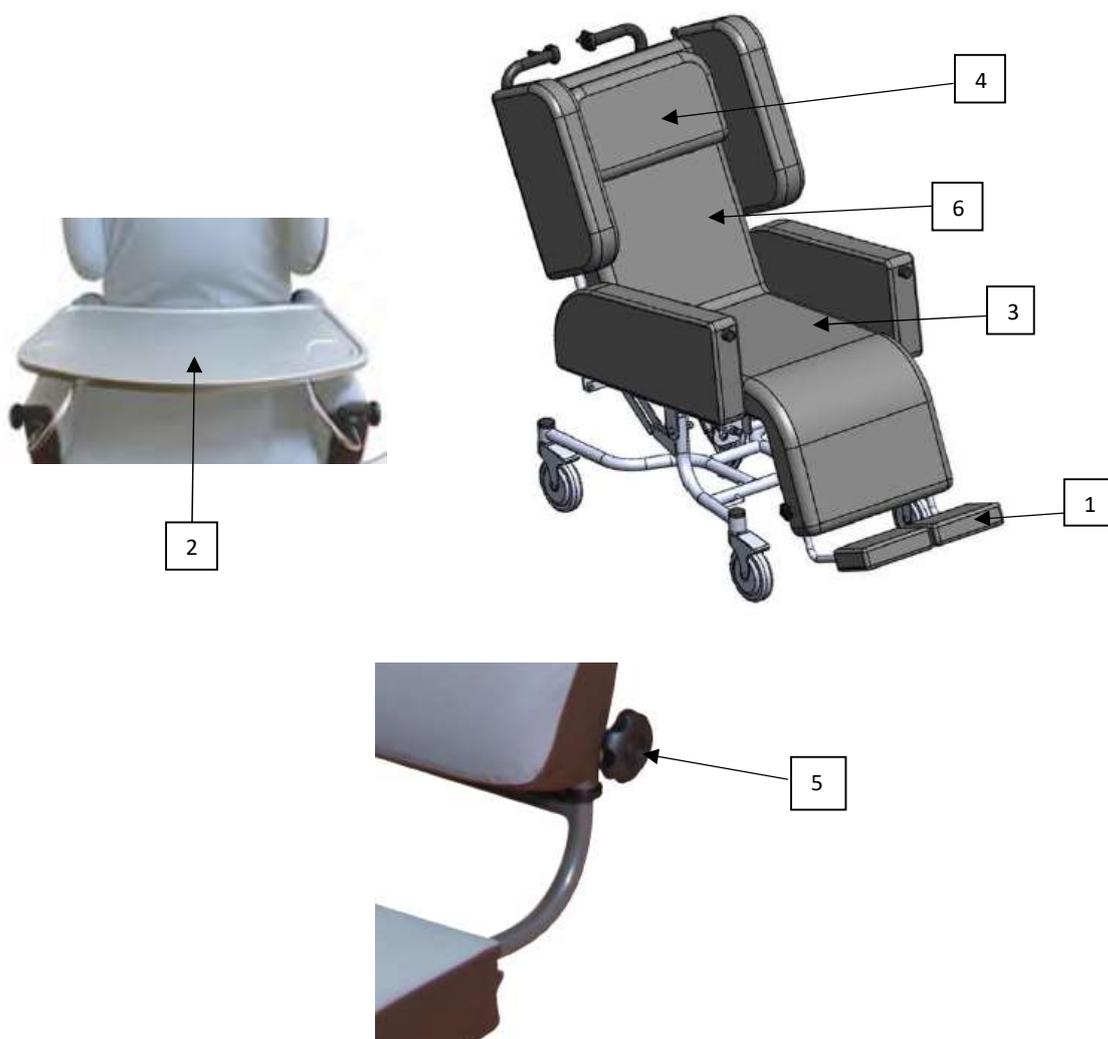

No.	Product Code	Product Description
1	CHP198860	Padded Footrest
2	CHP198861	Meal Tray
3	GSP400710	Patient Surface Assembly (Foam Core) - Small
3	GSP400715	Patient Surface Assembly (Foam Core) - Large
3	GSP400750	Seating Surface Cushion Cover - Small
3	GSP400755	Seating Surface Cushion Cover - Large
3	GSP400780	Air Cushion / Bladder - Seat – Large
3	GSP400781	Air Cushion / Bladder – Seat – Small
4	GSP400720	Head Rest Cushion - Small
4	GSP400725	Head Rest Cushion - Large
5	GSP400840	Footrest Adjustment Screw
6	GSP400786	Contoured Foam Insert – Back – Large
6	GSP400787	Contoured Foam Insert – Back – Small

8. Inner Frame Covers (Under support) – Seat, Back and Leg



No.	Product Code	Product Description
	GSP400860	Inner Frame Cover (under support) - Seat - Small - Aspire MOBILE Air Chair
	GSP400862	Inner Frame Cover (under support) - Seat - Large - Aspire MOBILE Air Chair
	GSP400864	Inner Frame Cover (under support) - Back - Small - Aspire MOBILE Air Chair
	GSP400866	Inner Frame Cover (under support) - Back - Large - Aspire MOBILE Air Chair
	GSP400868	Inner Frame Cover (under support) - Leg - Aspire MOBILE Air Chair