

Aspire Lift Recline Chair PORTO - Spare Parts Diagram



5



4



8.1

9

8




10



2

11

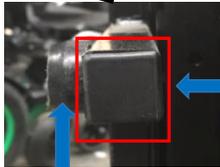

6



3



1



12/13

Product Code	Product Description
CHP227200	Chair Powerlift Recline - Aspire Porto Petite - Fawn - Space Saver - 150kg

No.	Item Number	Item Description
-----	-------------	------------------

1	GSP510072	Actuator Back / Seat - Zero Gravity - Aspire Lift Recline Chair PORTO
1.1	GSP510075	Actuator Tilting - Space Saver - Aspire Lift Recline Chair PORTO
1.2	GSP510070	Actuator - Single Motor - Aspire Lift Recline Chair PORTO (for older models only - SRN: Q19030502 00001 to Q19030502 00100)
2	GSP510150	Castor - Locking - Aspire Lift Recline Chair PORTO
3	GSP510200	Castor - Standard - Aspire Lift Recline Chair PORTO
4	GSP510456	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair PORTO
5	GSP510456	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair PORTO
6	GSP510352	Plastic Foot - Aspire ALTITUDE Vertical / Porto / Signature 2 Chair
7	GSP510358	Plastic End Cap - Aspire ALTITUDE Vertical / Single Action Recliner / Porto Chair
8	GSP510391	Handset - Aspire Lift Recline Chair MONTANA/VERONA - NEW - Aspire Print
8.1	GSP510372	Extension Cable - for Control Box to Handset - Aspire Lift Recline Chair PORTO
9	GSP510409	Mains Power Plug (AU) - Aspire Lift Recline Chair PORTO
10	GSP510416	Power Pack / Transformer - Aspire Lift Recline Chair PORTO
11	GSP511260	Base Frame - Aspire Lift Recline Chair PORTO
12	GSP400648	Transformer ONLY - for Battery Backup - All Aspire Lift Recline Chair / ComfiMotion Care / LUX Beds
13	GSP400649	Battery ONLY - for Battery Backup - All Aspire Lift Recline Chair / ComfiMotion Care / LUX Beds