

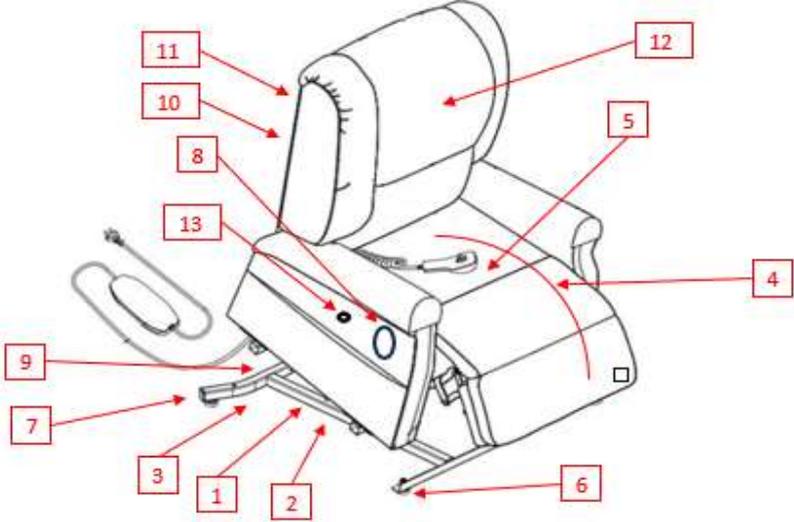
Chair Power Lift Recline - Aspire PostureFit Spare Parts Manual



Item Code	Description
CHP198250	Chair Powerlift Recline - Aspire PostureFit - Medium - 3 Motor - 160kg
CHP198260	Chair Powerlift Recline - Aspire PostureFit - Large - 3 Motor - 160kg

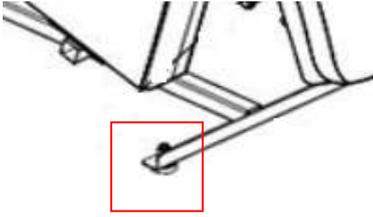
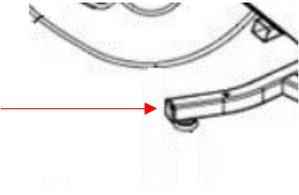
This manual is for internal use only

1 Frame, Seat, Cushion, Backrest, Castors and Hardware

Ref	Item Number	Item Description	Image	Type
				

Ref	Item Number	Item Description	Image	Type
1	GSP873100	Base Frame - CF46 - 520 - Aspire PostureFit Recliner - Medium		Spare Part
1	GSP873102	Base Frame - CF46 - 570 - Aspire PostureFit Recliner - Large		Spare Part
2	GSP873104	Trolley Assembly - Aspire PostureFit Recliner		Spare Part
3	GSP873106	Rear Castor Wheel - Aspire PostureFit Recliner		Spare Part

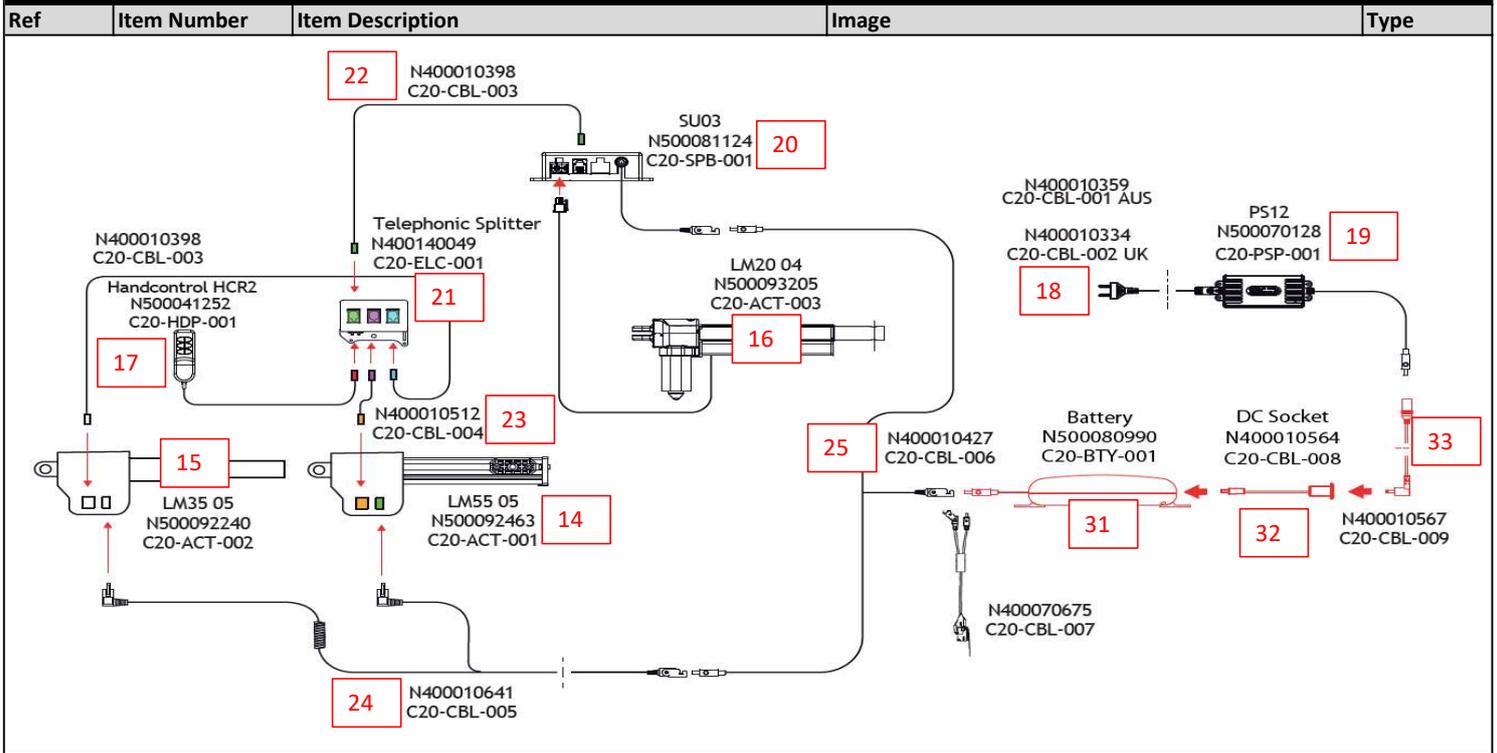
1 Frame, Seat, Cushion, Backrest, Castors and Hardware				
Ref	Item Number	Item Description	Image	Type
3A	GSP873196	Rear Castor Fork Only - LEFT - Aspire PostureFit Recliner		Spare Part
3A	GSP873198	Rear Castor Fork Only - RIGHT - Aspire PostureFit Recliner		Spare Part
4	GSP873136	Seat Detachable - Overlay - Aspire PostureFit Recliner - Medium - Black		Spare Part
4	GSP873138	Seat Detachable - Overlay - Aspire PostureFit Recliner - Large - Black		Spare Part
5	GSP873140	Seat Cushion - Aspire PostureFit Recliner - Medium - Black		Spare Part
5	GSP873142	Seat Cushion - Aspire PostureFit Recliner - Large - Black		Spare Part

1 Frame, Seat, Cushion, Backrest, Castors and Hardware				
Ref	Item Number	Item Description	Image	Type
6	GSP873144	Metal Foot - Aspire PostureFit Recliner		Spare Part
7	GSP873146	End Cap - Plastic - Aspire PostureFit Recliner		Spare Part
8	GSP873148	Handset Loop - PU Black - Aspire PostureFit Recliner		Spare Part
9	GSP873150	Front Castor Only - w/o trolley - Aspire PostureFit Recliner		Spare Part
10	GSP873152	Hand Bolt - M6 X 20mm - for Backrest Assembly - Aspire PostureFit Recliner		Spare Part
11	GSP873170	Backrest Wings - Pair - Aspire PostureFit Recliner - Small/Medium		Spare Part

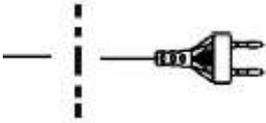
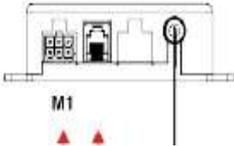
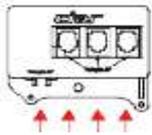
1 Frame, Seat, Cushion, Backrest, Castors and Hardware				
Ref	Item Number	Item Description	Image	Type
11	GSP873175	Backrest Wings - Aspire PostureFit Recliner - Tall Medium/ Large		Spare Part
11A	GSP873154	Hand Bolt - M6 X 60mm - for Wings - Aspire PostureFit Recliner		Spare Part
12	GSP873156	Waterfall Backrest Cushions - Set of 3 - Aspire PostureFit Recliner - Medium - Black-Black		Spare Part
12	GSP873158	Waterfall Backrest Cushions - Set of 3 - Aspire PostureFit Recliner - Large - Black-Black		Spare Part
12	CHP198290	Backrest Cushion - Classic Style - Aspire PostureFit Chair - Small/Medium		Accessory
12	CHP198291	Backrest Cushion - Classic Style - Aspire PostureFit Chair - Tall Medium/Large		Accessory

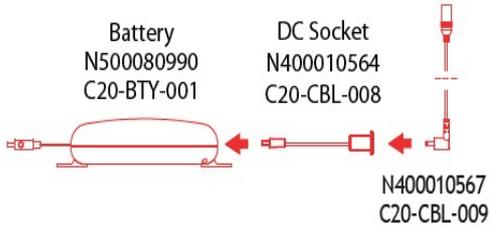
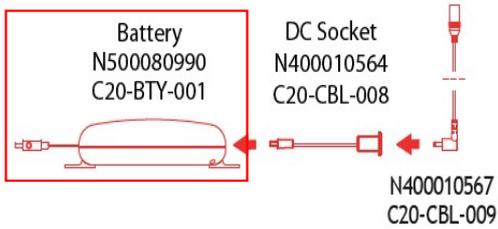
1 Frame, Seat, Cushion, Backrest, Castors and Hardware				
Ref	Item Number	Item Description	Image	Type
12	CHP198295	Backrest Cushion - Advance Immersion - Aspire PostureFit Chair - Small/Medium		Accessory
12	CHP198296	Backrest Cushion - Advance Immersion - Aspire PostureFit Chair - Tall Medium/Large		Accessory
13	GSP873160	Quick Release Pin - for Armrest - Aspire PostureFit Recliner		Spare Part

2 Electronics



Ref	Item Number	Item Description	Image	Type
14	GSP873108	Actuator - Main - Aspire PostureFit Recliner	<p>LM55 05 N500092463 C20-ACT-001</p>	Spare Part
15	GSP873110	Actuator - Back Recline - Aspire PostureFit Recliner	<p>LM35 05 N500092240 C20-ACT-002</p>	Spare Part
16	GSP873112	Actuator - Head - Aspire PostureFit Recliner	<p>LM20 04 N500093205 C20-ACT-003</p>	Spare Part

2 Electronics				
Ref	Item Number	Item Description	Image	Type
17	GSP873114	Handset - Aspire PostureFit Recliner	<p>Handcontrol HCR2 N500041252 C20-HDP-001</p> 	Spare Part
18	GSP873116	Mains Power Cable - Aspire PostureFit Recliner - AU Plug	 <p>N400010359 C20-CBL-001 AUS</p>	Spare Part
19	GSP873120	Power Pack/Transformer - Aspire PostureFit Recliner	<p>PS12 N500070128 C20-PSP-001</p> 	Spare Part
20	GSP873122	Supply Box - Head - Aspire PostureFit Recliner	 <p>SU03 N500081124 C20-SPB-001</p>	Spare Part
21	GSP873126	Splitter - Aspire PostureFit Recliner	<p>Telephonic Splitter N400140049 C20-ELC-001</p> 	Spare Part
22	GSP873128	Cable for splitter 50cm (n 2 pcs) - Aspire PostureFit Recliner		Spare Part

2 Electronics				
Ref	Item Number	Item Description	Image	Type
23	GSP873130	Cable for splitter 10cm - Aspire PostureFit Recliner		Spare Part
24	GSP873132	Y-Cable for actuators - Aspire PostureFit Recliner		Spare Part
25	GSP873134	Y-Cable for PS12 Supply Box - Aspire PostureFit Recliner		Spare Part
26	GSP873135	Limit Switch w/Cable - for Trolley - Aspire PostureFit Recliner (Not on Dwg.)		Spare Part
30	GSP873188	Conversion Kit - for Battery Backup - Aspire PostureFit Recliner	 <p>Battery N500080990 C20-BTY-001</p> <p>DC Socket N400010564 C20-CBL-008</p> <p>N400010567 C20-CBL-009</p>	Spare Part
31	GSP873190	Battery - Aspire PostureFit Recliner	 <p>Battery N500080990 C20-BTY-001</p> <p>DC Socket N400010564 C20-CBL-008</p> <p>N400010567 C20-CBL-009</p>	Spare Part

2 Electronics				
Ref	Item Number	Item Description	Image	Type
32	GSP873192	Cable - DC Socket - for Battery Back up - Aspire PostureFit Recliner	<p>Battery N500080990 C20-BTY-001</p> <p>DC Socket N400010564 C20-CBL-008</p> <p>N400010567 C20-CBL-009</p>	Spare Part
33	GSP873194	Cable - link to power supply - for Battery Backup - Aspire PostureFit Recliner	<p>Battery N500080990 C20-BTY-001</p> <p>DC Socket N400010564 C20-CBL-008</p> <p>N400010567 C20-CBL-009</p>	Spare Part