

Spare Parts Manual



Item Number	Item Description
CHP231600	Chair Pressure Care - Aspire PosturePro - Teal - 140kg
CHP231610	Chair Pressure Care - Aspire PosturePro PLUS - Teal - 140kg

This manual is for internal use only

Cushions and Hand Grips



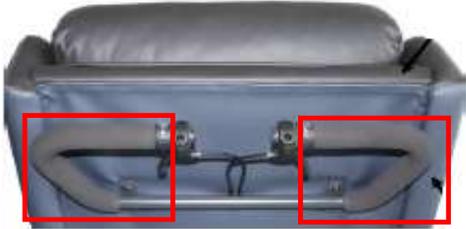
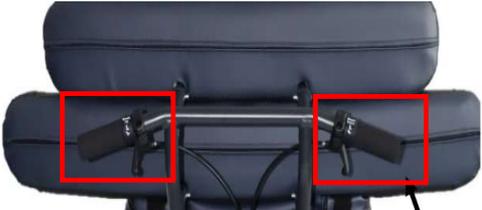
Posture Pro



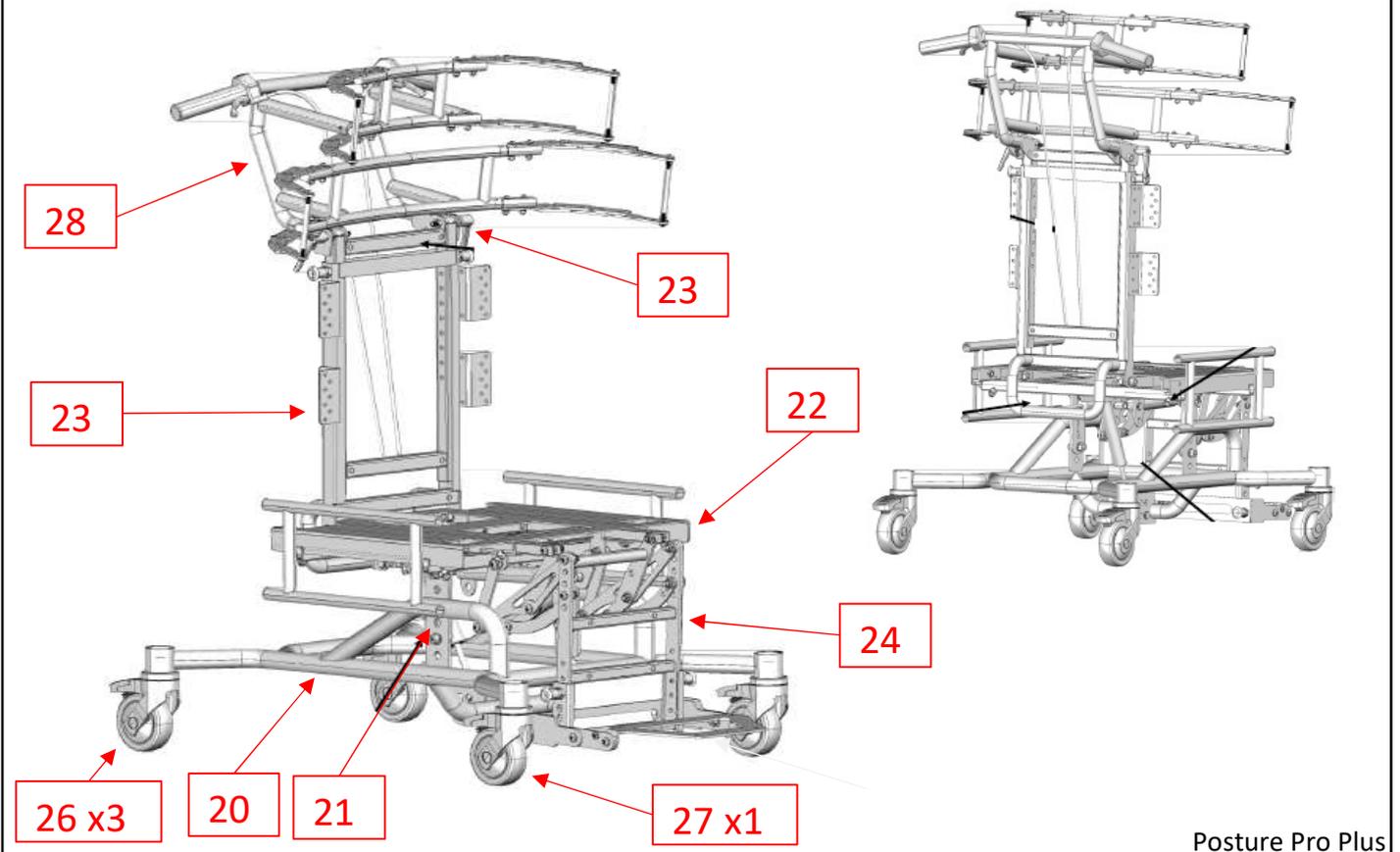
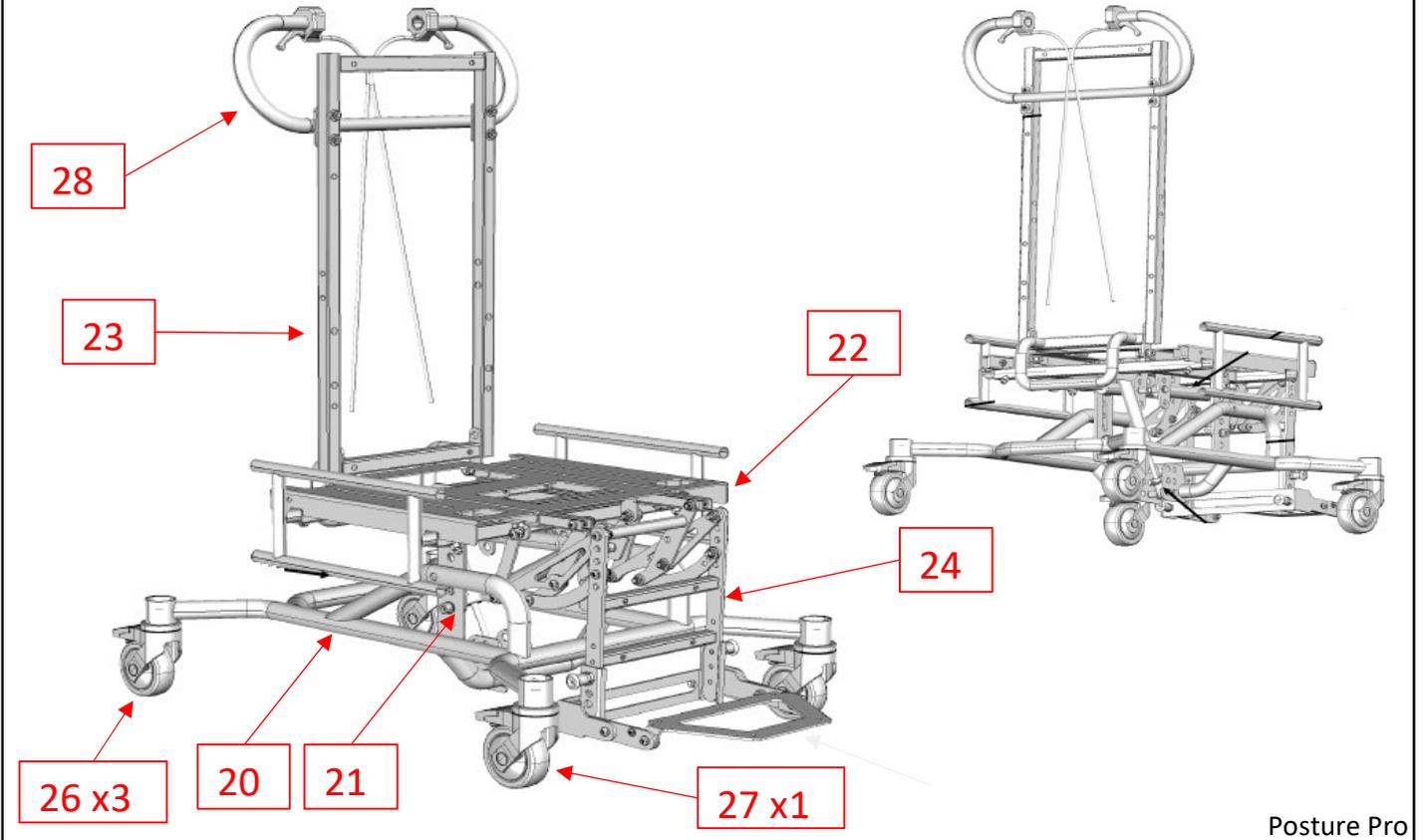
Posture Pro Plus

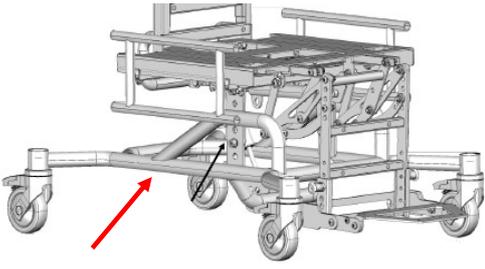
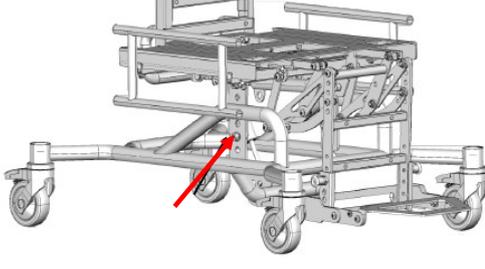
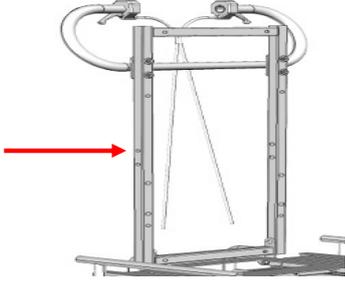
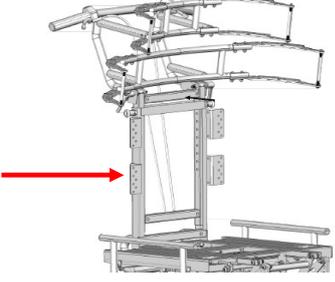
1 Cushions and Hand Grips				
Ref	Item Number	Item Description	Image	Type
1	GSP047500	Armrest - Complete - Right - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
1	GSP047502	Armrest - Complete - Left - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
2	GSP047504	Chair Back Support - Aspire PosturePro Chair		Spare Part
2	GSP047506	Chair Back Support - Aspire PosturePro PLUS Chair		Spare Part
3	GSP047508	Wing Assembly - Left & Right - Aspire PosturePro Chair		Spare Part
3	GSP047510	Wing / Headrest Assembly - Upper - Aspire PosturePro PLUS Chair		Spare Part

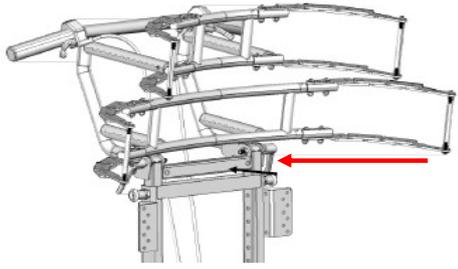
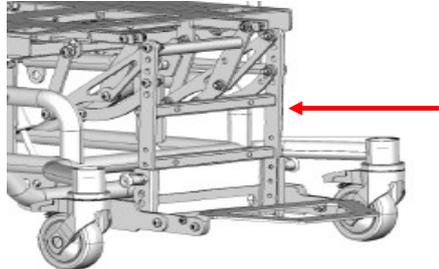
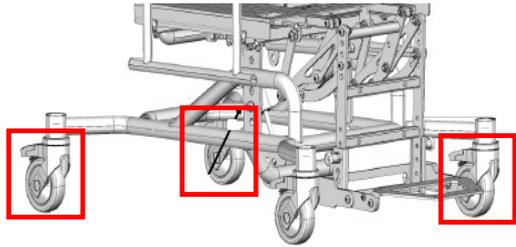
1 Cushions and Hand Grips				
3	GSP047512	Wing / Headrest Assembly - Lower - Aspire PosturePro PLUS Chair		Spare Part
3A	GSP047590	C Clip - Upper Headrest - Aspire PosturePro PLUS Chair		Spare Part
4	GSP047514	Footrest Assembly - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
4	GSP047516	Footrest Pad - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
5	GSP047518	Headrest / Upper Cushion - Aspire PosturePro Chair		Spare Part
5	GSP047520	Back Cover - Aspire PosturePro PLUS Chair		Spare Part

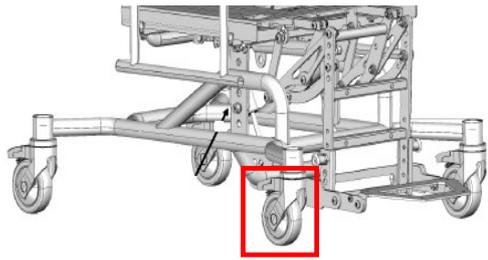
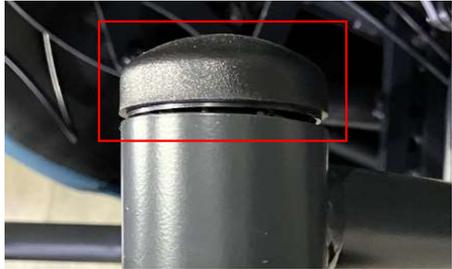
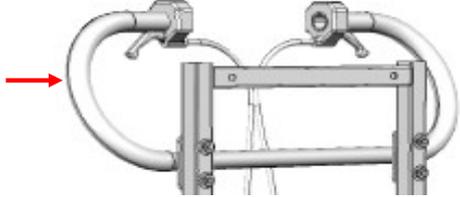
1 Cushions and Hand Grips				
6	GSP047522	Backrest Middle Cushion - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
7	GSP047524	Backrest Lower Cushion - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
8	GSP047526	Seat Cushion - w/Air Cell - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
9	GSP047528	Seat Base and Calf Cushion - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
10	GSP047530	Hand Grip - Pair - Aspire PosturePro Chair		Spare Part
10	GSP047532	Hand Grip - Pair - Aspire PosturePro PLUS Chair		Spare Part

Frame and Castors

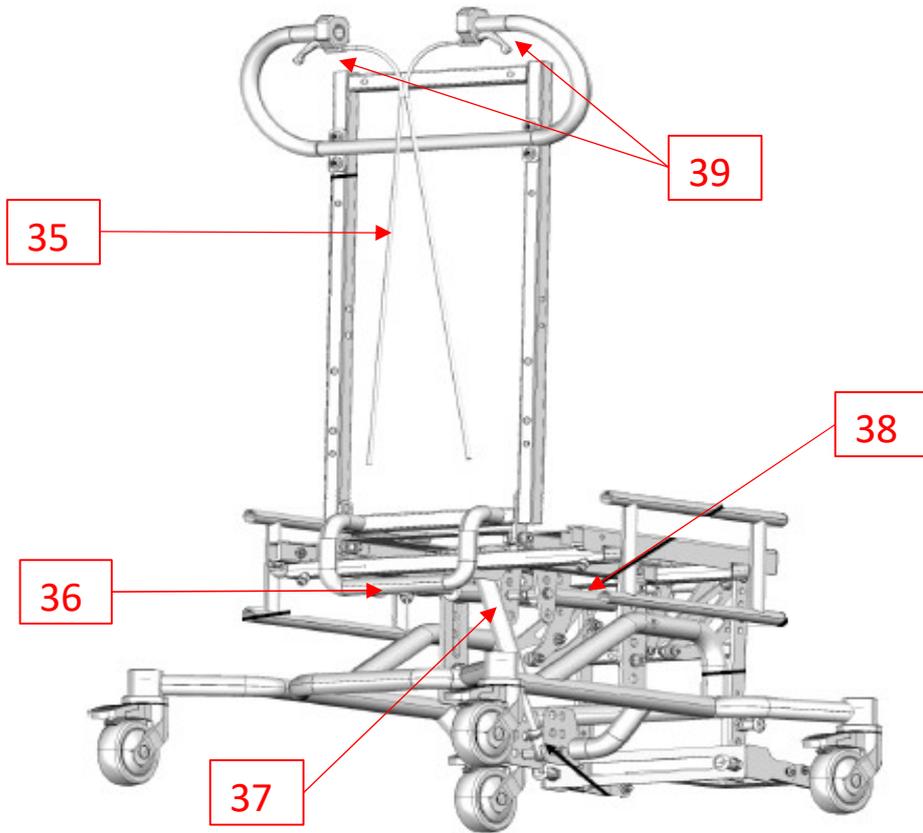


2 Frame and Castors				
Ref	Item Number	Item Description	Image	Type
20	GSP047540	Base Frame - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
21	GSP047542	Connecting Bracket - w/Hardware - Pair - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
22	GSP047544	Seat Frame - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
22A	GSP047598	Hand Screw - Seat Base - Aspire PosturePro / PosturePro PLUS Chair	Image Coming Soon	Spare Part
23	GSP047546	Backrest Frame - Aspire PosturePro Chair		Spare Part
23	GSP047548	Backrest Frame - Lower - Aspire PosturePro PLUS Chair		Spare Part

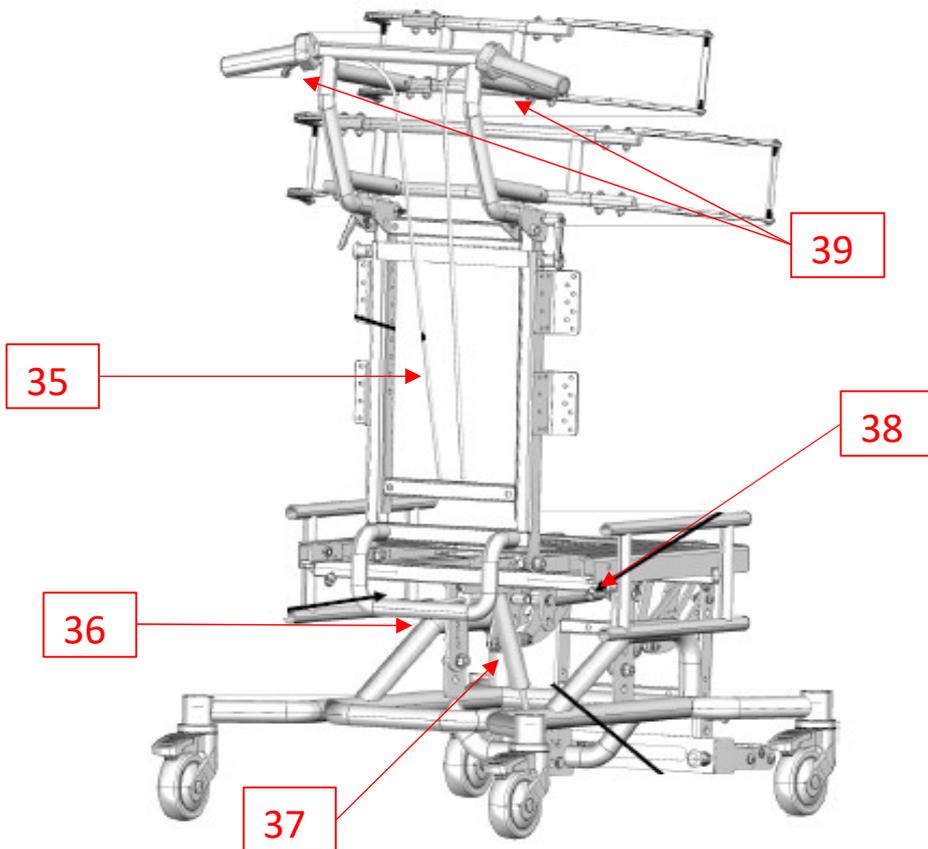
2 Frame and Castors				
23	GSP047550	Backrest Frame - Adjustable - Upper - Aspire PosturePro PLUS Chair		Spare Part
23A	GSP047586	Locking Pin Mechanism - Complete - Upper Backrest Tilt - Aspire PosturePro PLUS Chair		Spare Part
23B	GSP047588	Locking Pin Mechanism - Complete - Upper B/rest Ht /Footrest/Leg Ht - Aspire PosturePro PLUS Chair		Spare Part
24	GSP047552	Legrest Frame Assembly - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
24A	GSP047596	Hand Screw - Leg Extension / Arm - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
26	GSP403365	Castor - Total Lock - 100mm - Aspire COVE / PosturePro / PosturePro PLUS Chair (3 x Total Lock Castors on Chair)		Spare Part

2 Frame and Castors				
27	GSP403370	<p>Castor - Directional Lock - 100mm - Aspire PosturePro / PosturePro PLUS Chair</p> <p>(1 x Directional Castor on Chair)</p>		Spare Part
27A	GSP047600	<p>Cap - Castor Hub - Aspire PosturePro / PosturePro PLUS Chair</p>		Spare Part
28	GSP047558	<p>Push Handle - Aspire PosturePro Chair</p>		Spare Part
28	GSP047560	<p>Push Handle - Aspire PosturePro PLUS Chair</p>		Spare Part

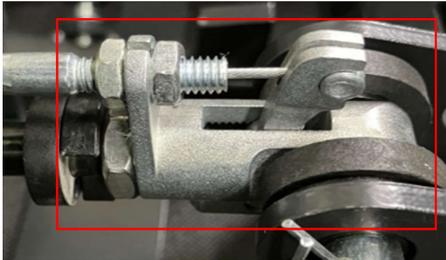
Levers, Gas Struts and Cables



Posture Pro



Posture Pro Plus

3 Levers, Gas Struts and Cables				
Ref	Item Number	Item Description	Image	Type
34	GSP047568	Cable - Legrest - Aspire PosturePro / PosturePro PLUS Chair	Image Coming Soon	Spare Part
35	GSP047570	Cable - Tilt or Recline - Aspire PosturePro / PosturePro PLUS Chair	Image Coming Soon	Spare Part
36	GSP047572	Gas Strut - Recline - Aspire PosturePro / PosturePro PLUS Chair	Image Coming Soon	Spare Part
37	GSP403330	Gas Strut - Tilt - Aspire COVE / PosturePro / PosturePro PLUS Chair	Image Coming Soon	Spare Part
38	GSP047574	Gas Strut - Legrest - Aspire PosturePro / PosturePro PLUS Chair	Image Coming Soon	Spare Part
38A	GSP047592	Boot & Lever - Complete - suits all Gas Struts - Aspire PosturePro / PosturePro PLUS Chair		Spare Part

3 Levers, Gas Struts and Cables				
38B	GSP047594	Mounting Hardware - suits all Gas Struts - Pin, Clip, Washers - Aspire PosturePro / PosturePro PLUS Chair		Spare Part
39	GSP047576	Release Lever - Aspire PosturePro Chair		Spare Part
39	GSP047578	Release Lever - Aspire PosturePro PLUS Chair		Spare Part