


Aspire Lift Recline Chair RAPHAEL/ RENOIR - Spare Parts Diagram

Chairs AFTER the serial number listed below has leg extension, Please check the chair serial number before ordering actuators	
Small	RS2-PO685853-0001
Medium	RM2-PO685853-0001
Large	RL2-PO685853-0001
Maxi	RMX2-PO685853-0001

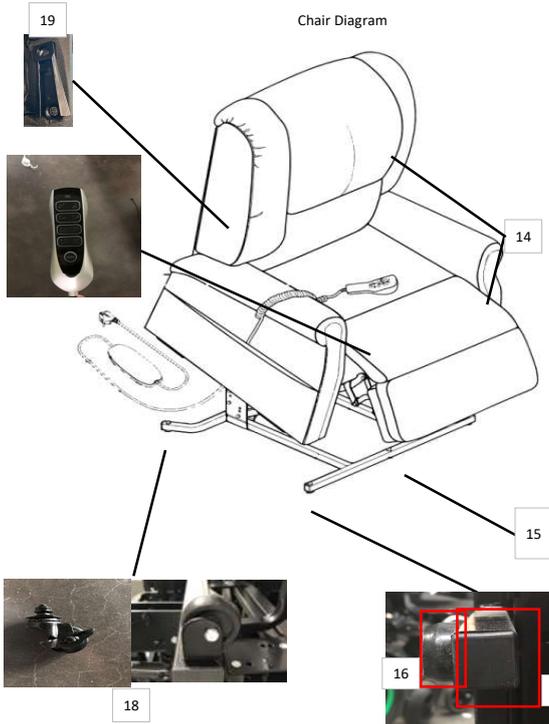


Product Code	Product Description
CHP227290	Chair Powerlift Recline - Aspire Raphael Quattro - SMALL - Bronze - 4 Motor - 130kg
CHP227291	Chair Powerlift Recline - Aspire Raphael Quattro - SMALL - Marina Grey - 4 Motor - 130kg
CHP227300	Chair Powerlift Recline - Aspire Raphael Quattro - MEDIUM - Bronze - 4 Motor - 130kg
CHP227301	Chair Powerlift Recline - Aspire Raphael Quattro - MEDIUM - Marina Grey - 4 Motor - 130kg
CHP227302	Chair Powerlift Recline - Aspire Raphael Quattro - MEDIUM - Nordic Stone Vinyl - 4 Motor - 130kg
CHP227310	Chair Powerlift Recline - Aspire Raphael Quattro - LARGE - Bronze - 4 Motor - 130kg
CHP227311	Chair Powerlift Recline - Aspire Raphael Quattro - LARGE - Marina Grey - 4 Motor - 130kg
CHP227312	Chair Powerlift Recline - Aspire Raphael Quattro - LARGE - Nordic Stone Vinyl - 4 Motor - 130kg
CHP227320	Chair Powerlift Recline - Aspire Raphael Quattro - MAXI - Bronze - 4 Motor - 220kg
CHP227321	Chair Powerlift Recline - Aspire Raphael Quattro - MAXI - Marina Grey - 4 Motor - 220kg
CHP227322	Chair Powerlift Recline - Aspire Raphael Quattro w/Leg Extension - MAXI - Marina Grey - 4 Motor - 180kg
CHP227319	Chair Powerlift Recline - Aspire Raphael Quattro w/Leg Extension - MAXI - Bronze - 4 Motor - 180kg
CHP227575	Chair Powerlift Recline - Aspire Renoir - Quattro - Small - Dusk - 130kg
CHP227580	Chair Powerlift Recline - Aspire Renoir - Quattro - Medium - Dusk - 130kg
CHP227585	Chair Powerlift Recline - Aspire Renoir - Quattro - Large - Dusk - 130kg

No.	Item Number	Item Description
1	GSP510010	Actuator - Backrest - Aspire Lift Recline Chair RAPHAEL
1	GSP510011	Actuator - Backrest - Aspire Lift Recline Chair RAPHAEL / RENOIR Leg Extension
2	GSP510030	Actuator - Footrest / Lift STANDARD - Aspire Lift Recline Chair RAPHAEL
2	GSP510031	Actuator - Footrest / Lift - HEAVY DUTY - 6000N - Aspire Lift Recline Chair RAPHAEL
2	GSP510032	Actuator - Footrest / Lift STANDARD - Aspire Lift Recline Chair RAPHAEL / RENOIR Leg Extension
3	GSP510040	Actuator - Headrest - Aspire Lift Recline Chair RAPHAEL / RENOIR
4	GSP510050	Actuator - Lumbar - Aspire Lift Recline Chair RAPHAEL / RENOIR
5	GSP510110	Splitter Cable - Headrest and Lumbar Actuator - Aspire Lift Recline Chair RAPHAEL / RENOIR
6	GSP510120	Extension Cable - Control Box to Handset - Aspire Lift Recline Chair RAPHAEL / RENOIR
7	GSP510230	Control Unit - Aspire Lift Recline Chair RAPHAEL / RENOIR
8	GSP510360	Handset - Aspire Lift Recline Chair RAPHAEL / RENOIR
9	GSP510400	Mains Power Plug (AU) - Aspire Lift Recline Chair RAPHAEL / RENOIR
10	GSP510414	Power Pack / Transformer - Aspire Lift Recline Chair RAPHAEL / RENOIR
10a	GSP510412	Cable - (from transformer to control box) - Aspire Raphael Lift Recline Chair
11	GSP400648	Transformer ONLY - for Battery Backup - All Aspire Lift Recline Chair / ComfiMotion Care / LUX Beds
12	GSP400649	Battery ONLY - for Battery Backup - All Aspire Lift Recline Chair / ComfiMotion Care / LUX Beds

No.	Item Number	Item Description
13	GSP510450	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - SMALL - Marina Grey
13	GSP510458	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - MAXI - Bronze
13	GSP510460	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - MAXI - Marina Grey
13	GSP510462	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - MAXI - Nordic Stone Vinyl
13	GSP510464	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - MED / LRG - Bronze
13	GSP510466	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - MED / LRG - Marina Grey
13	GSP510468	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - MED / LRG - Nordic Stone Vinyl
13	GSP510470	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - SMALL - Bronze
13	GSP510472	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RAPHAEL - SMALL - Nordic Stone Vinyl
13	GSP510474	Cover Set - Armrest / Headrest - Aspire Lift Recline Chair RENOIR - Dusk

No.	Item Number	Item Description
14	GSP510480	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - SMALL - Bronze
14	GSP510482	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Small - Bronze
14	GSP510484	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Small - Marina Grey
14	GSP510488	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Medium - Bronze
14	GSP510490	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - MEDIUM - Bronze
14	GSP510492	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - MEDIUM - Marina Grey
14	GSP510494	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - MEDIUM - Nordic Stone Vinyl
14	GSP510496	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Medium - Marina Grey
14	GSP510497	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Medium - Nordic Stone Vinyl
14	GSP510498	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Large - Bronze
14	GSP510500	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - LARGE - Bronze
14	GSP510501	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - Leg Extension - MAXI - Nordic Stone Vinyl
14	GSP510502	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - LARGE - Marina Grey
14	GSP510503	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - Leg Extension - SMALL - Nordic Stone Vinyl
14	GSP510504	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - LARGE - Nordic Stone Vinyl
14	GSP510506	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Large - Marina Grey
14	GSP510508	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Large - Nordic Stone Vinyl
14	GSP510510	Seat Detachable - Aspire Lift Recline Chair RAPHAEL - Maxi - Bronze
14	GSP510512	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Maxi - Bronze
14	GSP510514	Seat Detachable - Extension Seat - Aspire Lift Recline Chair RAPHAEL - Maxi - Marina Grey
14	GSP510516	Seat Detachable - Aspire Lift Recline Chair RENOIR - SMALL - Dusk
14	GSP510517	Seat Detachable - Aspire Lift Recline Chair RENOIR - MEDIUM - Dusk
14	GSP510518	Seat Detachable - Aspire Lift Recline Chair RENOIR - LARGE - Dusk

Electronics Schematics

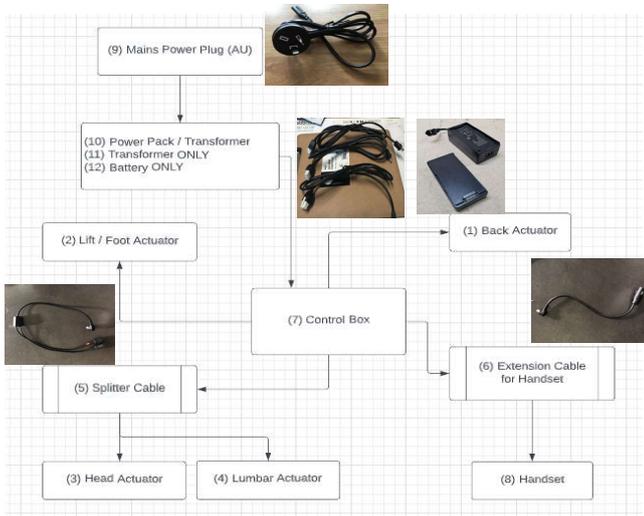

No.	Item Number	Item Description
15	GSP511270	Base Frame - Aspire Lift Recline Chair RAPHAEL - MAXI
15	GSP511280	Base Frame - Aspire Lift Recline Chair RAPHAEL / RENOIR - MEDIUM / LARGE
15	GSP511290	Base Frame - Aspire Lift Recline Chair RAPHAEL / RENOIR - SMALL
16	GSP510350	Plastic Foot - Aspire Lift Recline Chair MAUI/MONTANA/RAPHAEL / RENOIR/VERONA
17	GSP510355	Plastic End Cap - Side Frame - Aspire Lift Recline Chair MAUI/MONTANA/RAPHAEL / RENOIR/VERONA
18	GSP510130	Castor - Locking - Aspire Lift Recline Chair MAUI/RAPHAEL/VERONA/RENOIR
18	GSP510180	Castor - Standard - Aspire Lift Recline Chair MAUI/RAPHAEL/VERONA/RENOIR
19	GSP005340	Slide-in Bracket - for Backrest - Aspire Lift Recline Chair RAPHAEL / RAPHAEL X