

Aspire Rehab RX/RX Junior

Spare Parts Manual



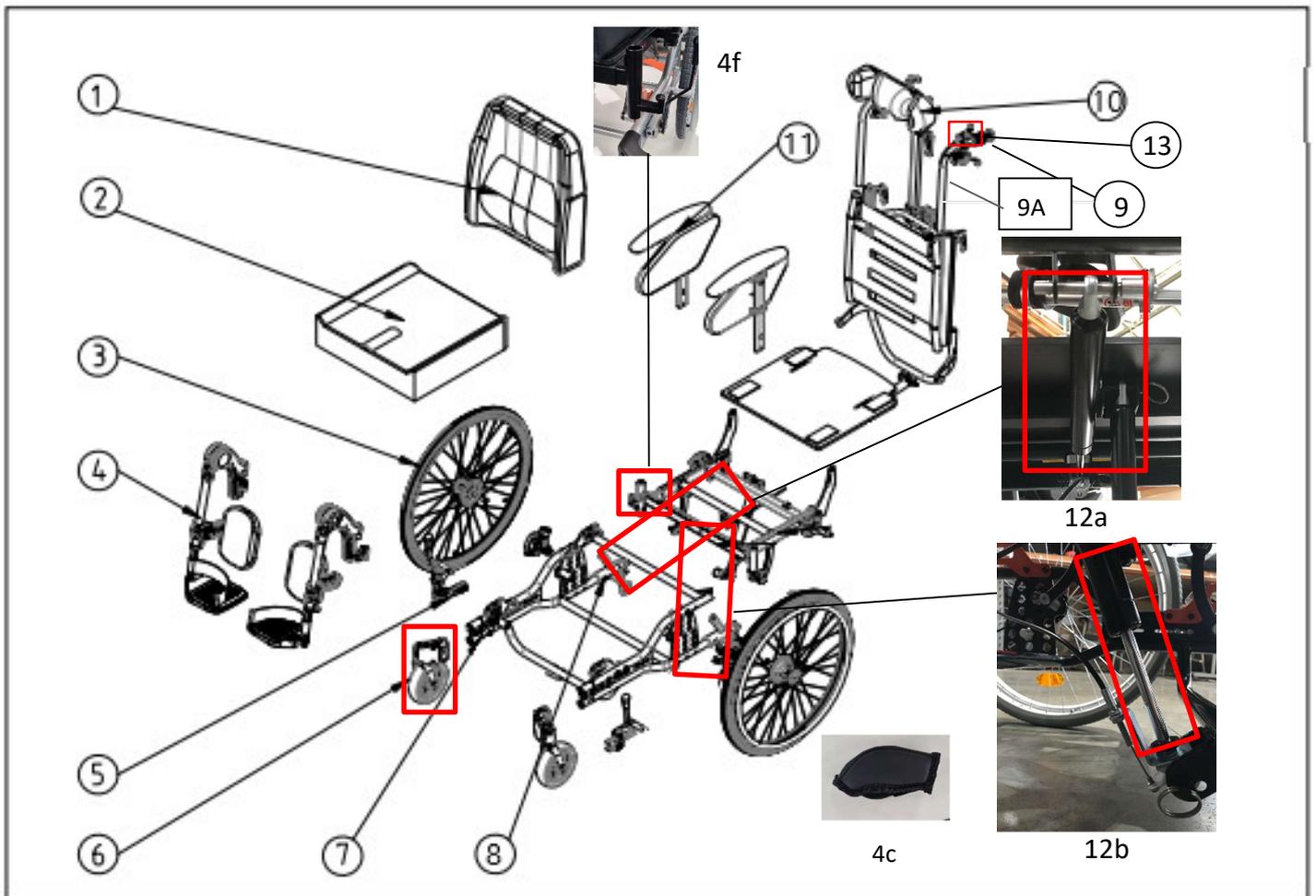
Rehab RX



Rehab RX JNR

Aidacare Code	Aidacare Description
MWS449630	Aspire Rehab RX JNR – 320 to 370mm
MWS449710	Aspire Rehab RX – 410 to 460mm
MWS449720	Aspire Rehab RX – 460 to 510mm
MWS449740	Aspire Rehab RX – 510 to 560mm
MWS449750	Aspire Rehab RX2 JUNIOR - 320mm
MWS449760	Aspire Rehab RX2 - 410mm
MWS449770	Aspire Rehab RX2 - 460mm
MWS449780	Aspire Rehab RX2 - 510mm

Exploded Diagram



Please Read:

- Aspire Rehab RX is a Steel Frame and RX2 is an Aluminium Frame.
- Most spare parts are compatible between RX & RX2 Wheelchairs except for the spare parts marked RX2 in this Spare Parts Manual.
- please see below for RX2 Spare parts, (also added in further pages in the Spare parts manual):

WCA848542	WChair Depth Extension Kit - 200mm - Silver - Aspire Rehab RX2
GSP492882	WChair Gas Strut - Tilt - Aspire Rehab RX2
WCA848544	WChair Depth Extension Kit - 150mm - Black - Aspire Rehab RX2
GSP492884	WChair Backrest Cane - Aspire Rehab RX2
GSP492886	WChair Gas Strut - Recline - Aspire Rehab RX2

Accessories



14



15



16



4d



17

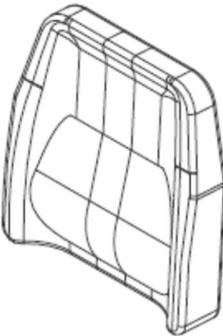
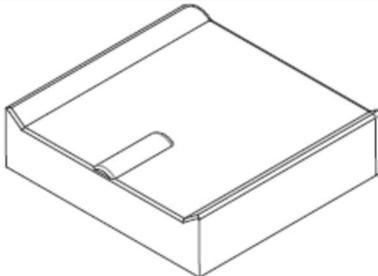


18



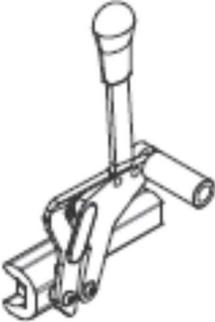
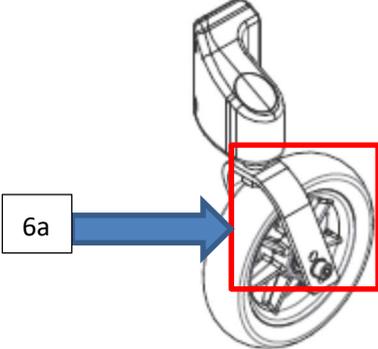
19

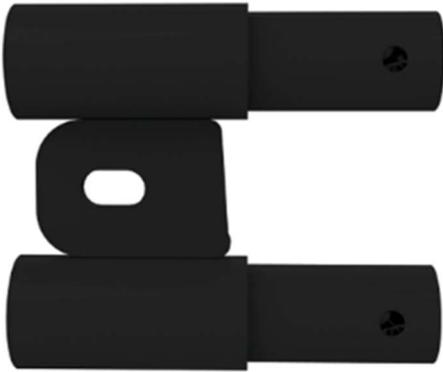
Spare Parts List

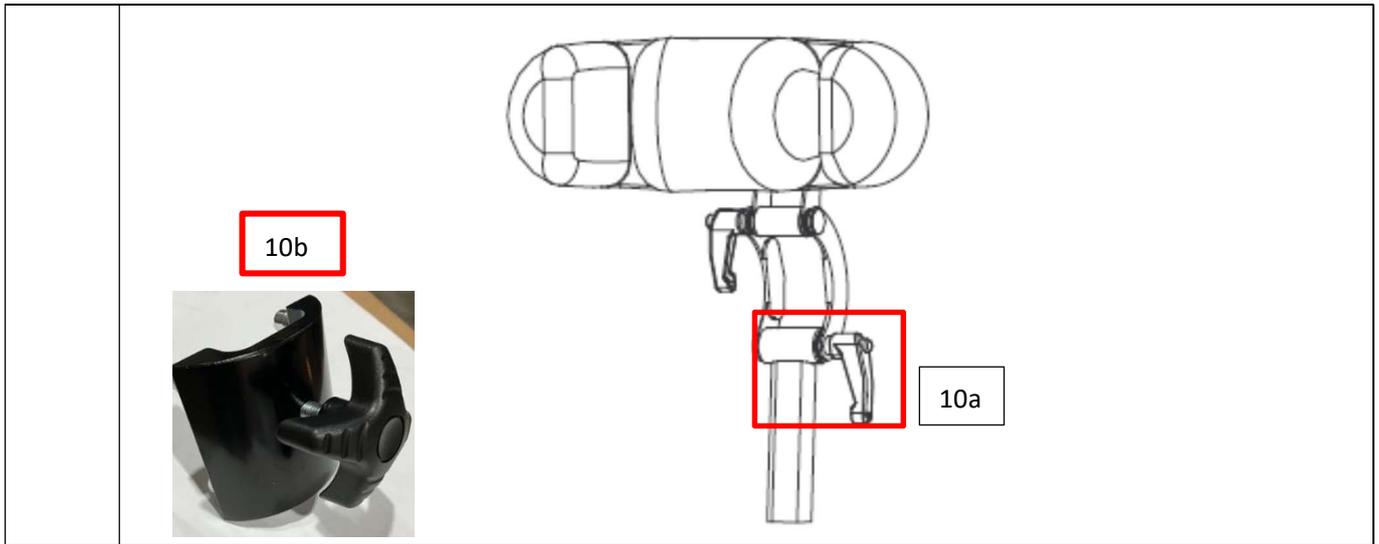
Pos.	item no.	Item Description	Type
1	GSP492650	Back Cushion – 320mm – RX JNR	Spare Part
1	GSP492660	Back Cushion – 410mm – RX	Spare Part
1	GSP492670	Back Cushion – 460mm – RX	Spare Part
1	GSP492680	Back Cushion – 510mm – RX	Spare Part
1a	GSP492720	Back Cushion Cover – 320mm –RX Junior	Spare Part
1a	GSP492730	Back Cushion Cover – 410mm – RX	Spare Part
1a	GSP492740	Back Cushion Cover – 460mm – RX	Spare Part
1a	GSP492750	Back Cushion Cover – 510mm – RX	Spare Part
			
2	GSP492510	Seat Cushion – 320mm – RX JNR	Spare Part
2	GSP492520	Seat Cushion – 410mm – RX	Spare Part
2	GSP492530	Seat Cushion – 460mm – RX	Spare Part
2	GSP492540	Seat Cushion – 510mm – RX	Spare Part
2a	GSP492580	Seat Cushion Cover – 320mm – RX JNR	Spare Part
2a	GSP492590	Seat Cushion Cover – 410mm – RX	Spare Part
2a	GSP492600	Seat Cushion Cover – 460mm – RX	Spare Part
2a	GSP492610	Seat Cushion Cover – 510mm – RX	Spare Part
			
3	WCA848525	Drum Brake Kit – 24in Wheels – RX/RX JNR	Accessory
3	WCA848510	Drum Brake Kit - 16in Wheels – RX/RX JNR	Accessory
3	WCA848520	Drum Brake Kit - 20in Wheels – RX	Accessory
3a	GSP492400	Brake handle – Attendant – Left – RX/RX JNR	Spare Part
3a	GSP492402	Brake Handle – Attendant – Right – RX/RX JNR	Spare Part

3b	GSP492404	Brake Cable – RX/RX JNR	Spare Part
3c	GSP492442	Rear Wheel – 24in – Solid – RX	Spare Part
3c	GSP492436	Rear Wheel – 20in – Solid – RX/RX JNR	Spare Part
3d	GSP492464	Quick Release Axle – RX/RX JNR	Spare Part
3e	GSP492444	Wheelchair Rear Wheel Bushing	Spare Part
4	WCA848160	Elevating Legrest - Left - Advanced - 320 to 410mm – RX/RX JNR	Accessory
4	WCA848170	Elevating Legrest - Right - Advanced - 320 to 410mm – RX/RX JNR	Accessory
4	WCA848180	Elevating Legrest - Left - Advanced - 450 to 510mm – RX	Accessory
4	WCA848190	Elevating Legrest - Right - Advanced - 450 to 510mm – RX	Accessory
4	WCA848100	Elevating Legrest – Left – Standard - 320 to 410mm – RX/RX JNR	Accessory
4	WCA848110	Elevating Legrest – Right – Standard - 320 to 410 mm – RX/RX JNR	Accessory
4	WCA848120	Elevating Legrest – Left – Standard - 450 to 510 mm – RX	Accessory
4	WCA848130	Elevating Legrest – Right – Standard - 450 to 510 mm – RX	Accessory
4a	GSP492432	Elevating Legrest – Knee Pad – RX/RX JNR	Spare Part
4b	GSP492433	Calf Pad Assembly – Left – 410 to 510mm	Spare Part
4b	GSP492435	Calf Pad Assembly – Right – 410 to 510mm	Spare Part

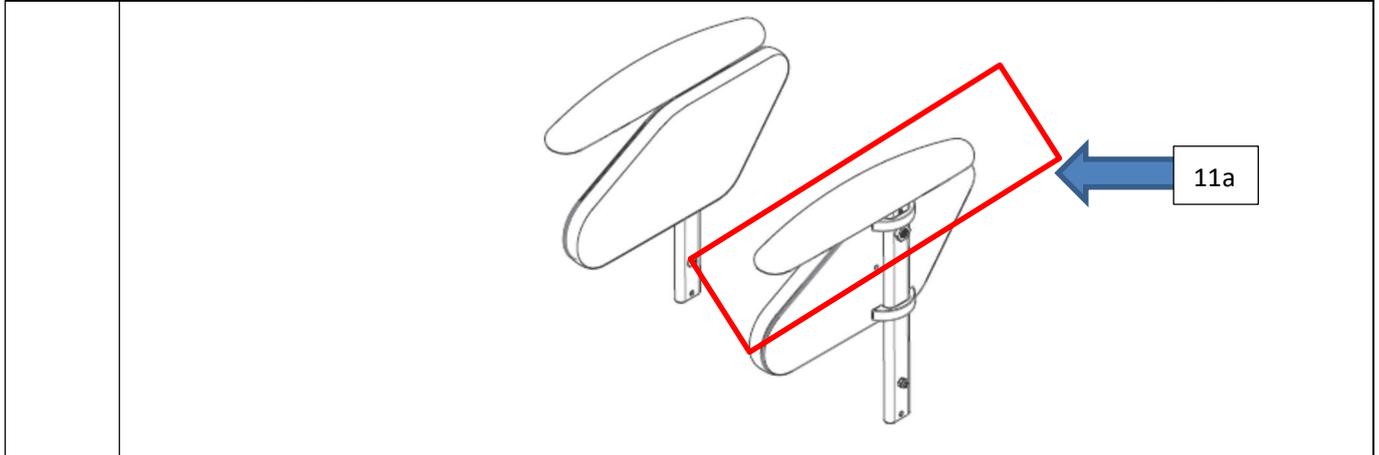
4b	GSP492366	Calf Pad Assembly – Left – 320 to 360mm	Spare Part
4b	GSP492365	Calf Pad Assembly – Right – 320 to 360mm	Spare Part
4c	GSP492431	Red Latch Plastic – for Adjusting Legrest – RX/RX JNR	Spare Part
4d	WCA848580	Calf Strap – RX/RX JNR	Accessory
4e	GSP492870	Soft Pad for Leg Rest, Left	Spare Part
4e	GSP492875	Soft Pad for Leg Rest, Right	Spare Part

4f	GSP492880	Elevating Legrest - Locking Arm	Spare Part
			
5	GSP492396	Hand Brake - Left – RX/RX JNR	Spare Part
5	GSP492398	Hand Brake - Right – RX/RX JNR	Spare Part
			
6	GSP492450	Front Wheel – Castor – RX/RX JNR	Spare Part
6a	GSP492456	Front Wheel Fork – RX/RX JNR	Spare Part
			
7	WCA848530	Rehab Depth Extension Kit - 50mm – RX	Accessory
7	WCA848540	Rehab Depth Extension Kit - 100mm – RX	Accessory
7	WCA848542	WChair Depth Extension Kit - 200mm - Silver - Aspire Rehab RX2	Accessory
7	WCA848544	WChair Depth Extension Kit - 150mm - Black - Aspire Rehab RX2	Accessory

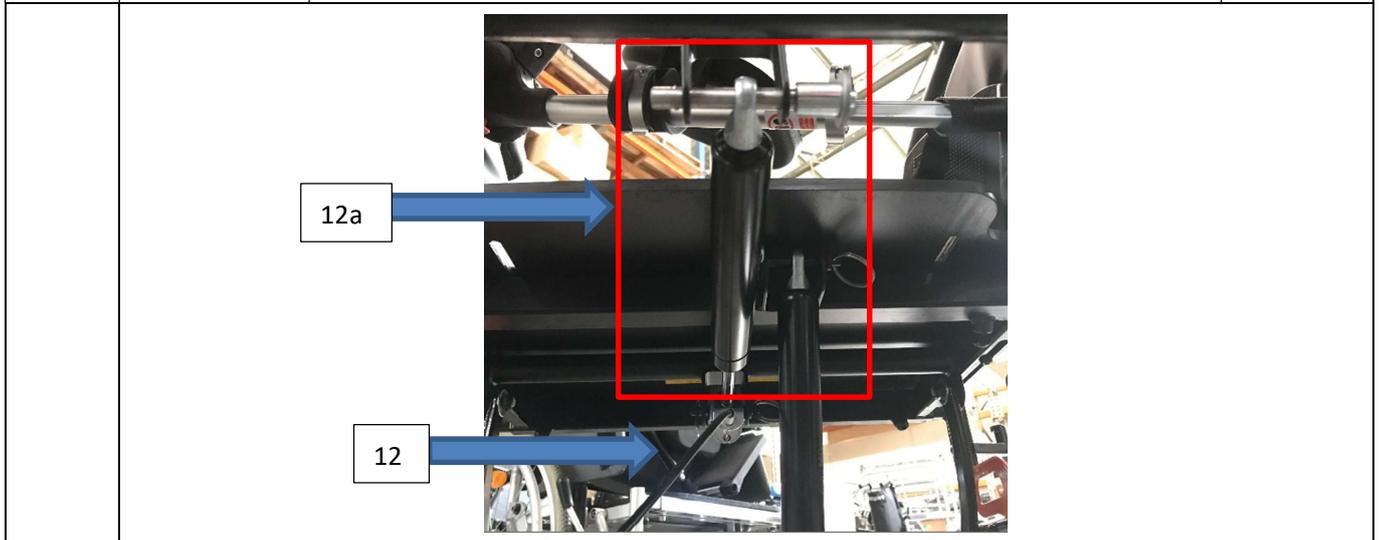
			
8	WCA848270	Anti-Tip Bar – RX/RX JNR	Accessory
			
9	GSP492842	Hand Grip – RX/RX JNR	Spare Part
			
9A	GSP492883	WChair Backrest Cane - Aspire Rehab RX	Spare Part
9A	GSP492884	WChair Backrest Cane - Aspire Rehab RX2	Spare Part
10	GSP492852	Headrest Assembly – RX/RX JNR	Spare Part
10a	GSP492845	Wchair Headrest Adjustment Clamp Lever - Aspire Rehab RS/RX	Spare Part
10b	GSP492853	WChair Headrest Mounting Bracket with Handwheel and Bolts	Spare Parts



11	GSP492380	Armrest Assembly - Left – RX/RX JNR	Spare Part
11	GSP492382	Armrest Assembly - Right – RX/RX JNR	Spare Part
11a	GSP492386	Armrest Pad – RX/RX JNR	Spare Part



12	GSP492840	Gas Strut Cable – RX/RX JNR	Spare Part
12a	GSP492836	Gas Strut – Recline – RX/RX JNR	Spare Part
12a	GSP492886	WChair Gas Strut - Recline - Aspire Rehab RX2	Spare Part



12c	GSP492835	Release Plate for Gas Strut	Spare Part
12D	GSP491730	Mounting Pin and Hardware – for Gas Strut	Spare Part
12E	GSP491732	Mounting Plate – for Gas Strut	Spare Part
12b	GSP492838	Gas Strut – Tilt – RX/RX JNR	Spare Part
12b	GSP492882	WChair Gas Strut - Tilt - Aspire Rehab RX2	Spare Part
13	GSP492834	Tilt & Recline Lever Assembly – RX/RX JNR	Spare Part

14	GSP492858	Tool Kit – RX/RX JNR	Spare Part
			
15	WCA848280	IV Pole – RX/RX JNR	Accessory
			
16	WCA848300	Lap Belt – Metal Seatbelt Buckle – RX/RX JNR	Accessory
			
17	WCA848550	Therapist Kit for Hand Adjustment – RX/RX JNR	Accessory
			

18	WCA848560	Tray – 320 to 410mm – RX/RX JNR	Accessory
18	WCA848570	Tray – 460 to 510mm – RX	Accessory
			
19	WCA848220	Stump Support – Left – Silver – 320 to 450mm – RX/RX JNR	Accessory
19	WCA848230	Stump Support – Right – Silver – 320 to 450mm – RX/RX JNR	Accessory
19	WCA848240	Stump Support – Left – Silver – 500 to 610mm – RX	Accessory
19	WCA848250	Stump Support – Right – Silver – 500 to 610mm – RX	Accessory
			
20	GSP492843	Backrest Cane Stopper – RX/RX JNR	Spare Part
			

