


Spare Parts Breakdown - Aspire Shell Chairs

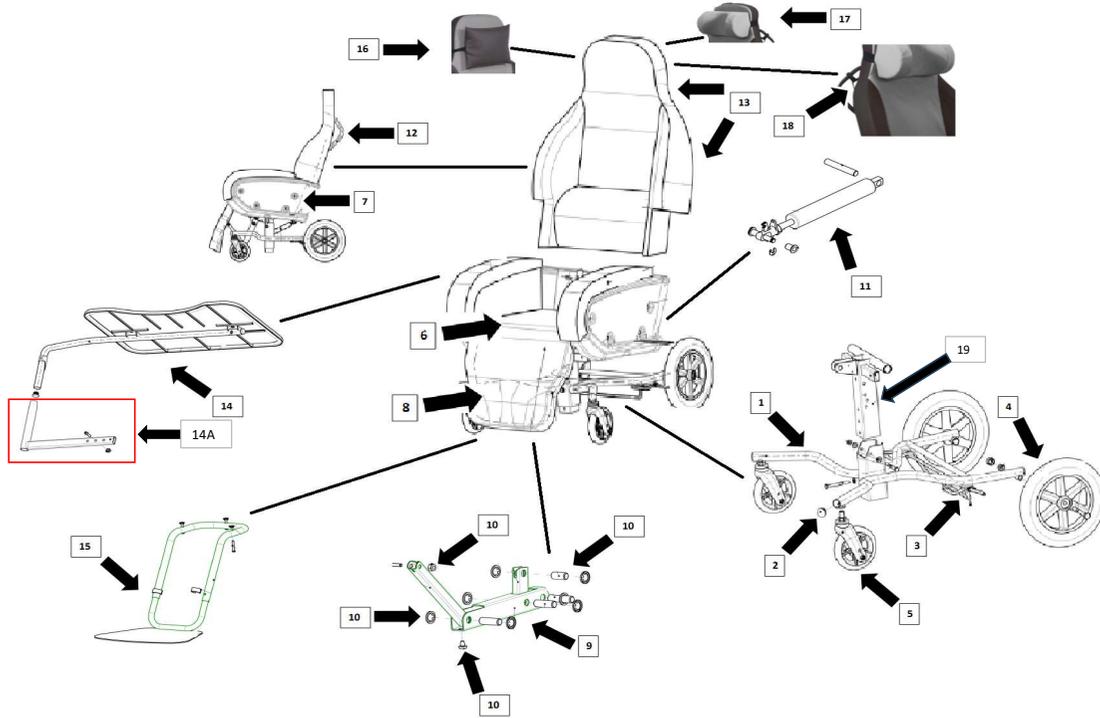

CHP198910	Aspire Shell Chair - 450mm Wide
CHP198915	Aspire Shell Chair - 510mm Wide
1	GSP401100 Complete Base Frame (No Wheels)
2	GSP401105 Base Frame End Cap
3	GSP401110 Brake Pedal & Hardware
3	GSP401185 Brake Level Spring
4	GSP401115 Rear Buggy Wheel & Mounting Hardware
5	GSP401120 Front Castor Wheel & Mounting Hardware
6	GSP401125 Seat Cushion - 450mm Wide
6	GSP401130 Seat Cushion - 510mm Wide
7	GSP401170 Replacement Cover - 450mm - Whole Piece
7	GSP401171 Replacement Cover - 510mm - Whole Piece
8	GSP401145 Legrest Cushion Assembly & Mounting Hardware - 450mm
8	GSP401146 Legrest Cushion Assembly & Mounting Hardware - 510mm
9	GSP401150 Legrest Pivot Arm Assembly & Mounting Hardware
10	GSP401155 Legrest Pivot Arm Mounting Hardware
11	GSP401160 Gas Strut & Mounting Hardware
12	GSP401165 Push Handle & Mounting Hardware
13	GSP401170 Replacement Cover - 450mm
13	GSP401171 Replacement Cover - 510mm
14	CHP198920 Front Tray (optional accessory)
14A	GSP401200 Mounting Hardware Bracket - for Front Tray
15	CHP198925 Footplate Kit (optional accessory)
16	CHP198930 Pillow Headrest (optional accessory)
17	GSP401175 Contoured Headrest
18	GSP401180 Tilt Lever & Cable Assembly
19	GSP401202 Vertical Pillar - Aspire Shell Chair