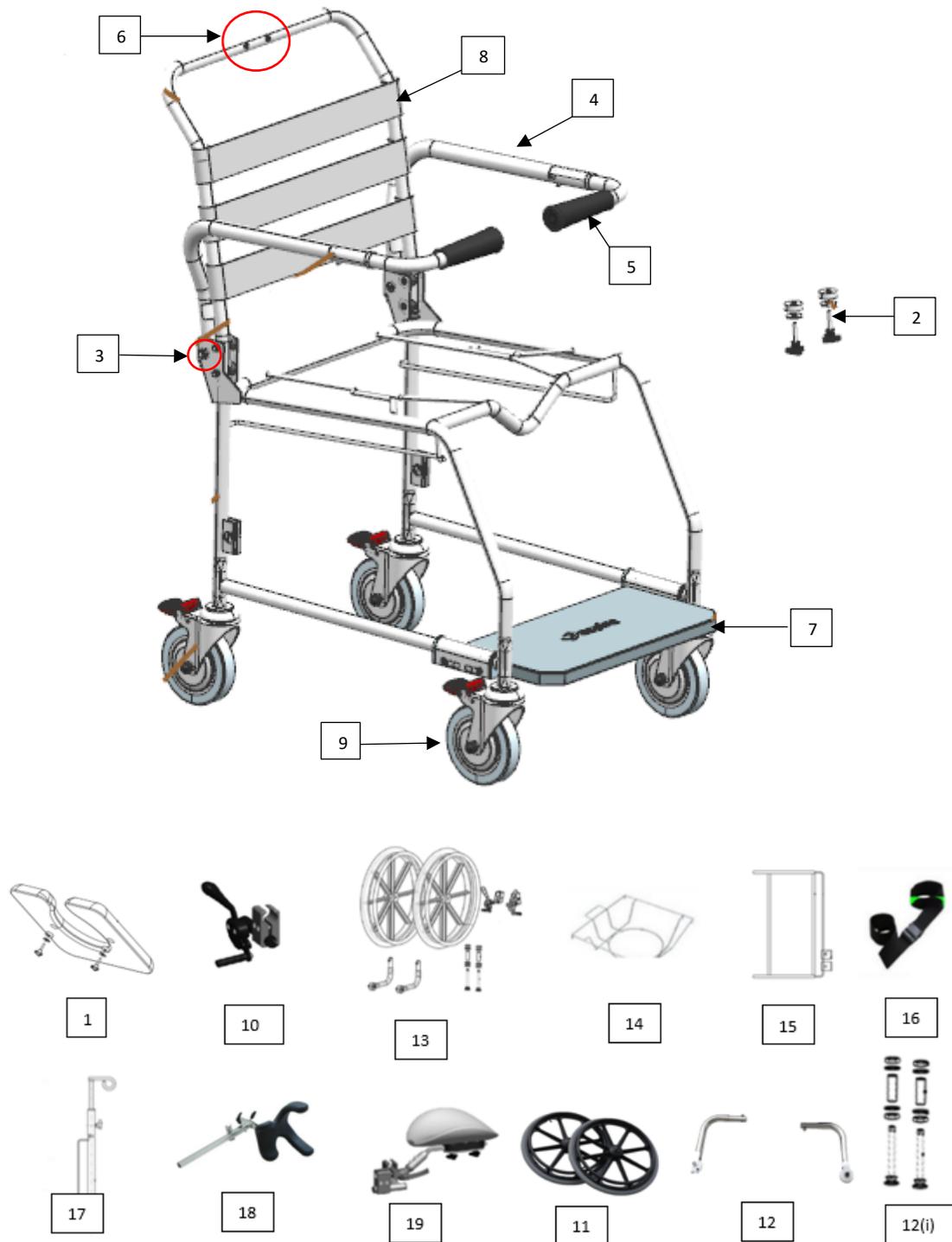


**ASPIRE 460 & 530MM WEIGHT BEARING PLATFORM SHOWER  
COMMODE (200KG CAPACITY)**

**Spare Parts Manual**

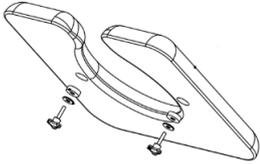
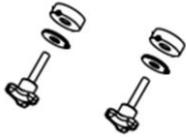
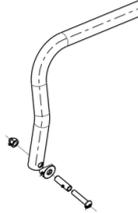
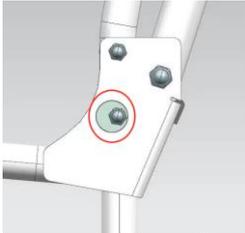


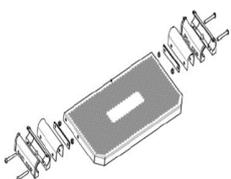
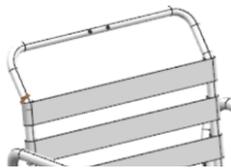
## Aspire Shower Commode Weight Bearing – Exploded Parts Diagram

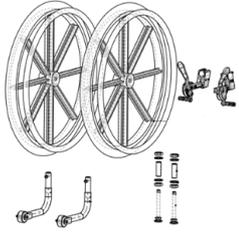
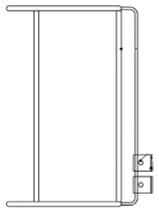


Aidacare Product Code	Aidacare Product Description
BTC066000	Shower Commode – Aspire - Weight Bearing - 460mm – Attendant Propelled
BTC066200	Shower Commode – Aspire – Weight Bearing – 460mm – Self Propelled
BTC066050	Shower Commode – Aspire – Weight Bearing – 530mm – Attendant Propelled
BTC066055	Shower Commode – Aspire – Weight Bearing – 530mm – Self Propelled

## Spare Parts List

Ref #	Item No.	Item Description	Type
1	BTC066600	Shower Commode Seat - Aspire - Open 460mm	Accessory
1	BTC066700	Shower Commode Seat - Aspire - Closed 460mm	Accessory
1	BTC066750	Shower Commode Seat - Aspire - Open 530mm	Accessory
			
2	GSP342405	Tri Nut Assembly – M6 – Used to fix seat to frame – Small Packet	Spare Part
			
3	GSP342412	Armrest Mounting Kit – NOT TIS or 600mm WB Model	Spare Part
			
3	GSP342398	Backrest Angle Assembly Kit	Spare Part
			
4	GSP342465M	Armrest Tube Replacement – Left - 460/530mm WB – Matte Finish	Spare Part
4	GSP342466M	Armrest Tube Replacement – Right - 460/530mm WB – Matte Finish	Spare Part
4	GSP342465S	Armrest Tube Replacement – Left - 460/530mm WB – Shiny Finish	Spare Part
4	GSP342466S	Armrest Tube Replacement – Right - 460/530mm WB – Shiny Finish	Spare Part
			

5	GSP342409M	Safety Swing Armrest (Handle section only) – Left – Matte Finish	Spare Part
5	GSP342411M	Safety Swing Armrest (Handle section only) – Right –Matte Finish	Spare Part
5	GSP342409S	Safety Swing Armrest (Handle section only) – Left – Shiny Finish	Spare Part
5	GSP342411S	Safety Swing Armrest (Handle section only) – Right – Shiny Finish	Spare Part
			
6	GSP342399	Plug Rubber – for Headrest Mount	Spare Part
			
7	GSP342428	WB Platform Assembly – 460mm WB	Spare Part
7	GSP342439	WB Platform Assembly – 530mm WB	Spare Part
			
8	GSP342449	Backrest Strap 1pce with Rivets – 460mm	Spare Part
8	GSP342450	Backrest Strap 1pce with Rivets – 530mm	Spare Part
			
9	GSP342416	Braking Castor – Red – 460, 530 & 600mm (not TIS/600mm WB)	Spare Part
9	GSP342416T	Braking Castor w Expander Adaptor - TENTE - 460/530/600mm (not TIS/600mm WB)	Spare Part
			
10	GSP342422	Brake Assembly – Left – for Self-Propelling Commode	Spare Part
10	GSP342423	Brake Assembly – Right – for Self-Propelling Commode	Spare Part
			

11	GSP342390	Self Propelling Wheel – 1pce - 22 inch	Spare Part
			
12	GSP342391	Anti-Tippers - Pairs	Spare Part
			
12(i)	GSP342392	Quick Release Axle – for Self Propelling Kit	Spare Part
			
13	BTC082900	Self-Propelling Kit – 22in Wheels	Accessory
			
14	BTC066400	Bowl & Pan Carrier – 460/530/600mm	Accessory
			
15	BTC067200	Oxygen Bottle Carrier	Accessory
			

16	BTC067100	Pelvic Strap – Adjustable	Accessory
			
17	BTC067103	IV Pole – Adjustable	Accessory
			
18	BTC067050	Advanced Headrest	Accessory
			
19	BTC067105	Stump Support (Left/Right) – Clamp On	Accessory
			