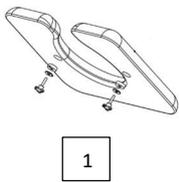
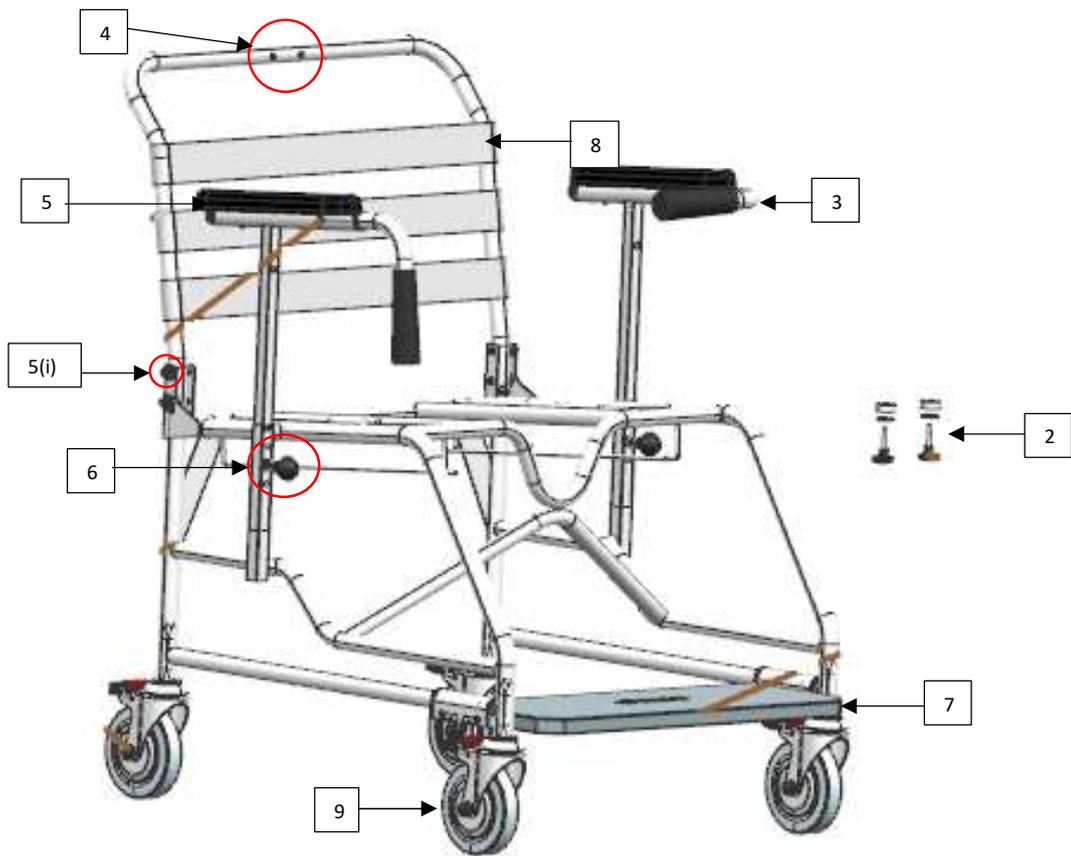


**ASPIRE 600MM WEIGHT BEARING PLATFORM SHOWER COMMODE  
(400KG CAPACITY)**

**Spare Parts Manual**



**Aspire Shower Commode Weight Bearing – Exploded Parts Diagram**



1



10



11



12



13



14



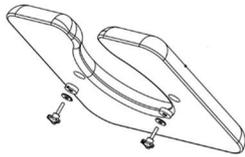
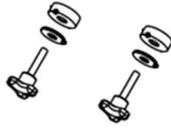
15

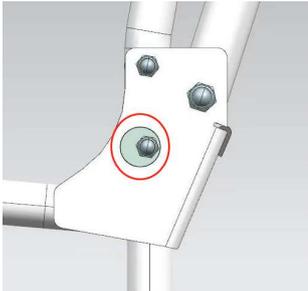
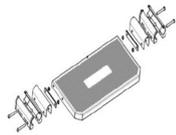
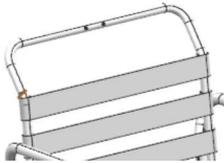


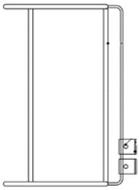
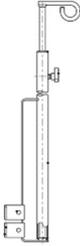
16

Aidacare Product Code	Aidacare Product Description
BTC061400	Shower Commode – Aspire – Weight Bearing – 600mm – Attendant Propelled

## Spare Parts List

Ref #	Item Number	Item Description	Type
1	BTC066800	Shower Commode Seat - Aspire - Open 600mm	Accessory
			
2	GSP342405	Tri Nut Assembly – M6 - Used to fix seat to frame – Small Packet	Spare Part
			
3	GSP342409M	Safety Swing Armrest (handle section only) – Left – Matte Finish	Spare Part
3	GSP342411M	Safety Swing Armrest (handle section only) – Right – Matte Finish	Spare Part
3	GSP342409S	Safety Swing Armrest (handle section only) – Left – Shiny Finish	Spare Part
3	GSP342411S	Safety Swing Armrest (handle section only) – Right – Shiny Finish	Spare Part
			
4	GSP342399	Plug Rubber – for Headrest Mount	Spare Part
			
5	GSP342426M	Armrest with Arm Pad – Left – 600mm WB (BTC061400) – Matte	Spare Part
5	GSP342427M	Armrest with Arm Pad – Right – 600mm WB (BTC061400) – Matte	Spare Part
5	GSP342426S	Armrest with Arm Pad – Left – 600mm WB (BTC061400) – Shiny	Spare Part
5	GSP342427S	Armrest with Arm Pad – Right – 600mm WB (BTC061400) - Shiny	Spare Part
			

5(i)	GSP342398	Backrest Angle Assembly Kit	Spare Part
			
6	GSP342421	Armrest Spring Lock Bolt – 600mm WB only (BTC061400)	Spare Part
			
7	GSP342429	WB Platform Assembly – 600mm WB	Spare Part
			
8	GSP342447	Backrest Strap 1 pce with Rivets – 600mm WB only (BTC061400)	Spare Part
			
9	GSP342386	Braking Castor – Red - 600mm WB only (BTC061400)	Spare Part
			
10	BTC067000	Armrest Padded Offset – 600mm WB only (BTC061400)	Accessory
			
11	BTC066400	Bowl & Pan Carrier – 460/530/600mm	Accessory
			

12	BTC067200	Oxygen Bottle Carrier	Accessory
			
13	BTC067100	Pelvic Strap – Adjustable	Accessory
			
14	BTC067103	IV Pole – Adjustable	Accessory
			
15	BTC067050	Advanced Headrest	Accessory
			
16	BTC067105	Stump Support (left/right) – Clamp on	Accessory
			