


Spare Parts Breakdown - Aspire Shower Stool

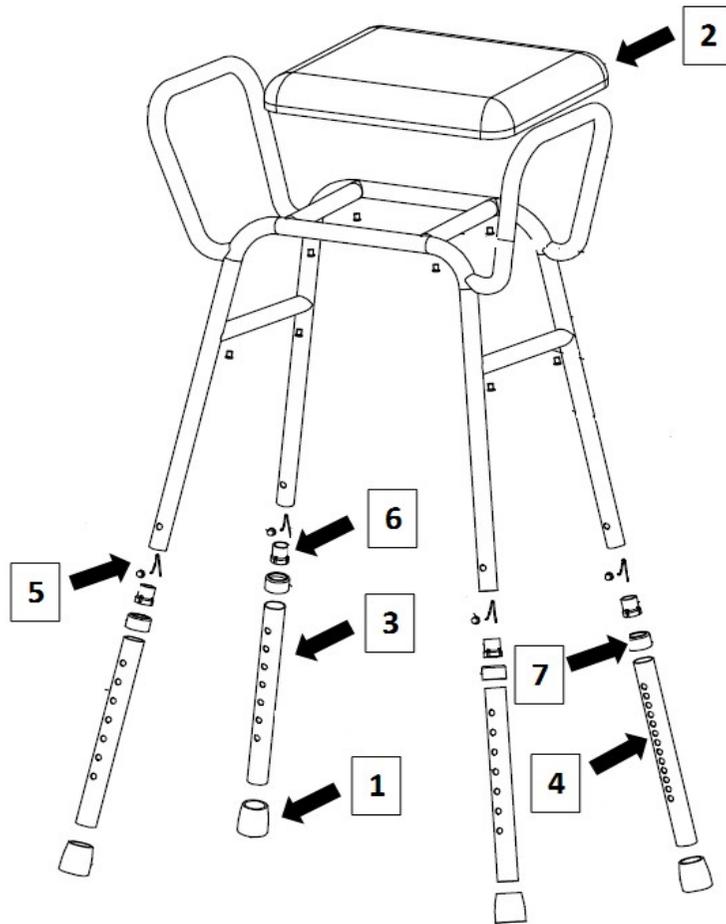

BTS119000	Shower Stool w/Arms - Aspire - 495mm - Treated Steel - 175kg
BTS119005	Shower Stool w/Arms - Aspire - 495mm - Aluminium - 175kg
BTS119010	Shower Stool w/Arms - Aspire - Wide - 525mm - Treated Steel - 175kg
BTS119015	Shower Stool w/Arms - Aspire - Wide - 525mm - Aluminium - 175kg
BTS119020	Shower Stool w/Arms and Padded Seat - Aspire - 495mm - Treated Steel - 175kg
BTS119025	Shower Stool w/Arms and Padded Seat & Back - Aspire - 495mm - Treated Steel - 175kg

1	GSP390800	Rubber Ferrule/Tip
2	BTS119100	Clip on Plastic Seat - Standard
2	BTS119105	Clip on Plastic Seat - Wide (not pictured)
3	GSP390802	Height Adjustable Leg (Single Holes)
4	GSP390803	Height Adjustable Leg (Double Holes)
5	GSP390797	Push Button & Spring
6	GSP390796	Plastic Leg Insert
7	GSP390801	Plastic Silencer
8	BTS119110	Shower Stool w/Arms - Clip on Padded Seat Only - Aspire - White
9	BTS119115	Clip On Padded Backrest (not pictured)