


Aspire Socialite - Spare Parts Diagram

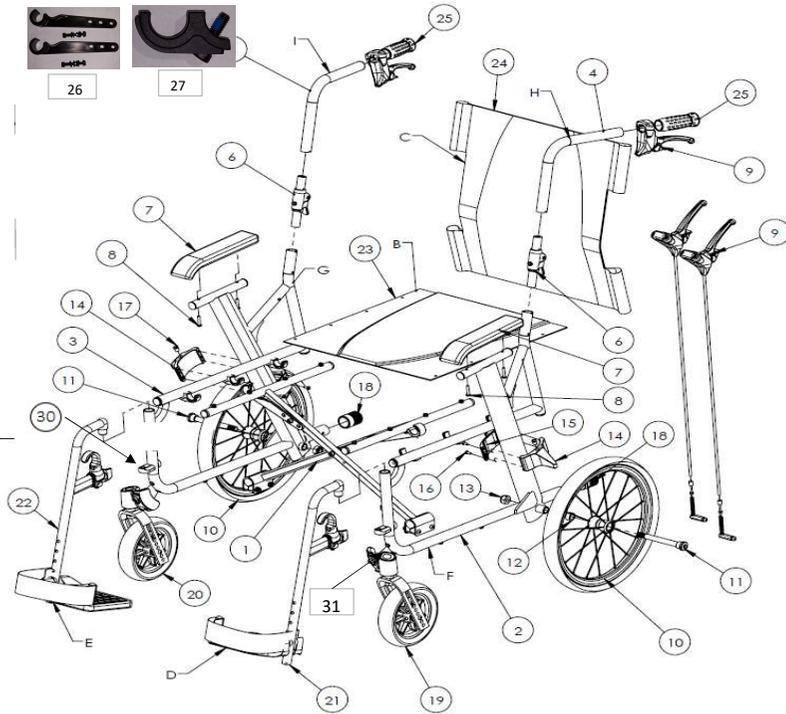

GSP491652 and GSP491653 - Brake Handle / Cable and Pad Assembly - Aspire SOCIALITE SP Wheelchairs.
OLD / NEW Design Brakes are compatible and Interchangeable.
 only LHS or RHS brakes can also be replaced if needed to.

Aidacare Code	Aidacare Description
MWS449800	WChair Folding - Aspire SOCIALITE AP - 450mm - 115kg - Silver ^
MWS449810	WChair Folding - Aspire SOCIALITE AP - 450mm - 115kg - Red ^
MWS449820	WChair Folding - Aspire SOCIALITE SP - 450mm - 115kg - Silver ^
MWS449830	WChair Folding - Aspire SOCIALITE SP - 450mm - 115kg - Red ^

Item	Aidacare Code	Aidacare Description
4,25	GSP491605	Backrest Cane with Brake and Grip Handle - Aspire SOCIALITE - Left - Red
4,25	GSP491610	Backrest Cane with Brake and Grip Handle - Aspire SOCIALITE - Left - Silver
4,25	GSP491615	Backrest Cane with Brake and Grip Handle - Aspire SOCIALITE - Right - Red
4,25	GSP491620	Backrest Cane with Brake and Grip Handle - Aspire SOCIALITE - Right - Silver
6	GSP491625	Mounting Hardware - Aspire SOCIALITE - For Backrest Cane - Old Models with Dimple
6	GSP491626	Mounting Hardware - Aspire SOCIALITE - For Backrest Cane - LEFT - V3
6	GSP491627	Mounting Hardware - Aspire SOCIALITE - For Backrest Cane - RIGHT - V3
7,8	GSP491630	Armrest Pad and Mounting Hardware - Aspire SOCIALITE
10	GSP491635	Rear Wheel Assembly - Aspire SOCIALITE - Attendant Prop
10	GSP491638	Rear Wheel Assembly - Aspire SOCIALITE - Self Prop
11	GSP491640	Bolt and End Cap - Aspire SOCIALITE - for Rear Wheel
11	GSP491637	Quick Release Axle - Aspire SOCIALITE - Self Prop
12,13	GSP491645	Mounting Washer Assembly - Aspire SOCIALITE - for Rear Wheel
9,14,15,16,17	GSP491650	Brake Handle / Cable and Pad Assembly - Aspire SOCIALITE - Attendant Prop
9,14,15,16,17	GSP491652	Brake Handle / Cable and Pad Assembly - Aspire SOCIALITE - LEFT - Self Prop (see details below)
9,14,15,16,17	GSP491653	Brake Handle / Cable and Pad Assembly - Aspire SOCIALITE - RIGHT - Self Prop (see details below)
18	GSP491655	End Cap - Aspire SOCIALITE - for End Stopper Tube
19	GSP491660	Front Castor and Fork Assembly - Aspire SOCIALITE - Left
20	GSP491665	Front Castor and Fork Assembly - Aspire SOCIALITE - Right
21	GSP491670	Swing Away Legrest - Aspire SOCIALITE - Left - Silver
22	GSP491675	Swing Away Legrest - Aspire SOCIALITE - Right - Silver
21	GSP491680	Swing Away Legrest - Aspire SOCIALITE - Left - Red
22	GSP491685	Swing Away Legrest - Aspire SOCIALITE - Right - Red
23	GSP491690	Seat Upholstery - Aspire SOCIALITE - Red
24	GSP491695	Backrest Upholstery - Aspire SOCIALITE - Red
23	GSP491700	Seat Upholstery - Aspire SOCIALITE - Silver
24	GSP491705	Backrest Upholstery - Aspire SOCIALITE - Silver
25	GSP492842	Wchair Handgrip
26	GSP491710	Hook Cross Bar - Aspire SOCIALITE
27	GSP491715	Mounting Seat Hardware - Aspire SOCIALITE
30	GSP491718	Legrest Hanger Boss - w/Washer and Bolt - Aspire SOCIALITE Wheelchair
31	GSP492453	Wchair End Cap - for Front Wheel Fork - Aspire Evoke2 / Socialite